

# Quick Reference Guide

## DPM™ | 4



**Note:** The DPM4 Quick Reference Guide is not intended as a replacement to the Operating Instructions. Prior to operating equipment, the user must be familiar with the Operating Instructions Manual contents. This document is a guideline only to be used as an aid to comprehensive Inservice Training.

# Table of Contents

---

Admit a Patient without Patient Information .....	2
Admit a Patient and Enter Patient Information .....	2
Discharge a Patient .....	2
Set a NIBP Interval .....	3
Set Alarm Limits .....	3
Silence Active Alarm(s) .....	4
Pause All Alarms .....	4
Set Screen Configuration (Standard, Trends, OxyCRG, or Large Fonts) .....	4
Change Waveform Configuration .....	4
Change Waveform Order .....	5
Initiate Standby .....	5
Resume Monitoring from Standby .....	5
Record Real Time Recording Strip .....	5
Record Trend Table .....	6
Record NIBP Measurements .....	6
Record Alarm Events .....	6

## Admit a Patient without Patient Information

Using Control knob on control panel

- Scroll and select **Menu**
- Scroll and select **Patient Setup**
- Select **Quick Admit**
- Select **Yes** to confirm clearing prior data and
- Select **Patient type** (size)
- Select **OK** to confirm and return to normal screen

## Admit a Patient and Enter Patient Information

Using Control knob on control panel

- Scroll and select **Menu**
- Scroll and select **Patient Setup**
- Select **Admit**
- Select **Yes** to confirm clearing prior data and
- Scroll and select to enter patient information
- Select **OK** to confirm and return to normal screen

## Discharge a Patient

Using Control knob on control panel

- Scroll and select **Menu**
- Scroll and select **Patient Setup**
- Select **Discharge Patient (Clear Patient Data)**
- Select **Yes** to confirm clearing prior data and return to normal screen

**Note:** Clear Patient Data is present if patient information is omitted.

## Set a NIBP Interval

Using Control knob on control panel

- Scroll and select NIBP tile
- Scroll and select **Interval**
- Scroll to select time
- Scroll and select **Exit** to return to normal screen

## Set Alarm Limits

Using Control knob on control panel (Option 1)

- Scroll and select Parameter tile
- Scroll and select **Alm HI/Alm LO**
- Scroll to adjust alarm limit
- Scroll and select **Exit** to return to normal screen

OR

Using Control knob on control panel (Option 2)

- Scroll and select **Menu**
- Scroll and select **System Setup**
- Scroll and select **Alarm Setup**
- Select **Alm Sel**
- Select parameter's **Alm Setup**
- Scroll and select **Alm HI/Alm LO**
- Scroll to adjust alarm limit
- Scroll and select **Exit** to return to normal screen

## Silence Active Alarm(s)

Press and hold Silence button >2 seconds. If new alarms occur, system silence will be terminated

## Pause All Alarms

Press Silence button. All alarms are paused for a preconfigured time (1, 2, or 3 minutes). **All visual and audio alarm notification is suspended.** Alarm notification will resume if Silence is pressed a second time or once the preconfigured time has elapsed.

## Set Screen Configuration (Standard, Trends, OxyCRG, or Large Fonts)

Using Control knob on control panel

- Scroll and select **Menu**
- Scroll and select **System Setup**
- Scroll and select **Face Select**
- Scroll to select screen
- Scroll and select **Exit** to return to normal screen

## Change Waveform Configuration

Using Control knob on control panel

- Scroll and select **Menu**
- Scroll and select **System Setup**
- Scroll and select **Trace Setup**
- Scroll and select waveform(s) to be displayed (A ✓ in the waveform box indicates waveform is selected)

## Change Waveform Order

Using Control knob on control panel

- Scroll and select **Menu**
- Scroll and select **System Setup**
- Scroll and select **Trace Setup**
- Scroll and select **Wave Sequence**
- Scroll to select channel to change
- Select parameter
- Scroll and select to **Exit** to return to normal screen

## Initiate Standby

Using Control knob on control panel

- Scroll and select **Standby**
- Scroll and select **Yes** to confirm

## Resume Monitoring from Standby

Press any key on front panel and Scroll and select **Yes** to confirm

## Record Real Time Recording Strip

- Select Record key on control panel
- Select Record key to stop (if continuous mode is enabled)

## Record Trend Table

Using Control knob on control panel

- Scroll and select **Menu**
- Scroll and select **Trend Table**
- Scroll and select **Resolution**
- Scroll to adjust time interval
- Scroll and select **REC** to record
- Scroll and select **Exit** to return to normal screen

## Record NIBP Measurements

Using Control knob on control panel

- Scroll and select **Menu**
- Scroll and select **NIBP Recall**
- Scroll and select **REC** to record
- Scroll and select **Exit** to return to normal screen

## Record Alarm Events

Using Control knob on control panel

- Scroll and select **Menu**
- Scroll and select **Alarm Recall**
- Scroll to adjust **Start/End** time
- Scroll and select **Alarm Recall Event** to select sorting criteria
- Scroll and select **Alarm Recall** to view alarm event or **UP-Down** to view other alarm events
- Scroll and select **REC** to record
- Scroll and select **Exit** to return to normal screen