

Diagnostic Ultrasound System DICOM Quick Guide FAQ Manual

Intellectual Property Statement

©2014-2021 Shenzhen Mindray Bio-medical Electronics Co., Ltd. All Rights Reserved.

Content

1. Preset.....	5
1.1 DICOMLocal Preset.....	5
1.2 DICOM Service Preset	7
1.2.1 DICOM Storage Preset.....	7
1.2.2 DICOM Print Preset	9
1.2.3 Worklist Preset.....	11
1.2.4 MPPS Preset	13
1.2.5 Storage Commitment Preset.....	15
1.2.6 Query/Retrieve Preset	17
1.2.7 HL7 Query Preset	18
2. Services	19
2.1 Verify Connectivity	19
2.2 DICOM Storage	20
2.2.1 Storage Operation.....	20
2.2.2 Structured Report (SR)	24
2.2.3 Encapsulated PDF	25
2.2.4 Send DCM File.....	26
2.2.5 Common Reasons of Failure and Troubleshooting	27
2.3 DICOM Print.....	28
2.3.1 Send to DICOM Print Manually.....	28
2.3.2 Send to DICOM Print Automatically	29
2.4 DICOM Worklist	30
2.5 HL7Query.....	31
2.6 MPPS.....	32
2.7 Storage Commitment	32
2.8 Query/ Retrieve.....	32
2.9 DICOM Media Storage and Review.....	34
2.9.1 DICOM Media Storage.....	34
2.9.2 DICOMDIR Review	35
2.10 Task Management	36
3. Common Failures and Troubleshooting.....	36
3.1 Common Failure: Service Verification Failed.....	36
3.2 Common Failure: Server disconnects the ultrasound system during implementing storage	37
3.3 Common Failure: Multiframe transmission is failed, but single frame is succeeded.	37

3.4	Common Failure: Network disconnected (shown by network icon) during transmission	37
3.5	Common failure: Worklist query succeeded, but no patient is shown	38
3.6	Common failure: Printing succeeded but the film is blank.....	38
3.7	Common Failure: Verification of connecting DICOM printer is succeed but printing is failed.....	38
3.8	Common Failure: Printed image is too small.....	39
3.9	Common Failure: MPPS message is not sent out correctly or not received by the server	39
3.10	Common Failure: Storage commitment is failed.....	39
3.11	Common Failure: Query is succeeded, but retrieve is failed, with the prompting that the remote server is wrong.....	40
3.12	Common Failure: Error messages in DICOM Task Management	41
3.12.1	Association aborts	41
3.12.2	Network error: Socket sending error	42
3.13	Print Service Examples.....	42
3.13.1	Example 1	42
3.13.2	Example 2	45

Note: Before using DICOM, please read the electronic file DICOM CONFORMANCE STATEMENT along with the device. At the same time, please verify that the SCU services provided by ultrasound machine can be supported by the SCP services provided by the systems such like PACS/RIS/HIS and so on.

The chapter is confined to the preset, connection verification and DICOM services of the DICOM-configured ultrasound machine, not including SCP configurations like PACS/RIS/HIS.

The DICOM package is optional, so the description here is only applicable for the system configured with the DICOM package.

This system supports the following DICOM functions:

- Verify Connectivity
- DICOM Storage
- DICOM Print
- DICOM Worklist
- MPPS (Modality Performed Procedure Step)
- Storage Commitment
- DICOM Medium Storage (DICOMDIR Review)
- DICOM Task Management

Workflows of DICOM Applications are briefly described as follows:

1. DICOM/HL7 preset (Localhost DICOM Service Property, Server Setting, and Set DICOM Service).
2. Verify connectivity.
3. DICOM application (Image Storage, Print, MPPS, Worklist, Storage Commitment, DICOMDIR Review, Q/R and Task Management).

Abbreviation introduction:


The followings are the abbreviations in this chapter.

Abbreviation	Description
DICOM	Digital Imaging and Communications in Medicine
AE	Application Entity
MPPS	Modality Performed Procedure Step
PDU	Protocol Data Unit
SCU	Service Class User (DICOM client)
SCP	Service Class Provider (DICOM server)
SOP	Service-Object Pair
SR	Structured Report
HL7	Health Level 7

1. Preset

1.1 DICOMLocal Preset

To open the DICOMLocal preset screen:

1. Select  on the upper right corner of the screen, and select [Setup] on the pop-up menu to enter setup screen.
2. Select [Network] page to open the DICOM Local screen, you can set up the Local host DICOM Service Property, Device Settings, as shown in Figure 1 DICOM Local Preset Screen

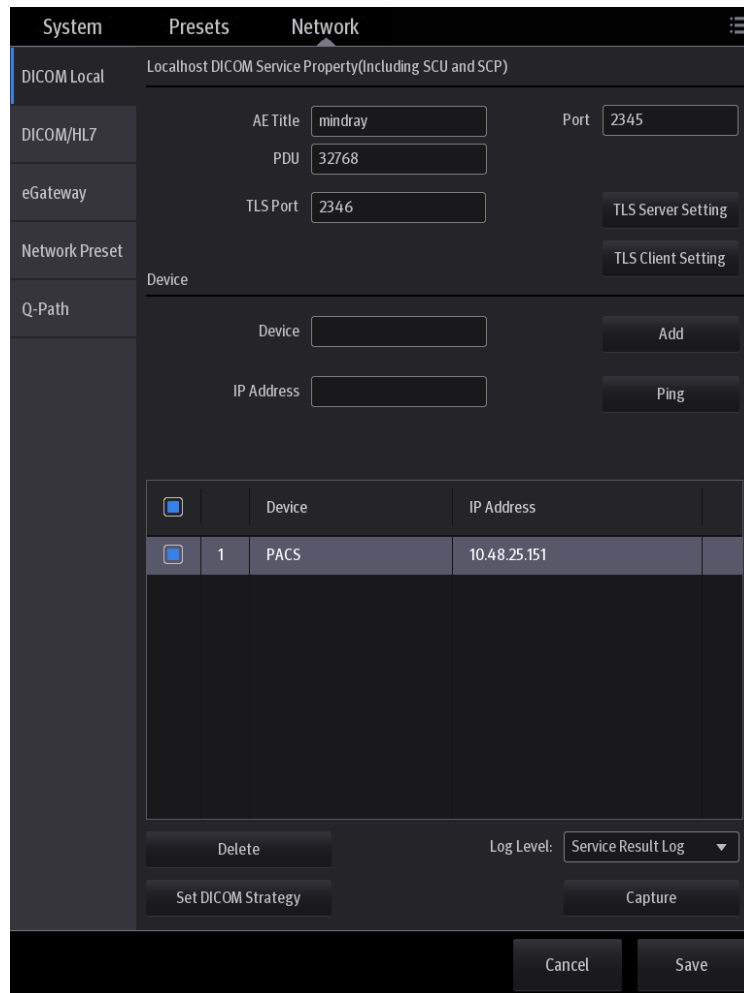


Figure 1 DICOM Local Preset Screen

The main parameters of DICOM Local preset are described as follows:

Name	Description
AE Title	Application Entity title.
Port	DICOM communication port, 2345 is default.

TLS port	DICOM TLS communication port,2346 is default.
PDU	Maximum PDU data package size, ranging from 16384 to 65536; if the value is less than 16384 or greater than 65536, the system automatically sets it to the value 32768.
TLS Server Setttng	Sever-side certificate management.
TLS Client Setttng	<p>Verify Certificate: The precondition of the DICOM TLS services. The Dicom services checked TLS will Verify Certificate in term of selected "Verify Certificate".</p> <p>Import: Pop "Import TLS certificates" dialogue, the Type combobox value have PEM and CER.</p> <p>Delete: Delete the imported TLS certificates.</p>
Device	Name of the device supporting DICOM services.
IP Address	IP address of the server.
Ping	Check whether IP Address can connect normally or not.
Add	Press to add server (s) to the Device List.
Delete	Press to delete the selected server (s) in the device list.
Log Level	Set log level about dicom services, including No log、 Service Result Log、 Service Process Log、 All Log.
Set DICOM Strategy	Set DICOM strategy for doppler to be compatible with other vendors' PACS
Capture	Capture the dicom network packet.


In the above screen, please enter AE Title, Port, and PDU according to the actual situation, and then press [Save] to exit the screen.

Tip:

- AE Title should be the same with the SCU AE Title preset in the server (PACS/RIS/HIS), for example, if the AE Title of the server preset in the print server is Printer, and the AE Title of the accepted SCU is preset as Machine, then in the figure above, the AE Title of Local should be Machine, and the AE Title of print server should be Printer.
- The device name can be entered to your preference. If the currently entered name has already existed, the system will show a tip: "The server name exists!" Press[OK] to enter another name.
- Port must not be set as 4001, 6000, 3001, 6555.
- IP address must be the actual IP address of the server.
- Set DICOM Strategy: The DICOM strategy must be configured by qualified personnel with good knowledge of DICOM standards. The qualified personnel must ensure the validity of the DICOM strategy. DICOM strategy support input/output.
- Capture: Mainly used for engineers to grasp packet data and locate problems.

1.2 DICOM Service Preset

To open the DICOM/HL7 Service screen:

1. Select  on the upper right corner of the screen, and select [Setup] on the pop-up menu to enter setup screen.
2. Select [Network] page, select [DICOM/HL7] to set the attributes of services.

After DICOM is installed and DICOM Worklist, MPPS, and Query/Retrieve are optionally configured, there will be related screen for presetting.

1.2.1 DICOM Storage Preset

Figure 2 DICOM Storage Preset Screen

The main parameters of DICOM Storage are described as below:

Name	Description
Devices	After you set the server (s) in DICOMLocal preset screen, the name (s) will appear in the drop-down list, select the name of the storage server.
Service name	Default is xxx-Storage, user-changeable.

AE Title	Application Entity title, here, it should be consistent with that of the storage server.
Port	DICOM communication port, 104 is default. Here, the port should be consistent with that of the storage server port.
Maximum Retries	0—9 times are optional, the default is 3, if the DICOM storage task that is sent to this server is failed, the task will be retried automatically, the retry times is set here.
Interval Time (Sec)	5—1800 seconds, 15 seconds is default. (Reserved)
Timeout (Sec)	Refers to the amount of time after which the system will stop trying to establish a connection to the service. Value: 5-60s, in increments of 5s, and 15 is default.
TLS	Transmission encryption when it is selected.
Color Mode	Three options are available, Color means the images sent from system to server will be coded by 24bit, Mixed means the images sent from system to server will be coded by 8bit or 24 bit depends on whether the images themselves are captured in color imaging mode or colorize switch is in "on" state or not, if they do so, will be coded by 24bit, else 8bit. Gray means the images sent from system to server will be coded by 8bit. The default is Color.
Compression Mode	Uncompressed(The image is uncompressed), RLE(The image is lossless compressed), JPEG(Please refer to the information of provider to confirm whether SCP can support JPGE compression. If not, please do not select it) are optional, the default is JPEG.
Compression Ratio	High, Middle and Low, three kinds of Compression Ratio are optional. Compression Ratio is in inverse proportion to image quality.
SR Storage Option	Three options are available when Structured Reporting is needed to send. You can select "Attach SR When Store Images", "Only Store SR", or "Not Store SR".
Encapsulated PDF	Encapsulate a PDF document within the DCM file. If SCP supports this function, then select it.
Allow Multiframe	If SCP supports this function, then select it.
Max Framerate	25,30,35, Full are available, user-changeable
Storage Mode	Parallel file, Parallel frame are optional, Parallel file is default. Please refer to the information of SCP to confirm whether SCP can support empty Basic Offset Table (zero length) of encapsulated Multi-Frame image. If not, please do not select it
Transducer Tracking	Record the probe S/N number in the dcm files sent to the server
Strategy Name	Set dicom Strategy for chosen dicom services.
Add	Add the DICOM service to the service list.
Cancel	Press to cancel parameter preset.
Update	Select an item in the service list, change the parameters in the above area, and press [Update] to update the item in the service

	list.
Delete	Press to delete the selected service in the service list.
Verify	Press to verify if the two DICOM application entities are normally connected.
Default	Select an item in the service list, press [Default] and you can see "Y" in the Default column.
Cancel	Press to exit the setup screen.
Save	Press to save settings and exit the setup screen.

Tip:

- JPEG is one of the storage types, but not all SCPs can support it. Please refer to the electronic file DICOM CONFORMANCE STATEMENT to check whether SCP supports this function.
- AE Title and Port must be consistent with those of server.
- Select the item in service list to modify the parameters and select [Update] to update them.
- You can press [Delete] to delete the item in service list.

1.2.2 DICOM Print Preset

The screenshot shows the DICOM Print Preset configuration interface. The 'Network' tab is selected, and the 'Print' sub-tab is active. The configuration fields include:

- Device: PACS
- Service Name: PACS-Print
- AE Title: PRINTSCP
- Port: 104
- Maximum Retries: 3
- Interval Time(Sec): 15
- Timeout(Sec): 15
- Copies: 1
- Settings: RGB
- Film Orientation: PORTRAIT
- Film Size: 8INX10IN
- Display Format: STANDAR...
- Priority: HIGH
- Medium Type: PAPER
- Destination: MAGAZINE
- Trim: NO
- Min Density: 0
- Max Density: 300
- Magnification Type: NONE
- Strategy Name: (empty)
- Configuration Info: (empty)

At the bottom, there is a table with the following data:

	Device	Service Name	AE Title	Port	Default	
<input checked="" type="checkbox"/>	1	PACS	PACS-Print	PRINTSCP	104	N

Figure 3 DICOM Print Preset Screen

The main parameters of DICOM Print are described as below:

Name	Description
Devices	After you set the server (s) in DICOMLocal preset screen, the name (s) will appear in the drop-down list, select the name of the print server.
Service name	Default is xxx-Print, user-changeable.
AE Title	Application Entity title, here, it should be consistent with that of the print server.
Port	DICOM communication port, 104 is default. Here, the port should be consistent with that of the print server port.
Maximum Retries	0—9 times are optional, the default is 3. If the DICOM storage task that is sent to this server is failed, the task will be retried automatically, the retry times is set here.
Interval Time (Sec)	5—1800 seconds, 15 seconds is default. (Reserved)
Timeout (Sec)	Refers to timeout during association establishment. Value: 5-60s, in increments of 5s, and 15 is default.
TLS	Transmission encryption when it is selected.
Copies	Refer to copies of printed files. You can select among 1 through 5, or directly enter the numeral.
Settings	The system supports RGB (color printing) and MONOCHROME2 (black and white printing). Please select the type the printer supports.
Film Orientation	Select between LANDSCAPE and PORTRAIT.
Priority	High、 Middle、 Low
Film Size	Select film size among the selections listed in the drop down list.
Display Format	Specify quantity of printed files, e.g. STANDARD\2, 3 indicates 6 images are printed for each page.
Media Type	Specify print medium: Paper, Clear Film, Blue Film; select Blue Film or Clear Film for black and white printing; select Paper for color printing.
Trim	Specify whether you want a trim box to be printed around each image on the film: Yes or No.
Min Density	Enter the minimum density of the film. The default is 0.
Max Density	Enter the maximum density of the film. The default is 300.
Magnification Type	Select how the printer magnifies an image to fit the film. Replicate: interpolated pixels belong to duplicate of adjacent pixels); Bilinear: interpolated pixels are generated from bilinear interpolations between adjacent pixels; Cubic: interpolated pixels are generated from cubic interpolations between adjacent pixels; None: without interpolation.
Destination	Specify where the file is exposed: MAGAZINE (stored in the

	magazine), or, PROCESSOR (exposed in the processor).
Strategy Name	Set dicom Strategy for chosen dicom services.
Configuration Info	Enter configuration information in the field.
Add	Add the DICOM service to the service list.
Delete	Press to delete the selected service in the service list.
Cancel	Press to cancel parameter preset
Update	Select an item in the service list, change the parameters in the above area, and press [Update] to update the item in the service list.。
Verify	Press to verify if the two DICOM application entities are normally connected.
Default	Select an item in the service list, press [Default] and you can see "Y" in the Default column.
Cancel	Press to exit the setup screen.
Save	Press to save settings and exit the setup screen.

Tips:

The parameters preset should be meet the requirements of printer, for instances:

1. If the printer cannot support the [Film Size] of 8IN*10IN, please modify it that the printer can support.
2. It will implement color printing when [Settings] is set as RGB, black and white printing when it is set as MONOCHROME2.
3. For [Media Type], select Blue Film or Clear Film for black and white printing; select Paper for color printing.
4. Other parameters can use defaulted values. If there are special requirements for printer, please modify it.

1.2.3 Worklist Preset

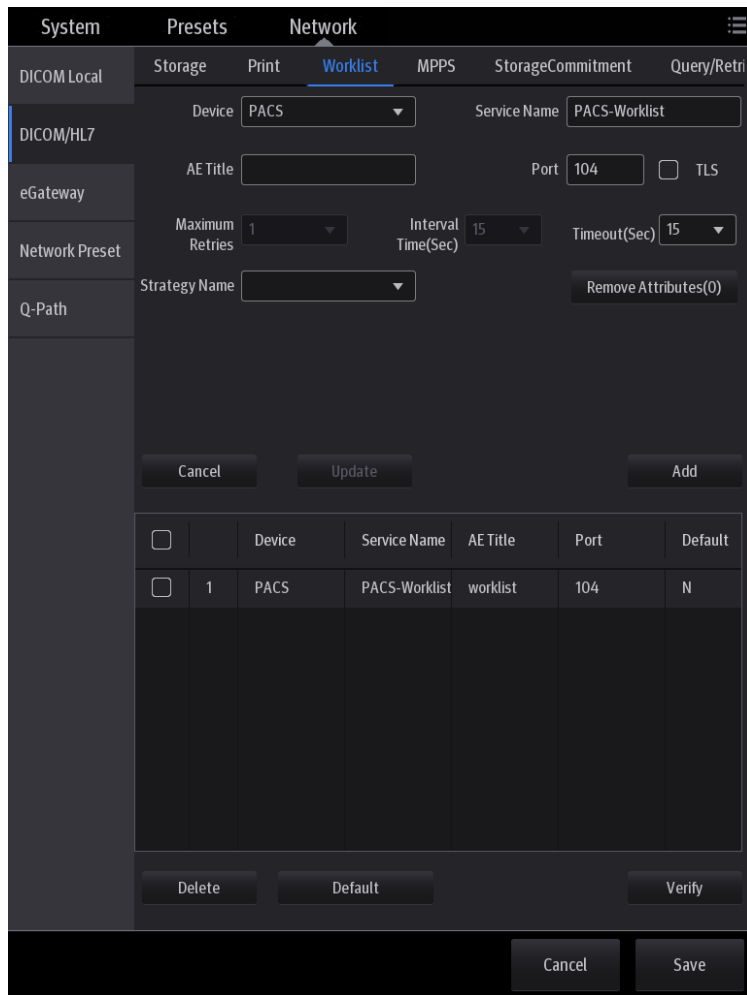


Figure 4 DICOM Worklist Preset Screen

The main parameters of DICOM Worklist are described as below:

Name	Description
Devices	After you set the server (s) in DICOMLocal preset screen, the name (s) will appear in the drop-down list, select the name of the Worklist server.
Service name	Default is xxx-Worklist, user-changeable.
AE Title	Application Entity title, here, it should be consistent with that of the Worklist server.
Port	DICOM communication port, 104 is default. Here, the port should be consistent with that of the Worklist server port.
Maximum Retries	0—9 times are optional, the default is 3. (Reserved)
Interval Time (Sec)	5—1800 seconds, 15 seconds is default. (Reserved)
Timeout (Sec)	Refers to timeout during association establishment. Value: 5-60s, in increments of 5s, and 15 is default.
TLS	Transmission encryption when it is selected.
Strategy Name	Set dicom Strategy for chosen dicom services.
Add	Press it to add the Worklist service to the service list.

Cancel	Press to cancel parameter preset.
Update	Select an item in the service list, change the parameters in the above area, and press [Update] to update the item in the service list.
Delete	Press to delete the selected service in the service list.
Verify	Press to verify if the two DICOM application entities are normally connected.
Default	Select an item in the service list, press [Default] and you can see "Y" in the Default column.
Cancel	Press to exit the setup screen.
Save	Press to save settings and exit the setup screen.

1.2.4 MPPS Preset

	Device	Service Name	AE Title	Port	Default	
<input type="checkbox"/>	1	PACS	PACS-MPPS	SCP	104	N

Figure 5 DICOM MPPS Preset Screen

The main parameters of DICOM MPPS are described as below:

Name	Description
Devices	After you set the server (s) in DICOMLocal preset screen, the name (s) will appear in the drop-down list, select the name of the MPPS server.

Service name	Default is xxx-MPPS, user-changeable.
AE Title	Application Entity title, here, it should be consistent with that of the MPPS server.
Port	DICOM communication port, 104 is default. Here, the port should be consistent with that of the MPPS server port.
TLS	Transmission encryption when it is selected.
Maximum Retries	0–9 times are optional, the default is 3. If the DICOM storage task that is sent to this server is failed, the task will be retried automatically, the retry times is set here.
Interval Time (Sec)	5–1800 seconds, 15 seconds is default. (Reserved)
Timeout (Sec)	Refers to timeout during association establishment. Value: 5-60s, in increments of 5s, and 15 is default.
Add	Add the DICOM service to the service list.
Cancel	Press to cancel parameter preset.
Update	Select an item in the service list, change the parameters in the above area, and press [Update] to update the item in the service list.
Delete	Press to delete the selected service in the service list.
Verify	Press to verify if the two DICOM application entities are normally connected.
Default	Select an item in the service list, press [Default] and you can see “Y” in the Default column.
Cancel	Press to exit the setup screen.
Save	Press to save settings and exit the setup screen.

Tip: You should set MPPS as default server when you use it.

1.2.5 Storage Commitment Preset

Figure 6 DICOM Commitment Preset Screen

The main parameters of storage commitment are described as below:

Name	Description
Devices	After you set the server (s) in DICOMLocal preset screen, the name (s) will appear in the drop-down list, select the name of the server.
Service name	Default is xxx-SC, user-changeable.
AE Title	Application Entity title, here, it should be consistent with that of the storage commitment server.
Port	DICOM communication port, 104 is default. Here, the port should be consistent with that of the storage commitment server port.
TLS	Transmission encryption when it is selected.
Maximum Retries	0—9 times are optional, the default is 3. (Reserved)
Interval Time	5—1800 seconds, 15 seconds is default. (Reserved)
Timeout	Refers to timeout during association establishment.

	Value: 5-60s, in increments of 5s, and 15 is default.
Associated Storage Service	The associated storage server is preset before storage commitment, only after the exam is sent out, can storage commitment be created.
Add	Add the DICOM service to the service list.
Cancel	Press to cancel parameter preset.
Update	Select an item in the service list, change the parameters in the above area, and press [Update] to update the item in the service list.
Delete	Press to delete the selected service in the service list.
Verify	Press to verify if the two DICOM application entities are normally connected.
Default	Unavailable
Cancel	Press to exit the setup screen.
Save	Press to save settings and exit the setup screen.

Tips:

1. Set storage server relating to storage commitment.
2. Storage commitment SCP sets the SCU port and IP corresponding to the port in DICOM Local screen and IP in Local Area Connection screen.

1.2.6 Query/Retrieve Preset

Figure 7 DICOM Query/Retrieve Preset Screen

The main parameters of Query/Retrieve are described as below:

Name	Description
Devices	After you set the server (s) in DICOMLocal preset screen, the name (s) will appear in the drop-down list, select the name of the storage commitment server.
Service name	Default is xxx-QueryRetrieve, user-changeable.
AE Title	Application Entity title, here, it should be consistent with that of the Query/Retrieve server.
Port	DICOM communication port, 104 is default. Here, the port should be consistent with that of the Query/Retrieve server port.
TLS	Transmission encryption when it is selected.
Maximum Retries	0—9 times are optional, the default is 3. (Reserved)
Interval Time	5—1800 seconds, 15 seconds is default. (Reserved)

Timeout	Refers to timeout during association establishment. Value: 5-60s, in increments of 5s, and 15 is default.
Add	Add the DICOM service to the service list.
Cancel	Press to cancel parameter preset.
Update	Select an item in the service list, change the parameters in the above area, and press [Update] to update the item in the service list.
Delete	Press to delete the selected service in the service list.
Verify	Press to verify if the two DICOM application entities are normally connected.
Default	Select an item in the service list, press [Default] and you can see "Y" in the Default column.
Cancel	Press to exit the setup screen.
Save	Press to save settings and exit the setup screen.

1.2.7 HL7 Query Preset

The screenshot displays the HL7 Query Preset configuration interface. It includes a left-hand navigation pane with categories such as System, Presets, Network, and HL7Query. The main configuration area is titled 'Network' and contains several input fields and dropdown menus for setting up the HL7 query parameters. These include 'Device' (set to PACS), 'Service Name' (PACS-HL7Query), 'AE Title', 'Port' (104), 'Maximum Retries' (0), 'Interval Time (Sec)' (15), and 'Timeout (Sec)' (15). There is also a 'Listen Port' field set to 304 and a 'Listen Mode' checkbox. Below the configuration fields, there are buttons for 'Cancel', 'Update', and 'Add'. At the bottom of the screen, there is a table with columns for 'Device', 'Service Name', 'AE Title', 'Port', and 'Default'. The table contains one entry with '1' in the first column, 'PACS' in the second, 'PACS-HL7Qu...' in the third, 'SCP' in the fourth, and 'N' in the fifth. Below the table are buttons for 'Delete' and 'Default'. At the very bottom of the screen, there are 'Cancel' and 'Save' buttons.

Figure 8 HL7Query Preset Screen

The main parameters of HL7Query are described as below:

Name	Description
Devices	After you set the server (s) in DICOMLocal preset screen, the name (s) will appear in the drop-down list, select the name of the HL7 server.
Service name	Default is xxx-HL7Query, user-changeable.
AE Title	Application Entity title, here, it's optional and empty by default.
Port	HL7 communication port, 104 by default. The port should be consistent with that of the HL7 server port.
Maximum Retries	0—9 times, and the default is 3. (Reserved)
Interval Time (Sec)	5—1800 seconds, and 15 seconds is default. (Reserved)
Timeout (Sec)	5-60s, in increments of 5s, and 15 is default. Timeout is the overtime of connecting Association, after the set time, TE7 will disconnect the TCP connection.
Lisen Port	HL7 Listen Port, the default is 304. This port is used to receive patient query result message.
Listen Mode	Under Listen Mode, Ultrasound Diagnostic System sends patient query message as client and receives query result from peer as server; Under non-Listen Mode, Ultrasound Diagnostic System sends and receives HL7 message in one TCP connection. The default is unselected.
Add	Press it to add the HL7 service to the service list.
Cancel	Press to cancel parameter preset changes.
Update	Select an item in the service list, change the parameters in the above area, and press [Update] to update the item in the service list.
Delete	Press to delete the selected service in the service list.
Default	Select an item in the service list, press [Default] and you can see "Y" in the Default column.
Cancel	Press to exit the setup screen.
Save	Press to save settings and exit the setup screen.

2. Services

2.1 Verify Connectivity

If you want to verify connectivity (it is not a must), you can press [Verify] button on Storage, Print or Worklist, MPPS, Storage Commitment and Query/Retrieve pages respectively. If the verification succeeded, the system prompts "xxx verification succeeded". Otherwise, it prompts "xxx verification failed", which means the connection is wrong. Please reconfirm it.

Tip:

Not all the SCPs can support verification; please consult SCP belongings to confirm whether SCP can support this service. If not, the verification won't pass.

2.2 DICOM Storage

DICOM Storage is used to send images (and SR) to DICOM storage server for remote storage. The entrances of storage include:

- ◆ Patient information management (iStation)
- ◆ Review status
- ◆ Auto Send Storage

Auto send storage DICOM storage supports:

- ◆ Single file transfer
- ◆ Exam transfer
- ◆ Batch transfer
- ◆ Static image and Cine file

Batch transfer means multiple exam records can be selected and sent for storage with one DICOM task for each exam. Multiple files in one exam will be sent in one DICOM connection.

The auto send function can be triggered by:

- ◆ Ending the exam (send to default storage service)
- ◆ Saving an image (image is sent to the default DICOM storage server as well as saved).

2.2.1 Storage Operation

The system supports automatic and manual storage.

The operations for manual storage service are described below:

Take "send to DICOM storage" in iStation page as an example, the operations are:

1. Configure the storage server according to the above descriptions about service preset.
2. Press [iStation] on the main screen to enter the patient information management page.
3. Select one or more exam records (that contain images. If not, the system will prompt), as shown in Figure 9 iStation Screen.

iStation

Find in results Filter All Data Source HD(D:)

<input type="checkbox"/>	ID	Name	Exam Type	Exam Date/Time ↓	Image	Clip
<input type="checkbox"/>	20141224-021404-6124	3	ABD	24/12/2014 02:14:25	2	0
<input type="checkbox"/>	20141224-011028-6124	2	ABD	24/12/2014 01:10:39	2	0
<input type="checkbox"/>	20141223-215813-6124	1	ABD	23/12/2014 22:08:01	3	0
<input checked="" type="checkbox"/>	20141222-165926-6124	Zhang,San	ABD	22/12/2014 16:59:26	2	0

Send To Delete Options Q/R Exit

Figure 9 iStation Screen

4. Press [Send To] to pop up the [Send To] dialog box.
5. Select the remote DICOM Server, press [Send], as shown in Figure 10 DICOM Send To Screen.

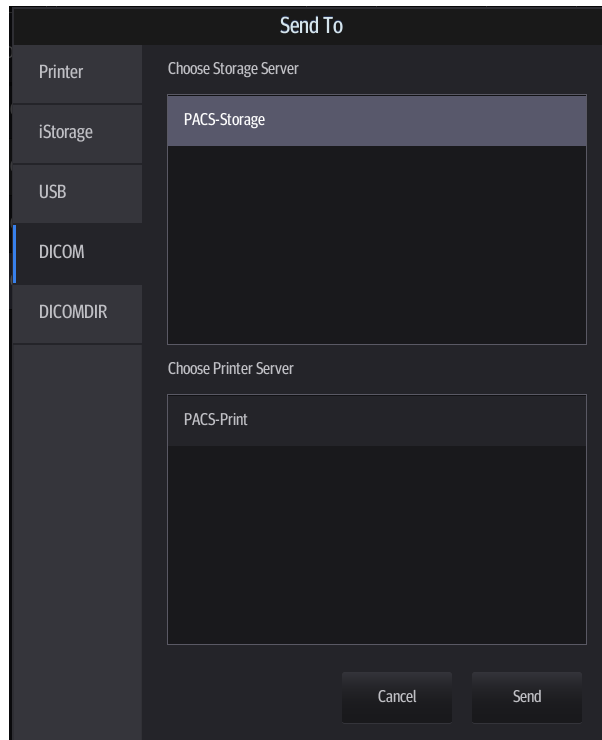



Figure 10 DICOM Send To Screen

6. Press  on the right corner of the screen to open Task Management, in which, you can review sending status. Select one record, and there is sending result in prompt dialog box, as shown in Figure 11 Task Management.

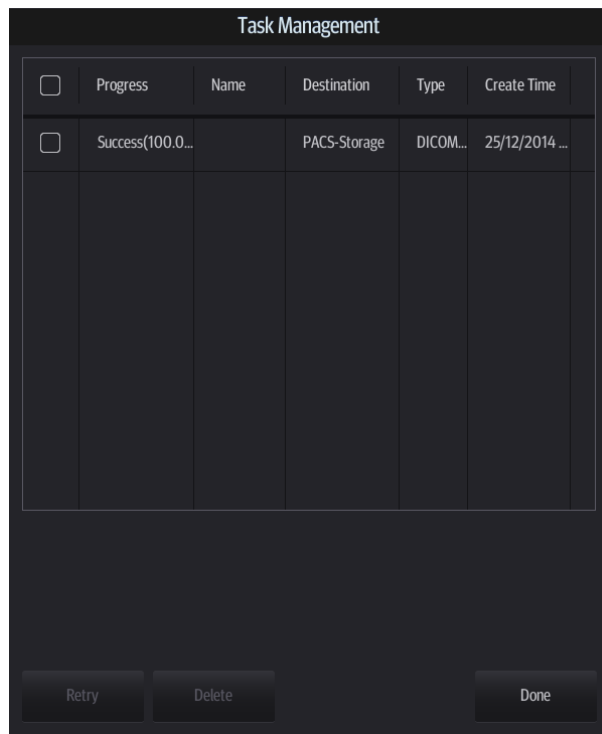


Figure 11 Task Management

7、 After sending images successfully, “Storage Commitment” column will be marked with "PACS received OK", otherwise, Storage Commitment” column be marked with "Send to PACS,OK".

Function paths of sending image to DICOM storage and sending images by user-defined key after ending exam automatically:

A. Send Image after end exam:

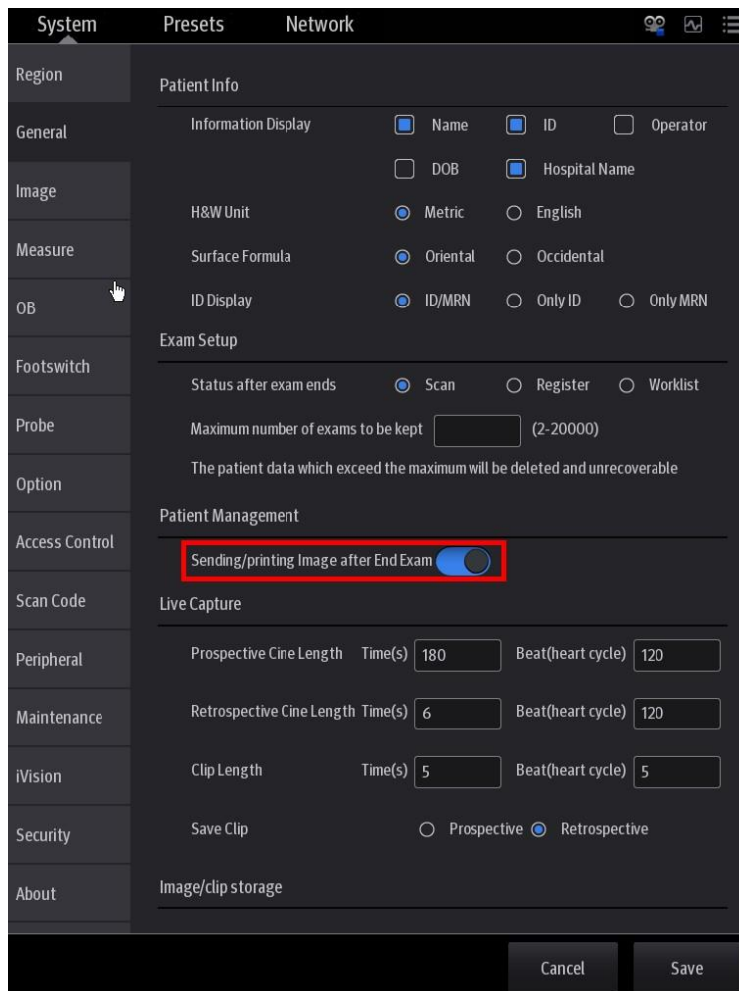


Figure 12 Send Image after End Exam

Steps:

1. In System Preset -> General, select “Sending/printing after End Exam”, as shown in Figure 12 Send Image after End Exam.
2. In DICOM/HL7 preset -> Storage Service, set the default storage server (one or more).
3. When pressing [End] with the current exam being active, all images (and SR) of the current exam are sent to the default DICOM storage server.

B. Send to DICOM Storage by user-defined shortcut key

1. In System Preset ->General, select “Send Image to DICOM Storage” and “Send Cine to DICOM Storage” while storing image/cine to hard disk, as shown in Figure 13 Send to DICOM Storage while storing image/cine.

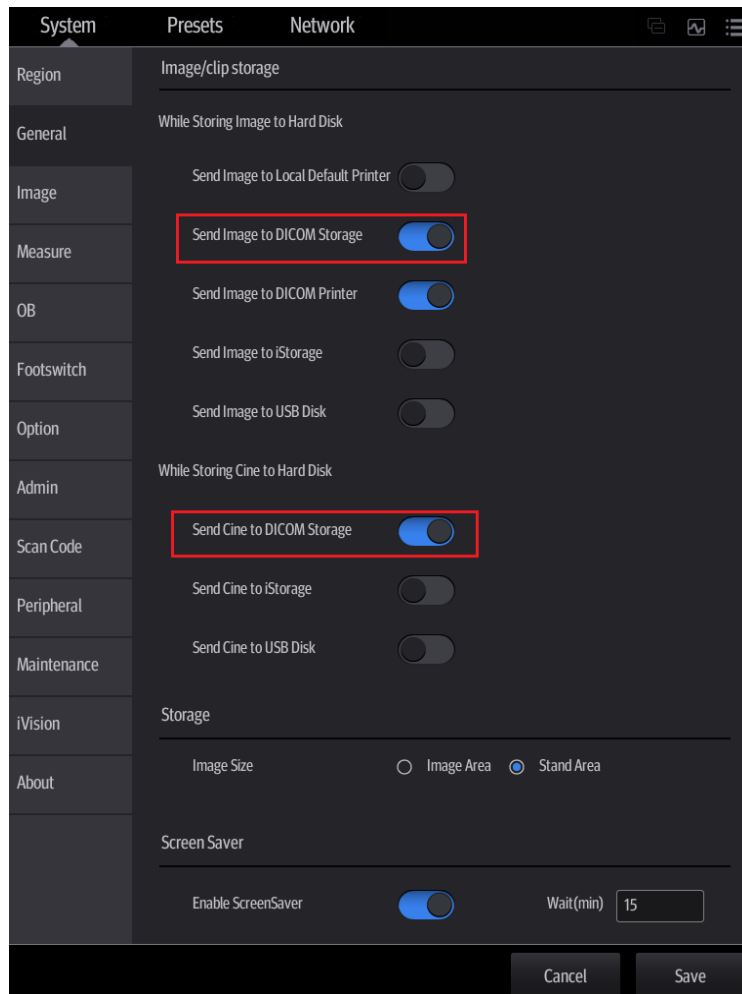


Figure 13 Send to DICOM Storage while storing image/cine

2. In DICOM/HL7 preset -> Print Service, set the default storage server (one or more).
3. In current exam, press [Save Image], the system will send a single-frame image to remote DICOM storage automatically. When pressing [Save Clip], the system will send a multi-frame image to remote DICOM storage

2.2.2 Structured Report (SR)

The system provides SRs in gynecology, obstetric, cardiac, vascular, breast and small parts exams that specified in the DICOM standard. The SR can not only be sent alone to a DICOM server but also be sent along with an exam record that has image(s) saved. The type of SR to be sent is determined by the application region of the exam:

- Send obstetric SR: gynecology and obstetrics exam;
- Send cardiac SR: cardiac exam;
- Send Vascular SR: vascular and urology exam. (The vascular SR contains the superficial and kidney vascular, application region of which corresponds to the vascular and urology exam respectively.)
- Send breast SR: breast exam;
- Send small parts SR: thyroid and testicle exam

The SR only could be sent in the circumstance as below:

- The exam type is obstetric, gynecology, cardiac, vascular, breast, thyroid, testicle
- The SR Key for the exam type must be installed in option preset UI.
- Exam is the unit to send.
- The status of the exam is "End".
- The DICOM storage service SCPs should be set as "Attach SR When Store Images" or "Only Store SR".

A. Attach SR When Store Images

Send image and gynecology and obstetric SR (or cardiac, vascular, breast, small parts SR) in iStation screen:

1. Select "Attach SR when Store Images" in Storage Preset -> SR Storage Option(See 1.2.1)
2. Start a new patient:
 - Press [Patient], enter the Patient ID, Patient Name etc, select [OB] (or GYN, CAR, VAS, BREAST, SMP) application region, press [Done]
 - Or, load a patient from the WorkList and start exam (commonly used in hospital)
3. Perform obstetric (gynecology, cardiac, vascular, breast, thyroid, testicles) measurements:
4. Save image(s)
 - Freeze the image, press [Save Image] to save single-frame file.
5. End the exam
6. Enter iStation screen; select the patient, press [Send To].A dialog box pops up, as shown in Figure 10 DICOM Send To Screen.
7. Select DICOM on the left side and select a server in the [Storage Server] list.
8. Press [Send], the status of sending task can be viewed in DICOM task management.

NOTE: For sending SR automatically, the same procedures are followed as in 2.2.1 Storage Operation.

B. Only Storage SR

Select "Only Store SR" in DICOM Storage Preset -> SR Store Option (See 1.2.1). The procedures are as the same as "Attach SR when Store Images".

2.2.3 Encapsulated PDF

Encapsulated PDF is a PDF document that has been encapsulated within a DICOM information object, which is used to encapsulate the report document in this system.

The Encapsulated PDF only could be sent in the circumstance as below:

- Exam is the unit to send.
- The status of the exam is "End".
- The "Encapsulated PDF" should be enabled in DICOM storage service preset.
- Measuring result should be existent in default report template, which is to say that measurement in the exam type must be done.

The operation to send Encapsulated PDF: After [Encapsulated PDF] is enabled in preset, select exam to send or send after ending exam automatically as 2.2.1.

2.2.4 Send DCM File

System allows saving images as DCM format to external media or device. The operations are described below:

A. Send to External Media

When the system is connected to external storage media, it allows saving the image as DCM format. The detailed operations are as follows:

1. Select one or more exams (with image) in iStation, press [Send To].
2. In the dialog box popped up, select [USB] and choose destination.
3. Select the Format Transfer “DCM”, press [Send], the selected exam can be sent to the specified external media with the format of DCM, as shown in Figure 14 Save as DCM Format.

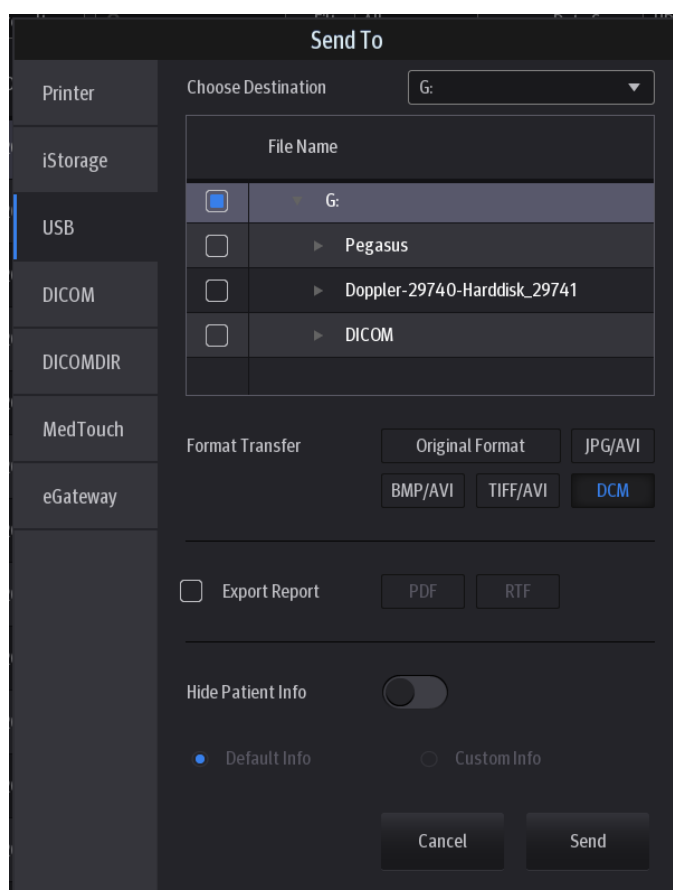


Figure 14 Save as DCM Format

B. Send to iStorage

When the system is connected to UltraAssist, it also allows sending files as DCM format. The detailed operations are as follows:

1. Configure the iStorage server in [Setup]/[Network]/[Network Preset].
2. Select one or more exams (with image) in iStation, press [Send To].
3. In the dialog box popped up, select the iStorage server.

4. Select the Format Transfer “DCM” press [Send], the selected exam can be sent to the specified iStorage server with the format of DCM, as shown in Figure 15 Send to iStorage.

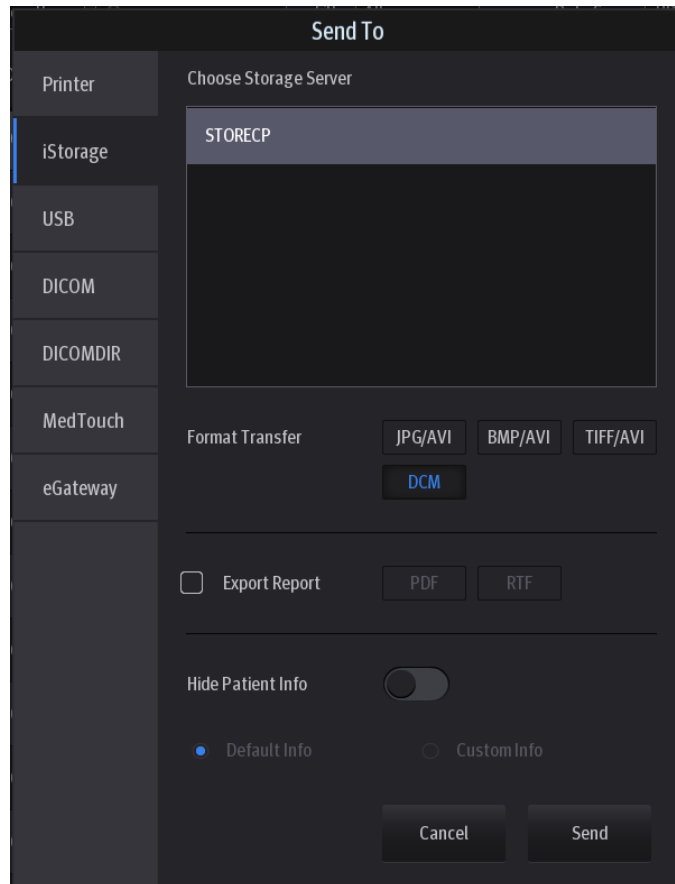


Figure 15 Send to iStorage

Note: The DCM files sent to external media or iStorage can be used as teaching files if “Hide Patient Information” on the send to dialog is selected, patient information will not be included in these DCM files.

2.2.5 Common Reasons of Failure and Troubleshooting

■ Failure:

The server disconnects to the system automatically.

● Potential Reasons:

The storage service of connected server cannot support JPEG compression, while which is selected in storage service in the system.

● Troubleshooting:

Modify the storage configuration of this server in Storage screen in the system. Select the service and select “Uncompressed”, and then press [Update] to update the latest configuration information.

■ Failure:

The net is disconnected during transmission.

● Potential Reasons:

The wireless network is disconnected during transmission.

● Troubleshooting:

Ensure normal wireless network.

■ Failure:

Multi-frame transmission is failed, but single frame is succeeded.

● Potential Reasons:

“Enable multi-frame image storage” is not selected in the dialog box of Storage preset.

● Troubleshooting:

Select “Enable multi-frame image storage” in the dialog box of Storage preset.

2.3 DICOM Print

DICOM print provides local image printing via the remote DICOM service or printer that supports the DICOM standard. The image displayed on screen can be printed via the selected DICOM print device during exam or image browsing. Print function entrances:

- Patient information management (iStation)
- Review status
- Auto send print

DICOM print supports:

- Single file transfer
- Exam transfer
- Batch transfer
- Static image file
- Succeeding frames printing is available if more than one frame is being sent for printing.

Batch transfer means multiple exam records can be selected and sent for printing with one DICOM task for each exam. Multiple files in one exam will be sent in one DICOM connection.

The auto send function can be triggered by (presettable):

- Ending the exam (send to default print service)
- Saving an image (image is sent to the default DICOM service as well as saved)

2.3.1 Send to DICOM Print Manually

Take “send to DICOM print” in iStation screen as an example, the operation steps are:

1. Configure the print server according to the upper descriptions of service preset.
2. Press [iStation] on the main screen to enter the patient information management screen.
3. Select one or more exam records (that contain images) in exam list.

4. Select [Send To] to pop up the [Send To] dialog box, select the Printer Server, press [Send] to print.

2.3.2 Send to DICOM Print Automatically

Function paths of sending to DICOM print automatically:

- Send to DICOM print when ending the exam
 - Steps:
 1. System Preset -> General, select “Sending/printing Image after End Exam”.
 2. In DICOM/HL7 preset -> Print Service, set the default print server.
 3. When pressing [End] with the current exam being active, all images of the current exam are sent to the default DICOM print server.
- Send to DICOM print when pressing user-defined key
 1. In System Preset -> General, select “Send Image to DICOM Printer” while storing image to hard disk, as shown in Figure 16 Send to DICOM print when saving image.

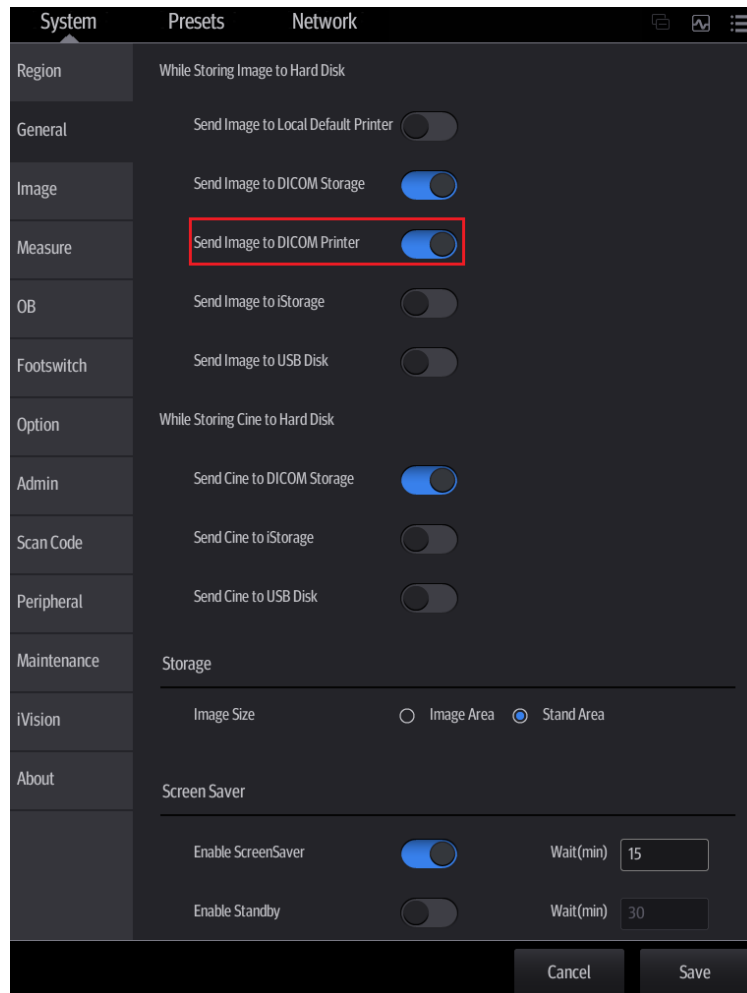


Figure 16 Send to DICOM print when saving image

2. In DICOM/HL7 Preset -> Print Service, set the default print server.
3. In current exam, press [Save Image], the system will save and send a single-frame image to remote DICOM printer automatically.

4. View the print status in the DICOM Task Management, check the DICOM preset and Local Area connection if a print error occurs.

2.4 DICOM Worklist

The user can set query conditions when finding exam record(s) from the RIS or other Worklist server, and import the patient information selected in the query result.

Steps:

1. Press [Patient] to open the Patient Info screen.
2. Select [Worklist] on the lower left corner to enter the WorkList screen.
3. Query the patient information by setting conditions such as Patient ID, Name, Exam Date, Modality, WorkList Server, Key words, Scheduled Station AE Title and so on. The default Exam Date is "today" of the system time. Patient records will be listed after pressing [Query].

The screenshot shows the 'Worklist' application interface. It features a search form with the following fields and controls:

- Patient ID:
- Exam Date: To:
- MRN:
- Server Type: PACS-Worklist:
- Patient Name: First Name: M.I.:
- Modality Type:
- Admit Date: Unit:
- Search Key:
- Scheduled Station AE Title: Accession #:
- Buttons: Clear, Setting, Query

Below the search form, it indicates '1 records are listed.' and displays a table with the following data:

Patient ID	Patient Name	MRN	Accession #	Exam Description	Exam Date	Gender
PatientID	ABC		123456789	RequestedProce...	06/07/2021	Female

At the bottom of the screen, there are 'Merge ID' and 'Done' buttons.

Figure 17 Query in WorkList

4. After the first query is finished, you can perform the second query within the range of the scheduled patients in the first time. The scheduled patients in the list will change in real time.
5. Select a patient record, the system will launch this patient into the Patient Info screen to edit and start the exam.

NOTE:

- The modality type can be entered by the user.
- If you set the “Scheduled Station AE Title” in Worklist Query Screen, then this item configured should be consistent with the one set in the server.
- Second query and detail information are still available during off-line. The query will be started automatically according to the default query conditions each time entering the [WorkList] screen if a default WorkList Server is set.
- Check the DICOM preset and Local Area Connection if a query error occurs.

2.5 HL7Query

The user can set query conditions when finding exam record(s) from the Patient Monitors or other HL7 server, and import the patient information selected in the query result.

Steps:

1. Press [Patient] to open the Patient Info screen.
2. Select [Import] on the lower left corner, and select [Worklist] on the pop-up menu to enter the WorkList screen.
3. Select the Server Type to be “HL7 Server”, Query the patient information by setting conditions such as Patient ID, Name or combination of them. Patient records will be listed after pressing [Query].

The screenshot shows the 'Worklist' application interface. At the top, there are several input fields for search criteria: Patient ID, MRN, Patient Name, Modality Type (set to 'US'), Exam Date (27/11/2019), Server Type (set to 'HL7 Server'), and Scheduled Station AE Title. Below these are fields for First Name, M.I., Admit Date (DD/MM/YYYY), Unit, Search Key (set to 'Requested Procedure'), and Accession #. There are 'Clear' and 'Query' buttons. Below the form is a table with the following data:

Patient ID	Patient Name	MRN	Accession #	Exam Description	Exam Date	Gender
PatientID	ABC	OtherPatientl...	123456789	ScheduledProced...	26/11/2019	Female

At the bottom right of the screen is a 'Done' button.

Figure 18 HL7 Query

4. After the first query is finished, you can perform the second query within the range of the scheduled patients in the first time. The scheduled patients in the list will change in real time.
5. Select a patient record, the system will launch this patient into the Patient Info screen to edit and start the exam.

NOTE:

- Only Patient ID and Patient Name can be input in HL7Query.
- Second query and detail information are still available during off-line status.
- The query will be executed automatically according to the default query conditions each time entering the [WorkList] screen if a default HL7 Query Server is set.
- Check the DICOM preset and Local Area Connection if a query error occurs.

2.6 MPPS

MPPS is to send the states information of exams to the configured server. Thus the exam states can be known and edited by other systems. Prerequisite: a default MPPS server is needed.

The MPPS function will be started by:

- Starting a new exam
- Ending an exam
- Activating an exam

The states are listed in the [Exam State] column in iStation screen. MPPS sending is hidden to the user.

2.7 Storage Commitment

Storage commitment is to ensure that the image data will be stored into the server correctly.

Prerequisite: a Storage Commitment server is needed, and a relative storage server must be set.

Steps:

1. A default storage commitment server is preset and a relative storage server is set.
2. Send image (and SR\Encapsulated PDF) to the storage server, the storage commitments are sent to the commitment server automatically.
3. If the storage commitment is succeeded, the storage commitment will return corresponding state and "Storage Commitment" column will be marked with "PACS received,OK". Otherwise the column will be marked with "Send to PACS,OK"

2.8 Query/ Retrieve

The query/ retrieve function is to query and get the patient exam record from a specified server.

Steps:

1. A query/ retrieve server is preset.
2. Select [iStation] -> [Query/ Retrieve].

3. Select the Source server, enter Search Key words and press [Query]. As shown in Figure 19 Query.

The screenshot shows a software interface titled "Query/Retrieve". At the top, there are three dropdown menus: "Source" set to "she-QueryRetrie...", "Destination" set to "Local Host", and "Level" set to "STUDY". Below these are input fields for "Patient ID", "ExamDate" (with a date range from "27/11/2012" to "27/11/2022"), "Patient Name", "Search Key" (with a dropdown menu set to "ModalitiesInStud"), and "Accession #". There are "Clear" and "Query" buttons. Below the input fields, a table titled "Patient(Source)" shows "1 records are listed." with columns for Patient ID, Name, Accession #, Exam Description, Exam Date/Time, Gender, and DOB. One record is visible with Patient ID "20191125-...", Name "123", Exam Date/Time "25/11/2019 ...", and Gender "Unknown". Below the table are "Cancel" and "Retrieve" buttons. At the bottom, a section titled "Patient(Destination)" shows "0 records are listed." with a table with the same columns as above. A "Done" button is at the bottom right.

Figure 19 Query

4. Select one or more records and press [Retrieve] to get the records down to local system, as shown in Figure 20 Retrieve.

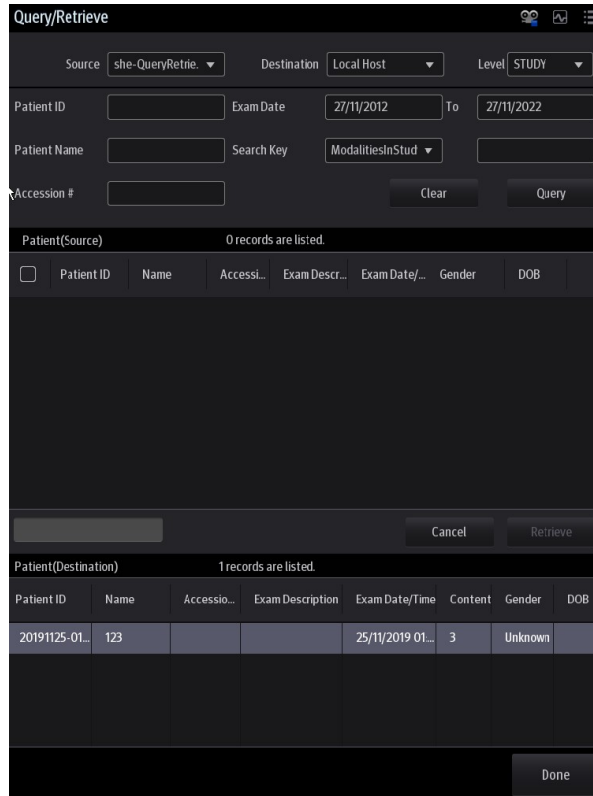


Figure 20 Retrieve

5. Press [Done], the obtained patient records can be viewed in iStation screen.

2.9 DICOM Media Storage and Review

2.9.1 DICOM Media Storage

DICOM media storage supports storing ultrasonic diagnostic data (patient and image information) into the media (USB flash drive or removable hard disk). Prerequisites for DICOM media storage and DICOMDIR:

- Normally read/ write of the USB ports on the ultrasound system
- File system format of removable device (USB flash drive) is FAT32 and the media should not be damaged

Steps:

1. Press [iStation] on the main screen to enter the patient information screen
2. Select one or more exams (with image) that is not active, select [Send To] to pop up the [Send To] dialog box.
3. Select [DICOMDIR] on the left side, and choose destination as shown in Figure 21 DICOM Media Storage.

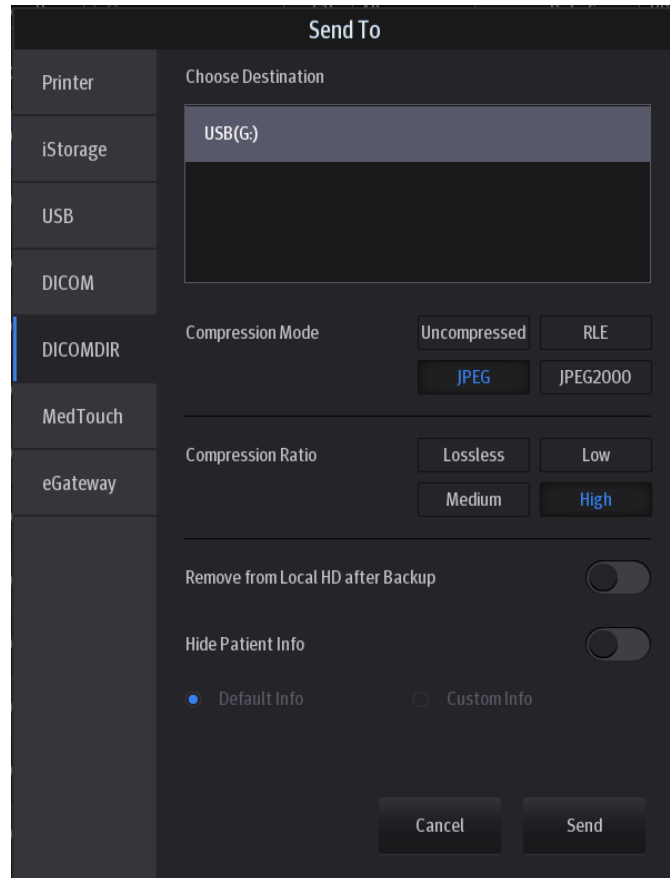


Figure 21 DICOM MediaStorage

4. Press [Send], to backup the image in current exam to the external media in DICOM format
5. If the backup is succeeded, [Backup] in the iStation screen will be marked with “√”; otherwise the column will not be marked.

NOTE: make sure enough space on the media; otherwise the media storage process will be failed due to insufficient space.

2.9.2 DICOMDIR Review

DICOM media review (DICOMDIR) function supports viewing DCM image(s) stored in external media.

Prerequisites for DICOM media storage and DICOMDIR:

- Configured with DVD recorder that working normally
- Normally read/ write of the USB ports on the ultrasound system
- File system format of CD/ DVD optical file should be ISO9660, and the optical disk should not be damaged
- File system format of DVD optical file should be UDF, and the optical disk should not be damaged
- File system format of removable device (USB flash drive) is FAT32 and the media should not be damaged

Istation review steps:


1. Press [iStation] in the main screen to enter the patient information screen
2. Select the external media from “Data Source”
3. The exam(s) and image(s) will display in the iStation screen if the media is read successfully.

External media ShowCase review steps:




1. Connect the external media to be browsed to the PC.
2. Double-click “Show_Studies.exe” in the root directory of the external media.
3. The exam(s) and image(s) will display in the ShowCase tool if the media is read successfully.

2.10 Task Management

The task management is a platform for the user to manage, view, delete or restart the background task, including DICOM tasks.

Press  icon on the upper right corner of the screen to open the DICOM Task Management dialog box, as shown in Figure 11 Task Management.

Task progress and status of storage task, print task and media storage task can be viewed in this dialog box. The task management icon indicates the status of the task:

1. Task in progress, the icon indicates in blue. 
2. No task is in progress with failed task exists, the icon indicates in red. 
3. No task is in progress without failed task, the icon is .

3. Common Failures and Troubleshooting

3.1 Common Failure: Service Verification Failed

Potential Reasons:

- Failed network connect, please check the network cable and connection;
- Verification service is not supported by the server;
- The verification is supported but not activated on the server; check the server setting if the verification service is activated.

Troubleshooting:

- Check the DICOM Conformance Statement if the SCP is supported;
- Enable the SCP service referring to the Server Manual or DICOM Conformance Statement.

- If the verification is still failed after all above steps and the operating system is Windows 7, please add the server port to the “Inbound Rules” of Windows Firewall. Steps:
 - Control Panel->Windows Firewall->Advanced settings-> Inbound Rules;
 - Click “New rules”, select Port in the pop-up dialog and then click “Next”;
 - Select “TCP” to which the rule applies and Input the server port in “Specific local ports” edit box, then click “Next”;
 - Select “Allow the connection” and click “Next”;
 - Specify the profiles for which the rule applies and click “Next”;
 - Input the Name of the rule and click “Finish”.

3.2 Common Failure: Server disconnects the ultrasound system during implementing storage

Potential Reasons:

- The JPG compression is enabled on local device but is not supported on the server;
- Wrong AE TITLE settings on local device and the server.

Troubleshooting:

- Select “uncompressed” in DICOM Service -> Storage;
- Set the correct local AE TITLE and server AE TITLE on local device according to the DICOM Conformance Statement or settings of the server.

3.3 Common Failure: Multiframe transmission is failed, but single frame is succeeded.

Potential Reasons:

- "Allow Multiframe" is not selected in the dialog box of Storage preset.

Troubleshooting:

- Select " Allow Multiframe " in the dialog box of Storage preset.

3.4 Common Failure: Network disconnected (shown by network icon) during transmission

Potential Reasons:

- The network cable or wireless network is disconnected during transmission;
- The connected switch, router or hub is down or in failure.

Troubleshooting:

- Connect the network;
- Reset the switch, router or hub, or replace a network device with higher performance.

3.5 Common failure: Worklist query succeeded, but no patient is shown

Potential Reasons:

- Scheduled Station AE Title in the preset is not consistent with the setting on server;
- Records on the server database are not accordant with the query conditions on local ultrasound system

Troubleshooting:

- Preset the right Scheduled Station AE Title on Worklist screen according to the server settings.
- The ultrasound system is defaulted only to inquire the patient information that the Modality is US, and the patient needs the server to provide US attribute. Change the query conditions (e.g. date or patient name) to make the server find the record accordingly. Based on DICOM standard, Patient ID cannot support fuzzy inquiry, only can support the whole word match. Please confirm the entered Patient ID is right or not.

3.6 Common failure: Printing succeeded but the film is blank

Potential Reasons:

- The printer doesn't support color printing which is performed by the ultrasound system;
- The film or paper of the printer doesn't support color printing.

Troubleshooting:

- Check the printer conformance statement if the printer can support color printing; if not, modify the "Settings" in Print Preset Screen -> Print to B/W type (MONOCHROME2);
- Generally, the "Medium Type" is set to "Blue File" for B/W printing while "Paper" or "Clear Film" for color printing

3.7 Common Failure: Verification of connecting DICOM printer is succeed but printing is failed

Potential Reasons:

- Wrong AE_TITLE settings on local device and the server.

- The parameters set in DICOM Service -> Print are not supported.
- IP address of printer server is the same with the system IP.

Troubleshooting:

- Set the correct local AE_TITLE and server AE_TITLE on local device according to the DICOM Conformance Statement or settings of the printer.
- Refer to the printer conformance statement for the printing parameters (Format, Film Size) value that supported. Check film size and type, and then correctly set Film Size, Format, Medium Type etc. in print service preset in ultrasound system.
- Set the correct IP address of the printer server.

3.8 Common Failure: Printed image is too small

Potential Reasons:

- The Magnification Type of local ultrasound system is set to NONE while this parameter on printer is set to same with the local ultrasound system.

Troubleshooting:

- Set the right Magnification Type in local ultrasound DICOM Service preset according to the Magnification Type specified in the printer conformance statement. If there is no printer conformance statement for reference, set the Magnification Type as Bilinear, CUBIC or REPLICATE.

3.9 Common Failure: MPPS message is not sent out correctly or not received by the server

Potential Reasons:

- The MPPS server is not set as default.

Troubleshooting:

- Set the MPPS server as default after setting the Port and AE Title correctly.

3.10 Common Failure: Storage commitment is failed

Potential Reasons:

- The associated storage commitment server is incorrect;
- The port and IP address settings on the commitment server are inconsistent with the local preset.

Troubleshooting:

- Check if the associated storage commitment server is correct;
- Check if the port and IP address settings on the commitment server are consistent with the local preset.

3.11 Common Failure: Query is succeeded, but retrieve is failed, with the prompting that the remote server is wrong

Potential Reasons:

- The client port, AE Title and IP address settings on the Q/R server are inconsistent with the local preset.
- The setting of the local storage SCP port is wrong.

Troubleshooting:

- Check if client port, AE Title and IP address settings on the Q/R server are consistent with the local preset.
- Some servers only support the specified port as the storage SCP port. Please confirm whether the server has this restriction. If it has, please modify the serial number of local port. See the Figure 22 DICOMLocal Preset.

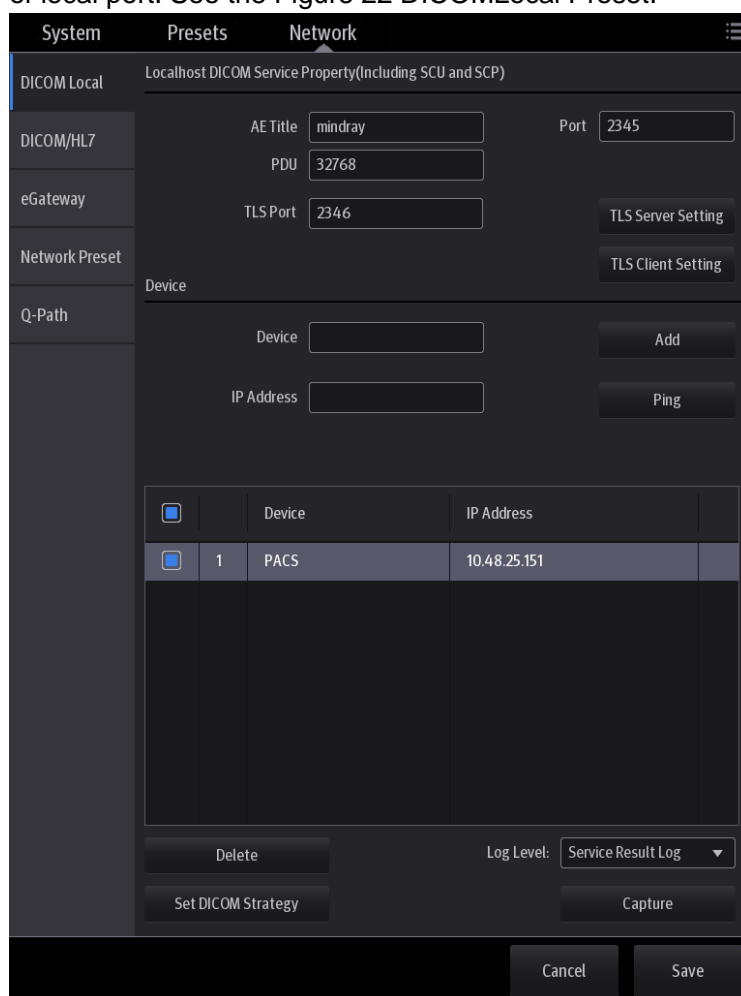


Figure 22 DICOMLocal Preset

3.12 Common Failure: Error messages in DICOM Task Management

3.12.1 Association aborts

Potential Reasons: The association is denied by the SCP due to incorrect setting of AE TITLE on the SCU. See Figure 23 Association aborts.

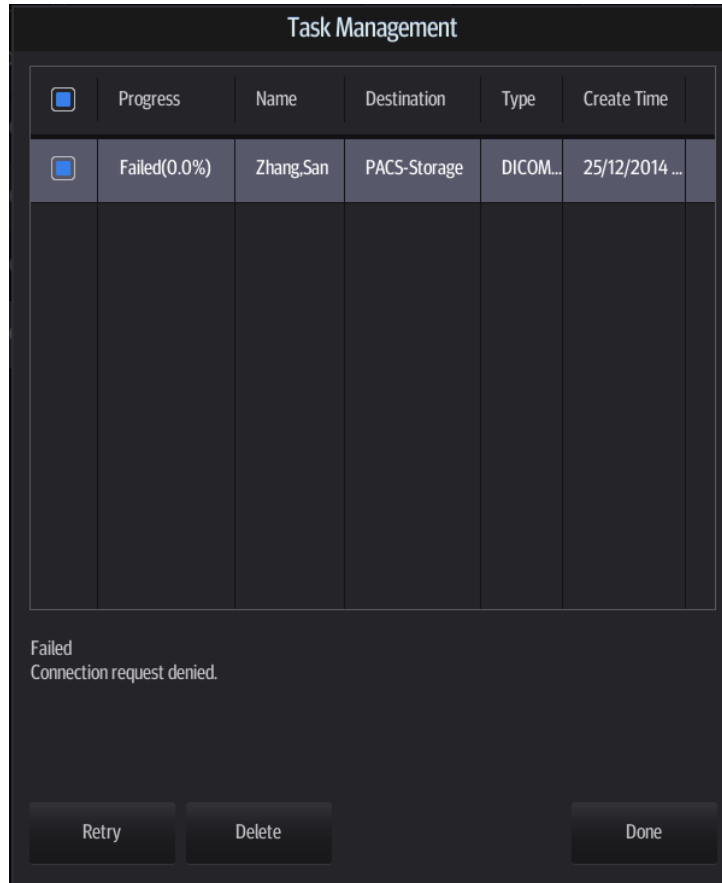


Figure 23 Association aborts

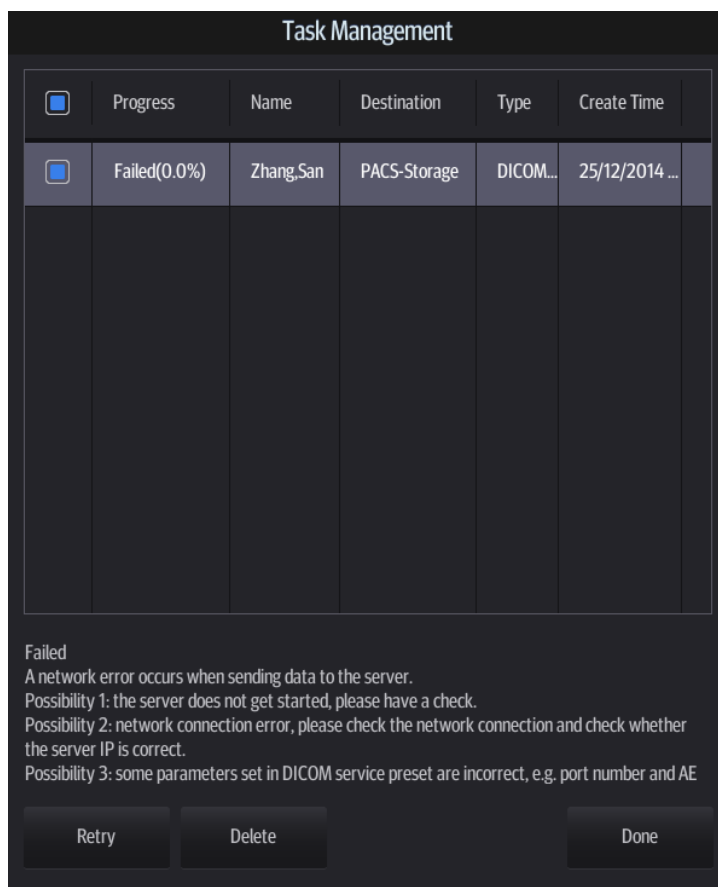


Figure 24 Network error

3.12.2 Network error: Socket sending error

Potential Reasons: Network cable unplugged or network equipment is failed during data transmission. See Figure 24 Network error.

3.13 Print Service Examples

Whether the printing will be succeeded is determined by:

1. Printer settings on local ultrasound system should consist with that specified on printer conformance statement;
2. The operations being performed are supported by the printer.

The following are two examples.

3.13.1 Example 1

The medium type supported by the Horizon CI DICOM printer in color and B/W printing is shown in table below:

Medium values)	Type (standard values)	Grayscale negotiated	PMM	SOP	Color negotiated	PMM	SOP

PAPER	DV Paper	CV Paper
Blue Film	DV Blue Film	CV Film (clear)
CLEAR FILM	DV Clear Film	CV Film (clear)

The MAGNIFICATION Types supported by Horizon CI DICOM printer conformance statement are listed in table below:

MAGNIFICATION TYPE	NONE REPLICATE BILINEAR CUBIC <i>BICUBIC</i> <i>MITCHELL</i> <i>LANCZOS</i> <i>SHARP1</i> <i>SHARP2</i> <i>SHARP3</i>
--------------------	--

AE TITLE settings that specified in Horizon CI DICOM printer conformance statement are listed in table below:

System Job Settings File	Corresponding Use
Print_SCP	Used for DICOM. Does not contain any print parameter settings. If the media type or size is not included in the DICOM attributes sent to the imager, the imager detects the image type and selects the media type and size as follows: For grayscale images, uses the media set for Grayscale in the Default Media menu at the control panel. For color images, uses the media set for Color in the Default Media menu at the control panel.
Default Grayscale	Imager uses the media set for Grayscale in the Default Media menu at the control panel
Default Color	Imager uses the media set for Color in the Default Media menu at the control panel
8x10-dvf	8x10 DirectVista film (blue)
8x10-dvfb	8x10 DirectVista film (blue)
8x10-dvfc	8x10 DirectVista film (clear)
8x10-dvp	8x10 DirectVista paper
8x10-cvf	8x10 ChromaVista film
8x10-cvt	8x10 ChromaVista film
11x14-dvf	11x14 DirectVista film (blue)
11x14-dvfb	11x14 DirectVista film (blue)
11x14-dvfc	11x14 DirectVista film (clear)
11x14-dvp	11x14 DirectVista paper

11x14-dvp	11x14 DirectVista paper
14x17-dvf	14x17 DirectVista film (blue)
14x17-dvfb	14x17 DirectVista film (blue)
14x17-dvfc	14x17 DirectVista film (clear)
14x17-dvp	14x17 DirectVista paper
a-dvp	A-size DirectVista paper
a-cvf	A-size ChromaVista film
a-cvt	A-size ChromaVista film
a4-dvp	A4-size DirectVista paper
a4-cvp	A4-size ChromaVista paper
a4-cvf	A4-size ChromaVista film
a4-cvt	A4-size ChromaVista film
priority-high	Sets print job priority to high
priority-medium	Sets print job priority to medium
priority-low	Sets print job priority to low
receive-1	Forces sheet output to receive tray 1
receive-2	Forces sheet output to receive tray 2
receive-3	Forces sheet output to receive tray 3
syngofilm	Special OEM-specific job settings

The Horizon CI DICOM printer screen displays:

A4 CV PAPER

8X10 DV Blue

14X17 DV Blue

● **The following are the correct settings of the ultrasound system DICOM print preset:**

■ Color printing:

- (1) Settings: RGB
- (2) Medium Type: Paper
- (3) Film Size: A4
- (4) MAGNIFICATION TYPE: NONE/CUBIC/ REPLICATE/BILINEAR
- (5) AE Title: Print_Scp (Recommended)

■ B/W printing

1. Method 1

- (1) Settings: MONOCHROME2
- (2) Medium Type: Blue Film
- (3) Film Size: 8X10
- (4) MAGNIFICATION TYPE: NONE/CUBIC/ REPLICATE/BILINEAR
- (5) AE Title: Print_Scp (recommended)

2. Method 2

- (1) Settings: MONOCHROME2
- (2) Medium Type: Blue Film
- (3) Film Size: 14X17
- (4) MAGNIFICATION TYPE: NONE/CUBIC/ REPLICATE/BILINEAR
- (5) AE Title: Print_Scp (recommended)

● **The following are the incorrect settings of the ultrasound system DICOM print preset:**

■ Color printing:

- | | |
|-----------------------------|---|
| 1) Settings:
MONOCHROME2 | Incorrect.
"RGB" should be selected in color printing while "MONOCHROME2" for B/W printing |
| 2) Medium Type: Blue Film | Incorrect.
Paper and Clear file are supported by color printer according to the printer conformance statement; but only Paper is available as displayed in the printer screen. |
| 3) Film Size: A3 | Incorrect.
Only three film size are available according to printer screen display:
A4
14X17
8X10 |
| 4) MAGNIFICATION TYPE | Can be set as: NONE/ CUBIC/ REPLICATE/ BILINEAR |
| 5) AE Title: 14x17-dvfb | Incorrect.
Setting AE Title as "14x17-dvfb" will enforce the 14x17 Direct Vista film (blue) printing. |

3.13.2 Example 2

This example is being performed with FUJI Film drypix4000 printer connected with the ultrasound system.

■ DICOM print settings on ultrasound system:

- 1) Settings: MONOCHROME2
- 2) Medium Type: Blue Film
- 3) Film Size: 14X17
- 4) FORMAT: STANDARD\3, 5
- 5) Film Orientation: Landscape

Result: failed

■ Modify the DICOM print settings:

- 1) Settings: MONOCHROME2
- 2) Medium Type: Blue Film
- 3) Film Size: 14X17

4) FORMAT: STANDARD\3, 5

5) Film Orientation: Portrait

Result: succeeded.

■ Reasons:

Settings of Film orientation along with the FORMAT that supported by FUJI Film drypix4000 printer are specified in the conformance statement. The print setting of 14X17 Landscape STANDARD\3,5 is unavailable.

According to the printer conformance statement, Film orientation and FORMAT table supported by FUJI Film drypix4000 are listed in table below:

Film Size and Orientation	FORMAT
14".17" Portrait	STANDARD\1,1
	STANDARD\1,2
	STANDARD\1,3
	STANDARD\2,1
	STANDARD\2,2
	STANDARD\2,3
	STANDARD\2,4
	STANDARD\3,3
	STANDARD\3,4
	STANDARD\3,5
	STANDARD\3,6
	STANDARD\4,4
	STANDARD\4,5
	STANDARD\4,6
	STANDARD\4,7
	STANDARD\4,8
	STANDARD\5,5
	STANDARD\5,6
	STANDARD\5,7
	STANDARD\5,8
	STANDARD\6,6
	STANDARD\6,7
	STANDARD\6,8
	STANDARD\6,9
	STANDARD\6,10
	STANDARD\7,7
	STANDARD\7,8
	STANDARD\7,9
	STANDARD\7,10
	STANDARD\8,8
STANDARD\8,9	
STANDARD\8,10	

<p>14"·17" Landscape</p>	<p>STANDARD\1,1 STANDARD\2,1 STANDARD\3,1 STANDARD\1,2 STANDARD\2,2 STANDARD\3,2 STANDARD\4,2 STANDARD\3,3 STANDARD\4,3 STANDARD\5,3 STANDARD\6,3 STANDARD\4,4 STANDARD\5,4 STANDARD\6,4 STANDARD\7,4 STANDARD\8,4 STANDARD\5,5 STANDARD\6,5 STANDARD\7,5 STANDARD\8,5 STANDARD\6,6 STANDARD\7,6 STANDARD\8,6 STANDARD\9,6 STANDARD\10,6 STANDARD\7,7 STANDARD\8,7 STANDARD\9,7 STANDARD\10,7 STANDARD\8,8 STANDARD\9,8 STANDARD\10,8</p>
--------------------------	--