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AED Tool

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Installation and Use Guide

mindray

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
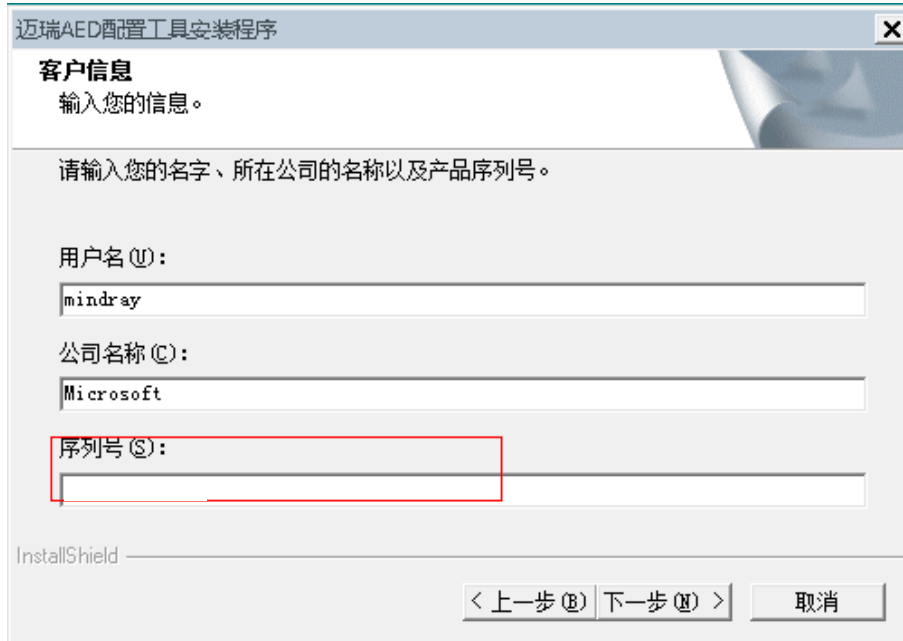
The screenshot shows a Windows-style dialog box titled "迈瑞AED配置工具安装程序" (Mindray AED Configuration Tool Installation Program). The main heading is "客户信息" (Customer Information) with the instruction "输入您的信息。" (Enter your information.). Below this, it says "请输入您的名字、所在公司的名称以及产品序列号。" (Please enter your name, the name of your company, and the product serial number.). There are three input fields: "用户名 (U):" (Username) containing "mindray", "公司名称 (C):" (Company Name) containing "Microsoft", and "序列号 (S):" (Serial Number) which is currently empty and highlighted with a red rectangular border. At the bottom, there are navigation buttons: "< 上一步 (B)" (Previous Step), "下一步 (N) >" (Next Step), and "取消" (Cancel). The "InstallShield" logo is visible in the bottom left corner.

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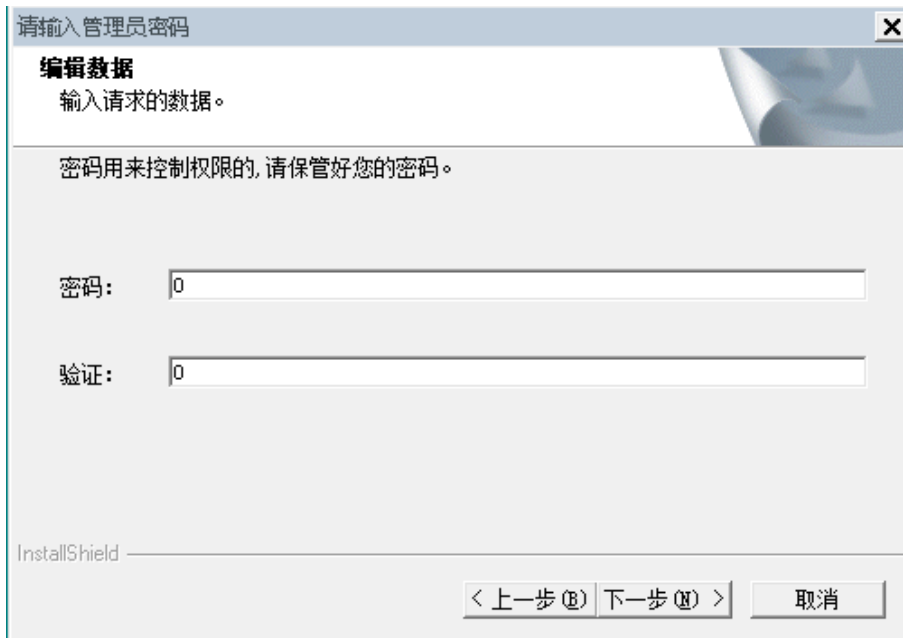The screenshot shows a dialog box titled "请输入管理员密码" (Please Enter Administrator Password). The main heading is "编辑数据" (Edit Data) with the instruction "输入请求的数据。" (Enter the requested data.). Below this, it says "密码用来控制权限的, 请保管好您的密码。" (The password is used to control permissions, please keep your password safe.). There are two input fields: "密码:" (Password) and "验证:" (Verify), both containing a single "0" character. At the bottom, there are navigation buttons: "< 上一步 (B)" (Previous Step), "下一步 (N) >" (Next Step), and "取消" (Cancel). The "InstallShield" logo is visible in the bottom left corner.

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AED Tool

© Copyright 2019-2021 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved.

For this Operator's Manual, the issue date is 2021-06.

1.1 Installation Environment

1. The operating environment of AED tools is compatible with Windows 10 and Windows 7 Professional SP1 systems.
2. Connection method: support the device to connect the AED tools through the USB data cable; the red box in the figure below is the micro USB port of the device:

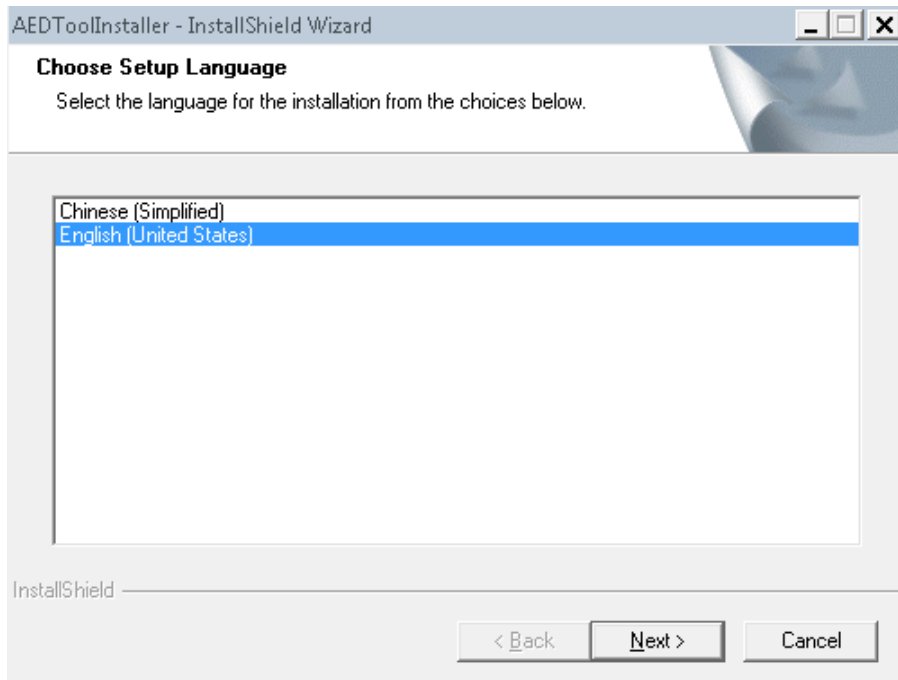
1.2 Software Installation

1.2.1 Installing the Software


1. Double click **AedToolInstaller.exe** and install the software.

Note: For 32-bit operating systems, first install the VCOM driver, double-click `lpc_driver_setup.exe` to install, and then install the configuration tool software `AedToolInstaller.exe`.


2. Choose the setup language.

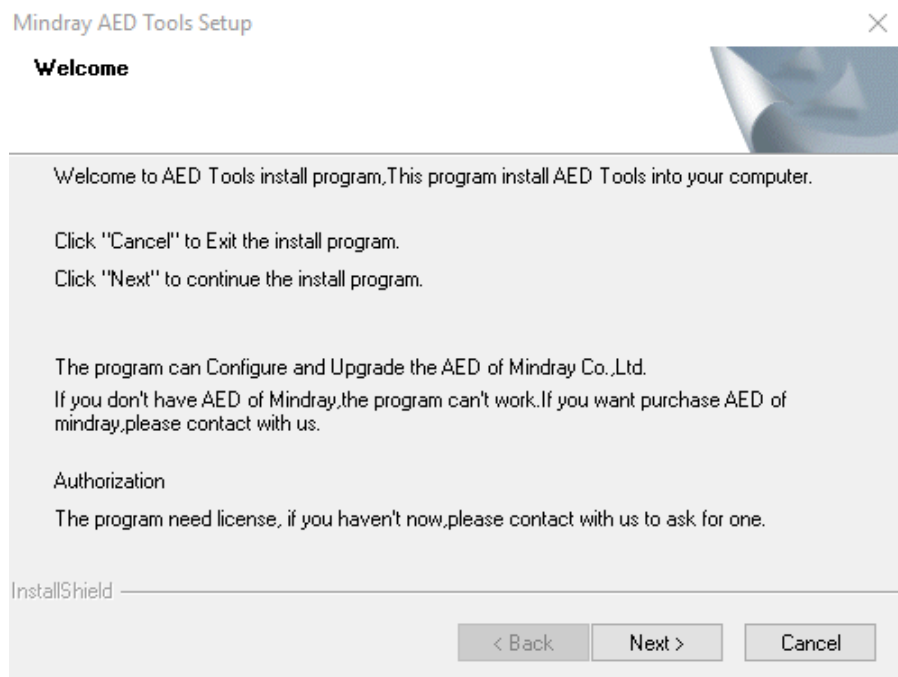3. Click **Next**, and be ready to install.

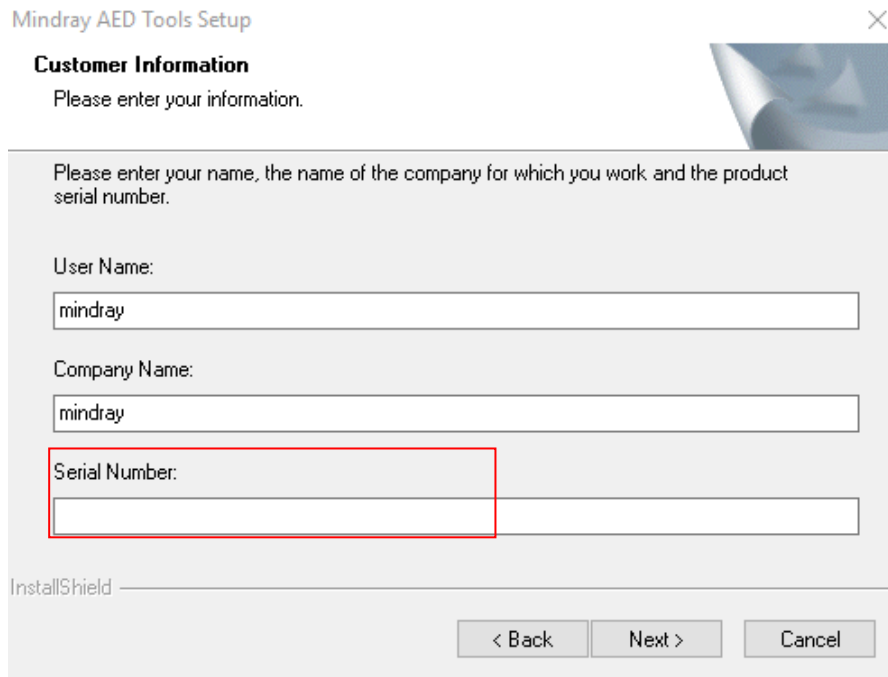4. Click **Next**, and enter **Customer Information** and enter the serial number.
 - ◆ If you enter the administrator serial number, you may have administrator permissions: configuration management, device information, factory maintain, factory test, data export, and user test.
 - ◆ If you enter the common user serial number, you may have common user permissions: configuration management, device information, data export, and user test.

Mindray AED Tools Setup

Customer Information

Please enter your information.

Please enter your name, the name of the company for which you work and the product serial number.

User Name:

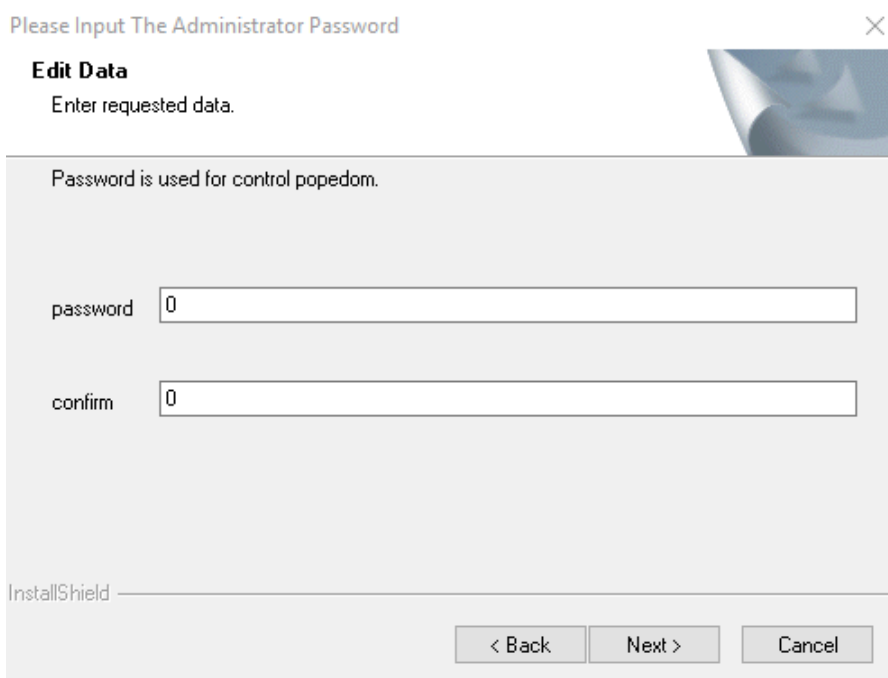
Company Name:

Serial Number:

InstallShield

< Back Next > Cancel

5. Click **Next**, and set a login password for the configuration tool:

Please Input The Administrator Password

Edit Data

Enter requested data.

Password is used for control popedom.

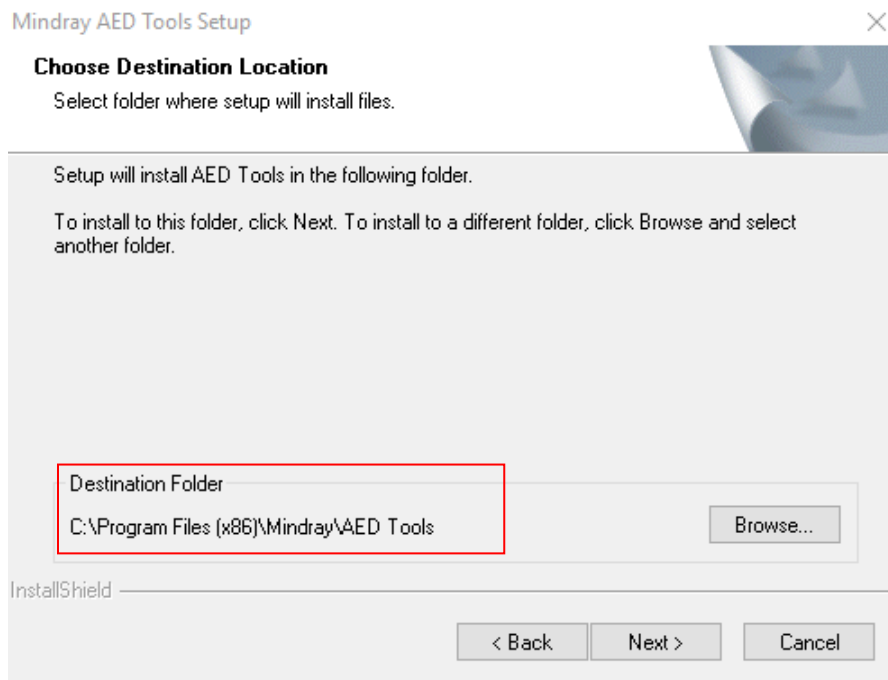
password

confirm

InstallShield

< Back Next > Cancel

6. Click **Next**, and select the installation directory.


7. Click **Next**, until the USB driver is installed successfully.
8. After the installation is complete, follow the prompts to choose whether you need to restart your computer now.



1.2.2 Updating the VCOM Driver

Note: The Windows 10 operating system can automatically recognize the VCOM driver. You may not need to perform VCOM driver operations. The Windows 7 operating system needs this operation.

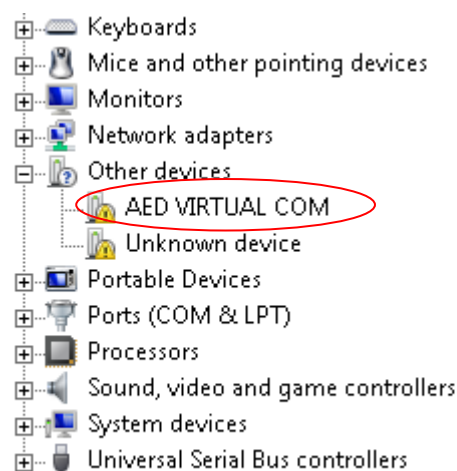
1. Open the AED lip to power it on. Use the USB cable to connect the PC and the USB micro port of the device. The device manager has the following device.

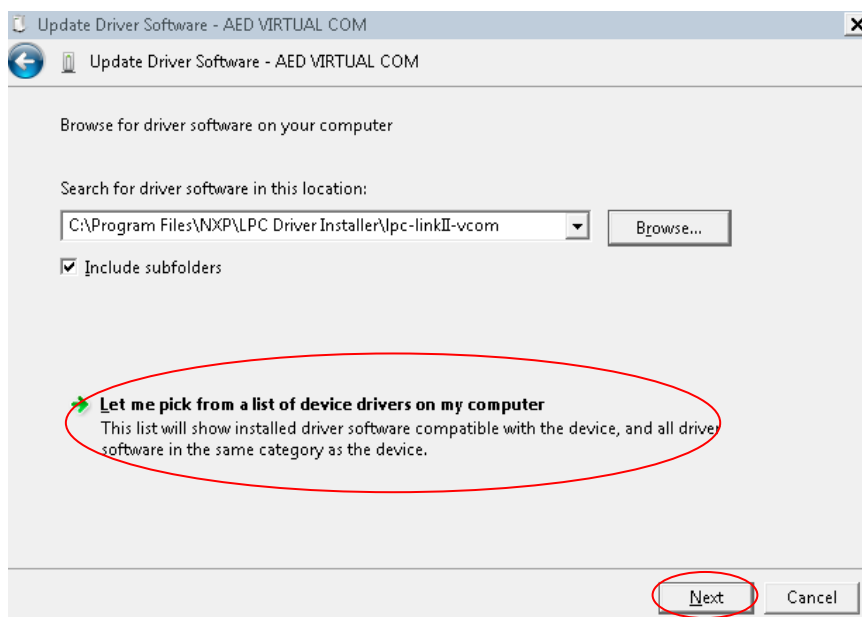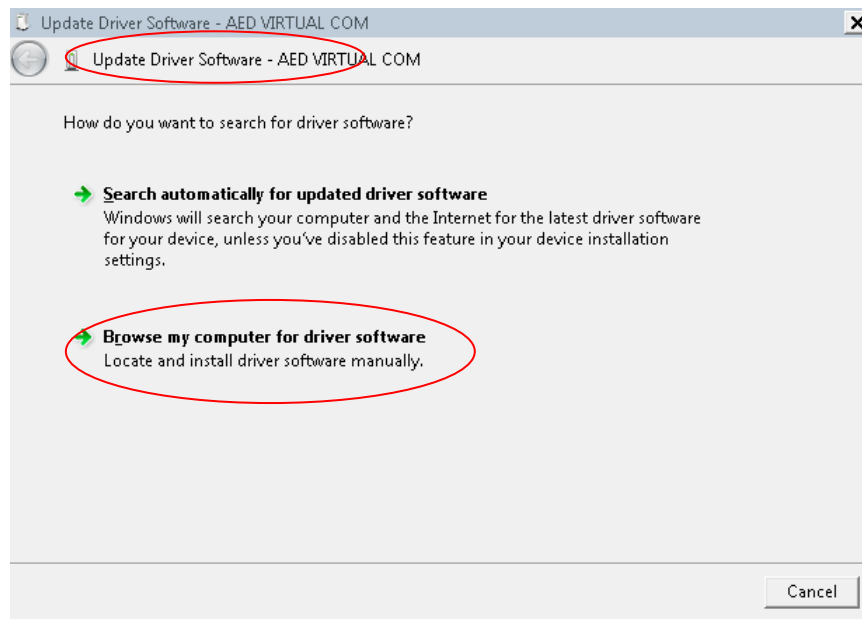
2. Right-click to select **Update Driver Software**.

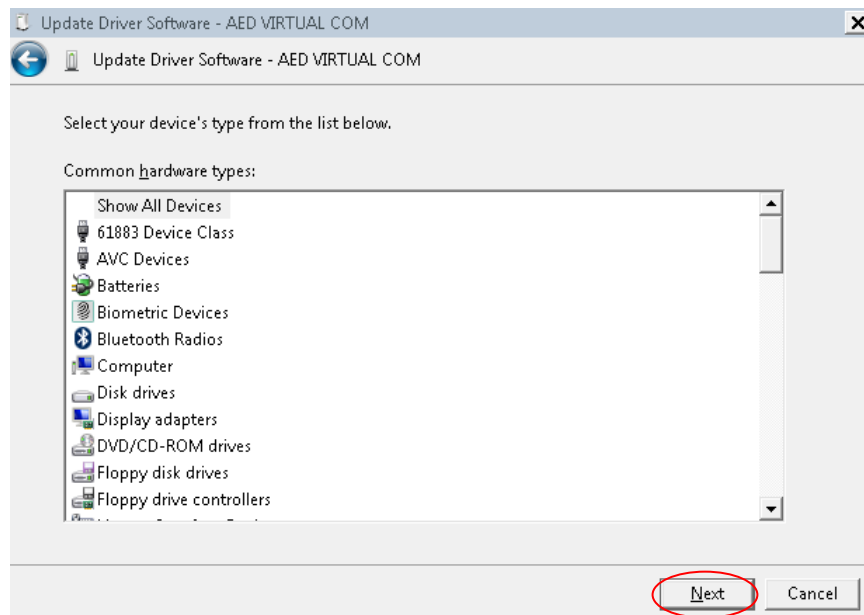
3. Directly select **Next**.


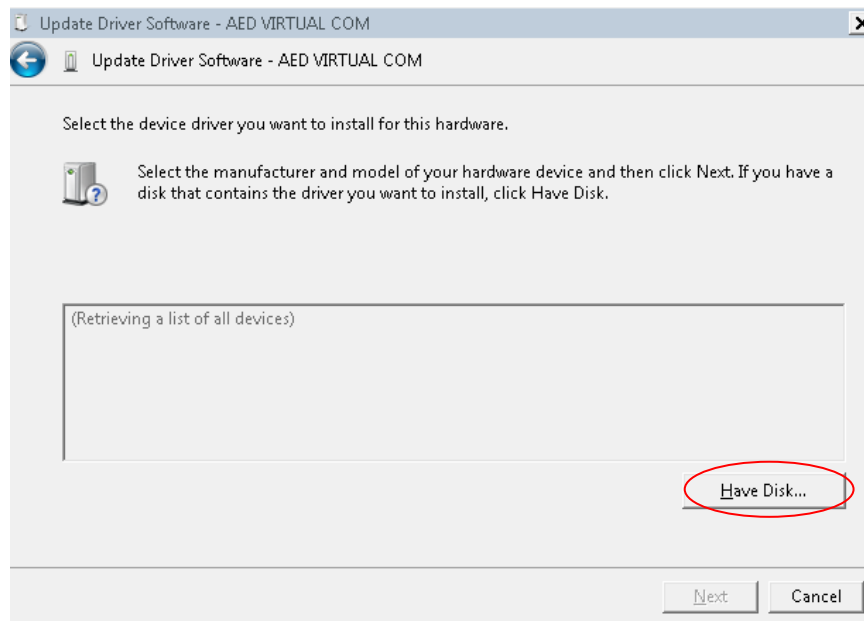
4. Select **Have Disk...**

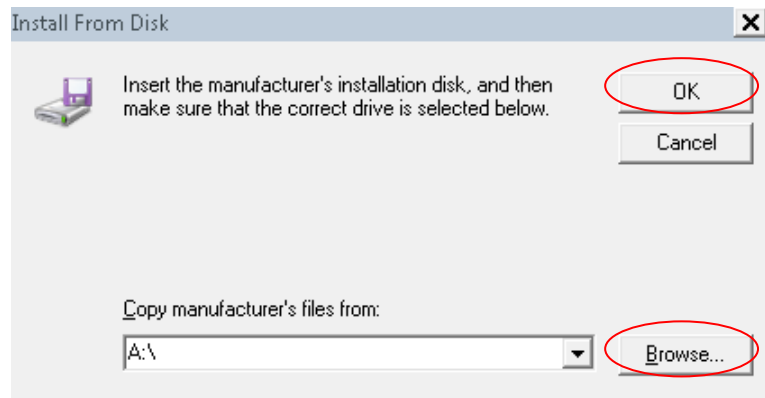
5. Browse to select the vcom installation directory.

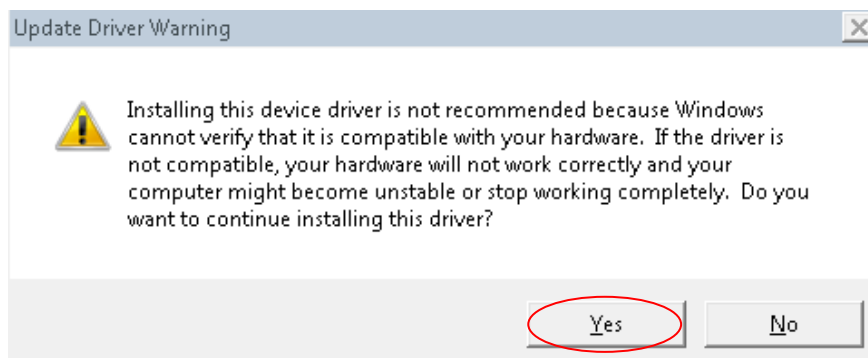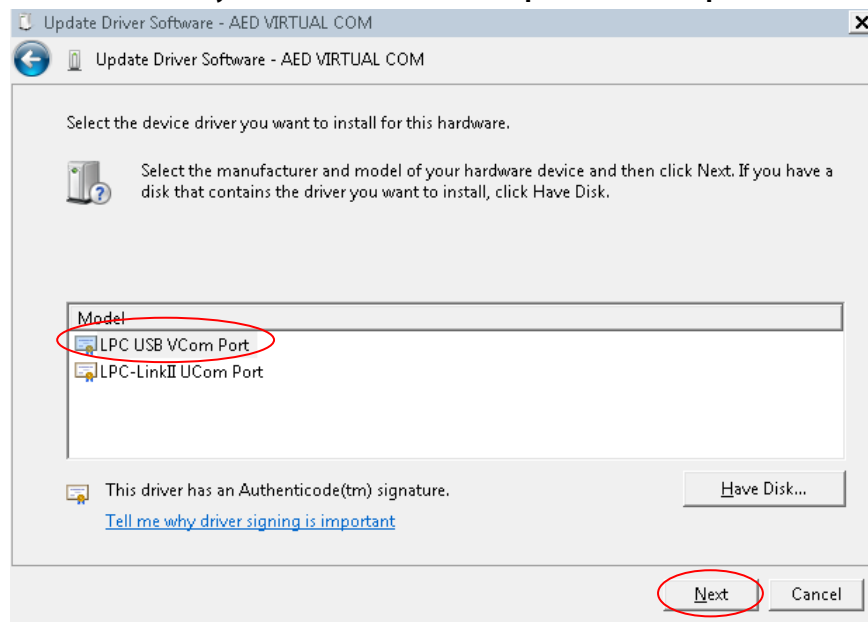
6. Select **Installation directory\NXP\LPC Driver Installer\lpc-linkII-vcom\lpc-vcom.inf**.

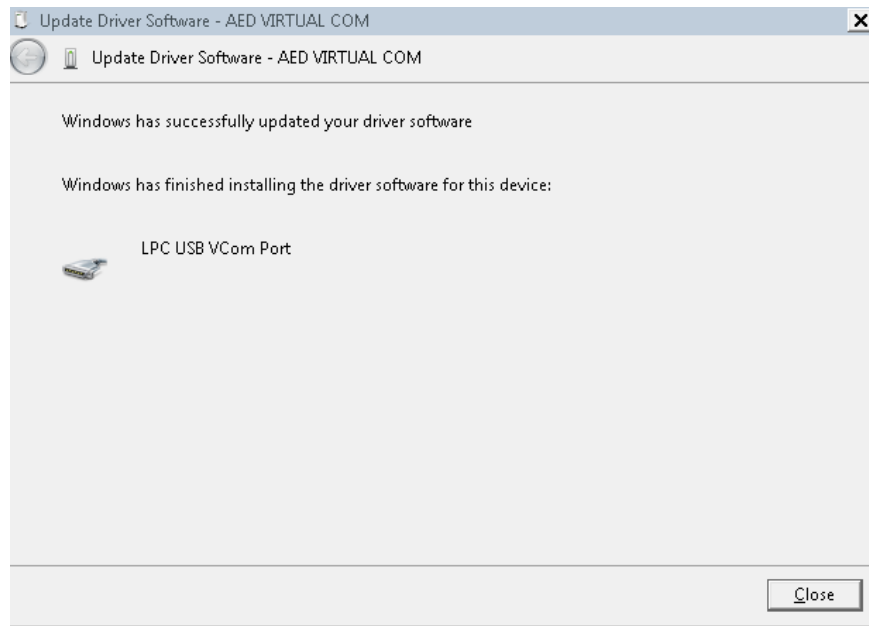7. Wait for updates, and close the window

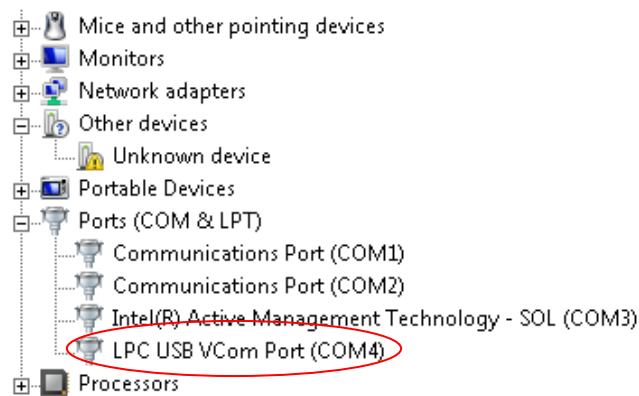8. After the completion, the device manager is as shown below, indicating that the VCOM recognition is successful.

1.2.3 Uninstalling the Software

1. Open the **Start** menu on the PC and enter the **Control Panel-> Programs and Functions** page. Select Mindray AED tools software, and click **Uninstall**.
2. An uninstallation prompt pops up. Select **Yes**, and follow the prompts to uninstall.

1.3 Software Operations

1.3.1 Connecting the Device

Power on the AED device, and use a USB cable to connect the AED device to the PC. After the connection, the AED Tool prompts that **“The AED device is detected, enter the configuration mode?”**. If you select **Yes**, the device enters the configuration mode and is connected with the AED Tool. If you select **No**, the device remains in non configuration mode, and is not connected with the AED Tool.

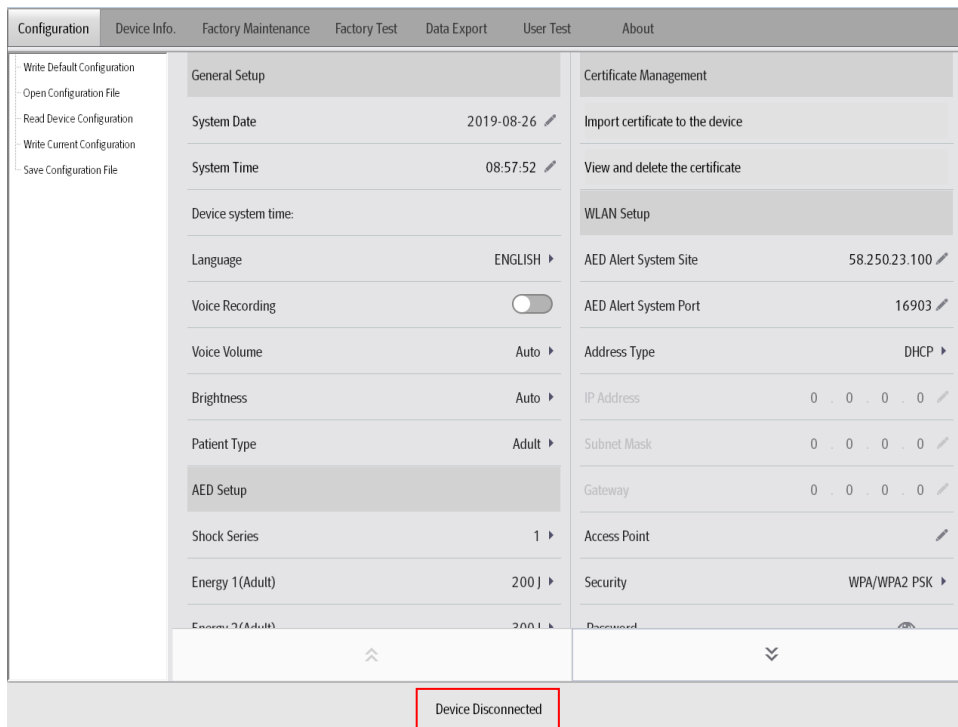
Note:

- **After you use the USB cable to connect the device with the AED Tool, you should wait for the prompt “The AED device is detected, enter the configuration mode?” about 30s to 60s. If this prompt is not displayed for a long time, you are recommended to re-plug the USB cable or replace the USB cable.**
- **For the Windows 10 operating system, you should right click and operate the AED Tool as an administrator.**

1.3.2 Identifying the Connection Status


- **Disconnected Device**

Start the AED Tool software. If the device is not connected or the device is disconnected, there is a prompt **“Device Disconnected”** as shown below:

■ Connected Device:

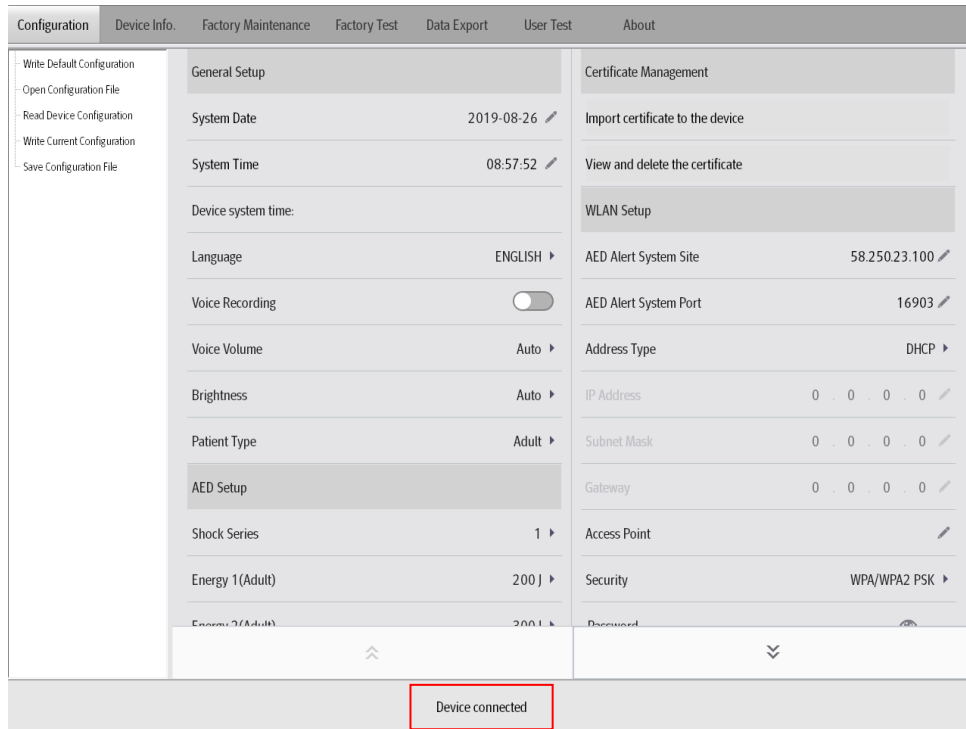Turn on the device and use the USB cable to connect it to the AED Tool. Select **Yes** to enter the configuration mode. There is a prompt **Device Connected** as shown below.

1.3.3 Managing Configurations

On the **Configuration** page, you can write the default configurations, open the configuration file, read the device configurations, write the current configurations, save the configuration file, view the configuration items through the up and down arrows, and edit the configurations.

■ Method 1 to write configurations:

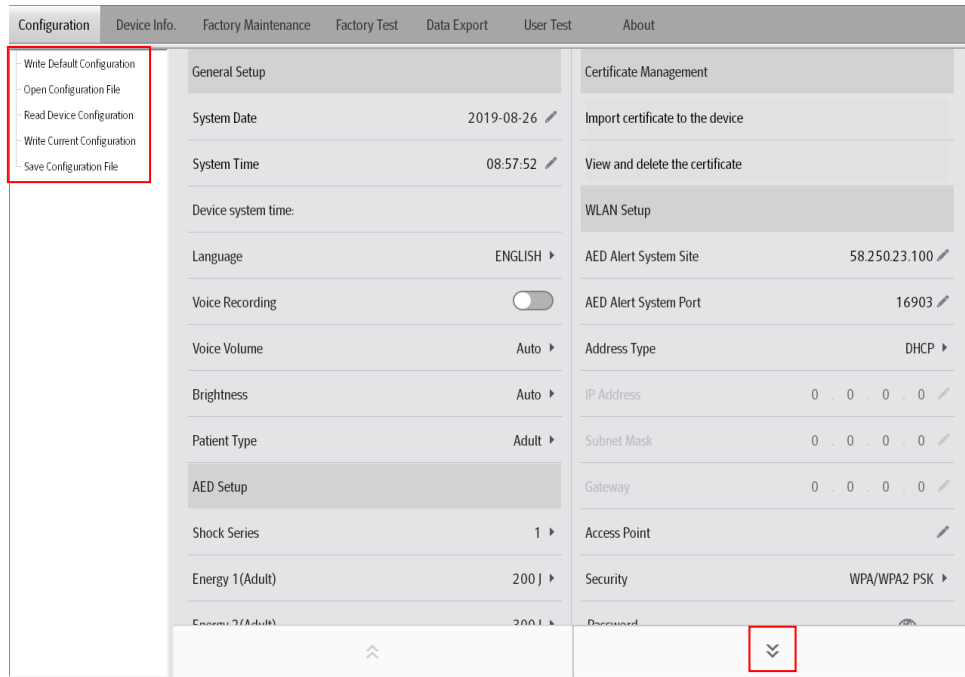
1. After the device is connected to the AED Tool, enter the **Configuration** page, click **Read Device Configuration**, and the prompt “**Reading the configuration file successfully!**” appears.
2. Close the above prompt box, edit **System Time**, **Language**, **Shock Series**, **WLAN Setup**, and other items, click **Write Device Configuration**, the prompt “**Writing the configuration file successfully! Effective after the device is restarted**” appears.
3. Restart the AED device and the written configurations take effect.

■ Method 2 to write configurations:

1. Enter the **Configuration** page, click **Open Configuration File**, select the directory to be configured, and the prompt “**Opening the configuration file successfully!**” appears.
2. Close the above prompt box, edit **System Time**, **Language**, **Shock Series**, **WLAN Setup**, and other items, click **Write Device Configuration**, the prompt “**Writing successfully! Effective after the device is restarted**” appears.
3. Restart the AED device and the written configurations takes effect.


■ Method 3 to write configurations:

1. Enter the **Configuration** page, click **Write Default Configuration**, and the prompt **"Are you sure to restore factory settings?"** appears. Click **OK**, and the prompt **"Writing successfully! Effective after the device is restarted"** appears.
2. The **System Time**, **Language** and **WLAN Setup** are not changed after restoring the default configurations. You are recommended to confirm these configurations by clicking **Read Device Configuration** after restarting the device.

1.3.4 Viewing the Device Information

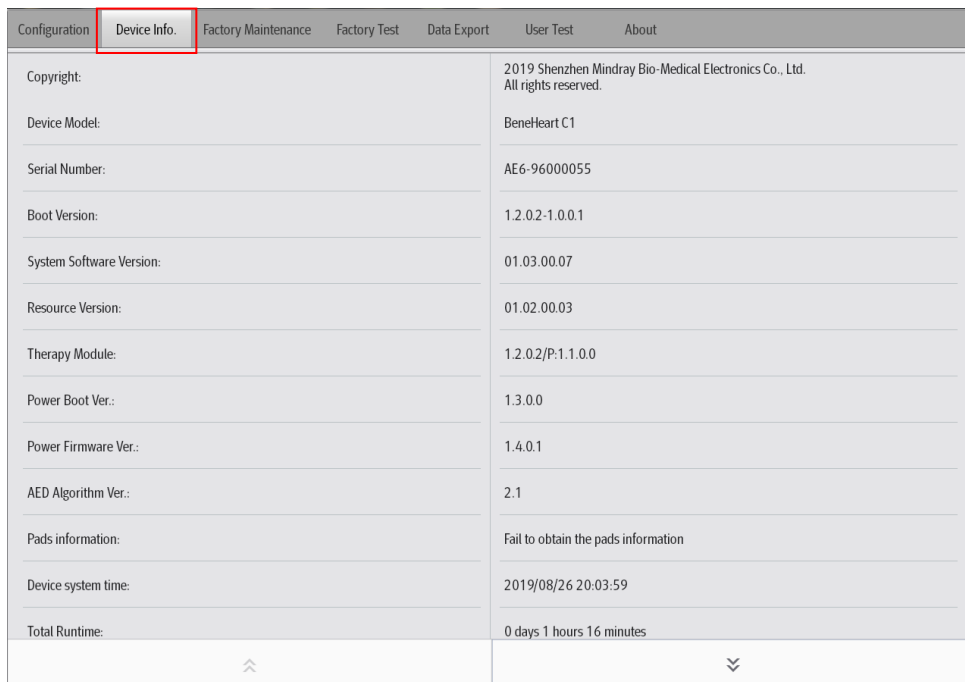
Enter the **Device Info.** page and read the device information:

1.3.5 Exporting Data


Enter the **Data Export** page, select the latest file or all files, and click **Data Export** to export all selected data. Select **Delete Data After Exporting**, the data in the device is deleted after the data is exported completely.

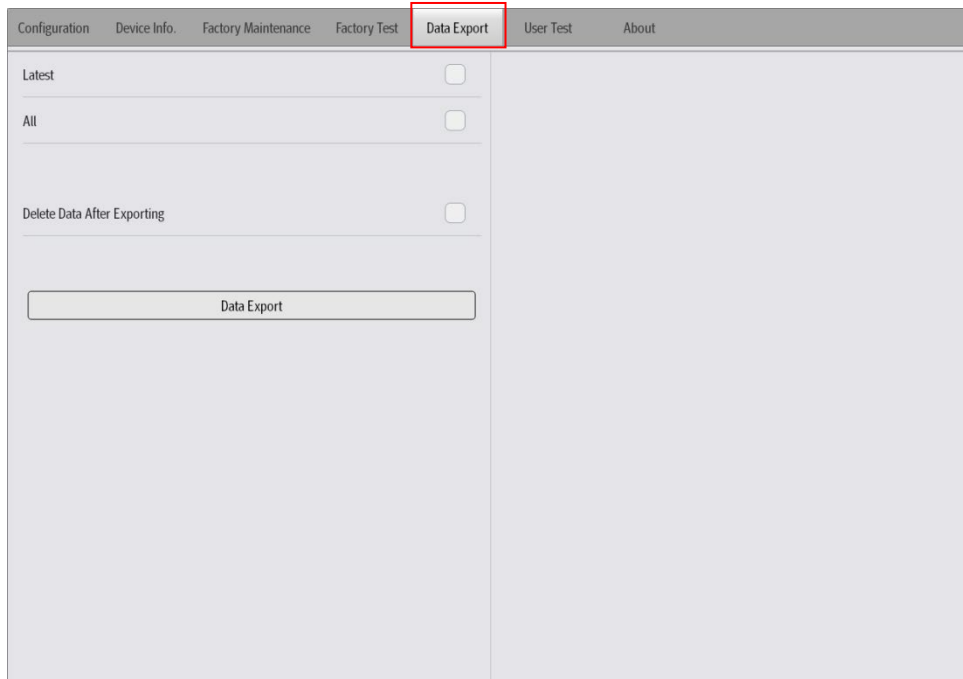Note: The exported data is stored in the \ImportData\Savior folder under the installation location directory of the AED Tool, and this folder is named with the serial number of the device.

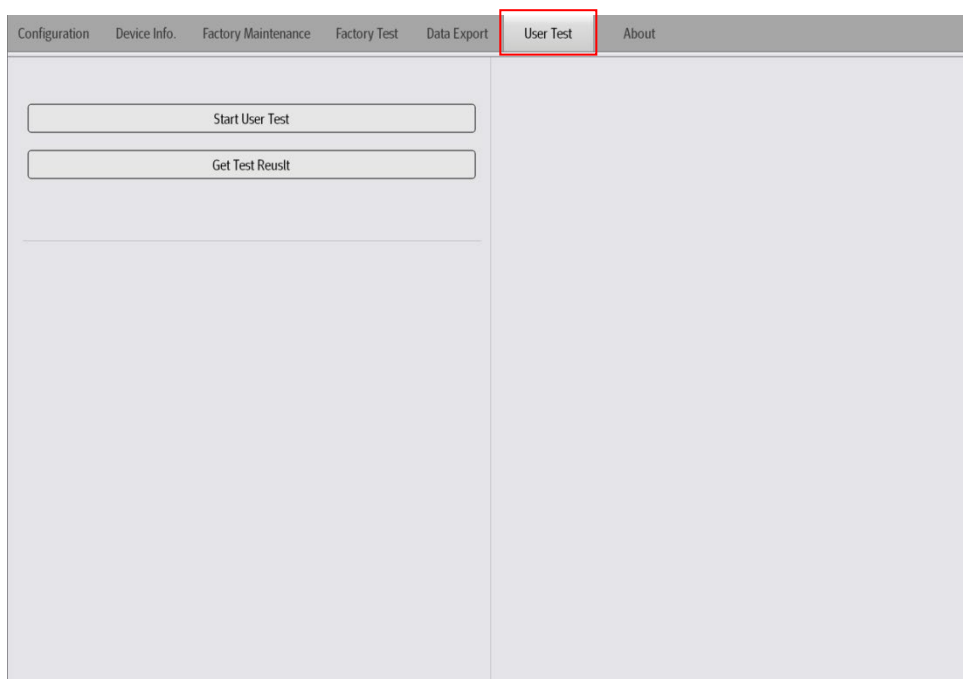1.3.6 Performing the User Test

With the device lid opened, enter the **User Test** page to perform the user test and obtain test results. Click **Start User Test**, the device starts the user test, and you should can perform the user test followed the voice and text prompts. After the test is completed, click **Get Test Result** to read the test results.

1.3.7 Performing the Network Test

Enter the **Configuration** page and find **WLAN Setup** by the up and down arrows.

For the device configured with the 4G module, after setting **AED Alert System Site** and **AED Alert System Port**, click **Wireless Network Test** and wait for the wireless network test to be completed.

WLAN Setup	
AED Alert System Site	10.6.144.28 
AED Alert System Port	16903 
Wireless test	

Wireless test 

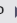
Access Technology: FDD LTE

Channel: LTE BAND 3

Signal Strength (dBm): RSRP:-100 dBm,RSRQ:-10 dB,RSSI:-69 dBm

Normal Network

For the device configured with the Wi-Fi module, after setting **AED Alert System Site**, **AED Alert System Port** and other Wi-Fi items, click **Wireless Network Test** and wait for the wireless network test to be completed.

WLAN Setup	
AED Alert System Site	58.250.23.100 
AED Alert System Port	16903 
Address Type	DHCP 
IP Address	0 . 0 . 0 . 0 
Subnet Mask	0 . 0 . 0 . 0 
Gateway	0 . 0 . 0 . 0 
Access Point	
Security	WPA/WPA2 PSK 
Password	
WLAN Band	2.4G 
Wireless test	

Wireless test
✕

Network Name:		Trigger:	-75 dBm <input type="text"/> (-100 to -1)
Channel:		Delta:	15 dB <input type="text"/> (1 to 100)
RSSI:		Scan Period:	30s <input type="text"/> (1 to 100)
AP MAC Address:		WiFi Regulatory Domain:	UNITED STATES ▾
Device IP Address:	0.0.0.0	<input type="button" value="Roaming Parameter Write in"/>	
MAC Address:	90-0C-84-00-E9-12	Ping IP	1 . 0 . 0 . 0 <input type="text"/>
Connection Status:	WiFi Not Connected		
Security Type:	WPA/WPA2 PSK		<input type="button" value="Wireless test"/>

1.3.8 Viewing the Software Version

Enter the **About** page, and view the detailed version information of the AED Tool.

Configuration
Device Info.
Factory Maintenance
Factory Test
Data Export
User Test
About

AED Tool Version:01.02.00.02

SVN: 11415

Build Time:Aug 15 2019 17:26:34

Copyright: (C) 2019 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved.

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© Copyright 2019-2021 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. Все права защищены.