

Value sheet of Mindray BS Measurement System

Русский: Таблица результатов измерительной системы BS компании Mindray

Português: Planilha de valores do Sistema de Medição BS da Mindray

Español: Hoja de valores del sistema de medición Mindray BS

Italiano: Scheda dei valori del sistema di misurazione BS di Mindray

Türkçe: Mindray BS Ölçüm Sistemi'nin değer sayfası



The data of each group is same.

Русский: Данные совпадает во всех группах.

Português : A dados de cada grupo é a mesma.

Español : la datos de cada grupo es la misma.

Italiano : la dati di ogni gruppo è la stessa.

Türkçe : her grubun veri aynıdır.

1.BS-200: BS-200, BS-220, BS-330, BS-350;

10.BS-430: BS-430, BS-450,BS-460;

2.BS-200E: BS-200E, BS-220E,;

11.BS-480: BS-480, BS-490;

3.BS-240: BS-230, BS-240;

12.BS-600: BS-600, BS-620;

4.BS-240E: BS240E, BS240Pro;

13.BS-600M: BS-600M, BS-620M;

5.BS-300: BS-300, BS-320;

6.BS-330E:BS-330E(Serial Number starts with "XQ-"),
BS-350E(Serial Number starts with "XS-")

14.BS-800: BS-800, BS-820, BS-800M, BS-820M, BS-1800,

7.BS-360E: BS-360E, BS-370E, BS-350S, BS-360S, BS-

BS-1800plus;

330E(V35.00)(Serial Number starts with "W8-" and
software version starts with "35.00"), BS-

15.BS-2000: BS-2000, BS-2200, BS-2000M, BS-2200M;

350E(V35.00) (Serial Number starts with"W9-" and
software version starts with "35.00");

16.BS-2800M:BS-2600M, BS-2800M;

8.BS-380: BS-380, BS-390;

17.For applicable analyzers of the analyte, please subject to

9.BS-400: BS-400, BS-420;

the parameter sheet and instrument.

LOT : 044624003

: 2025-09-15

English	Abbreviated name	Calibration Value	Level	Model
Русский	сокращенное наименование	Эталонное значение	Уровень	модель
Português	Nome abreviado	Valor de calibração	Nível	Modelo
Español	nombre abreviado	Valor de calibración	Nivel	modelo
Italiano	abbreviazione	Valore di calibrazione	Livello	modelli
Türkçe	kısaltılmış ad	Kalibrasyon Değeri	Düzye	model

English	Expanded Uncertainty
Русский	Расширенная неопределенность
Português	Incerteza Expandida
Español	Incertidumbre expandida
Italiano	Incertezza estesa
Türkçe	Daha Uzun Süreli Belirsizlik

HbA1c/Hb	
English	Hemoglobin A1c/ Hemoglobin
Русский	гемоглобин A1c / гемоглобин
Português	Hemoglobina A1c /Hemoglobina
Español	hemoglobina A1c /hemoglobina
Italiano	Emoglobina A1c/Emoglobina
Türkçe	Hemoglobin A1c / Hemoglobin

Abbreviated name		Hb		Calibration Rule		Two-point linear			
Model		BS-200 ¹		BS-200E ²		BS-240 ³		BS-240E ⁴	
Level	Unit	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty
S1	µmol/L	91.5	2.1	91.5	2.1	91.5	2.1	91.5	2.1
S2		142	3	141	3	141	3	141	3

Model		BS-300 ⁵		BS-330E ⁶		BS-360E ⁷		BS-380 ⁸	
Level	Unit	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty
S1	μmol/L	91.5	2.1	91.5	2.1	91.5	2.1	91.5	2.1
S2		141	3	141	3	141	3	141	3
Model		BS-400 ⁹		BS-430 ¹⁰		BS-480 ¹¹		BS-600 ¹²	
Level	Unit	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty
S1	μmol/L	91.5	2.1	91.5	2.1	91.5	2.1	91.5	2.1
S2		141	3	141	3	141	3	141	3
Model		BS-600M ¹³		BS-800 ¹⁴		BS-2000 ¹⁵		BS-2800M ¹⁶	
Level	Unit	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty
S1	μmol/L	91.5	2.1	91.5	2.1	92.2	2.1	91.6	2.1
S2		141	3	141	3	142	3	142	3
Abbreviated name		HbA1c		Calibration Rule		Two-point linear			
Model		BS-200 ¹		BS-200E ²		BS-240 ³		BS-240E ⁴	
Level	Unit	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty
S1	μmol/L	3.70	0.09	3.80	0.10	3.70	0.09	3.70	0.09
S2		15.3	0.4	15.0	0.4	15.4	0.4	15.4	0.4
Model		BS-300 ⁵		BS-330E ⁶		BS-360E ⁷		BS-380 ⁸	
Level	Unit	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty
S1	μmol/L	3.80	0.10	3.80	0.10	3.70	0.09	3.80	0.10
S2		15.0	0.4	15.0	0.4	15.4	0.4	15.0	0.4
Model		BS-400 ⁹		BS-430 ¹⁰		BS-480 ¹¹		BS-600 ¹²	
Level	Unit	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty
S1	μmol/L	3.80	0.10	3.70	0.09	3.70	0.09	3.70	0.09
S2		15.0	0.4	15.4	0.4	15.4	0.4	15.4	0.4
Model		BS-600M ¹³		BS-800 ¹⁴		BS-2000 ¹⁵		BS-2800M ¹⁶	
Level	Unit	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty	Calibrator Value	Expanded Uncertainty
S1	μmol/L	3.70	0.09	3.70	0.09	3.70	0.09	3.70	0.09
S2		15.3	0.4	15.4	0.4	15.5	0.4	15.4	0.4

mindray

HbA1c Calibrator

For use on: BS-2000, BS-2800M,
BS-600M, BS-800

LOT 044624003

🕒 2025-09-15

