

BeneFusion mNP

Enteral Feeding Pump

Operator's Manual



© Copyright 2023 Shenzhen Mindray Scientific Co., Ltd. All rights reserved.

Release date: December 2023


Revision: 1.0

Intellectual Property Statement

SHENZHEN MINDRAY SCIENTIFIC CO., LTD. (hereinafter called Mindray Scientific) owns the intellectual property rights to this product and this manual. This manual may refer to information protected by copyrights or patents and does not convey any license under the patent rights of Mindray Scientific, nor the rights of others.

Mindray Scientific intends to maintain the contents of this manual as confidential information. Disclosure of the information in this manual in any manner whatsoever without the written permission of Mindray Scientific is strictly forbidden.

Release, amendment, reproduction, distribution, rental, adaptation, translation of this manual in any manner whatsoever without the written permission of Mindray Scientific is strictly forbidden.

mindray,  **MINDRAY**, and **BeneFusion** are the registered trademarks or trademarks owned by Mindray in China and other countries. Mindray Scientific is authorized by Mindray to use above registered trademarks or trademarks.

Responsibility on the Manufacturer Party

Contents of this manual are subject to changes without prior notice.

All information contained in this manual is believed to be correct. Mindray Scientific shall not be liable for errors contained herein nor for incidental or consequential damages in connection with the furnishing, performance, or use of this manual.

Mindray Scientific is responsible for the effects on safety, reliability and performance of this product, only if:

- all installation operations, expansions, changes, modifications and repairs of this product are conducted by Mindray Scientific authorized personnel;
- the electrical installation of the relevant room complies with the applicable national and local requirements;
- the product is used in accordance with the instructions for use.

Warranty

THIS WARRANTY IS EXCLUSIVE AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE.

Exemptions

Mindray Scientific's obligation or liability under this warranty does not include any transportation or other charges or liability for direct, indirect or consequential damages or delay resulting from the improper use or application of the product or the use of parts or accessories not approved by Mindray Scientific or repairs by people other than Mindray Scientific authorized personnel.

This warranty shall not extend to

- Malfunction or damage caused by improper use or man-made failure.
- Malfunction or damage caused by unstable or out-of-range power input.
- Malfunction or damage caused by force majeure such as fire and earthquake.
- Malfunction or damage caused by improper operation or repair by unqualified or unauthorized service people.
- Malfunction of the instrument or part whose serial number is not legible enough.
- Others not caused by instrument or part itself.

Company Contact



Manufacturer: Shenzhen Mindray Scientific Co., Ltd.
Address: 6/F, Bldg 2, 1203 Nanhuan Avenue, Yutang Block,
Guangming District, 518106 Shenzhen, P.R.China
Website: www.mindray.com
E-mail Address: service@mindray.com
Tel: +86 755 81888998
Fax: +86 755 26582680



EC-Representative: Shanghai International Holding Corp. GmbH (Europe)
Address: Eiffestraße 80, 20537 Hamburg, Germany
Tel: 0049-40-2513175
Fax: 0049-40-255726

Notification of Adverse Events

As a health care provider, you may report the occurrence of certain events to SHENZHEN MINDRAY SCIENTIFIC CO., LTD., and possibly to the competent authority of the Member state in which the user and / or patient is established.

These events, include device-related death and serious injury or illness. In addition, as part of our Quality Assurance Program, SHENZHEN MINDRAY SCIENTIFIC CO., LTD. requests to be notified of device failures or malfunctions. This information is required to ensure that SHENZHEN MINDRAY SCIENTIFIC CO., LTD. provides only the highest quality products.

Preface

Manual Purpose

This manual contains the instructions necessary to operate the product safely and in accordance with its function and intended use. Observance of this manual is a prerequisite for proper product performance and correct operation and ensures patient and operator safety.

This manual is based on the maximum configuration and therefore some contents may not apply to your product. If you have any question, please contact us.

This manual is an integral part of the product. It should always be kept close to the equipment so that it can be obtained conveniently when needed.

Intended Audience

This manual is geared for clinical professionals who are expected to have a working knowledge of medical procedures, practices and terminology as required for monitoring of critically ill patients.

Illustrations

All illustrations in this manual serve as examples only. They may not necessarily reflect the setup or data displayed on your equipment.

Conventions

- ***Italic text*** is used in this manual to quote the referenced chapters or sections.
- ***Bold text*** is used to indicate the screen texts.
- → is used to indicate operational procedures.

Contents

1 Safety	1 - 1
1.1 Safety Information	1 - 1
1.2 Equipment Symbols	1 - 3
2 Equipment Introduction	2 - 1
2.1 Intended Purpose	2 - 1
2.2 Indirect Benefit	2 - 2
2.3 Applied Part	2 - 2
2.4 Main Unit	2 - 2
2.5 Screen Display	2 - 4
3 Equipment Preparation	3 - 1
3.1 Equipment Preparation Safety Information	3 - 1
3.2 Installation	3 - 2
3.3 Setting Up the Equipment	3 - 2
4 Getting Started	4 - 1
4.1 Turning on the Pump	4 - 1
4.2 Preparing the IV Container	4 - 1
4.3 Loading the Infusion Set	4 - 1
4.4 Purge	4 - 3
4.5 Starting Infusion	4 - 3
4.6 Bolus Infusion	4 - 4
4.7 Changing the Infusion Parameters	4 - 4
4.8 Pausing the Infusion	4 - 5
4.9 Setting Keep Tube Open (KTO) Rate	4 - 5
4.10 Replacing the IV Container	4 - 5
4.11 Unloading the Infusion Set	4 - 6
4.12 Viewing the Infused Volume	4 - 6
4.13 Entering the Standby Mode	4 - 7
4.14 Turning Off the Pump	4 - 7
5 Alarms	5 - 1
5.1 Alarm Safety Information	5 - 1
5.2 Understanding the Alarms	5 - 1
5.3 Alarm Screen	5 - 2
5.4 Resetting Alarms	5 - 2
5.5 Pausing Alarm Sound	5 - 3
5.6 Alarm Solutions	5 - 3

5.7 Occlusion Alarm	5 - 6
6 Menu Options	6 - 1
6.1 General Option	6 - 1
6.2 Department Management	6 - 2
6.3 System Options	6 - 2
6.4 User Maintenance	6 - 2
7 Infusion Modes	7 - 1
7.1 Continuous Feeding/Time Feeding/Nourishing Feeding	7 - 1
7.2 Intermittent Feeding	7 - 1
8 Feeding Library	8 - 1
9 Networked Communication	9 - 1
9.1 Network Safety Information	9 - 1
9.2 Connecting the Equipment to the CMS	9 - 1
9.3 Connecting the Equipment to the eGateway	9 - 2
10 Maintenance	10 - 1
10.1 Maintenance Safety Information	10 - 1
10.2 Maintenance and Testing Schedule	10 - 2
10.3 Maintaining the Battery	10 - 3
10.4 Checking the History Record	10 - 4
10.5 Disposing of the Equipment	10 - 4
11 Care and Cleaning	11 - 1
11.1 Care and Cleaning Safety Information	11 - 1
11.2 Cleaning the Equipment	11 - 2
11.3 Disinfecting the Equipment	11 - 2
11.4 Cleaning the Pole Clamp and DC Adapter	11 - 5
11.5 Disinfecting the Pole Clamp and DC Adapter	11 - 5
11.6 Sterilization	11 - 6
12 Accessories	12 - 1
A Product Specifications	A - 1
A.1 Specifications	A - 1
A.2 Wireless Network	A - 2
A.3 Infusion Specifications	A - 3
A.4 Recommended Infusion Sets	A - 4
A.5 Occlusion Alarm Delay and Bolus Volume	A - 5
A.6 Infusion Accuracy Graphs	A - 6

A.7 Operating Environment A - 9

B EMC and Radio Regulatory ComplianceB - 1

 B.1 EMCB - 1

 B.2 Radio Regulatory ComplianceB - 9

C AbbreviationsC - 1

This page intentionally left blank.

1 Safety

1.1 Safety Information

WARNING

- Indicates a potential hazard or unsafe practice that, if not avoided, could result in death, serious injury or damage to product/property.
-
-

CAUTION

- Indicates a potential hazard or unsafe practice that, if not avoided, could result in minor personal injury, product malfunction or damage to product/property.
-
-

NOTE

- Provides application tips or other useful information to ensure that you get the most out of the product.
-
-

1.1.1 Warnings

WARNING

- To avoid risk of electric shock, the equipment must only be connected to mains power with protective earth. If a protective earth conductor is not provided, operate it on battery power, if possible.
- To avoid explosion hazard, do not use the equipment in the presence of oxygen-rich atmospheres, flammable anesthetics, or other flammable agents.
- The equipment is not intended to be used within the Magnetic Resonance (MR) environment.
- Do not use the multiple portable socket outlets (MPSO) or AC mains extension cords. Ensure that the sum of the individual ground leakage currents does not exceed the allowable limits.

- **Do not open the equipment housings. All servicing and future upgrades must be carried out by trained and authorized personnel. Moreover, the servicing must be done only after the AC power supply is disconnected.**
 - **Do not place the equipment or accessories in any position that might cause it to fall on the patient.**
 - **Do not start an infusion unless the setup was verified to be correct.**
 - **To avoid inadvertent disconnection, route all cables in a way to prevent a stumbling hazard. Wrap and secure excess cabling to reduce risk of entanglement by patients or personnel.**
 - **Clearing the occlusion result from line kinks, filter coagulation, etc. may cause extra bolus to patients. Appropriate measures should be taken.**
 - **Do not touch the patient and device connectors simultaneously. Otherwise leakage current may result in patient injury.**
 - **To avoid electric shock, do not touch patient and other non-defibrillation proof equipments during defibrillation. Defibrillation will not affect the performance of the equipment.**
 - **Keep a distance of at least 20 cm away from the patient when operating the pump.**
 - **Ensure that the pumps are being operated correctly according to the warnings in the manual. If not, the devices should be observed that they are working normally.**
-
-

1.1.2 Cautions

CAUTION

- **When several infusion lines are connected to the same vascular access, there may be back flow or prolonged response time of occlusion alarm. Therefore, use check valve at the line end or follow local hospitals' instructions while in connection with other infusion system.**
- **When using this equipment for enteral nutrition, do not use enteral fluids for intravenous infusion to avoid patient injury, and use only dedicated disposable enteral feeding sets for enteral nutrition.**
- **Ensure that the equipment is supplied with continuous electric power during work. Sudden power failure may cause data loss. If it is not sure whether the power supply is stable, it is recommended to prepare another equipment in case of power failure.**
- **Electromagnetic fields may affect equipment performance. This makes it necessary for other equipment used in the vicinity of this equipment to meet EMC standards. Mobile phones, X ray and MRI equipment are all potential interference sources because of their high-intensity electromagnetic radiation.**

- Always install or carry the equipment properly to avoid damage caused by drop, impact, strong vibration or other mechanical force. The equipment should be observed to verify normal operation after fall, otherwise it cannot be used.
- Dry the equipment immediately in case of rain or water spray.
- Some settings are password protected and can only be changed by authorized personnel. Contact your department manager or biomedical engineering department for the passwords used at your facility.











1.1.3 Notes





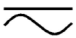











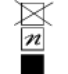


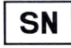NOTE

- The software was developed in compliance with IEC62304.
- The equipment provides power-down storage. Alarms limit setting and history record are saved and will be maintained if the equipment is powered down suddenly. The storage time is equals to the equipment's service life. The alarm limit settings before power-down are reloaded when the equipment is restarted.
- This manual describes all features and options. Your equipment may not have all of them.



1.2 Equipment Symbols

Some symbols may not appear on your equipment.

	Refer to instruction manual/ booklet		Caution
	Alternating current		Input/output
	Both direct and alternating current		Direct current
	Battery		USB connector
	Stand-by		Stop

	Non-ionizing electromagnetic radiation		General warning sign
	Unique device identification		Medical Device
	Date of manufacture		Manufacturer
	Serial number		Authorized representative in the European Community
IP44	Protected against solid foreign objects with a diameter no less than 1.0 mm in diameter. Protected against splashing water.		DEFIBRILLATION-PROOF TYPE CF APPLIED PART
	Dispose of in accordance to your country's requirements		Temperature limitations
	Atmospheric pressure limitations		Humidity limitations
	THIS WAY UP		Keep dry
	Fragile, handle with care		STACKING LIMIT BY NUMBER
	CLASS II equipment		
	The product bears CE mark indicating its conformity with the provisions of the REGULATION (EU) 2017/745 on medical devices and fulfills the general safety and performance requirements of Annex I of this regulation. Note: The product complies with the Council Directive 2011/65/EU.		

The general meaning assigned to geometric shapes, safety colors and contrast colors for safety signs are as follows:

Geometric shape	Meaning	Safety color	Contrast color	Graphical symbol color
	Mandatory action	Blue	White	White
	Warning	Yellow	Black	Black

This page intentionally left blank.

2 Equipment Introduction

2.1 Intended Purpose

The enteral feeding pump is intended for use for the delivery of enteral nutrition for enteral therapy with feeding set.

WARNING

- **This pump is intended for use only by clinical professionals or under their guidance. It must only be used by persons who have received adequate training in its use. Anyone unauthorized or untrained must not perform any operation on it.**
-
-

NOTE

- **According to the conclusion of clinical evaluation and residual risk evaluation, for the intended patients, there is no known side effects that can occur during or after the use of the medical device. And there is no need for the operator to make extra preparations. Thus, no residual risk associated with using the medical device should be disclosed.**
-

2.1.1 Indication for Use

The enteral feeding pump is intended for use for the delivery of enteral nutrition for enteral therapy with feeding set.

2.1.2 Intended Users

The enteral feeding pump is intended to be used by trained healthcare professionals.

2.1.3 Intended Patient Population

The enteral feeding pump can be used for adults and pediatrics. Not for use with neonates.

2.1.4 Intended Medical Conditions

The enteral feeding pump is expected to be used in institutes or units with healthcare capabilities.

2.1.5 Contra-indications

The enteral feeding pump can not be used for arterial or intravenous route.

2.1.6 Side-effects

None.

2.2 Indirect Benefit


Through SOTA analysis, since the enteral feeding pump is not directly used to treat diseases, it will not produce the direct clinical benefits, and its clinical benefits are mainly indirect clinical benefits (a positive impact on patient management): precise infusion.

2.3 Applied Part


The applied part of the equipment is the feeding set.

2.4 Main Unit

2.4.1 Front View

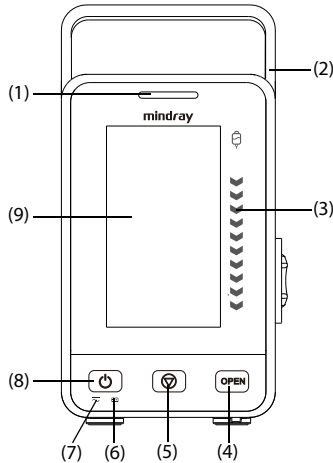


- | | |
|-------------------------------|----------------------|
| (1) Alarm light | (2) Handle |
| (3) Infusion status indicator | (4) Door opening key |
| (5) Stop key | (6) Battery LED |
| (7) External power LED | (8) Power switch |
| (9) Display | |

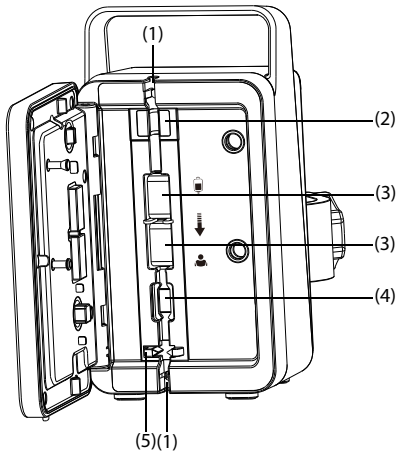

- | | |
|--|------------------------|
| (1) Tubing channel notches | (2) Air in line sensor |
| (3) Pumping mechanism (with waterproof membrane) | (4) Pressure sensor |
| (5) Anti free-flow clamp | |

2.4.2 Rear View

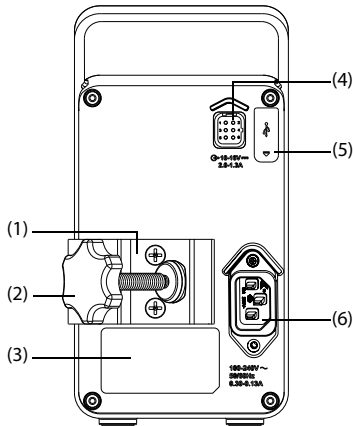

- | |
|-----------------------|
| (1) Pole clamp |
| (2) Pole clamp handle |
| (3) Product label |

- (4) Multifunctional connector (optional)
 - Uses as a DC power input connector.
 - Uses as a RS232 connector.
 - Connects to the hospital's nurse call system.
- (5) USB connector
Connects USB devices, for example the USB drive.
- (6) AC power input connector

2.5 Screen Display









The screen may look slightly different in different infusion modes. The infusion screen includes alarm information and system status information area, infusion status area, pressure status area and key area.




The pressure status area on the screen indicates the condition of the current pressure:

- Green: Pressure is normal.
- Yellow: Pressure is near the threshold for the infusion.
- Red: Pressure is beyond the threshold for the infusion.

2.5.1 On-screen Symbols





The following table lists the on-screen symbols:





Symbol	Description	Symbol	Description
	Audible alarm tones are paused.		Alarms are acknowledged and the alarm is reset.
	Alarms are acknowledged and the reminder sound is given.		Night mode
	Wireless network is connected. The solid part indicates network signal strength.		Wireless network is not connected.
	The battery works correctly. The solid portion represents the remaining charge.		The battery is being charged.

Symbol	Description	Symbol	Description
	The battery has critically low charge and needs to be charged immediately. Otherwise, the equipment will automatically shut down.		No battery is installed, battery fault, battery communication fault, or battery charging fault. Contact service personnel for help.
	The battery has low power and needs to be charged.		

2.5.2 Operation Keys

The equipment provides operation keys for you to access some functions. The following table shows available operation keys.

Symbol	Label	Function	Symbol	Label	Function
	AudioPause	Pauses alarm sound.		AlarmReset	Acknowledges the ongoing alarms.
	Lock	Locks the touchscreen.		Purge	Initiate a purge.
	Volume	Enters the Volume menu.		Menu	Enters the Menu .
	Link Code	Enters the Link Code menu.		End Prescription	Ends the current prescription.
	Exit	Returns to the main screen.		Bolus	Initiate a Bolus infusion.
	Start	Starts an infusion.		Stop	Pause an infusion.
	Back	Returns to the previous screen or the parameter setup screen.		Home	Returns to the main screen.

Symbol	Label	Function	Symbol	Label	Function
	Setup	Enters the Standby Time setup menu or the parameter setup screen.		Cancel	Cancels the shutdown and returns to the main screen.
	Turn Off	Turn off the pump.		Standby	Enters Standby.

2.5.3 Using the Touchscreen





You can use the touchscreen to select a screen element by pressing directly on the pump's screen.

To avoid misuse, the touchscreen is locked automatically if no operation is detected in the preset time. To manually lock the touchscreen, swipe the touchscreen from top down, and select **Lock**.

To unlock the touchscreen, select  on the touchscreen and swipe the slide as instructed.

2.5.4 Using the On-Screen Keyboard

The on-screen keyboard enables you to enter information:

- Enter the information by selecting one character after another.
- Select the Backspace key  to delete single characters.
- Select the Caps Lock key  to switch uppercase letters and lowercase letters.
- Select the Enter key  to confirm the entry and close the on-screen keyboard.
- Select the Space bar  to enter a space.

3 Equipment Preparation

3.1 Equipment Preparation Safety Information

WARNING

- Use only installation accessories specified by Mindray Scientific.
 - The equipment software copyright is solely owned by Mindray Scientific. No organization or individual shall resort to modifying, copying, or exchanging it or to any other infringement on it in any form or by any means without due permission.
 - Connect only approved devices to this equipment. Devices connected to the equipment must meet the requirements of the applicable IEC standards (e.g. IEC 60950 safety standards for information technology equipment and IEC 60601-1 safety standards for medical electrical equipment). The system configuration must meet the requirements of the IEC 60601-1 medical electrical systems standard. Any personnel who connect devices to the equipment's signal input/output port are responsible for providing evidence that the safety certification of the devices has been performed in accordance to the IEC 60601-1. If you have any questions, please contact Mindray Scientific.
 - If it is not evident from the equipment specifications whether a particular combination with other devices is hazardous, for example, due to summation of leakage currents, please consult the manufacturer or an expert in the field. A determination must be made that the proposed combination will not negatively affect the devices themselves or the patient's safety.
 - Ensure that the equipment is properly secured and positioned. Position change and severe shock may lead to changes to the delivery accuracy.
-
-

CAUTION

- The equipment should be installed by the authorized personnel.
 - Before use, verify whether the packages are intact. In case of any damage, do not apply it to patient.
-

NOTE

- **Save the packing case and packaging material as they can be used if the equipment must be reshipped.**
-

3.2 Installation

3.2.1 Pole Clamp Installation

By turning the pole clamp handle, the pole clamp secures the pump to either a horizontal or vertical bar of the medical supply unit or IV pole.

3.2.2 IV Pole Installation

To mount the pumps to an IV Pole, ensure that the number of the pumps does not exceed any of the following combinations:

- MIP-300 IV Pole: maximum two pumps
- MIP-600 IV Pole: maximum two pumps
- MIP-500 IV Pole: maximum four pumps

NOTE

- **After two pumps are mounted to a MIP-300 IV Pole, ensure that the maximum load of IV container does not exceed 4 kg. Failure to do so can result in system becoming unbalanced.**
 - **Remove the IV containers from the IV Poles before transporting the equipment. Carry each component separately. Failure to do so can result in system becoming unbalanced.**
-

3.3 Setting Up the Equipment

Before getting started, ensure that the pump is properly set up:

- The pump is placed on a stable surface, or properly mounted to an IV pole using the pole clamp.
- The pump is plugged into a properly-grounded AC power outlet. When AC mains is connected, the external power LED is illuminated in green.
- If the pump is run on battery power, ensure that the battery is adequately charged.
- To ensure the history records are stored correctly, the system date and time should be checked before first use.

WARNING

- **Always use the accompanying power cord delivered with the pump.**
 - **Before connecting the equipment to the AC mains, check that the voltage and frequency ratings of the power line are the same as those indicated on the equipment.**
 - **Do not touch the power connector with wet hand. Eliminate the liquid or any residue inside of or at the surroundings of the AC power input connector and power cord connectors.**
 - **Use the battery if the integrity of the protective earth conductor or the protective earthing system in the installation is in doubt.**
-
-

CAUTION

- **When connected with the adapter, it is specified as a part of the equipment. Use only the specified adapter.**
-

This page intentionally left blank.

4 Getting Started

4.1 Turning on the Pump

Press the power switch to turn on the pump. The pump automatically performs a self test at startup. Check that the alarm tone is heard and the alarm lamp illuminates, one after the other, in red and yellow. This indicates that the visible and audible alarm indicators function correctly. The loading guide screen displays.

WARNING

- **Before putting the system into operation, the operator must verify that the equipment, connecting cables and accessories are in correct working order and operating condition.**
 - **Check that visual and auditory alarm signals are presented correctly when the equipment is powered on. Do not use the equipment if you suspect it is not working properly, or if it is mechanically damaged. Contact your service personnel or us.**
-

NOTE

- **Stay within 1 meter (39 inches) of the pump while setting it up and operating it, making sure that you have a clear view of the pump interface.**
 - **The equipment uses a mains plug as isolation means to the mains power. Do not locate the equipment in a place difficult to operate the mains plug.**
-

4.2 Preparing the IV Container


The height between the IV container and the pump is critical to the flow accuracy. To prepare the IV container, follow this procedure:

1. Hang the IV container.
2. Adjust the height of the pump so that there is 51 ± 5 cm (20 ± 2 in.) between the fluid line and the middle of the pump. Distances outside of this tolerance can adversely affect flow accuracy.

4.3 Loading the Infusion Set

1. Close the roller clamp or Robert clamp.
2. Press the door opening key to open the pump door.

3. Avoiding any slack, insert the infusion line into the slot, following the flow direction indicator (1). Ensure that the infusion set is straightly and firmly clipped into the tubing channel notches (2) on both sides of the casing.

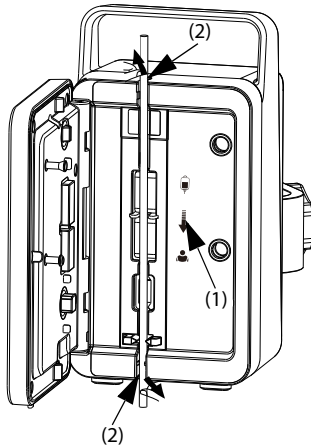

4. Close the pump door.

WARNING



- To ensure the accuracy of air bubble detection, check and remove the remained fluid in the infusion set slot before loading the infusion set.
 - While loading the infusion set, do not touch the free-flow clamp to avoid being hurt.
 - This pump uses standard, single use infusion set with Luer lock connections.
 - The pump must be mounted to the same level as the patient's heart. The most accurate pressure monitoring in the infusion set is achieved when the pump is positioned close to the patients heart level.
 - We recommend you to use an infusion set stated in this manual. If a non-recommended infusion set must be used or a different set needs to be changed, perform the calibration and performance test before use. Otherwise, the accuracy of the infusion and the performance of the pump may be adversely affected.
 - To ensure the accuracy of rate and alarm detection, the infusion set should be calibrated in this pump before first use.
 - Single use accessories are not designed to be reused. Reuse may cause a risk of contamination and affect the measurement accuracy.
-

NOTE

- **Take care that your hands are not squeezed when you close the pump door.**
 - **Make sure that the infusion set is located in both sides of the tubing channel notches after loading the infusion set.**
-

4.4 Purge

The infusion set should be purged prior to being connected to a patient. If the infusion set is not purged before being loaded into the pump, proceed as follows to purge the line:


1. Ensure that the pump is disconnected from the patient.
2. Open the roller clamp or Robert clamp.
3. Swipe the touchscreen from top down and select .
4. Select  to start purging.

NOTE

- **If required, set the purge rate after the purge is started. The initial purge rate is 1200 ml/h.**
 - **The Air in Line alarm will not given during purging.**
 - **The volume used for purging is not added to the infused volume.**
-

4.5 Starting Infusion

To start infusion, follow this procedure:

1. Set infusion parameters, and check the following:
 - ◆ Verify parameter settings according to the prescriber's order.
 - ◆ Verify that the displayed infusion set brand and type correspond with the currently used infusion set.
 2. Connect the infusion set to the patient access device.
 3. Press  to start infusion.
-
-

WARNING

- **Do not connect patient until disposables have been purged and loaded into the pump. Connecting to patient before disposables are loaded and purged can cause serious injury or death.**

- **Check that no drops are falling in the drip chamber before infusion starts or stops. If drips are falling, close the roller clamp or the Robert clamp, do not use the equipment, and contact your service personnel.**
-
-




NOTE

- **Always discharge the previous patient before starting a new infusion for new patient. Failure to do so can lead to data being attributed to the wrong patient.**
 - **If the infusion set type is Nutrition or the selected drug type is Enteral Nutrition, “Air detection will be closed” is always displayed in the system information area, “[Enteral feeding] Air detection will be closed” is displayed before starting the infusion or bolus infusion.**
 - **The infusion could not be started when the door is open.**
 - **Monitor the connection of infusion set, tubing, pump and patient, and the drug information on a regular basis. Start infusion according to the instructions in this manual.**
-

4.6 Bolus Infusion

Bolus infusion is a controlled volume of fluid or drug being delivered at an increased rate for diagnostic or therapeutic purposes. The pump should be connected to the patient during bolus infusion.

To perform bolus infusion, follow this procedure:

1. Select  from the main screen.
2. Set the bolus volume in the popup dialog.
3. Select  to start an automatic bolus infusion, or press and hold  to start a manual bolus infusion.

NOTE


- **The delivered bolus volume will be added to the total infusion volume and subtracted from the volume to be infused (VTBI).**
 - **The pump gives a beep every time a 0.5 ml bolus volume is infused.**
-

4.7 Changing the Infusion Parameters


You can modify rate or drug name without stopping the infusion. This function is called titration.


1. Select the above parameters in the infusion running screen.
2. Reconfigure the parameters in the popup dialogs.

To change other infusion parameters, follow this procedure:

1. Press  to pause the infusion.
2. Select the desired parameter area, and reconfigure parameters as per the prescriber's order.

4.8 Pausing the Infusion

Press  to temporarily stop a running infusion.

Press  again to restart the infusion after the infusion solution change.

4.9 Setting Keep Tube Open (KTO) Rate

At the end of infusion, the pump continues to infuse at a very low rate. KTO is used to keep the patient's vein open, to prevent back flow or vascular occlusion.

The default KTO rate is 0.5 ml/h. To edit the KTO rate, follow this procedure:


1. Swipe the touchscreen from top down → select **Menu** → select **General Option**.
2. Set the **KTO Rate**. If **KTO Rate** is zero, the pump will not initiate a KTO infusion when the preset volume is complete.

NOTE

- **If the KTO rate is greater than the infusion rate, the pump will continue to infuse at the set infusion rate.**
 - **The pump runs for 30 minutes at a KTO rate. At the completion of the KTO infusion, the pump stops infusion, and gives a KTO Finish alarm.**
 - **The volume used during KTO infusion will be added to the total infusion volume.**
-

4.10 Replacing the IV Container

To replace the IV container, follow this procedure:


1. Press  to pause the infusion.
2. Close the roller clamp or Robert clamp to avoid free flow.
3. Remove the IV container from the IV pole, and replace a new container.

NOTE

- **Check the infusion status after replacing the IV container. If the pump is running a KTO infusion, reconfigure the infusion parameters before restarting the infusion.**
-

4.11 Unloading the Infusion Set

To unload the infusion set, follow this procedure:

1. In the main screen, select  to stop the infusion.
2. Close the roller clamp or Robert clamp.
3. Disconnect the patient from the infusion set.
4. Press the door opening key to open the pump door.
5. On the outside of the pump, grasp the tubing on both sides of the pump and pull the tubing straight out of the pumping channel.

WARNING


- **To prevent free flow, ensure that the roller clamp or Robert clamp is closed before removing the infusion set from the pump.**
 - **It is recommended that the infusion sets be changed every 24 hours (or as per national hygiene regulations or manufacturer's instructions). If infusion sets not recommended by this manual are used, adjust the fixing site every four hours.**
 - **It is recommended that the disposable enteral nutrition sets be changed every 24 hours.**
 - **If the infusion sets are not changed within the recommended time, the accuracy of the infusion may be affected.**
-
-

4.12 Viewing the Infused Volume

The **Volume** dialog allows you to review the infused volume of up to 24 hours. You can also view the infused volume of the configured time interval and time length.

Choose either of the following ways to enter the **Volume** dialog:


- Swipe the touchscreen from top down → select **Volume**.
- Select **Volume** from the **Pause** screen.

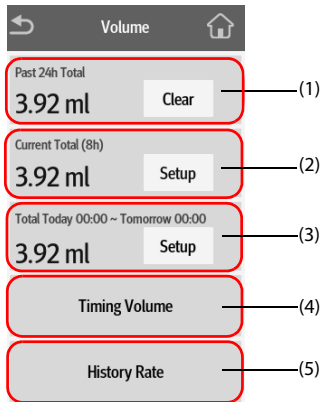


- (1) **Past 24h Total:** view the total infused volume in the past 24 hours. The display range is 0 ml to 99999.99 ml. Select **Clear** to clear the infused volume.
- (2) View the recent total infused volume. Configure the time before viewing the total infused volume within the configured time.
- (3) View the total infused volume in the configured time period. Configure the time period before viewing the total infused volume in the configured time period.
- (4) **Timing Volume:** view the total infused volume of the configured timing interval. Configure the **Timing Interval** before viewing the total infused volume of each interval.
- (5) **History Rate:** view the history rate.

4.13 Entering the Standby Mode

To enter the standby mode, hold the power switch and select **Standby**.

While the pump is in the standby mode, select  to set the standby time. The maximum standby time is 24 hours. When the configured standby time is expired, the pump triggers the **Standby Time Expired** alarm and exits the standby mode automatically.

To manually exit the standby mode, select .

4.14 Turning Off the Pump

To turn off the pump, press and hold the power switch and select **Turn Off**. If the infusion set is not removed from the pump, the pump gives prompt message **Remove infusion set to turn off**. Select **OK**, and unload the infusion set.

CAUTION

- **Press and hold the power switch for no less than 10 seconds to forcibly shut down the pump if it could not be shut down normally. This may cause loss of patient data.**
-

NOTE

- **Turning off the pump does not disconnect the pump from the AC mains. To completely disconnect the power supply, unplug the power cord.**
-

5 Alarms

5.1 Alarm Safety Information

WARNING

- A potential hazard can exist if different alarm presets and default settings are used for the same or similar equipment in the same care area, for example an intensive care unit or cardiac operating room.
 - The equipment in your care area may each have different alarm settings to suit different patients. Always check that the alarm settings are appropriate for your patient before start infusing.
 - When the alarm sound is paused, the equipment gives no alarm tones even if a new alarm occurs. Be careful about whether to pause the alarm sound or not. When the alarm sound is paused, observe the patient frequently.
 - Do not rely exclusively on the audible alarm system during an infusion. Adjustment of alarm volume to a low level may result in a hazard to the patient. Always make sure that the audio alarm volume level is adequate in your care environment. Always keep the patient under close surveillance.
 - Fully evaluate the risk before changing the alarm mode setting. New alarms may be failed to be detected if the operator is not familiar with the new sound.
-
-

5.2 Understanding the Alarms

By severity, the alarms are classified into the high priority alarms and low priority alarms. When an alarm occurs, the equipment indicates it visually and audibly. For more information, see the following table.

Alarm priority	Alarm lamp color	Alarm lamp flashing frequency	Alarm sound interval	Alarm message	Alarm priority indicator	Duty Cycle
High priority alarm	Red	2.0 ± 0.6 Hz	5s (±2s)	White text or symbol inside a red box	!!!	20% to 60%


Alarm priority	Alarm lamp color	Alarm lamp flashing frequency	Alarm sound interval	Alarm message	Alarm priority indicator	Duty Cycle
Low priority alarm	Yellow	Not flashing	20s ($\pm 2s$)	Black text or symbol inside a yellow box	!	100%

NOTE


- The tones of the alarm sound and the reminder sound are different.
- The frequency of the reminder sound and the bolus sound is 600Hz, which is different from the frequency of alarm sound.
- When multiple alarms occur simultaneously, the alarm messages are displayed circularly, and the sound and light of the higher priority alarm are given.

5.3 Alarm Screen

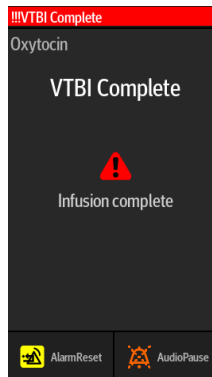

When an alarm occurs, the alarm screen is displayed to help you identify the problem.



5.4 Resetting Alarms

When an alarm occurs, press  to acknowledge and reset the alarm. The alarm reset state has the following features:



- The alarm sound is silenced, and the alarm screen disappears.

- A  appears before the alarm message, indicating that the alarm is acknowledged.
- The alarm reset symbol  is displayed after the alarm message.


For the **VTBI Complete**, **KTO Finish**, **Standby Time Expired** and **Empty** alarms, when they are reset, all the alarm indications (alarm sound, alarm message, and alarm light) disappear.


5.5 Pausing Alarm Sound



To enter the audio pause state, choose one of the following ways:

- Select  in the alarm screen.
- Swipe the touchscreen from top down, and select .

The audio pause state has the following features:

- Except for the **Battery Depleted** alarm, the sound of all alarms are silenced for two minutes.
- The audio pause symbol  is displayed in the system information area.
- If a new alarm is triggered during the audio pause state, the sound of the new alarm will also be silenced.

When the audio pause time expires, the audio paused state is automatically deactivated. You can also cancel the audio paused state by pressing  again.


For the **Low Battery**, **Reminder**, and **Time Near End** alarms, press  and the pump gives a reminder sound every 5 minutes. The symbol  is displayed after the alarm message.






5.6 Alarm Solutions


WARNING

- **When an alarm occurs, check the pump's status and handle the alarm as soon as possible. If the alarms do not conform with the actual situation, contact your service personnel.**
-
-

Alarm	Priority	Causes	Solutions
Air in Line	High	Single air bubble in the infusion set exceeds the preset Bubble Size limit.	<ul style="list-style-type: none"> • Disconnect the patient from the pump, and purge the line. • Check the single bubble limit is properly setup.

Alarm	Priority	Causes	Solutions
Upstream Occlusion	High	An occlusion occurred at the supply end (the tubing between the container and the pump).	<ul style="list-style-type: none"> • Check if kinks are in tubing and tubing isn't damaged. • Check the infusion set patency.
Downstream Occlusion	High	An occlusion occurred at the patient end (the tubing between the patient and the pump). The preset pressure limit is exceeded.	<ul style="list-style-type: none"> • Check if kinks are in tubing and tubing isn't damaged. • Check the infusion set patency. • Check the pressure limit is properly setup.
Infusion Set Disengaged	High	The infusion set is disengaged from the pump.	Close the roller clamp or the Robert clamp to stop the infusion, and reload the infusion set.
No Infusion Tube	High	The infusion set is not loaded properly, or start infusion when the infusion set is not loaded.	Close the roller clamp or the Robert clamp to stop the infusion, and reload the infusion set.
Infusion Set Error	High	Abnormal pressure is detected after loading the infusion set.	Reload the infusion set.
Long time fast infusion	High	The pump runs for 2.5 hours at a rate higher than 1600ml/h.	Pause the infusion.
Door Open	High	Door is opened abnormally while the pump is infusing or not infusing.	Close the pump door.
Door Error	High	The pump door faults.	Close the pump door.
Battery Depleted	High	The battery is depleted.	Connect the pump to the external power source.
VTBI Complete	High	The preset VTBI is completed.	<ul style="list-style-type: none"> • Press  to reset the alarm. • Continue therapy or select new therapy.

Alarm	Priority	Causes	Solutions
KTO Finish	High	The KTO infusion is running for thirty minutes.	<ul style="list-style-type: none"> Press  to reset the alarm. Continue therapy or select new therapy.
System Error	High	The pump system faults, such as battery error, etc.	Stop using the pump, and contact your service personnel.
Start Frequently	High	Start infusion 5 times within 10 seconds.	Press  to clear the alarm.
KTO Running	Low	The infusion is completed and the pump continues infusion at the KTO rate.	<ul style="list-style-type: none"> The alarm is cleared after the KTO infusion reaches 30 minutes. Press  to pause the KTO infusion. Complete the infusion or prepare for a new therapy.
Battery in Use	Low	The external power source has been disconnected and the pump runs on battery power.	<ul style="list-style-type: none"> Press  to reset the alarm. Connect the pump to the external power source.
Battery Error	Low	Battery fault, such as battery over heat, charging failure, etc.	Contact your service personnel.
CMS/eGW Disconnected	Low	The pump is disconnected from the CMS, the wireless network connection symbol disconnects.	Reconnect the pump with the central station, the wireless network connection symbol restores.
Standby Time Expired	Low	The preset standby time is reached.	Press  to reset the alarm.
System Time Error	Low	The real time clock (RTC) reset or RTC fault.	Reset the system time.
Time Near End	Low	The remaining infusion time reaches the configured time near end.	Complete the infusion or prepare for a new therapy.

Alarm	Priority	Causes	Solutions
Reminder	Low	No operation is detected after the preset Reminder Time is reached.	Turn off the pump or enter the standby.
Low Battery	Low	Low battery.	Connect the pump to the external power source.
Para. Unconfirmed	Low	No operation is detected for 10 seconds in the parameter edit state.	<ul style="list-style-type: none"> • Press  to acknowledge the alarm. • Edit and confirm the parameter setting.

NOTE

- **The pump stops infusion when a high priority alarm is triggered.**
- **The pump continues infusion when a low priority alarm is triggered.**
- **The pump stops infusion after the first Battery Depleted alarm occurs, and the shutdown delay is at least three minutes.**

5.7 Occlusion Alarm

Signals collected by the built-in pressure sensor is used for pressure calculation by the internal Central Processing Unit (CPU). The calculated pressure value is compared with the set occlusion alarm limit, the pump gives prompt message **Pressure increasing.Occlusion?** when the downstream pressure continuously increases and exceeds the set limit. The pump stops the infusion and gives an **Occlusion** alarm when the pressure exceeds the set limit.

The pump restarts the infusion when the pressure that caused the alarm is reduced. When the number of auto restarts has been reached, the infusion will not restart after an occlusion alarm. A bolus reduction is automatically initiated by the pump after the restart is failed or the occlusion alarm is reset.

NOTE

- **If this pump is running at 0.1 ml/h, and respectively configure the occlusion pressure alarm limit to the lowest level and highest level, the occlusion alarm delay time may reach up to one hour and 20 hours. Adjust the pressure limit to a lower level for a low rate infusion.**

6 Menu Options

6.1 General Option

Swipe the touchscreen from top down → select **Menu** → select **General Option**.

Menu Item	Range	Function
Bubble Size	50, 500, 1000, 1500, 2000, 4000 μ l	Set the alarm limit for the size of single air bubble.
Time Near End	Off, 1, 2, 3, 4, 5, 6, 7, 8, 9, 10, 15, 20, 25, 30 min	Set for how long the Time Near End alarm is triggered since the infusion is completed.
Reminder Time	Off, 1, 2, 3, 4, 5 min	Set for how long the Reminder alarm is triggered since the pump is last operated.
Lock Time for No Infusion	Off, 1, 2, 3, 4, 5 min	Set for how long the touchscreen automatically locks since the pump is last operated while the pump is not infusing.
Lock Time in Infusion	Off, 15 sec, 30 sec, 1min, 2min, 3min, 4min, 5min	Set for how long the touchscreen automatically locks since the pump is last operated while the pump is infusing.
Common Mode	Each infusion mode	Check or uncheck the infusion mode. The checked infusion mode will be displayed in the infusion mode list of the infusion status area.
Large Font	Primary: Rate	Set the primary parameter can be displayed in the large font screen.
	Param 1: Time, VTBI, Volume, Weight/BSA	Set the parameter can be displayed in the large font screen.
	Param 2: Time, VTBI, Volume, Weight/BSA	Set the parameter can be displayed in the large font screen.

6.2 Department Management


Menu Item	Function
Applied Department	The checked department can be displayed in the title area of the drug selection screen. The drug in the drug selection screen switches to the drug of corresponding department.
Drug Management	Add drug, modify drug, and delete drug. Note: The build-in drug is not allowed to be deleted.
Config Management	Modify the parameter settings of the applied departments. After the parameters of configuration management are changed, the settings of general option and system options of the corresponding department will be changed synchronously.

6.3 System Options

Swipe the touchscreen from top down → select **Menu** → select **System Options**.

Menu Item	Function
Sound Volume	Set the sound volume. The set range is 1 to 8.
Brightness	Set the screen brightness. The set range is 1 to 8.
Brightness On Battery	Set the screen brightness when the pump runs on battery power. The set range is 1 to 8.
History Record	View the history record.
Export History Record	Export the history record.
Night Mode	Set the night mode switch, start time, end time, system volume, and screen brightness.
Nurse Call	Set the nurse call switch, signal type, trigger type, and alarm level.
Version Information	View the software version, brand library, and Wi-Fi version.

6.4 User Maintenance

Swipe the touchscreen from top down → select **Menu** → select **User Maintenance** → input the required password → select .

CAUTION

- **The maintenance settings can only be changed by authorized personnel. Contact your department manager or biomedical engineering department for the passwords used at your facility.**
-

Menu Item	Function
Device Management	Set the facility, department, device name, QR code type, QR code prefix, and asset number. Scan the QR code from the screen to accept prescription.
Patient Information	Select the patient location and whether patient is discharged automatically after the pump is turned off.
Network Setup	Set the WLAN, WLAN IP, Central Station Setup, and device discover.
Brand Management	Add brand, delete brand, and modify brand.
Time and Language	Set the date, time, date format, time format, and language. Note: This setting is effective after the pump has been restarted.
Unit Setup	Set the pressure unit. The options include: mmHg, kPa, bar, and psi. Set the weight unit. The options include: kg and lb. Set the height unit. The options include: cm and inch.
Alarm	Set the alarm sound mode and the sensitivity for detecting the Empty alarm.
Bolus Volume Unit	Sets the unit of bolus volume. The options include: ml.
Bolus Limit	Set the upper limit of the auto bolus volume setting, and the maximum volume of a manual bolus infusion.
Purge Limit	Set the maximum volume of the purge.
Prescription Setup	Set the infusion mode after the prescription is accepted.
Concentration Config	Set the concentration parameter for infusion modes.
Modify Password	Modify the password for accessing the User Maintenance menu.
Import and Export	Import configuration file or brand library from the USB drive. Export configuration or brand library to the USB drive.

This page intentionally left blank.

7 Infusion Modes

The pump supports continuous feeding, time feeding, nourishing feeding, and intermittent feeding.

7.1 Continuous Feeding/Time Feeding/Nourishing Feeding

In continuous feeding, time feeding, and nourishing feeding, the IV drug therapy continues to infuse at a set rate. Nourishing feeding is typically use for low rate infusions for pediatric patients. The infusion modes offer four parameters: rate, time, VTBI and Conc. When two of rate, time and VTBI are entered, the third is calculated.

NOTE

- **When infusing in the continuous feeding, time feeding, and nourishing feeding, you must set rate, but time and VTBI settings are optional.**
-

7.2 Intermittent Feeding

In the intermittent feeding, intermittent infusion and maintenance are performed alternately and circularly.

- Intermittent stage: the pump runs the high rate infusion at the set **Rate** and **Intmt. Vol.**
- Maintenance stage: the pump runs the low rate infusion at the set **Maintain Rate** and **Intmt. Time**. The pump does not infuse at this stage if the **Maintain Rate** is not set.

NOTE

- **Total VTBI and Maintain Rate are optional parameters. If the Maintain Rate is not set, infusion stops at the maintenance stage. If the Total VTBI is not set, the infusion stops when the IV container is empty.**
-

This page intentionally left blank.

8 Feeding Library

The pump can be configured with a feeding library, which predefines drugs, concentrations, occlusion pressure levels and other infusion parameters. Using a feeding library simplifies the infusion operation, and reduces the risk of operation fault.

The feeding library is created, edited, and imported via the PC program. It has the following features:

- Saving at least 5000 drug names.
- At least 30 colors are available for drug marking.
- Supporting at least 30 drug categories.
- Predefining drugs, concentrations, occlusion pressures, KTO rate, bolus volume limit.
- The feeding library can be imported to this pump after being created via the PC program. If the set value exceeds the limit, the pump gives prompts.
- You can predefine the infusion modes and corresponding parameters in the feeding library. When the drug is selected, the pump automatically load the infusion mode and corresponding parameters.

CAUTION

- **The feeding library should be created by professionals. Checked that the drug and parameter settings are suitable for the care area before use.**
 - **The facility is responsible for performing initial checks to ensure that the proper feeding library is loaded.**
-

NOTE

- **The predefined parameters can be changed during a therapy. This does not affect the embedded library.**
-

This page intentionally left blank.

9 Networked Communication

The equipment can be connected to the BeneFusion nCS Infusion Supervision System and BeneVision Central Monitoring System (hereafter both referred to as “CMS”), and the eGateway.

9.1 Network Safety Information

CAUTION

- **Wireless network designing, deploying, debugging, and maintenance should be executed by the service personnel or authorized technicians.**
 - **Always set the wireless network according to local wireless regulations.**
 - **Data communication for all network functions must be performed within a closed network or within a virtually isolated network provided by a hospital. The hospital is responsible for ensuring the security of the virtually isolated network.**
 - **Keep network authentication information, for example password, safe, protecting the network from being accessed by unauthorized users.**
 - **Do not connect non-medical devices to the network.**
 - **If wireless network signal is poor, there may be a risk of CMS data loss.**
 - **RF interference may result in wireless network disconnection.**
 - **Disconnecting from the network may result in CMS data loss and function failure. Check the patient in case of network disconnection and solve the network problem as soon as possible.**
 - **Ensure that the IP address setting is correct. Changing the network settings may result in network disconnection. Contact your service personnel if you have any problems on setting the IP address.**
-

9.2 Connecting the Equipment to the CMS

The equipment can be connected to the CMS through the wireless network. When connected to the CMS, the system provides the following function:

- The equipment can transmit infusion information, alarm information, and equipment information (such as battery and network) to the CMS.
- Patient information can be synchronized between the equipment and the CMS.

- Patient can be admitted or discharged by the CMS, and patient information can be transmitted to this equipment.

9.3 Connecting the Equipment to the eGateway

You can connect the equipment to the eGateway to implement interaction between the equipment and the Hospital Information System (HIS) via the HL7 protocol. When connected to the eGateway, the system provides the following functions:

- The equipment can transmit infusion information and drug information to the eGateway.
- Patient information can be synchronized between the equipment and the eGateway.

10 Maintenance

Regular maintenance is essential to ensure that the equipment functions properly. This chapter contains information on periodic testing and maintenance.

10.1 Maintenance Safety Information

WARNING

- **To avoid electric shock, stop using the equipment if you find the housing of the equipment has signs of broken. Contact the service personnel for help in that case.**
 - **Failure on the part of the responsible individual hospital or institution using this equipment to implement a recommended maintenance schedule may cause undue equipment failure and possible health hazards.**
 - **No modification of this equipment is allowed.**
 - **This equipment contains no user serviceable parts.**
 - **The safety checks or maintenance involving any disassembly of the equipment should be performed by professional service personnel. Otherwise, undue equipment failure and possible health hazards could result.**
 - **The service personnel must be properly qualified and thoroughly familiar with the operation of the equipment.**
-
-

CAUTION

- **The equipment and accessories shall not be served or maintained while in use with a patient.**
 - **If you discover a problem with the equipment, such as the product label falls off or illegible, contact your service personnel.**
-

NOTE

- **If needed, contact the manufacture for circuit diagrams, component part lists, descriptions, calibration instructions, or other information concerning the repair of the equipment.**

- **The maintenance personnel should confirm the accuracy of the pump every three years. If the accuracy not meet the claimed accuracy, perform the accuracy calibration and verify whether the accuracy meets the claimed specification.**
- **After the maintenance personnel complete the pressure calibration, the pressure accuracy must be tested to confirm whether the calibration is valid.**

10.2 Maintenance and Testing Schedule

Follow the maintenance and testing schedule or local regulations to perform testing and maintenance. Make sure to clean and disinfect the equipment before taking any tests and maintenance. The following table lists the maintenance and testing schedule:

Test/Maintenance Item		Recommended Frequency
Performance Tests		
Tests required by IEC 60601-2-24:2012		<ul style="list-style-type: none"> • Once every three years. • When you suspect that the occlusion alarm is abnormal. • When you suspect that the rate is abnormal.
Safety Tests		
Electrical safety tests		<ul style="list-style-type: none"> • Once every three years, or if required. • When the power board is repaired or replaced. • When the main board is replaced. • When the equipment drops to the ground.
Other Tests		
Visual inspection		Every day, before first use.
Power-on test		Each time the equipment is powered on.
Battery check	Functionality test	<ul style="list-style-type: none"> • When the battery is first installed. • When the battery is replaced.
	Performance test	Every four months or if the battery runtime reduces significantly.
Pressure calibration, and accuracy calibration.		If the performance test fails. For more information, see the service manual.

Except the regular check and battery check, all other test and maintenance tasks should be performed by the qualified service personnel only. If your equipment needs a safety test and performance test, contact the service personnel.

10.3 Maintaining the Battery

To optimize performance, a fully or nearly fully discharged battery should be charged as soon as possible. The battery is recharged automatically when the equipment is connected to AC mains power.

The service life of a battery depends on how frequent it is used. When properly used, the lithium-ion battery has a service life of three years. If improperly used, its service life can be shorten. We recommend replacing the battery every three years.

The performance of the battery deteriorates over operating time or storage time. You should condition the battery every four months.

WARNING

- **Use only specified battery. Use of a different battery may present a risk of fire or explosion.**
- **The battery must only be installed and replaced by service personnel trained and authorized by Mindray Scientific.**
- **Do not crush, drop or puncture the battery. Mechanical abuse can lead to internal damage and internal short circuits. If a battery has been dropped or banged against a hard surface, whether damage is externally visible or not, remove the battery from use and dispose of it properly.**
- **If the battery shows signs of damage or signs of leakage, replace it immediately. Use caution in removing the battery. Avoid contacting the leakage.**
- **Extremely high ambient temperature may cause battery overheat protection, resulting in equipment shutdown.**
- **The lithium-ion battery has a service life. Replace your battery when it reaches the end of its service life. Failure to replace the battery may cause serious damage to your equipment from battery overheating.**
- **Do not open the battery, heat the battery above 60 °C, incinerate battery, or short battery terminals. They may ignite, explode, leak or heat up, causing personal injury.**

NOTE

- **Remove the battery if it will not be used for an extended period of time.**
- **The battery should be charged only in this equipment.**
- **Storing the battery at high temperature for an extended period of time will significantly shorten their life expectancy.**
- **Storing the battery in a cool place can slow the aging process. Ideally the battery should be stored at 15 °C.**

- **If the battery is not conditioned for a prolonged time, its charge indication may not be accurate and you may wrongly evaluate the remaining battery runtime.**
 - **Do not use the pump for infusion during battery conditioning.**
 - **Do not interrupt battery conditioning.**
-

10.4 Checking the History Record

The **History Record** menu shows the history of pump activities, including the infusions, alarms, calibrations, maintenance configurations, and other operations.

NOTE

- **A total loss of power has no impact on the history records stored.**
 - **Alarms are saved as events and will remain if the equipment is powered down. The time of equipment power down is also recorded as an event.**
 - **The pump stores up to 5000 events. When the capacity is reached, earlier events will be overwritten by later ones.**
-

10.5 Disposing of the Equipment

The service life of this equipment is ten years. Dispose of the equipment when its service life is reached. Follow local regulations regarding the disposal of such product.

WARNING

- **For disposal of parts, batteries, packaging materials, and accessories, where not otherwise specified, follow local regulations regarding disposal of hospital waste.**
-
-

11 Care and Cleaning

In this chapter we only describe cleaning and disinfection of the pump, pole clamp and DC adapter. For the cleaning and disinfection of other reusable accessories, refer to their instructions for use.

11.1 Care and Cleaning Safety Information

WARNING

- **Use only the approved cleaners, disinfectants and methods listed in this chapter to clean and disinfect your equipment and accessories. Warranty does not cover damage caused by unapproved substances or methods.**
 - **Do not mix disinfecting solutions, as hazardous gases may result.**
 - **We make no claims regarding the efficacy of the listed chemicals or methods as a means for controlling infection. For the method to control infection, consult your hospital's infection control officer or epidemiologist.**
 - **The responsible hospital or institution shall carry out all cleaning and disinfection procedures specified in this chapter.**
-

CAUTION

- **Turn off the equipment and remove the power cord from the equipment before cleaning and disinfecting.**
 - **Never immerse any part of the equipment or accessories in liquids or allow liquid to enter the interior of the equipment or accessories.**
 - **Any contact of cleaners or disinfectants with connectors or metal parts may cause corrosion.**
 - **Do not pour or spray any liquid directly on the equipment or accessories or permit fluid to seep into connections or openings.**
 - **If you spill liquid on the equipment or accessories, disconnect the power supply, dry the equipment, and contact your service personnel.**
 - **Never use abrasive materials (such as steel wool or silver polish), or erosive cleaners (such as acetone or acetone-based cleaners).**
 - **Dilute and use the cleaners or disinfectants according to the manufacturer's instructions.**
-

- **Check the equipment after cleaning and disinfecting. If there is any sign of damage, remove it from use.**

11.2 Cleaning the Equipment

Clean the equipment on a regular basis. Before cleaning, consult your hospital's regulations.

To clean the equipment, follow this procedure:

1. Dampen a soft lint-free cloth with water or ethanol (70%).
2. Wring excess liquid from the cloth.
3. Wipe the display screen of the equipment.
4. Wipe the external surface of the equipment with the damp cloth, avoiding the connectors and metal parts.
5. Dry the surface with a clean cloth. Allow the equipment air dry in a ventilated and cool place.

11.3 Disinfecting the Equipment

Disinfect the equipment as required in your hospital's servicing schedule. Cleaning the equipment before disinfecting is recommended. Always dilute and use disinfectants according to the manufacturer's instructions. The following table lists approved disinfectants:

Product Name	Product Type	Manufacturer
Alpet® D2 Surface Sanitizing Wipes	Wipes	BEST SANITIZERS INC™.
CIDEX® OPA	Liquid	Gilag GmbH International Advanced Sterilization products
Clorox Dispatch® Hospital Cleaner Disinfectant Towels with Bleach	Wipes	Clorox professional products company
Clorox Healthcare® Bleach Germicidal Wipes	Wipes	Clorox professional products company
Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant Wipes	Wipes	Clorox professional products company
Diversey Oxivir® TB Wipes	Wipes	Diversey Inc
Metrex CaviCide1™	Liquid, spray	METERX® RESEARCH

Product Name	Product Type	Manufacturer
Metrex CaviWipes™	Wipes	METERX® RESEARCH
PDI Sani-Cloth® AF3 Germicidal Disposable Wipe	Wipes	PDI Inc.
PDI Sani-Cloth® Bleach Germicidal Disposable Wipe	Wipes	PDI Inc.
PDI Sani-Cloth® HB Germicidal Disposable Wipe	Wipes	PDI Inc.
PDI Sani-Cloth® Plus Germicidal Disposable Cloth	Wipes	PDI Inc.
PDI Super Sani-Cloth® Germicidal Disposable Wipe	Wipes	PDI Inc.
VIRAGUARD® Hospital Surface Disinfectant Towelette	Wipes	VERIDIEN corporation
Virex® II 256 (1:256)	Liquid	Diversey Inc
Virex® TB	Liquid, spray	Diversey Inc
JIAN ZHI SU Disinfectant Tablets	Tablet	Beijing ChangJiangMai Medical Science Technology Co. Ltd
JIAN ZHI SU Surface Disinfectant Spray	Liquid, spray	Beijing ChangJiangMai Medical Science Technology Co. Ltd
JIAN ZHI SU Disinfectant, Double-chain Quaternary Ammonium	Liquid	Beijing ChangJiangMai Medical Science Technology Co. Ltd
DIAN'ERKANG Surface Wipes	Wipes	Shanghai Likang Disinfectant Hi-Tech Co., Ltd
DIAN'ERKANG Surface Disinfectant	Liquid	Shanghai Likang Disinfectant Hi-Tech Co., Ltd
DIAN'ERKANG Disinfectant Spray	Liquid, spray	Shanghai Likang Disinfectant Hi-Tech Co., Ltd
Clinell® Universal Wipes	Wipes	GAMA Healthcare Ltd
Clinell® Sporicidal Wipes	Wipes	GAMA Healthcare Ltd

Product Name	Product Type	Manufacturer
Tristel Duo™	Liquid, foam	Tristel solutions Limited
Tristel Jet	Liquid, spray	Tristel solutions Limited
Tristel Fuse For Surfaces, 196ppm	Liquid	Tristel solutions Limited
Surfanios Premium, 0.25%	Liquid	ANIOS LABORATORIES
Surfa 'safe	Liquid, spray	ANIOS LABORATORIES
Wip' Anios premium	Wipes	ANIOS LABORATORIES
Aniosurf ND premium, 0.25%	Liquid	ANIOS LABORATORIES
Mikrobac® Tissues	Wipes	BODE Chemie GmbH
Cleanisept® Wipes	Wipes	Dr. Schumacher GmbH
mikrozid® PAA Wipes	Wipes	Schülke & Mayr GmbH
mikrozid® Sensitive Wipes	Wipes	Schülke & Mayr GmbH
Ecolab Incidin® OxyWipes	Wipes	Ecolab Deutschland GmbH
Glutaraldehyde, 2%	Liquid	/
Isopropanol, 70%	Liquid	/
Sodium hypochlorite bleach, 0.5%	Liquid	/
Hydrogen peroxide, 3%	Liquid	/
Rely+On™ Virkon® High Level surface Disinfectant, 1%	Powder	Antec International Ltd
1-Propanol, 50%	Liquid	/
Descosept® forte	Liquid	Dr. Schumacher GmbH
Descosept® AF	Liquid	Dr. Schumacher GmbH
Dismozon® plus, 0.4%	Powder	BODE Chemie GmbH

Product Name	Product Type	Manufacturer
mikrozid® AF Wipes	Wipes	Schülke & Mayr GmbH
Terralin® Liquid	Liquid	Schülke & Mayr GmbH
Perform® Classic Concentrate OXY, 0.5%	Powder	Schülke & Mayr GmbH

11.4 Cleaning the Pole Clamp and DC Adapter

Clean the pole clamp and DC adapter on a regular basis. To clean the pole clamp and DC adapter, follow this procedure:

1. Clean the pole clamp and DC adapter with a soft cloth moistened with water or ethanol (70%).
2. Wipe off the cleaner residue with a dry cloth.
3. Allow the pole clamp and DC adapter to air dry.

11.5 Disinfecting the Pole Clamp and DC Adapter

We recommend that the pole clamp and DC adapter should be disinfected only when necessary as determined by your hospital's policy.

Cleaning the accessories before disinfecting is recommended.

Product Name	Product Type	Manufacturer
Isopropanol, 70%	Liquid	/
Hydrogen peroxide, 3%	Liquid	/
Perform® Classic Concentrate OXY, 0.5%	Powder	Schülke & Mayr GmbH
Dismozon® plus, 0.4%	Powder	BODE Chemie GmbH
Descosept® AF	Liquid	Dr. Schumacher GmbH
Descosept® forte	Liquid	Dr. Schumacher GmbH
mikrozid® AF Wipes	Wipes	Schülke & Mayr GmbH
Rely+On™ Virkon® High Level surface Disinfectant, 1%	Powder	Antec International Ltd

Product Name	Product Type	Manufacturer
Terralin® Liquid	Liquid	Schülke & Mayr GmbH

CAUTION

- **To avoid long term damage, the accessories should be disinfected only when necessary as determined by your hospital's policy.**
-

11.6 Sterilization

Sterilization is not recommended for this equipment, related products, accessories, or supplies unless otherwise indicated in the Instructions for Use that accompany the products, accessories or supplies.

12 Accessories

The accessories listed in this chapter comply with the requirements of IEC 60601-1-2 when in use with the equipment. For details about the accessories, refer to the instructions for use provided with the accessory.

WARNING

- **Use accessories specified in this chapter. Using other accessories may cause damage to the equipment or not meet the claimed specifications.**
-

CAUTION

- **The accessories may not meet the performance specifications if stored or used outside the specified temperature and humidity ranges. If accessory performance is degraded due to aging or environmental conditions, contact your service personnel.**
 - **Check the accessories and their packages for any sign of damage. Do not use them if any damage is detected.**
 - **Use the accessories before the expiry date if their expiry date is indicated.**
-

PN	Description
0020-20-12522	Power cord, 10A, 250V, 2.5m, International
009-001791-00	Power cord, 250V, 16A, 3m, South Africa
009-002636-00	Power cord, 10A, 1.5m, Australia standard
009-007190-00	Power cord, 3m, India
DA8K-10-14452	Power cord, USA
DA8K-10-14453	Power cord, UK
DA8K-10-14454	Power cord, Europe
009-009838-00	Nurse call cable
115-098866-00	DC adapter
115-089928-00	Standard pole clamp
041-062198-00	Rotatable pole clamp

PN	Description
034-000998-00	MIP-300 IV Pole (Basic)
034-000999-00	MIP-600 IV Pole (Luxury)
034-001011-00	MIP-500 IV Pole (Syringe Pump Luxury)

A Product Specifications

A.1 Specifications

Classifications	<p>Connect to AC power source, type of protection against electrical shock: CLASS I EQUIPMENT, equipment energized from an internal electrical power source</p> <p>Connect to DC power source, type of protection against electrical shock: CLASS II EQUIPMENT, equipment energized from an internal electrical power source</p> <p>Degree of protection against electrical shock: Defibrillation-proof type CF applied part (direct cardiac application)</p> <p>Mode of operation: Continuous</p> <p>Degree of protection against harmful ingress of water: IP44</p> <p>Degree of mobility: Portable</p>
Operating conditions	<p>Temperature: 5°C to 40°C</p> <p>Relative humidity (noncondensing): 15% to 95%</p> <p>Barometric: 57.0 kPa to 107.4 kPa</p>
Storage conditions	<p>Temperature: -20°C to 60°C</p> <p>Relative humidity (noncondensing): 10% to 95%</p> <p>Barometric: 16.0 kPa to 107.4 kPa</p> <p>Corrosive-free and ventilated</p>
Power Supply	<p>AC Power Supply: 100 VAC to 240 VAC, 50/60 Hz, 0.30A to 0.13A</p> <p>DC Power Supply: 10 VDC to 16 VDC, 2.0A to 1.3A</p>
Battery run time	<p>At least 5.5 hours for normal battery and at least 11 hours for smart battery (operating at a rate of 5ml/h or 25ml/h, under standard operating conditions*)</p> <p>At least 1 hour for normal battery and at least 2.5 hours for smart battery (operating at a rate of 2000ml/h, under standard operating conditions*)</p> <p>*Operating with a fully charged new battery at 20°C ± 2°C, default screen brightness and volume, Wi-Fi disabled, without accessories.</p>
Battery charge time	<p>≤ 5 hours for normal battery and ≤ 6 hours for smart battery (the pump is off, and charged by the AC power supply).</p>
Shutdown delay	<p>Operating at a rate of 25ml/h, at least 30 minutes after first low battery alarm</p>
Main unit weight	<p>≤ 1.5 kg (with normal battery, without pole clamp)</p> <p>Pole clamp ≤ 0.15 kg</p>

Main unit (W × D × H)	100mm x 95mm x 200mm (without pole clamp, the error is ±3mm)
Display	4.3 inches Color TFT LCD, resolution ≥ 272x480 pixels
Speaker	Gives alarm tones (sound pressure 50 to 75 dB). Supports multi-level tone modulation. Alarm tones comply with IEC 60601-1-8.
USB connector	1, USB 2.0 protocol. Fixed time synchronization pulse specified by the USB protocol.
Multifunctional connector	1, Uart protocol

NOTE

- **The equipment is not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.**
- **When stored under temperature conditions beyond the defined operating conditions, the equipment needs to be placed under room temperature at least one hour before use.**
- **The equipment may not meet the performance specifications if stored or used outside the specified temperature and humidity ranges. If the performance of the equipment is degraded due to aging or environmental conditions, contact your service personnel.**

A.2 Wireless Network

Standards	IEEE 802.11a/b/g/n
Modulation mode	BPSK, QPSK, 16-QAM, 64-QAM
Operating frequency	2412MHz to 2472MHz 5180MHz to 5320MHz 5500MHz to 5700MHz 5745MHz to 5825MHz
Data rate	IEEE 802.11a: 6 to 54 Mbps IEEE 802.11b: 1 to 11 Mbps IEEE 802.11g: 6 to 54 Mbps IEEE 802.11n: MCS0 to MCS7
Transfer power	< 20 dBm (CE requirement: detection mode – RMS) < 30 dBm (FCC requirement: detection mode – PEAK)

Operating mode	Transmitting data through the wireless access point (AP)
Data security	Standard: OPEN and WPA/WPA2-PSK Encryption: TKIP and AES
System capacity	Number of the pumps supported by a single AP: ≤ 16
Data transmission delay between the pump and the CMS	Total data transmission delay time between the pump and the CMS is $\leq 8s$
Interruption number and time between the pump and the CMS	Total interruption duration $\leq 0.01*$ total communication time (Test within 24 hours, with 16 pumps, in which three pumps are roaming for 30 times)
Delay time of network disconnection alarm	$\leq 14 s$

A.3 Infusion Specifications

Accuracy	Infusion accuracy: $\leq \pm 5\%$ (use SHINVA ANDE Disposable Enteral Feeding Set EIS-22-A) Bolus accuracy: $\leq \pm 5\%$ or $\pm 0.02ml$, whichever is greater Note: Test in accordance with IEC60601-2-24:2012
Set range of the infusion rate/ purge rate/bolus rate	0.1ml/h to 2000ml/h Minimum resolution: 0.01ml/h (0.1 to 99.99ml/h) 0.1ml/h (100.0 to 999.9ml/h) 1ml/h (1000 to 2000ml/h) Range of the infusion rate for nourishing feeding: 0.1ml/h to 100ml/h
Occlusion pressure	75mmHg to 825mmHg Resolution: 1mmHg The maximum occlusion pressure is 990mmHg.
Occlusion pressure tolerance	$\leq \pm 20\%$ or $\leq \pm 125mmHg$, whichever is greater
Maximum volume (under single fault conditions)	$\leq 0.5ml$
KTO rate	0.1 to 50ml/h Minimum resolution: 0.01ml/h
Time set range	00:00:01 to 99:59:59 (h:min:sec)

VTBI set range	0.1 to 9999.99 ml Minimum resolution: 0.01ml Nourishing feeding: 0.1 to 1000ml
Conc. set range	0.001 to 9999.99
Conc. unit set range	ng/ml, ug/ml, mg/ml, g/ml, mU/ml, U/ml, kU/ml, EU/ml, mmol/ml, mol/ml, mcal/ml, cal/ml, kcal/ml, mEq/ml
Patient management	Discharging/admitting a patient, Editing/exporting/importing patient information
Prescription	Accepting and performing the prescription

WARNING

- **The infusion accuracy and pressure detection is affected by viscosity of liquids and disposables used (for example diameter, material, elasticity and needle).**
-

NOTE

- **The infusion accuracy tests and occlusion pressure tests are performed in accordance with IEC60601-2-24:2012 (test temperature: 20°C ± 2°C).**
-

A.4 Recommended Infusion Sets

Product Name	Type	Manufacturer
Disposable Enteral Feeding Set	Nutrition	SHINVA ANDE HEALTHCARE APPARATUS CO., LTD

NOTE

- **The pump will not affect the quality of disposables from other suppliers. Changes in quality may affect the technical data of the pump. Mindray Scientific is not responsible for such changes.**
-

A.5 Occlusion Alarm Delay and Bolus Volume

Rate (ml/h)	Occlusion alarm delay time (hh: mm: ss)	
	High occlusion alarm pressure level	Low occlusion alarm pressure level
1	< 01:50:00	< 00:04:00
25	< 00:03:00	< 00:00:30

Rate	Bolus volume after occlusion (ml)	
	High occlusion alarm pressure level	Low occlusion alarm pressure level
25ml/h	< 0.18	< 0.18

Test conditions:



- Infusion set brand: SHINVA ANDE EIS-22-A
- Anti-bolus: On
- Test temperature: 20°C ±2°C
- Infusion line length: 1 meter


WARNING

- Occlusion alarm pressure, alarm delays and bolus volume may vary depending on test conditions, temperature and tube length.
-


A.6 Infusion Accuracy Graphs



A.6.1 Infusion Accuracy at 1 ml/h

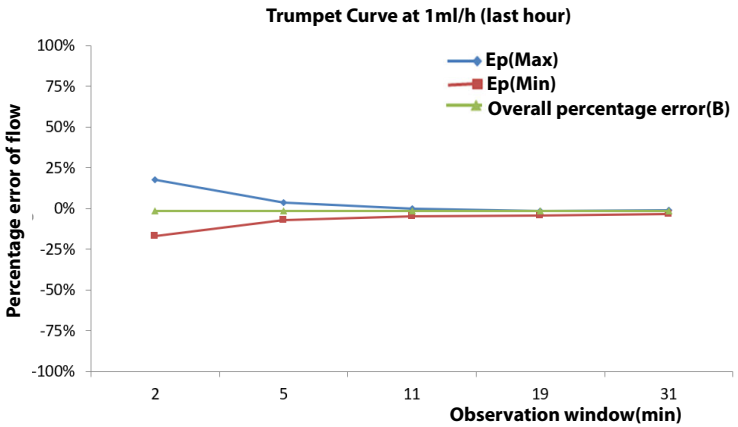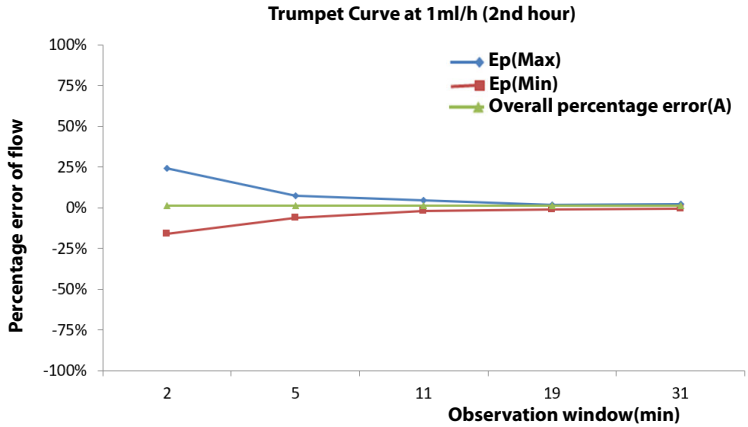

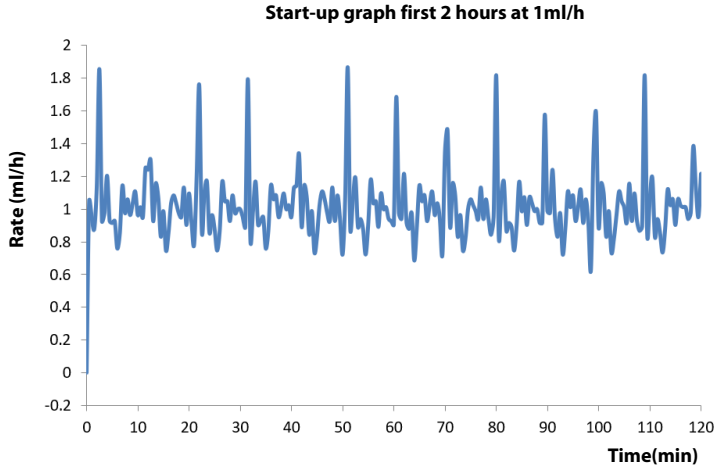

A.6.2 Infusion Accuracy at 25ml/h

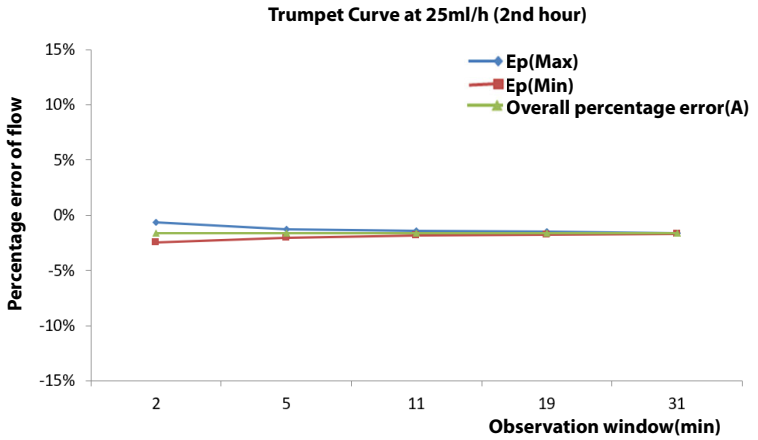

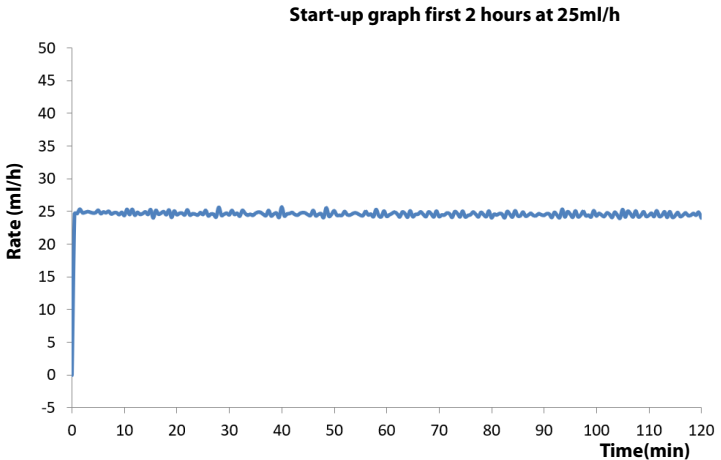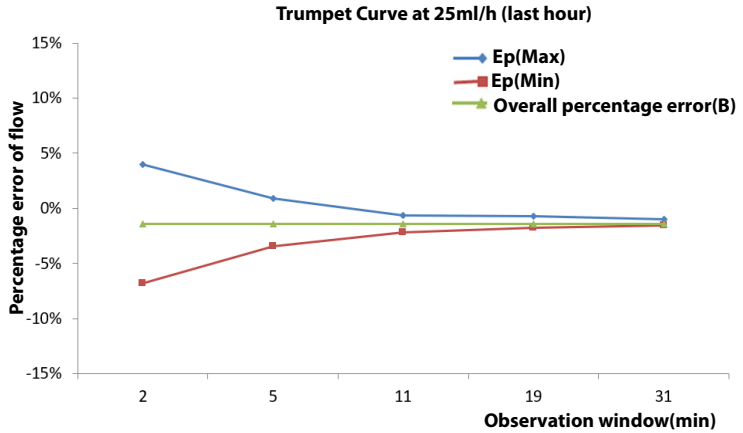

Test conditions:

- Infusion set brand: SHINVA ANDE EIS-22-A
- Test interval: $\Delta t = 0.5$ minute

WARNING

- Infusion accuracy may be influenced by the pump's environment (such as pressure, temperature, humidity, and any infusion consumables used).
-
-

A.7 Operating Environment

Operating system	FreeRTOS
Classification	OS Core
Version Information	9.0.0

This page intentionally left blank.

B EMC and Radio Regulatory Compliance

B.1 EMC

The system complies with the EMC standard IEC60601-1-2:2020.

WARNING

- **The use of unapproved accessories may diminish system performance.**
 - **Use of components, accessories, probes, and cables other than those specified may result in increased emission or decreased immunity of system.**
 - **The system needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided below.**
 - **Use of the system adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, the system and the other equipment should be observed to verify that they are operating normally.**
 - **Use of accessories, transducers and cables other than those specified or provided by the manufacturer of the system could result in increased electromagnetic emissions or decreased electromagnetic immunity of the system and result in improper operation.**
 - **Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the system, including cables specified by the manufacturer. Otherwise, degradation of the performance of the system could result.**
 - **Other devices may interfere with the system even though they meet the requirements of CISPR.**
 - **Use of portable or mobile communications devices can degrade the performance of the equipment.**
 - **The system is not intended for use in residential environments and can possibly not provide adequate protection to radio reception in such environments.**
-

If the system is operated within the electromagnetic environment listed in TABLE EMC-2, EMC-3, TABLE EMC-4 and TABLE EMC-5, the system will remain safe and will provide the following basic performances:

- Operating mode
- Accuracy
- Function
- Protection against UNINTENDED BOLUS volumes
- Occlusion
- ALARM CONDITIONS regarded
- Data stored


Table EMC-1

Guidance and Mindray Declaration - Electromagnetic Emissions		
The system is intended for use in the electromagnetic environment specified below. The customer or the user of system should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment - guidance
RF Emissions CISPR 11	Group 1	The system uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class B	The system is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/ Flicker Emissions IEC 61000-3-3	Complies	

Table EMC-2

Guidance and Mindray Declaration - Electromagnetic Immunity			
The system is intended for use in the electromagnetic environment specified below. The customer or the user of system should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact; ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact; ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast Transient / burst IEC 61000-4-4	±2 kV for power supply lines; ±1 kV for input/output lines	±2 kV for power supply lines;	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±0,5 kV, ±1 kV line(s) to line(s); ±0,5 kV, ±1 kV, ±2 kV line(s) to earth	±0,5 kV, ±1 kV line(s) to line(s); ±0,5 kV, ±1 kV, ±2 kV line(s) to earth	
Voltage dips, Short interruptions and voltage variation on power supply input voltage IEC 61000-4-11	0% U_T ; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U_T ; 1 cycle 70% U_T for 25/30 cycle at 0° 0% U_T ; 250/300 cycle	0% U_T ; 0,5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0% U_T ; 1 cycle 70% U_T for 25/30 cycle at 0° 0% U_T ; 250/300 cycle	Mains power quality should be that of a typical commercial or hospital environment. If you require continued operation during power mains interruptions, it is recommended that our product be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 HZ) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Note: U_T is the A.C. mains voltage prior to application of the test level.			

Table EMC-3

Guidance and Mindray Declaration - Electromagnetic Immunity			
The system is intended for use in the electromagnetic environment specified below. The customer or the user of system should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 0,15 MHz to 80 MHz	3 Vrms 0,15 MHz to 80 MHz	Portable and mobile RF communications equipment should be used no closer to any part of system, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1.2\sqrt{P}$ $d = 2\sqrt{P}$
	6 Vrms in ISM bands ^a between 0,15 MHz and 80 MHz	6 Vrms in ISM bands ^a between 0,15 MHz and 80 MHz	
Radiated RF IEC 61000-4-3	3V/m 80 MHz to 2.7 GHz	3V/m 80 MHz to 2.7 GHz	$d = 1.2\sqrt{P}$ 80 MHz to 800 MHz $d = 2.3\sqrt{P}$ 800 MHz to 2.7 GHz Where, P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^b , should be less than the compliance level in each frequency range ^c . Interference may occur in the vicinity of equipment marked with the following symbol: 
Note 1: At 80 MHz and 800 MHz, the higher frequency range applies. Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

^a The ISM (industrial, scientific, and medical) bands between 150 kHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz. The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz.

^bField strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the device.

^c Over the frequency ranges 150 kHz to 80 MHz, field strengths should be less than 3V/m.

Table EMC-4

Guidance and Mindray Declaration - Electromagnetic Immunity			
The system is intended for use in the electromagnetic environment specified below. The customer or the user of system should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Proximity magnetic fields IEC 61000-4-39	65 A/m 134,4 kHz Pulse modulation 2,1 kHz	65 A/m 134,4 kHz Pulse modulation 2,1 kHz	/
	7,5 A/m 13,56 MHz Pulse modulation 50 kHz	7,5 A/m 13,56 MHz Pulse modulation 50 kHz	

Table EMC-5 Test specifications and minimum distances

Recommended Separation Distances between Portable and Mobile RF Equipment and the System						
<p>The system is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the system can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the system as recommended below, according to the maximum output power of the communications equipment. Portable and mobile radio communications equipment (e.g. two-way radio, cellular/ cordless telephones and similar equipment) should be used no closer to any part of this system, including cables, than determined according to the following method:</p>						
Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	Immunity test level (V/m)
385	380 - 390	TETRA 400	Pulse modulation 18Hz	1.8	0.3	27
450	430 - 470	GMRS 460 FRS 460	FM \pm 5 kHz deviation 1 kHz sine	2	0.3	28
710	704 - 787	LTE Band 13,17	Pulse modulation 217 Hz	0.2	0.3	9
745						
780						
810	800 - 960	GSM 800/900, tetra 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	2	0.3	28
870						
930						

1720	1700 - 1990	GSM 1800, CDMA 1900, GSM 1900, DECT, LTE Band 1, 3, 4, 25, UMTS	Pulse modulati on 217 Hz	2	0.3	28
1845						
1970						
2450	2400 - 2570	Bluetooth , WLAN, 802.11 b/ g/n, RFID 2450, LTE Band 7	Pulse modulati on 217 Hz	2	0.3	28
5240	5100 - 5800	WLAN, 802.11 a/ n	Pulse modulati on 217 Hz	0.2	0.3	9
5500						
5785						

Table EMC-6

Recommended Separation Distances between Portable and Mobile RF Communications Device and the System				
The system is intended for use in an electromagnetic environment in which radiated RF disturbance are controlled. The customer or the user of system can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communication equipment (transmitters) and system as recommended below, according to the maximum output power of the communication equipment.				
Rated Maximum Output Power of Transmitter Watts (W)	Separation Distance According to Frequency of Transmitter			
	150 kHz to 80 MHz Out ISM bands $d = 1.2\sqrt{P}$	150 kHz to 80 MHz in ISM bands $d = 2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.7 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.2	0.12	0.23
0.1	0.38	0.64	0.38	0.73
1	1.2	2	1.2	2.3
10	3.8	6.4	3.8	7.3
100	12	20	12	23
For transmitters at a maximum output power not listed above, the recommended separation distanced in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. Note 1: At 80 MHz and 800 MHz, the higher frequency range applies. Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.				

Table EMC-7

No.	Name	Cable Length (m)	Shield or not	Remark
1	Power cord	2.5	Not shield	/
2	Nurse call cable	2.8	Not shield	with ferrite core
3	DC power cord	2.8	Not shield	/

B.2 Radio Regulatory Compliance

Refer to **A.2 Wireless Network** for the details of RF parameters.

WARNING

- **Keep a distance of at least 20cm away from the equipment when Wi-Fi function is in use.**
-
-

This page intentionally left blank.

C Abbreviations

Abbreviation	In Full
AC	Alternating Current
Anti-Bolus	Anti-Bolus
BOLUS	Bolus
CCU(CICU)	Cardiac Intensive Care Unit
CE	Conformité Européenne
CISPR	International Special Committee on Radio Interference
CPU	Central Processing Unit
DC	Direct Current
DEFS	Dose Error Reduction Systems
DPS	Dynamic Pressure System
EEC	European Economic Community
EMC	Electromagnetic Compatibility
EMI	Electromagnetic Interference
EtO	Ethylene oxide
ICU	Intensive Care Unit
ID	Identification
IEC	International Electrotechnical Commission

Abbreviation	In Full
IEEE	Institute of Electrical and Electronic Engineers
ISO	International Organization for Standardization
IV	Intravenous
KTO	Keep Tube Open
LED	Light Emitting Diode
Max	Maximum
MDD	Medical Device Directive
Min	Minimum
MRI	Magnetic Resonance Imaging
N/A	Not Applied
OR	Operating Room
SN	Series Number
USB	Universal Serial Bus
VTBI	Volume To Be Infused

