


Physical View

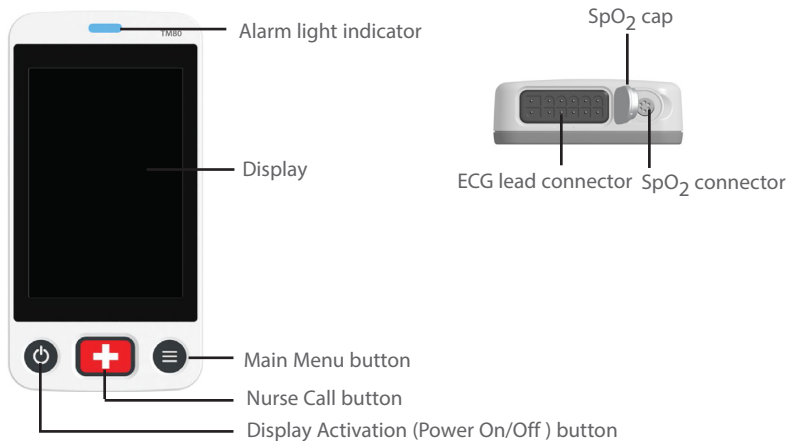

General Navigation


Switching Orientation

1. Swipe your finger down from the top of the main screen to display the orientation switch buttons.
2. Select the desired button.

Swiping for More Data



Quick Key Deployment

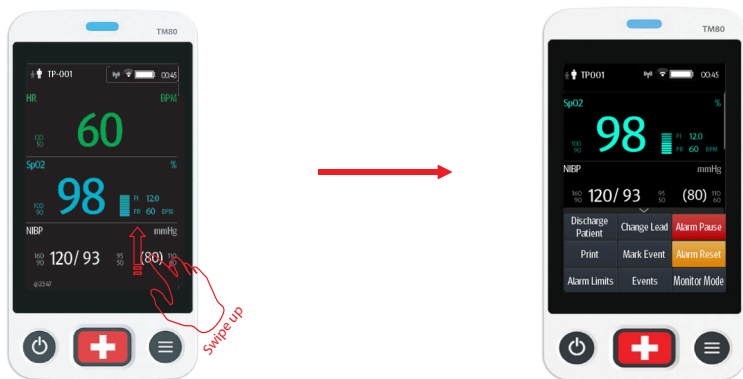

Note: The quick keys are configurable. These images are just representative.

ECG

Checking Paced Status


1. Tap the HR digital area or ECG waveform area to enter the ECG menu.
2. Set **Paced** to **Yes** or **No**.

Changing Lead

If **Change Lead** is configured in the quick keys area, tapping **Change Lead** changes the current first ECG lead waveform to the next ECG lead waveform that is available in sequential order.

RESP (Optional)

Enabling/Disabling the Resp Functionality


1. Press  to enter the main menu.
2. Tap **Parameter Setup**.
3. Enable or disable **Resp**.

Setting NCM Alarm Delay


1. Tap the **Resp** digital area or **Resp** waveform area to enter the **Resp** menu.
2. Set **NCM Alarm Delay**.

Monitoring with BP10

Pairing a TM80 with a BP10

1. Press the MPAN key on the right panel of the BP10.
2. Press  to enter the main menu of the TM80.
3. Tap **Wireless Modules** → tap **MPAN** →select **Connect to BP10** →select your desired device from the list of devices →tap **Connect**.

Unpairing via the TM80

1. Press  to enter the main menu of the TM80.
2. Tap **Wireless Modules** → from the devices list select the device you wish to disconnect → tap **Disconnect**, or
2. Tap **Wireless Modules** → tap **MPAN** → select **Off**.

Unpairing via the BP10

1. Press the MPAN key on the right panel of the BP10.
2. Select **Yes** when the prompt message “Are you sure you want to close MPAN and unpair BP10?” appears.

CAUTION:

Before pairing a BP10 with a new TM80, unpair it with the current TM80 first.

Before moving a TM80 or BP10 to another area, unpair them first.

Before admitting a new patient at a BP10, you need to unpair it with the TM80 first.



Discharging a Patient

Tap the **Discharge Patient** quick key. Or,

Press  to enter the main menu → tap **Discharge Patient**.


Alarms

Alarm Status Symbols


Symbol	Description
	All the alarms are paused.
	Audible alarm tones are turned off.

Symbol	Description
	Alarms are acknowledged and the alarm system is reset.

Changing Parameter Alarm Properties

1. Enter the **Alarm Limits** menu in either of the following ways:
 - ◆ Select the **Alarm Limits** quick key.
 - ◆ Press  to enter the main menu → tap **Alarms** → tap **Alarm Limits**.
2. Tap a parameter and set its alarm properties.

Changing Arrhythmia Alarm Settings

1. Enter the **Arrhythmia Alarm Setup** menu in either of the following ways:
 - ◆ Tap the **Arrhythmia Alarm Setup** quick key, if it is configured in the quick key area.
 - ◆ Press  to enter the main menu → tap **Alarms** → tap **Arrhythmia Alarm Setup**.
2. Tap a parameter and set its alarm properties.

Pausing Alarms

Tap the **Alarm Pause** quick key. Or,



Press  to enter the main menu → from the **Commands** section tap **Alarm Pause**.

Resetting Alarms


Tap the **Alarm Reset** quick key. Or,

Press  to enter the main menu → from the **Commands** section tap **Alarm Reset**.

Unlocking the Screen

1. If the display is off, press  or  to activate the display and access the **Screen Locked** menu.
2. Enter the passcode to unlock the screen.

Adjusting Volumes

1. Press  to enter the main menu.
2. Tap **Audio Volume**.
3. Select the category to adjust and drag the slider to the appropriate volume.

Review

Reviewing Events

Tap the **Events** quick key. Or,

Press  to enter the main menu → tap **Review** → tap **Events**.


Reviewing Tabular Trends

On the event review page tap **Tabular Trends**. Or,

Press  to enter the main menu → tap **Review** → tap **Tabular Trends**.

Entering/Exiting Standby Mode

Entering Standby Mode

1. Enter the standby dialog in either of the following ways:
 - ◆ Tap the **Standby** quick key, if it is configured in the quick keys area.
 - ◆ Press  to enter the main menu → tap **Standby**.
2. Tap **Yes**.

Exiting Standby Mode

Press  to exit standby mode.

Entering/Exiting Monitor Mode

Entering Monitor Mode

Tap the **Monitor Mode** quick key. Then, the TM80's display is always on and alarms are issued locally.

Exiting Monitor Mode

Tap the **Telemetry Mode** quick key.



046-007481-00(5.0)

mindray