



## SAFETY

### Safety checks


- General overview of system integrity, system and accessories are visually clean


### Cleaning

- Adhere to the **Operators Manual** for advice on cleaning agents
- Follow hospital policy, but with an awareness that agents utilised by the hospital may cause damage to the equipment over time

## PATIENT ADMISSION & DISCHARGE

### Admitting a patient

- Ensure that the bedside monitor is switched on, and not in **standby mode**
- Check that the previous patient has been **discharged** from the **Central Monitoring System**
- Within the **Multibed View**, click the mouse in the monitored patient bed number **BED 1** to open the drop down menu
- Select the patient symbol  in the drop down menu to open the **Patient Management** menu screen
- Enter appropriate patient data
- Select **Save** to confirm and save

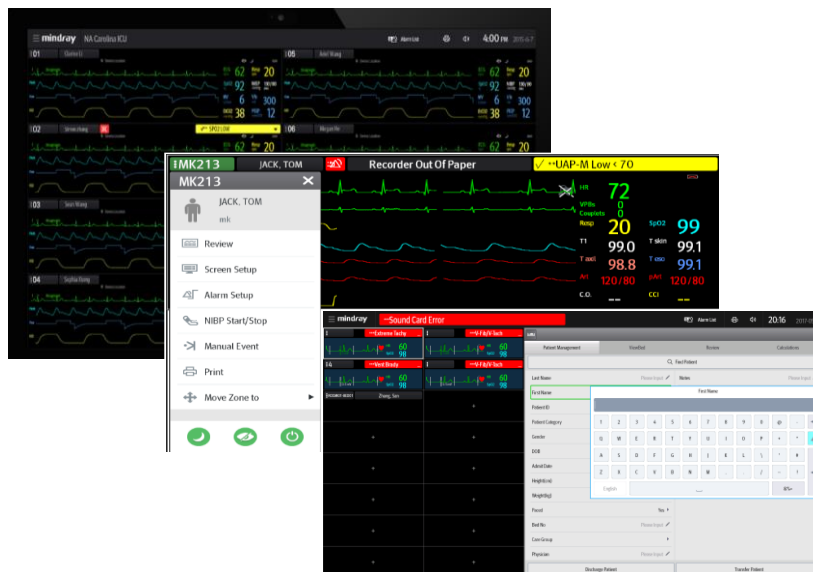

### Discharging a patient

- Within the **Multibed View**, click the mouse in the monitored patient tile to open the **View Bed** menu
- Select the **Patient Management** tab located in the upper left hand side of the **View Bed** menu screen
- Select **the Discharge Patient** tab located at the bottom of the **Patient Management** menu
- The **Discharge Patient** menu appears - select the appropriate option, followed by **OK**
- The previous patient has now been discharged
- Select **X** to close the menu

## PATIENT MANAGEMENT

### Assigning a patient to a care group

- Select **Patient management**
- Click on **care group**
- Choose a colour form the drop down menu
- Press **save**

### Transferring a patient to another bed space

- Ensure the previous patient in the destination monitor has been discharged
- Open the **Patient Management** menu screen
- Select the **Transfer Patient** tab located at the bottom of the **Patient Management** menu
- Within the **Transfer Patient** menu, select the destination bed, followed by **Ok**
- **Are you sure you want to transfer this bed to... → OK**
- **Transfer Patient Succeeded → OK**

**Please Note:** Where possible, use the T1 to transfer the patient to the next location

### Locating and re-admitting a patient

- **Patient Management → Find** → enter search criteria e.g. name or hospital number → Choose the patient to be readmitted and Select the  button

## ALARMS

### Setting alarm limits

- Within the **Multibed View**, click the mouse in the monitored patient bed number **BED 1** to open the Drop-down menu
- **Alarm Setup** → adjust **High** or **Low** limit as required → **X** to exit the **Alarm** menu



### Central Monitoring System Silence (depending on configuration)

- Select the  symbol at the top of the screen

**Please Note:** Silencing alarms at the CMS, only silences CMS sounds, not alarm sounds at bedside monitors


# REVIEWING PATIENT DATA

## Accessing an expanded view of monitored patient data

- Within the **Multibed View**, click the mouse in the monitored patient tile to open the **Expanded View** screen

**Please Note:** All patient parameters and trends can be viewed within this screen


- Click on the monitored patient **Review** tab, at the top of the expanded view window → click on **Tabular Trends** tab → select desired trend data display format

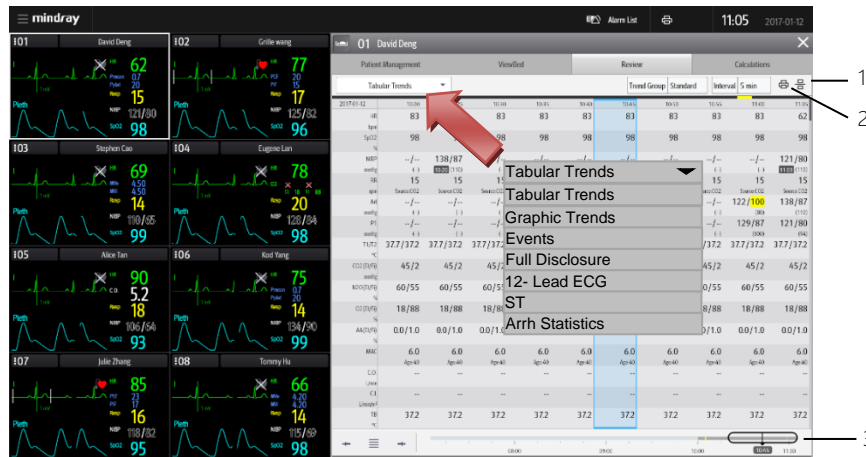

1. Allows viewing of data in two formats
2. Print icon
3. Viewable data slide bar

## Viewing compressed & full-sized ECG waveform data

- Within the review menu select **Full Disclosure** in the drop-down list to display compressed ECG waveform view → ECG portion to view → **Detail** located at the top of the menu view

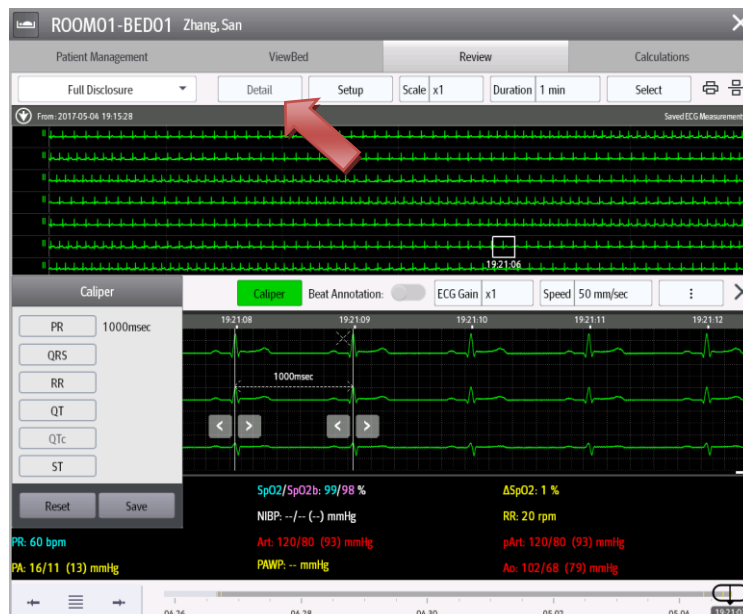

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions. The Illustrations may not necessarily reflect the setup or data displayed on your device.