

Defibrillator

Cleaning and Disinfection Protocol



Cleaning Protocol

Recommended Cleansing Agent

Water

Sodium hypochlorite bleach (10%, Sodium hypochlorite)

Hydrogen peroxide (3%)

Ethanol (75%)

Isopropyl alcohol (70%)

Perform® classic concentrate OXY (KHSO4 solution)

Main Unit

1. Shut down the equipment, disconnect the power cord and other cables and remove the batteries
2. Clean the display screen using a soft, clean cloth dampened with cleansing agent
3. Clean the exterior surface of the equipment using a soft, clean cloth dampened with cleansing agent
4. Clean the paddle tray using a soft, clean cloth dampened with cleansing agent
5. Wipe off all the cleaning solution with a dry cloth after cleaning if necessary
6. Dry your equipment in a ventilated, cool area

External Defibrillator Paddles

1. Wipe the surface of paddle with a cloth moistened with water or 75% ethanol
2. Remove excessive cleaning agent with a dry cloth if necessary
3. Allow the paddles to air dry or dry with a cloth



Disinfection Protocol

Recommended Disinfectant:

Ethanol (75%); Isopropyl alcohol (70%)

Main Unit

Disinfect the equipment as required in your hospital's servicing schedule. It is recommended that the equipment is cleaned before disinfection.

External Defibrillator Paddle

Disinfection may cause harm to the paddles. We advise disinfecting only when necessary to avoid long-term damage. It is recommended that the paddles are cleaned before disinfection.

1. Clean the paddles as instructed above
2. Disinfect the paddles with a cotton or soft cloth moistened with one of the recommended disinfectants
3. After disinfection, be sure to wipe off the disinfectant left on the paddles with a soft cloth moistened with water
4. Allow the paddles to air dry

Before cleaning and disinfecting your equipment, please read carefully the 'Preservation and Sanitation' chapter found in the device's operators manual.