

Importing Configurations

Mindray BeneHeart D3 Defibrillator

To configure your BeneHeart D3 defibrillator to a standard Resuscitation Council (UK) 2015 guideline configuration, follow this procedure:

- Visit www.mindrayuk.com/education
- Download the D3 defibrillator RC(UK) 2015 guideline configuration to a formatted USB drive.
- Once the configuration has downloaded, insert the USB drive into the USB port found on the back of the D3 defibrillator.
- Turn the device to **'Monitor'** mode.
- Push the **'Menu'** button.
- Select **'Others'**.
- Select **'Configuration'**
- Using the navigation dial, enter code **315666**.
- Select **'OK'** and **'OK'** again.
- Select the **'Config. Import'** softkey.
- Select **'Yes'** to confirm the command.
- A message stating **'Config. updated successfully'** will appear on screen.
- Select the **'Exit'** softkey and confirm the command.
- Turn the mode device to off.

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions.
The illustrations may not necessarily reflect the setup or data displayed on your device.