

Importing Configurations

Mindray BeneVision N Series Patient Monitor

How to download N15 Standard configurations from the Mindray UK website:


- Visit www.mindrayuk.com/education
- Find the N15 Standard Configuration zip file and download onto a PC. Please note an encrypted USB stick cannot be used
- Unzip the file and copy the **mrbigdipercfg** folder and the **mrbigdiperUser** folder to a USB stick

To import the configuration from the USB drive to the monitor, follow this procedure:

- Turn the N Series patient monitor on with power key
- Connect the USB drive to the monitor's USB port on the back of the monitor
- Select the **Main Menu** quick key




- Swipe to the **Configuration** column → select **Manage**

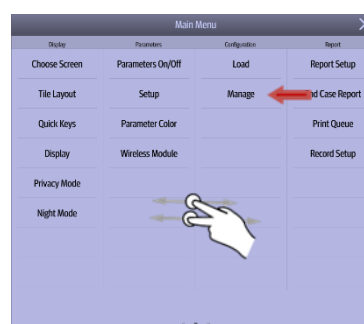


- Input the password **315666** and select

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions. The illustrations may not necessarily reflect the setup or data displayed on your device.

- Ensure the **Default Patient Category** is selected and whether **Default Use EWS** is **Yes** or **No**

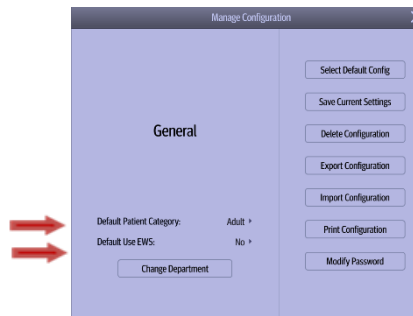


- Select **Import Configuration** → select **User Maintenance Settings, N15 Standard (Adult), N15 Standard (Ped)** and **N15 Standard (Neo)** → select **Import** → select **Back**

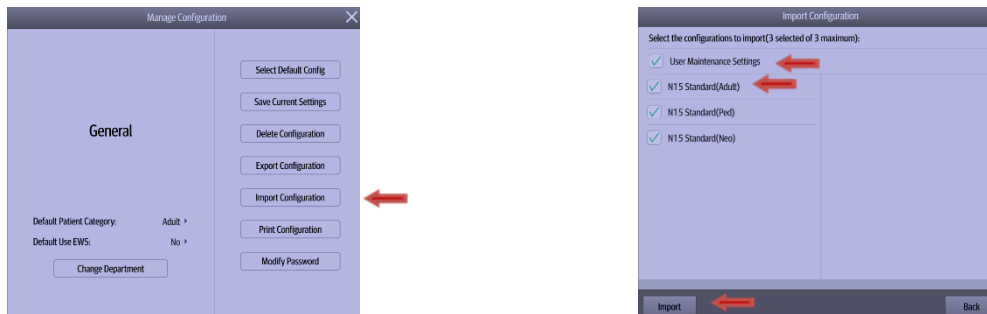

To set the default configuration, follow this procedure:

- In the **Manage Configuration** menu choose **Select Default Config** → select **Load Specified Config**

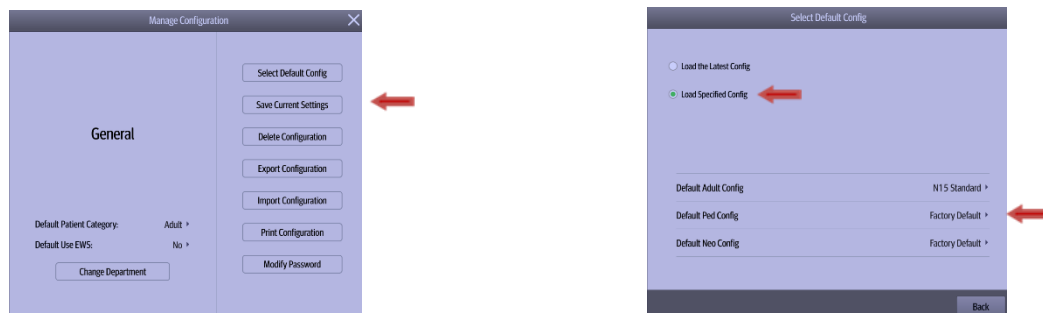


- Press the area labelled **Factory Default** located next to **Default Adult Config / Default Ped Config / Default Neo Config** → select **N15 Standard** for each of the three patient categories (Adult, Ped and Neo) → select **Back** → Press **'X'** to exit menu

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions. The illustrations may not necessarily reflect the setup or data displayed on your device.

To export the configuration to the USB drive from the monitor, follow this procedure:

- Connect the USB drive to the monitor's USB port
- Select the **Main Menu** quick key → swipe to the **Configuration** column → Select **Manage**

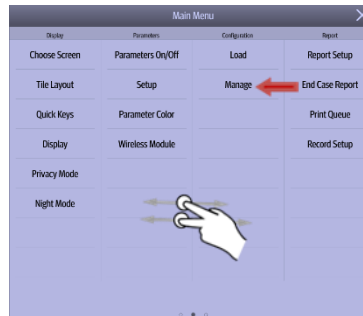

- Input the password **315666** and select
- Select **Export Configuration** → select the **Configurations** and **User Maintenance Settings** → select **Export** → select **Back**



In order to transfer configurations to an N1 via a USB stick, a dock is required.
For further information, please contact the Clinical Application team.

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions.
The illustrations may not necessarily reflect the setup or data displayed on your device.