



# Invasive Pressure Zeroing Quick Guide

- Turn the stopcock **OFF** to the patient and **OPEN** to air, in order to vent the transducer to atmospheric pressure

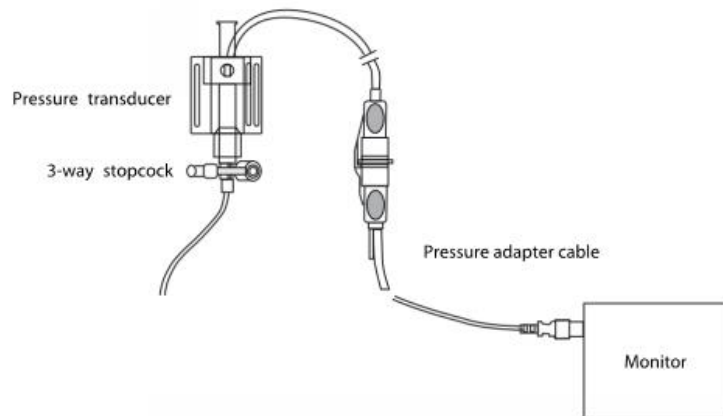

- Press the **Zero IBP**  quick key located on the touch screen
- The **Zero IBP** menu appears on the screen
- Select the pressure to be zeroed e.g **Art Zero** / **CVP Zero** / **Zero All Channels**
- During zero calibration, the selected button appears dimmed, recovering on completion of the calibration

**OR**

- Open the specific IBP menu by touching the waveform area and selecting the Zero option
- On completion of the zero calibration, **CLOSE** the stopcock to air and **OPEN** to the patient

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions.  
The illustrations may not necessarily reflect the setup or data displayed on your device.