


CO₂ Module

Measure & Standby modes

Quick Guide

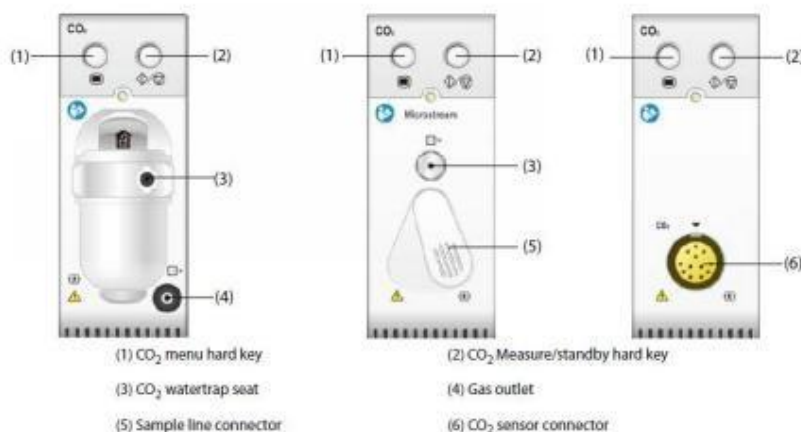

To enter or exit the Standby mode manually

- Select the hard key on the CO₂ module to enter or exit **Measure / Standby** mode
- Select **Operating Mode** in the **CO₂ Setup** menu and toggle between **Measure** and **Standby**

Information displayed when the CO₂ Module is in Standby mode



When the CO₂ Module is placed into standby mode, the sample intake pump sets to a sample flow rate of zero and the word **Standby** appears on the monitor display

When exiting **Standby** mode by pressing the CO₂ intake pump defaults to the preset flow rate

Please Note: The system configuration may be set to automatically place the CO₂ Module into standby mode following a predetermined period of inactivity

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions.
The illustrations may not necessarily reflect the setup or data displayed on your device.