

# VS-900 Vital Signs Monitor

## Quick Guide



Shenzhen Mindray Bio-Medical Electronics Co., Ltd.  
Mindray Building, Keji 12th Road South, Hi-tech Industrial Park,  
Nanshan, Shenzhen 518057 P.R. China

Website: [www.mindray.com](http://www.mindray.com)  
Tel. +86 755 81888998  
Fax: +86 755 26582680



Contents of this guide are subject to change without prior notice.

The Monitor is intended for monitoring physiologic parameters, including SpO<sub>2</sub>, PR, NIBP, TEMP and CO<sub>2</sub>, on adult, pediatric, and neonatal patients in health care facilities by clinical physicians or appropriate medical staff under the direction of physicians.

### WARNING:

The equipment must be connected to a properly installed power outlet with protective earth contacts only. If the installation does not provide for a protective earth conductor, disconnect it from the power line and operate it on battery power, if possible.






© 2013-2016 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved.

**mindray**

## VS-900 Vital Signs Monitor Quick Guide


This is a Quick Guide. Refer to VS-900 Operator's Manual (PN:046-004883-00) for details.

### Keys and Functions


	<p><b>Power ON/OFF Switch</b></p> <ul style="list-style-type: none"> <li>Press this key to turn the monitor on.</li> <li>Press this key to enter/exit Standby mode.</li> <li>When the monitor is on, press and hold this key for above 2 seconds to turn the monitor off.</li> </ul>
	<p><b>Alarm Reset Key</b></p> <ul style="list-style-type: none"> <li>Press this key to disable the audio of present alarms.</li> <li>Press and hold this key for more than 2 seconds to pause or restore alarms.</li> </ul>
	<p><b>NIBP Start/Stop Key</b></p> <ul style="list-style-type: none"> <li>Press to start or stop NIBP measurements.</li> </ul>
	<p><b>Admit Patient Key</b></p> <ul style="list-style-type: none"> <li>Press this key to admit a new patient.</li> <li>Press this key to return to main screen.</li> </ul>
	<p><b>Knob</b></p> <ul style="list-style-type: none"> <li>Rotate the knob clockwise or anti-clockwise to move the cursor.</li> <li>Press the knob to select one item, such as accessing a menu or confirming the selection.</li> </ul>

## Interfaces and Menu

### Side Connectors and Recorder

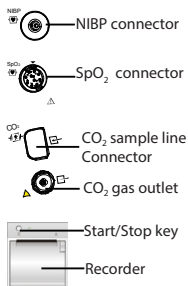

### Rear View

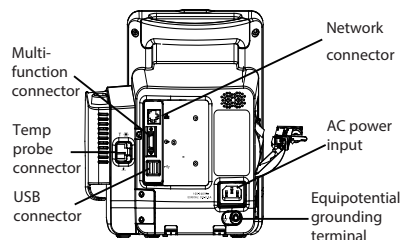

## Operating Modes

- Monitor Mode:** The monitor will automatically enter monitor mode after powers on. Monitor mode is a common mode for monitoring patient vital signs.
- Spot Check Mode:** The spot mode is intended for on-spot measurement in a short period. When spot mode is ON, the message area will display [**Spot Check**] on the top of screen.  
To enter Spot Check mode, select [**Main**]→[**Maintenance>>**]→[**User Settings>>**]→Enter required password→Set [**Spot Check**] to [**ON**].
- Standby Mode:** In standby mode, the monitoring for current patient is over, but the monitor still powers on.  
To enter Standby Mode, press the power button. Then a warning pops up. Select [**Yes**] to enter Standby mode.
- Demo Mode:** Demo mode is password protected, and it is used for demonstration purpose only.  
To enter Demo mode, select [**Main**]→[**Maintenance>>**]→[**Demo>>**]→Enter required password→Select [**OK**].