

WIFI capability provides you an added option for data communication together with bi-directional LIS, USB port and LAN port, barcode reader, printer and keyboard.

Technical Specifications

Principles
 Impedance method for WBC, RBC and PLT counting
 Cyanide free reagent for hemoglobin test

Performance

Parameter	Linearity Range	Precision (CV %)	Carryover
WBC($10^9/L$)	0-100	$\leq 3.5\%$ (4.0-6.9) $\leq 2.0\%$ (7.0-15.0)	$\leq 0.5\%$
RBC($10^{12}/L$)	0-8.00	$\leq 1.5\%$ (3.5-6.5)	$\leq 0.5\%$
HGB(g/L)	0-280	$\leq 1.5\%$ (100-180)	$\leq 0.5\%$
MCV(fL)		$\leq 1.0\%$ (70-110)	
PLT($10^9/L$)	0-1000	$\leq 5.0\%$ (100-149) $\leq 4.0\%$ (150-500)	$\leq 1.0\%$

Parameters
 19 parameters: WBC, Lymph#, Mid#, Gran#, Lymph%, Mid%, Gran%, RBC, HGB, HCT, MCV, MCH, MCHC, RDW-CV, RDW-SD, PLT, MPV, PDW, PCT
 3 histograms for WBC, RBC and PLT

Reagent
 M-30D DILUENT
 M-30CFL LYSE
 PROBE CLEANSER

Sample Volume
 Prediluted mode 20 μ L
 Whole blood mode 9 μ L

Throughput
 40 samples per hour

Display
 8.4 inch TFT Touch Screen

Multi-language
 Chinese, English, Spanish, Portuguese, Russian, French, Bahasa Indonesia

Data Storage Capacity
 Up to 200,000 results including numeric and graphical information

Communication
 LAN Port supports HL7 protocol
 Support bi-directional LIS

Interface
 4 USB port (for external printer, software upgrade, barcode reader, WIFI adapter, keyboard and mouse), LAN port (1)

Printout
 Thermal recorder, 50 mm wide paper, various printouts formats
 External printer optional

Operating Environment
 Temperature: 10°C~40°C
 Humidity: 10%~90%
 Air pressure: 70kPa~106kPa

Power Requirement
 100V-240V
 $\leq 300VA$
 50Hz/60Hz

Dimension and Weight
 Dimension: Depth(410 mm) x width(300 mm) x height(400 mm)
 Weight: $\leq 20Kg$

BC-20s

Auto Hematology Analyzer

Minimum Size, Maximum Capability





What a 3-part should be

At Mindray we pride ourselves in our dedication and experience in developing better solutions for small labs. Our new line of 3-part hematology analyzers is the culmination of that effort. Compact yet powerful, full featured yet affordable, the BC-20s is what a 3-part analyzer should be.



Exclusive Feature

Detailed flag information never before seen on a 3-part analyzer. Provides information useful for diagnosis including WBC flag, RBC flag and PLT flag.


BC-20s Auto Hematology Analyzer

Sample 1 : BC-20s shows flag "Lymph increased" which means a high number of Lymphocytes and/or immature cells. Meanwhile R1 flag is also displayed. Two kinds of flag messages are both supported to ensure clinicians have better understanding of sample results.

Sample 2 : Flag "Anemia" and "HGB Abn./Interfere?" are displayed. They mean the sample has shows either signs of Anemia, Abnormal Hemoglobin or sample is interfered such as by high value of Leukocytes or Agglutinated Erythrocytes.

Sample 3 : Flag "Thrombopenia" indicator is shown together with Platelets low flag.

Different flag information provided according to parameter results together with histograms.

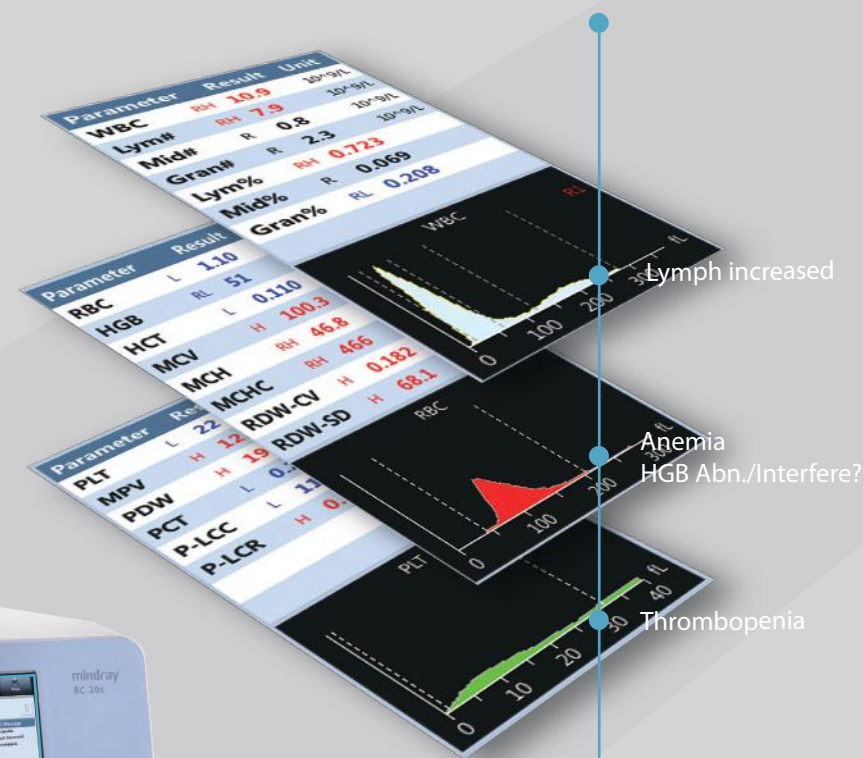

Better Usability

Minimum size with the footprint similar to that of a 17 inch laptop, with space saving design that allows internal storage of lyse giving small labs more space. 8.4 inch TFT touch screen together with our powerful software enhances user operations and experience.

Higher Efficiency


New technology that eliminates the need for cleanser and rinse, reducing the number of reagents needed while at the same time lowering overall reagent consumption.

Flexible packaging of reagents, with normal and small sizes to better cater to the needs of different daily sample volumes.



Enhanced Performance

Higher throughput at 40 tests per hour.

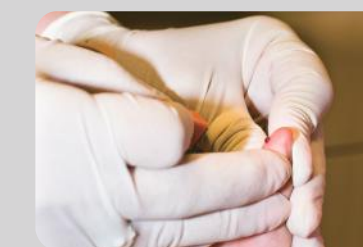

Micro sample volume at 9.0uL for whole blood mode with capillary whole blood samples supported, perfect for pediatric samples.