

Ultrasound System

Cleaning, Disinfection & Sterilization Guide

[Advanced]

Ultrasound system
Cleaning, Disinfection & Sterilization Guide
[Advanced]


Thank you for the purchase of Mindray's products. To ensure safe and proper operation of the system, read the guide carefully. Keep this guide carefully after reading, so that you can check it whenever necessary. Please go to <https://www.mindray.com/en/resources-center> for the latest user guide.

Intellectual Property Statement

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. (hereinafter called Mindray) owns the intellectual property rights to this Mindray product and this manual. This manual may refer to information protected by copyright or patents and does not convey any license under the patent rights or copyright of Mindray, or of others.

Mindray intends to maintain the contents of this manual as confidential information. Disclosure of the information in this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

Release, amendment, reproduction, distribution, rental, adaptation, translation or any other derivative work of this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

mindray,  and **MINDRAY** are the trademarks, registered or otherwise, of Mindray in China and other countries. All other trademarks that appear in this manual are used only for informational or editorial purposes. They are the property of their respective owners.

This posting serves as notice under 35 U.S.C. §287(a) for Mindray patents: <http://www.mindrayna.com/patents>.

Statement

Contents of this document are subject to changes without prior notice.

Mindray is responsible for the effects on safety, reliability and performance of this product, only if:

- All installation operations, expansions, changes, modifications and repairs of this product are conducted by Mindray authorized personnel;
- The electrical installation of the relevant room complies with the applicable national and local requirements and requirements of this guide;
- The product is used in accordance with the instructions for use.

Table of Contents

	Intellectual Property Statement.....	ii
	Statement	ii
	Important Information	1
	About this Guide.....	1
	Meaning of Signal Words.....	1
1	General Information About Cleaning, Disinfection and Sterilization.....	3
	1.1 Safety Precautions	3
	1.2 Brief Introduction to Materials	4
	1.2.1 Plastics	4
	1.2.2 Stainless Steel.....	5
	1.2.3 Silica Gel	5
	1.2.4 Epoxy Glue	5
	1.2.5 Glass	5
2	Inspection Before and After Use.....	7
	2.1 Appearance Inspection	7
	2.2 Use of Probe Sheath.....	7
	2.3 Wearing the Probe Sheath.....	8
3	Cleaning, Disinfecting/Sterilizing Probes	9
	3.1 Cleaning, Disinfection, and Sterilization Overview.....	9
	3.2 Selecting a Cleaning, Disinfection/Sterilization Method	10
	3.3 Processing Non-Critical Probes	11
	3.4 Processing Semi-Critical Probes.....	12
	3.4.1 Before Processing	12
	3.4.2 Cleaning	12
	3.4.3 High-Level Disinfection.....	13
	3.5 Processing Critical Probes	14
	3.5.1 Before Processing	14
	3.5.2 Cleaning	14
	3.5.3 Sterilization	15
4	Cleaning and Disinfecting the TEE Probe.....	17
	4.1 Before Processing.....	18

4.2	Cleaning	18
4.2.1	Compatible Detergents	18
4.2.2	Cleaning Procedure	18
4.3	Disinfection	19
4.3.1	Compatible Disinfectants	19
4.3.2	Disinfection Procedure	19
5	Cleaning and Sterilizing the Laparoscopic Probe	21
5.1	Before Processing	22
5.2	Cleaning	22
5.3	Sterilization	23
5.3.1	Sterilization using V-PRO Low Temperature Sterilization System	23
5.3.2	Sterilization using STERRAD Low Temperature Sterilization System	24
5.3.3	Sterilization using Solution	24
5.4	List of Cleaner, Disinfectant and Sterilant/Sterilization System	25
5.4.1	Validated Cleaner and Sterilant/Sterilization System	25
5.4.2	Material Compatible Disinfectant	26
6	Cleaning and Sterilizing the Needle-guided Bracket	27
6.1	Cleaning	27
6.2	Sterilization	28
6.2.1	Sterilization with a Sterilant	28
6.2.2	High-Pressure Steam Sterilization	28
7	Cleaning and Disinfecting the Main Unit	31
7.1	Cleaning	31
7.1.1	Cleaning the Display	31
7.1.2	Cleaning the Control Panel	32
7.1.3	Cleaning the Cover	32
7.1.4	Cleaning Other Parts	32
7.2	Disinfection	33
7.2.1	Identifying the Parts to Disinfect	34
7.2.2	Compatible Disinfectants	38
7.2.3	Disinfection Procedure	41
8	Cleaning Other Accessories	43
8.1	Cleaning the Holders	43
8.2	Cleaning the ECG Cables	43
8.3	Cleaning the Peripherals	43
Appendix A	Compatible Probe Cleaners and Disinfectants	45
A.1	Cleaners	45

A.2	Disinfectants.....	48
A.2.1	Wipes.....	48
A.2.2	Sprays	52
A.2.3	Solutions.....	55
A.2.4	Devices.....	58
A.2.5	Powders.....	59
Appendix B	Composition.....	61
B.1	Active Ingredients of the Cleaner	61
B.2	Active Ingredients of the Disinfectant.....	61
Appendix C	Registered Disinfectants in Canada.....	65
Appendix D	Registered Disinfectants in FDA Region	67

This page intentionally left blank.

Important Information

Thank you for your expression of confidence in the Mindray brand name. Mindray is constantly working on the further development of all products. Please appreciate that changes to the scope of supply in form, equipment and technology are possible for this reason. Therefore, no claims may be deduced from the information, figures and descriptions in this guide.

This guide contains warnings regarding foreseeable potential dangers, but you shall always be alert to dangers other than those indicated as well. Mindray shall not be liable for damage or loss that results from negligence or from ignoring the precautions and operating instructions described in this guide.

If the administrator for this system is changed, be sure to hand over this guide to the new manager.

WARNING

The probe and accessories supplied with it are not delivered disinfected or sterilized. Cleaning and disinfection (or sterilization) in accordance with this guide before use are required. After disinfection or sterilization, residual chemicals must be completely removed. Otherwise, personal injury or damage to the probes or accessories may occur.

About this Guide




Please follow the descriptions in the guide to clean and disinfect (or sterilize) the probes, needle-guided brackets, entire machine and accessories manufactured by Mindray. The guide is intended to provide information about cleaning and disinfecting (or sterilizing) the ultrasound system effectively, and also about protecting the ultrasound system from damages caused by incorrect cleaning, disinfection or sterilization procedures.



Mindray has verified that the disinfectants and disinfection equipment described in this guide are compatible with the ultrasound system. All the verified disinfectants are listed in disinfectant list of this guide.

The disinfectants and sterilants described in the guide are recommended by Mindray. For details on effectiveness and clinical application of these disinfectants and sterilants, refer to the instructions of disinfectant and sterilant manufacturers.

The functions and accessories described in the guide may vary with the specific system you purchased.

Meaning of Signal Words

In this guide, the signal words  **DANGER**,  **WARNING**,  **CAUTION**, **NOTE** and **Tip** are used regarding safety and other important instructions. The signal words and their meanings are defined as follows. Please understand their meanings clearly before reading this guide.

Signal word	Meaning
 WARNING	Indicates a potentially hazardous situation that, if not avoided, could result in death or serious injury.
 CAUTION	Indicates a potentially hazardous situation that, if not avoided, may result in minor or moderate injury.
NOTE	Indicates a potentially hazardous situation that, if not avoided, may result in property damage.

1 General Information About Cleaning, Disinfection and Sterilization

1.1 Safety Precautions

Follow the following precautions strictly to ensure the safety of the patient and the operator.

WARNING

- Do not subject the probe to shock. A defective probe may cause electric shock to the patient.
- Do not disassemble the probe to avoid the possibility of electric shock.
- Never immerse the probe connector into liquids such as water or disinfectant, for the connector is not waterproof. Immersion may cause electric shock or malfunction.
- In normal diagnostic ultrasound mode, there is no danger of a normal-temperature burn; however, keeping the probe on the same region of the patient for a long time may cause such a burn.

CAUTION

- The probe is only for use with the specified ultrasonic diagnostic system. Please refer to the ultrasonic diagnostic system operation manual to select the proper probe.
- For clinical applications of a semi-critical or critical nature (e.g., intraoperative, transrectal, transvaginal, transesophageal, or biopsy procedures), it is recommended, when appropriate, to use sterile, legally marketed probe sheaths.
- Damage to the probe may occur due to its contact with improper detergents or coupling gel. DO NOT dip the probe in the strong polar solution of ethanol, chloride of lime, ammonium chloride, acetone and formaldehyde. The probe should not make contact with solution or ultrasound gel containing oily medium such as mineral oil or lanoline.
- Coupling gel must be used during ultrasonic examination. Please use the ultrasound gel compliant with the relevant local regulations. And manage the ultrasound gel properly to ensure that it does not become a source of infection.
- Do not use the carrying case for storing the probe. If the carrying case is used for storage, it may become a source of infection.

- **The probe and accessories supplied with it are not delivered disinfected or sterilized. Cleaning and disinfection (or sterilization) in accordance with this guide before use are required.**
- **Disposable components are packaged sterile and can be used once only. Do not use if integrity of packaging violated or if expiration date has passed. Please use the disposable components compliant with the relevant local regulations.**
- **Use the disinfection or sterilization solution recommended in this guide only. Otherwise, Mindray will not be liable for damage caused by other solutions. If you have any questions, please contact Mindray Customer Service Department.**

NOTE:

- Read the following precautions to prevent the probe from malfunction:
- Before connecting or disconnecting the probe, freeze the ultrasonic image or turn off the diagnostic ultrasound system.
- After completing each examination, clean, disinfect or sterilize the probes as required. If necessary, repeat the cleaning, disinfection (or sterilization) process before next use.
- To prevent the probe from being damaged, do not use it where it will be exposed to:
 - Locations exposed to direct sunlight
 - Locations subject to sudden changes in environmental temperature
 - Dust
 - Excessive vibration
 - Locations near heat generators
- Cleaning cannot be replaced by sterilization. Probes and needle-guided brackets must be cleaned before sterilization.
- Repeated disinfection will eventually damage the probe, please check the probe performance periodically.
- Repeated disinfection will degrade the performance and safety of the needle-guided brackets.

1.2 Brief Introduction to Materials

1.2.1 Plastics

Plastic is material consisting of any of a wide range of synthetic or semi-synthetic organic compounds that are malleable and so can be molded into solid objects. It is made up of additives including synthetic resin, filler, plasticizer, stabilizer and pigment.

The plastic housing molded by injection molding process is anti-stamping, heat-resistant and stable in dimensions. However, some general engineering plastics (especially ABS) contain unsaturated double bonds. During the storage or application of these plastics, due to external factors such as illumination and oxidization, unsaturated double bonds will result in yellowing, degraded mechanical performance or even cracking.

Plastics in reusable medical devices manufactured by Mindray can be cleaned and disinfected (or sterilized) by using the recommended methods. For all plastics cannot withstand repeated actions of alcohols, do not disinfect or sterilize the plastic material in a way that is not recommended.

Plastic is mainly used in probe housings.

1.2.2 Stainless Steel

Stainless steel is corrosion resistant due to a natural passive layer which forms in the air. If this passive layer is destroyed by certain substances (solid, liquid or gaseous), there is no longer any corrosion resistance. The stainless steels may also corrode, particularly if they are in contact with blood or tissue residues.

Therefore make certain, not only for hygienic reasons, that organic remnants have been completely removed.

Stainless steel is mainly used in needle-guided brackets.

1.2.3 Silica Gel

Silica gel is excellent in thermo oxidative stability, weather resistance, and resistance to oxygen, ozone and ultraviolet radiation, for the backbone of silica gel consists of Si-O-Si bonds. However, polysiloxane has a helical structure and therefore has a good gas and vapor permeability.

The long-time immersion disinfection or sterilization may result in the inflation of polysiloxane. And the solution may permeate silica gel to damage the parts protected by silica gel. Therefore, disinfection should be performed in accordance with the immersion time specified by the disinfectant manufacturer.

Nevertheless, silica gel attracts dirt. Thus, make sure that organic residues are removed completely after cleaning and disinfection (or sterilization).

Silica gel is mainly used in sealant of acoustic lens and housing of probe.

1.2.4 Epoxy Glue

Epoxy glue, as one kind of thermosetting resins, has a good bonding effect on various interfaces, even in a high temperature of 110°C. It has resistance to ultraviolet radiation, erosion and weather.

The long-term immersion in peroxymonosulfuric acid, hydrogen peroxide, hypochlorite or glutaraldehyde may discolor epoxy glue slightly, but will not affect its performance.

Epoxy glue is mainly used in sealing of the probe housing.

1.2.5 Glass

Glass is mainly made up of various inorganic mineral substances, such as Zircon sand, borax and sodium carbonate. It also contains a little auxiliary materials. Its main ingredients are SiO₂ and other oxide. It has resistance to erosion.

Long-time immersion in disinfectant or sterilant solution will not affect its chemical properties.

Glass is mainly used in monitor and touch screen of the ultrasound system.

This page intentionally left blank.

2 Inspection Before and After Use

Brand new equipment should be unpacked and checked after delivery. If there is any transport damage, contact Mindray Customer Service Department immediately.

Every probe passes through an extensive cleaning and final check at the end of the manufacturing process. However, soiling is possible during transportation and storage. Therefore the probes must go through the complete cleaning and disinfection procedure before their first use.

Inspection before and after use must be performed as described below to ensure safe operation of the probe. If any abnormality is found, immediately stop using the probe and contact Mindray Customer Service Department or sales representative.

2.1 Appearance Inspection

Before and after each examination, confirm that there are no abnormalities of the probe surface or cable sheath, such as peeling, cracks, protruding parts, or looseness of the acoustic lens.

WARNING

Probe abnormalities may cause electric shock or injury to the patient. If any abnormality is found, immediately stop using the probe and contact Mindray Customer Service Department.

2.2 Use of Probe Sheath

Probe sheaths are available for use with all clinical situations where infection is a concern.

A sterile probe sheath is suggested to be used during intra-cavity examination or intra-operative examination.

Use a commercially available probe sheath.

To order probe sheath, contact:

CIVCO Medical Instruments Co.

102 First Street South, Kalona, IA 52247-9589 USA

Tel: 1-319-656-4447

E-mail: info@civco.com

<http://www.civco.com>

CAUTION


- **A probe sheath should be used once only to prevent infection.**

- **Use intact probe sheaths only.**
- **The probe sheath contains natural rubber latex and talc that can cause allergic reactions to some individuals.**
- **Use probe sheaths before the expiry date. Check if the probe sheath to be used is within the expiry period.**
- **Do not use pre-lubricated condoms as a sheath. Lubricant may not be compatible with the probe material and damage may result.**


2.3 Wearing the Probe Sheath

The method of using a probe sheath is shown as follows (for reference only):


1. Apply an appropriate amount of coupling gel inside the sheath or on probe surface. Poor imaging may occur if no gel is applied.

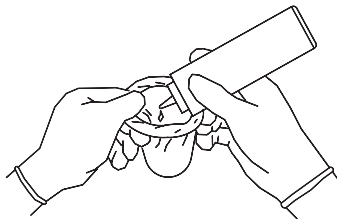

2. Use proper sterile technique to insert the probe into a probe sheath. Pull probe sheath tightly over probe surface to remove wrinkles and air bubbles, and take care to avoid puncturing cover.

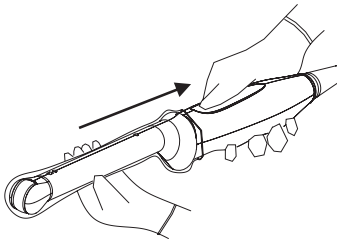

3. Secure the sheath with enclosed elastic bands.

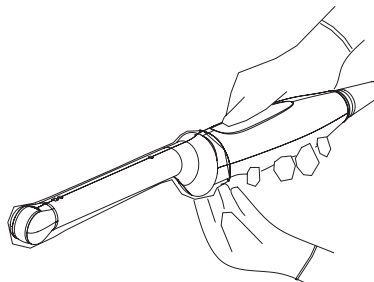

4. Inspect the sheath to ensure there are no holes or tears.

3 Cleaning, Disinfecting/Sterilizing Probes

After completing each examination, clean, disinfect or sterilize the probes as required. If necessary, repeat the cleaning, disinfection (or sterilization) process before next use. When biopsy procedures have been performed, be sure to sterilize the needle-guided bracket. Fail to do so may result in the probe and the needle-guided bracket to becoming sources of infection. Please follow the instructions in the manual for cleaning.

WARNING

Never immerse the probe connector into liquids such as water or disinfectant, for the connector is not waterproof. Immersion may cause electric shock or malfunction.

CAUTION

- **No cleaning and disinfecting may result in the probe becoming a source of infection.**
- **Please follow the disinfectant manufacturer's manual for performing cleaning and disinfection, including preparing sterile water and cleaning and disinfection time.**

NOTE:

- After the examination, wipe off the ultrasound gel thoroughly. Otherwise, the ultrasound gel may solidify and degrade the image quality of the probe.
- DO NOT make the probe to become overheated (more than 55 °C) during cleaning and disinfections. High temperature may cause the probe to become deformed or damaged.
- Observe the illustration graph carefully to immerse the probe. Only soak parts of the probe below the strain relief.
- Repeated disinfection or sterilization will eventually damage the probe, please check the probe performance periodically.
- Clean the probe thoroughly in accordance with the cleaning procedure before disinfection or sterilization.
- For details about probe types, refer to the operator's manuals of the ultrasound system.
- For details about recommended disinfectants for probes, see "Appendix A Compatible Probe Cleaners and Disinfectants".

3.1 Cleaning, Disinfection, and Sterilization Overview

Cleaning and disinfection refer to two distinct processes. According to the Centers for Disease Control and Prevention (CDC) "Guideline for Disinfection and Sterilization in Healthcare Facilities" (2008):


- Cleaning is the removal of visible soil (e.g. organic and inorganic material) from objects and surfaces and normally is accomplished manually or mechanically using water with detergents or enzymatic products. Thorough cleaning is essential before high-level disinfection and sterilization because inorganic and organic material that remains on the surfaces of instruments interfere with the effectiveness of these processes.
- Disinfection describes a process that eliminates many or all pathogenic microorganisms, except bacterial spores.
 - Low-Level Disinfection (LLD): Destruction of most bacteria, some viruses, and some fungi.
 - High-Level Disinfection (HLD): Destruction/removal of all microorganisms except bacterial spores.
- Sterilization describes a process that destroys or eliminates all forms of microbial life and is carried out in healthcare facilities by physical or chemical methods.

3.2 Selecting a Cleaning, Disinfection/Sterilization Method

Probes can be divided into three categories based on their intended use according to the standard ISO 17664-1:2021. Some probes may fall into more than one category (e.g. probes use for biopsy procedures). When selecting a disinfectant, determine the required level of disinfection based on intended use and possibility of cross-contamination.

- Non-critical items: come into contact with intact skin only or are devices not intended for direct patient contact. Probes that only come into contact with clean, intact skin are considered noncritical devices and require cleaning after every use. Cleaning may be followed by a low-level disinfectant spray or wipe. For details, see "3.3 Processing Non-Critical Probes".
- Semi-critical items: come into contact with mucous membranes. This category includes all endocavity probes - transvaginal, transrectal, and transesophageal (TEE). These semi-critical probes must be cleaned with an appropriate cleaner after use followed by high-level disinfection. For details, see "3.4 Processing Semi-Critical Probes".
- Critical items: enter normally sterile parts of the human body. These probes are considered critical and include all intraoperative probes. These probes must be cleaned with an appropriate cleaner after each use, followed by a sterilization process. For details, see "3.5 Processing Critical Probes".

Figure 3-1 Workflow of selecting a processing method

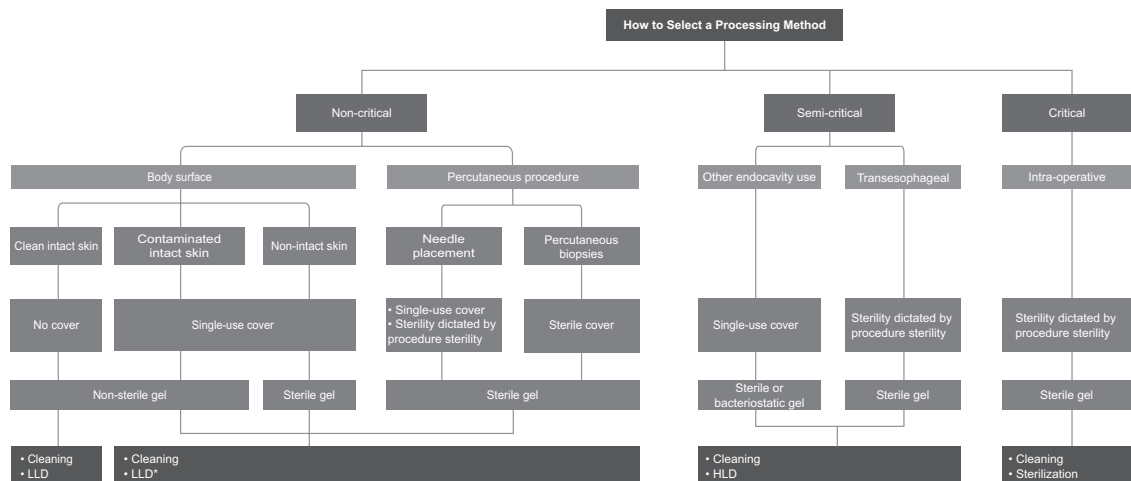

NOTE

LLD marked with * indicates that those categories must undergo low-level disinfectants that are effective against the mycobacteria and bloodborne pathogens. For details, consider referencing the position statement of the American Institute of Ultrasound in Medicine “Guidelines for Cleaning and Preparing External- and Internal-Use Ultrasound Transducers and Equipment Between Patients as well as Safe Handling and Use of Ultrasound Coupling Gel” at <https://www.aium.org/officialstatements/57>.

3.3 Processing Non-Critical Probes

Processing of non-critical probes requires a two-step process: Cleaning of the probe followed by low-level disinfection.

⚠ WARNING

Use protective eyewear when disinfecting the probe using sprays.

Perform the following procedure:

1. Wear a pair of gloves to prevent infection through the whole processing.
2. Disconnect the probe from the system. If the sheath is used, take off the sheath and discard it.
3. Clean the probe.
 - a. Select an appropriate low-level disinfectant wipe or a piece of disposable lint-free soft cloth soaked with a disinfectant spray. For details about recommended disinfectants for probes, see "Appendix A Compatible Probe Cleaners and Disinfectants".
 - b. Wipe all the surface of the probe according to the wiping duration specified in the operator’s manual provided by the manufacturer.

When necessary, clean and disinfect the seams or biopsy guide features by using disposable cotton swabs.
4. Disinfect the probe.

Prepare a new low-level disinfectant wipe or a piece of disposable lint-free soft cloth soaked with a disinfectant spray to wipe the probe again.

5. Inspect the probe. If visible dirt still exists, repeat the preceding steps to wipe the probe until it is all clean.
6. Allow the probe to air dry in a clean and well-ventilated place or dry the probe with a piece of disposable lint-free soft cloth or tissue.
Do not dry the probe by heating.
7. Check whether the probe has defects such as peeling, rifts, bumps, cracks, or liquid spill. If such defects exist, the probe has reached the end of its service life. In this case, stop using it and contact the Mindray service department.
8. Store the probe in a cool, clean and dry environment.

3.4 Processing Semi-Critical Probes

Processing of semi-critical probes requires a two-step process: Cleaning of the probe followed by high-level disinfection.

For detailed information on the TEE cleaning and disinfection, see "4 Cleaning and Disinfecting the TEE Probe".

3.4.1 Before Processing

This step is to remove the ultrasound gel or other visible dirt.

1. Wear a pair of gloves to prevent infection through the whole processing.
2. Disconnect the probe from the system. If the sheath is used, take off the sheath and discard it. Wipe off the ultrasound gel or other visible dirt on the surface of the probe by using a damp piece of disposable lint-free soft cloth or tissue.

3.4.2 Cleaning

Select wipes or detergent to clean the probe. For details about recommended cleaners or disinfectants for probes, see "Appendix A Compatible Probe Cleaners and Disinfectants".

Cleaning with Wipes

Perform the following procedure:

1. Use an approved cleaning or disinfectant wipe, cleaning sponge, or a soft cloth soaked in approved cleaner or disinfectant to clean all surfaces of the probe thoroughly.
2. Dry the probe with a piece of disposable lint-free soft cloth or tissue.
Do not dry the probe by heating.
3. Inspect the probe. If visible dirt still exists, repeat the preceding steps to wash the probe until it is all clean.


Cleaning with Detergent

Perform the following procedure:

1. Choose an appropriate cleaning agent including mild detergents, and enzymatic.

2. Immerse the probe fully in the cleaning fluid for at least 1 minute or according to manufacturer's instructions. Lightly clean the probe with a piece of lint-free soft cloth or soft sponge until no dirt is visible. When necessary, clean the seams or biopsy guide features by using disposable cotton swabs. Avoid using a brush to wash the lens because it may damage the probe.

Observe the graph here carefully to immerse the probe. Only soak parts of the probe below the strain relief.

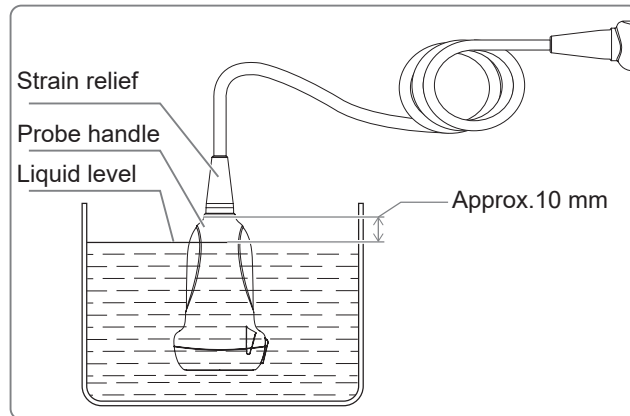


3. Rinse the probe thoroughly by using a large amount of clean water (about 7.5 L/2 gallons) at room temperature for about 1 minute to remove the residual dirt and cleaning solvent.
4. Dry the probe with a piece of disposable lint-free soft cloth or tissue.
Do not dry the probe by heating.
5. Inspect the probe. If visible dirt still exists, repeat the preceding steps to wash the probe until it is all clean.

3.4.3 High-Level Disinfection

Perform the following procedure:

1. Disinfect the probe by using an appropriate high-level disinfectant solution or device. For how to use a high-level disinfectant or device, see the operator's manual provided by the manufacturer. Prepare a disinfectant by using sterile distilled or softened water when necessary. For details about recommended disinfectants or devices for probes, see "Appendix A Compatible Probe Cleaners and Disinfectants".
 - Soaking: Immerse the probe head in the disinfectant and shake the probe appropriately to remove any bubbles on the probe surface. For details about the probe immersion duration, see the operator's manual provided by the manufacturer.

Observe the graph here carefully to immerse the probe. Only soak parts of the probe below the strain relief.

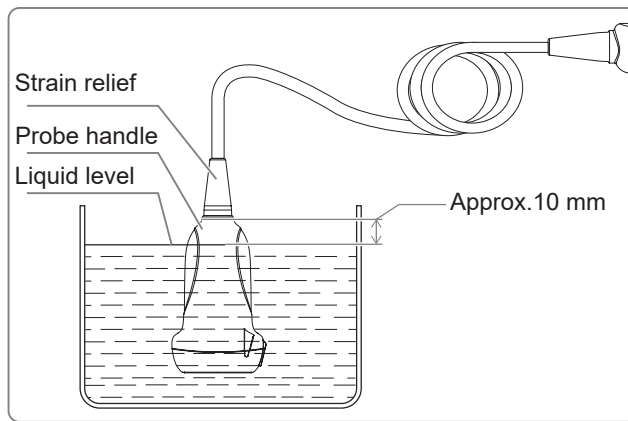

- Wiping: Use a market disinfection wipe product or sterile disposable lint-free soft cloth wetted with disinfection spray and wipe all surfaces of the probe for a duration according to the manufacturer instructions.
2. Rinse the probe thoroughly by using a large amount of clean water (about 7.5 L/2 gallons) at room temperature for about 1 minute to remove the residual disinfectant. Or follow the disinfectant manufacturer's instructions regarding rinsing. Dry the probe with a piece of disposable lint-free soft cloth or tissue.
Do not dry the probe by heating.
 3. Check whether the probe has defects such as peeling, rifts, bumps, cracks, or liquid spill. If such defects exist, the probe has reached the end of its service life. In this case, stop using it and contact the Mindray service department.
 4. Store the probe in a cool, clean and dry environment.

3.5 Processing Critical Probes

Processing of critical probes requires a two-step process: Cleaning of the probe followed by sterilization.

For detailed information on the laparoscopic cleaning and disinfection, see "5 Cleaning and Sterilizing the Laparoscopic Probe".

3.5.1 Before Processing

This step is to remove the ultrasound gel or other visible dirt.

1. Wear a pair of gloves to prevent infection through the whole processing.
2. Disconnect the probe from the system. If the sheath is used, take off the sheath and discard it. Wipe off the ultrasound gel or other visible dirt on the surface of the probe by using a damp piece of disposable lint-free soft cloth or tissue.

3.5.2 Cleaning

Select wipes or detergent to clean the probe. For details about recommended cleaners or disinfectants for probes, see "Appendix A Compatible Probe Cleaners and Disinfectants".

Cleaning with Wipes

Perform the following procedure:

1. Use an approved cleaning or disinfectant wipe, cleaning sponge, or a soft cloth soaked in approved cleaner or disinfectant to clean all surfaces of the probe thoroughly.


2. Dry the probe with a piece of disposable lint-free soft cloth or tissue.
Do not dry the probe by heating.
3. Inspect the probe. If visible dirt still exists, repeat the preceding steps to wash the probe until it is all clean.

Cleaning with Detergent

Perform the following procedure:

1. Choose an appropriate cleaning agent including mild detergents, and enzymatic.
2. Immerse the probe fully in the cleaning fluid for at least 1 minute or according to manufacturer's instructions. Lightly clean the probe with a piece of lint-free soft cloth or soft sponge until no dirt is visible. When necessary, clean the seams or biopsy guide features by using disposable cotton swabs. Avoid using a brush to wash the lens because it may damage the probe.

Observe the graph here carefully to immerse the probe. Only soak parts of the probe below the strain relief.

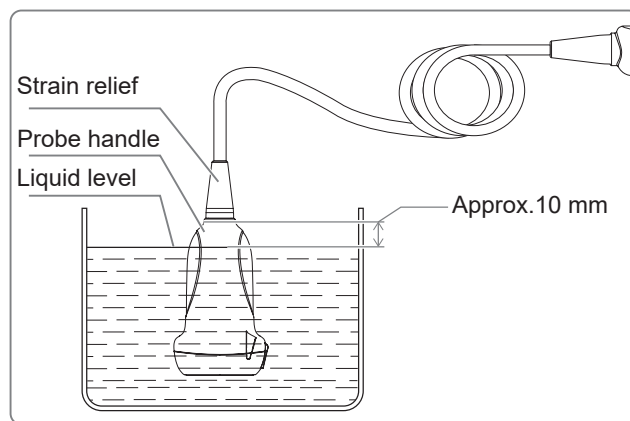

3. Rinse the probe thoroughly by using a large amount of clean water (about 7.5 L/2 gallons) at room temperature for about 1 minute to remove the residual dirt and cleaning solvent.
4. Dry the probe with a piece of disposable lint-free soft cloth or tissue.
Do not dry the probe by heating.
5. Inspect the probe. If visible dirt still exists, repeat the preceding steps to wash the probe until it is all clean.

3.5.3 Sterilization

For intra-operative probes, they have to be thoroughly cleaned and sterilized after completing each examination.

Perform the following procedure:

1. Sterilize the probe by using an appropriate sterilant or device. For details about recommended sterilants or devices for probes, see "Appendix A Compatible Probe Cleaners and Disinfectants".


For how to use a device, see the operator's manual provided by the manufacturer.

When using a sterilant, follow the following steps:

- a. Prepare a sterilant by using sterile distilled or softened water when necessary.

- b.** Immerse the probe head in the sterilant and shake the probe appropriately to remove any bubbles on the probe surface.

For details about the probe immersion duration, see the operator's manual provided by the manufacturer.

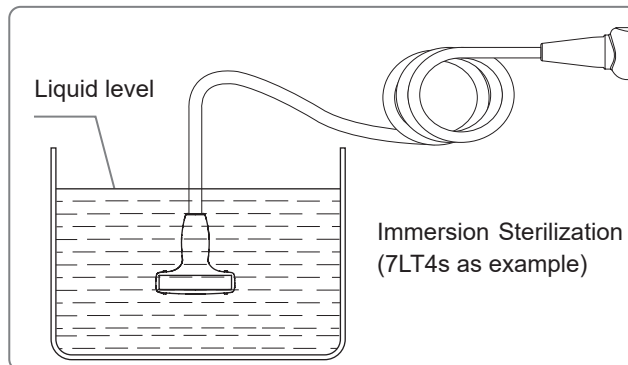

- c.** Rinse the probe thoroughly by using a large amount of sterile distilled or softened water (about 7.5 L/2 gallons) at room temperature for about 1 minute to remove the residual disinfectant. Or follow the sterilant manufacturer's instructions regarding rinsing. Dry the probe with a piece of sterile disposable lint-free soft cloth.

Do not dry the probe by heating.

- 2.** Check whether the probe has defects such as peeling, rifts, bumps, cracks, or liquid spill. If such defects exist, the probe has reached the end of its service life. In this case, stop using it and contact the Mindray service department.
- 3.** Store the probe in a cool, clean and dry environment.

4 Cleaning and Disinfecting the TEE Probe

This section describes the methods and precautions for cleaning and disinfection of TEE probes. After completing each examination, clean, disinfect the TEE probes as required. If necessary, repeat the cleaning, disinfection process before next use.

WARNING

- **Keep the control handle and system connector out of any cleaning or disinfection solutions. The control handle and cable may be cleaned with a damp cloth, but only the distal end of the transducer up to the 100 cm marker on the shaft (70 cm for P8-3Ts) may be placed into a disinfection solution.**
- **Do not use other disinfection methods like Iodine, Steam, Heat or Ethylene Oxide.**
- **To avoid injury to the patient, you must follow the manufacturer's recommendation for rinsing.**

CAUTION

- **To avoid damaging the transducer, the transducer should not be exposed to the disinfectant longer than specified to achieve the desired effect, but never longer than one hour.**
- **To avoid damaging the transducer, do not steam autoclave or subject the transducer to Ethylene Oxide (ETO).**
- **To avoid damaging the transducer, do not immerse the transducer in a solution containing ethanol.**
- **After disinfection, rinse the transducer thoroughly with clean water to remove all chemical residues. Chemical residues on the transducer may be harmful to the human body.**
- **The efficacy of disinfectants solutions is not guaranteed by MINDRAY. Contact the manufacturers for information on the activity of the products.**

NOTE:

- **After the examination, wipe off the ultrasound gel thoroughly, otherwise, the ultrasound gel may solidify and degrade the image quality of the transducer.**
- **Do not permit the transducer to become overheated (more than 55°C) during cleaning and disinfections. High temperature may cause the transducer to become deformed or damaged.**
- **Clean the probe thoroughly in accordance with the cleaning procedure before disinfection.**

4.1 Before Processing

This step is to remove the ultrasound gel or other visible dirt.

1. Wear a pair of gloves to prevent infection through the whole cleaning, disinfection process.
2. Disconnect the probe from the system. If the sheath is used, take off the sheath and dispose it as directed by the hospital.
3. Wipe away the ultrasound gel or other visible dirt on the surface of the probe by using a damp piece of disposable lint-free soft cloth or tissue.

4.2 Cleaning

Only the following cleaners are recommended by Mindray to clean the TEE transducers. For the biological effectiveness and the correct use of the cleaners, see the information of the cleaner's manufacturer.

4.2.1 Compatible Detergents

Cleaner	Active ingredient	Concentration	Applicable for
Cidezyme/Enzol	Proteolytic enzymes	< 5%	P7-3T
EMpower	Proteolytic enzymes	< 2%	P7-3Ts
Metrizyme	Proteolytic enzymes	< 2%	P7-3TU
Neodisher	Trisodium nitrilotriacetate	5-10%	P7-3TE
MediClean forte	2,2-Iminodiethanol diethanolamine	1-2%	P8-3Ts
Prolystica 2x conc. Enzymatic Presoak & cleaner	Ethanolamine	1-5%	P7-3T P7-3Ts
	Protease	0.1-1%	P7-3TU
	Ethoxylated alcohol	1-5%	P7-3TE
	Polyalkylene glycol	1-5%	P8-3Ts
	Glycerine	1-5%	
WIP'ANIOS CLEAN'UP	Association of surfactants, excipients	/	P8-2Ts/P8-2TU

4.2.2 Cleaning Procedure

1. Select an appropriate cleaner include cleaning wipes and sprays, mild detergents, enzymatic cleaners and specially designed enzymatic sponges.
2. Follow the manufacturer's instructions to prepare and use the cleaner. Select an appropriate method:
 - Soaking: Soak the probe in the cleaner solution for at least 1 minute or follow the manufacturer's instructions. Wipe and wash the probe surface gently by using a piece of lint-free soft cloth or soft sponge until no dirt is visible. When necessary, wash the locating groove and other items by using disposable cotton swabs.

Avoid using a brush to wash the lens because it may damage the probe.

Only the section from distal end to the 100 cm marker (70 cm for pediatric TEE probe) on the flexible shaft can be immersed in the cleaner solution.

- Wiping: Use a commercially available cleaning wipe, cleaning sponge, or a soft cloth soaked with a cleaning spray to wipe and clean all surfaces of the probe thoroughly for at least 1 minute or follow the manufacturer's instructions until the probe is clean. When necessary, wash the locating groove and other items by using disposable cotton swabs.
3. Rinse the probe insertion part thoroughly with plenty of clean flowing water (about 7.5L) at room temperature for about 1 minute to remove the residual dirt and cleaning solvent. Or follow the rinsing method specified by the manufacturer. Use moistened soft cloth to wipe the residual cleaners on the handle, cable, and connector.
 4. Dry the probe with a piece of disposable lint-free soft cloth or tissue.
Do not dry the probe by heating.
 5. Inspect the probe. If visible dirt still exists, repeat the preceding steps to wash the probe until it is all clean.

4.3 Disinfection

Only the following disinfectants are recommended by Mindray to disinfect the TEE transducers. For the biological effectiveness and the correct use of the disinfectants, see the information of the disinfectants' manufacturer.

4.3.1 Compatible Disinfectants

Disinfectant	Active ingredient	Concentration	Applicable for
Cidex (applicable for FDA region only)	Glutaraldehyde	2.55%	P7-3T P7-3Ts
Cidex OPA (applicable for FDA region only)	Ortho-phthalaldehyde	0.55%	P7-3TU P7-3TE
Metricide	Glutaraldehyde	2.60%	P8-3Ts
Metricide 28	Glutaraldehyde	2.50%	P8-2Ts
Metricide OPA Plus (applicable for FDA region only)	Ortho-phthalaldehyde	0.60%	P8-2TU
Revital-Ox Resert XL HLD	Hydrogen Peroxide	1.4-2.3%	P7-3T
	2-Fluoroic Acid	≤2.50	P7-3Ts P7-3TU P7-3TE P8-3Ts
Rapicide OPA-28	Ortho-phthalaldehyde	0.575%	P8-2Ts P8-2TU

4.3.2 Disinfection Procedure

1. Select an appropriate high-level disinfectant to disinfect the probe. Follow the manufacturer's instructions to use the high-level disinfectant. Prepare a disinfectant by using sterile distilled or softened water when necessary.
2. Select an appropriate method:

- Soaking: Soak the probe in the disinfectant solution and shake the probe properly to remove bubbles on the surface of the probe. For the probe soaking duration, see the disinfectant manufacturer's instructions.
Only the section from distal end to the 100 cm marker (70 cm for pediatric TEE probe) on the flexible shaft can be immersed in the disinfectant solution.
 - Wiping: Use a commercially available disinfecting wipe, or a disposable sterile lint-free soft cloth soaked with a disinfecting spray to wipe and disinfect all surfaces of the probe thoroughly for the time specified in the manufacturer's instructions.
3. Rinse the probe insertion part thoroughly with plenty of clean flowing water (about 7.5L) at room temperature for about 1 minute to remove the residual dirt and disinfectants. Or follow the rinsing method specified by the manufacturer. Use moistened soft cloth to wipe the residual disinfectants on the handle, cable, and connector.
 4. Dry the probe with a piece of disposable lint-free soft cloth or tissue.
Do not dry the probe by heating.
 5. Check whether the probe has defects such as peeling, rifts, bumps, cracks, or liquid spill. If such defects exist, the probe has reached the end of its service life. In this case, stop using it and contact the Mindray service department.
 6. Store the probe in a cool and dry environment.

 **CAUTION**

- **The lens may be discolored; the label on the transducer may fade. These are not abnormalities.**
- **Repeated disinfection will eventually damage the transducer, please check the transducer's performance periodically.**
- **Disinfecting incorrectly or with chemicals not recommended by Mindray will void the warranty.**

 **WARNING**

To avoid injury to the patient, if residue from the disinfectant is not removed it can cause irritation and/or burning of the mouth and esophageal tissue.

5

Cleaning and Sterilizing the Laparoscopic Probe

This section describes the methods and precautions for cleaning and sterilization of laparoscopic probes. After completing each examination, clean, sterilize the laparoscopic probes as required. If necessary, repeat the cleaning, sterilization process before next use.

WARNING

- **Keep the control handle and system connector out of any cleaning or sterilant solutions. The control handle and cable may be cleaned with a damp cloth, but only the distal end of the transducer up to the hard shaft (under the strain relief) may be placed into a sterilant solution.**
- **To avoid injury to the patient, you must follow the manufacturer's recommendation for rinsing.**
- **If the probe is not cleaned and sterilized, it may become the source of infection.**

CAUTION

- **To avoid damaging the transducer, the transducer should not be exposed to the sterilant longer than specified to achieve the desired effect.**
- **To avoid damaging the transducer, do not immerse the transducer in a solution containing ethanol.**
- **After sterilization, rinse the transducer thoroughly with sterile distilled water to remove all chemical residues. Chemical residues on the transducer may be harmful to the human body.**
- **The laparoscopic probe, as a critical probe, must undergo cleaning and sterilization completely after each use.**

NOTE:

- **After the examination, wipe off the ultrasound gel thoroughly, otherwise, the ultrasound gel may solidify and degrade the image quality of the transducer.**
- **Do not permit the transducer to become overheated (more than 55°C) during cleaning and sterilization. High temperature may cause the transducer to become deformed or damaged.**

- Since the probe connector is not waterproof, during the cleaning and sterilization (soaking) process, be sure to put the seal cover on the probe connector to prevent the liquid from entering the connector. Do not use the seal cover during sterilization using systems to avoid damage to the probe due to device air pressure of the sterilization system.

5.1 Before Processing

This step is to remove the ultrasound gel or other visible dirt.

Perform the following procedure:


1. Wear protective devices such as surgical caps, masks, gloves, goggles or face shield, and dedicated lab suit when cleaning and sterilizing the laparoscopic probe.
2. After using the probe, freeze the image, power off the ultrasound system, and disconnect the probe from the ultrasound system, to prevent data loss due to hot plug. If the sheath is used, take off the sheath and dispose it as directed by the hospital. Cleaning and sterilization are required even if the sheath is used.
3. Wipe away the ultrasound gel or other visible dirt on the surface of the probe by using a damp piece of disposable lint-free soft cloth or tissue.
4. Place the contaminated probe in an appropriate hard transport container and transport it to the decontamination room. During transportation, avoid colliding or squeezing the probe. Do not touch the acoustic head with heavy objects. Keep the probe moist during transportation to prevent body fluid from drying up on the probe surface. If cleaning cannot be performed immediately, immerse the insertion part of the probe in the detergent or water to avoid drying for more than 30 minutes.

5.2 Cleaning

Perform the following procedure:

1. Select an appropriate cleaner. For details, see "5.4.1 Validated Cleaner and Sterilant/ Sterilization System".
2. Follow the manufacturer's instructions to prepare and use the cleaner. Select an appropriate method:
3. Cleaning the insertion part: soak the probe insertion part thoroughly in the cleaner solution for at least 5 minutes or follow the manufacturer's instructions.
4. Wipe and wash the probe surface gently by using a piece of lint-free soft cloth or soft sponge until no dirt is visible. When necessary, wash the locating groove and other items by using disposable cotton swabs. Avoid using a brush to wash the lens because it may damage the probe.

Only the section from distal end to the hard shaft can be immersed in the cleaner solution.

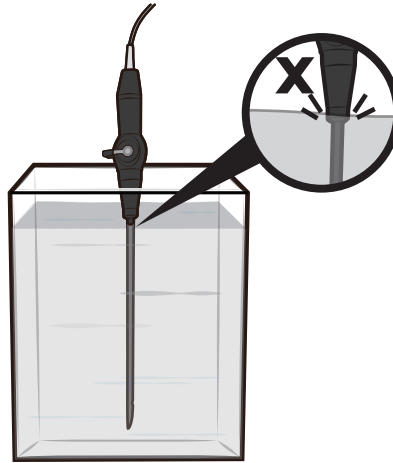

5. Cleaning other parts (except for the insertion part): use a disposable soft cloth soaked with cleaner solution and screw it dry until no liquids drip to wipe the strain relief, control handle, deflection lever, cable, and probe connector for at least 1 minute or follow the manufacturer's instructions until the probe is clean.

Avoiding touching the internal pins of the connector with any cleaner.

6. Rinse the probe insertion part thoroughly with plenty of clean flowing water (about 7.5L) at room temperature for about 1 minute to remove the residual dirt and cleaner solution. Or follow the rinsing method specified by the manufacturer. Use moistened dust-free soft cloth to wipe the residual dirt or cleaners on the parts except for the insertion part.
7. Dry the probe with a disposable lint-free soft cloth or tissue.
Do not dry the probe by heating.
8. Inspect the probe. If visible dirt still exists, repeat the preceding steps to clean the probe until it is all clean.
9. Check whether the probe has defects such as peeling, rifts, bumps, cracks, or liquid spill. If such defects exist, the probe has reached the end of its service life. In this case, stop using it and contact the Mindray service department.

5.3 Sterilization

NOTES:

- Clean the probe thoroughly in accordance with the cleaning procedure before sterilization. Sterilization using systems or sterilant both can achieve sterilization effect. Select an appropriate sterilization method as required.
- This chapter only introduces the basic operation procedures of the sterilization system. For details about using the sterilization system, refer to the manufacturer's instructions.

5.3.1 Sterilization using V-PRO Low Temperature Sterilization System

Perform the following procedure:

1. Place the probe into a STERIS Sterilization Tray and wrap the probe with sterilization wrap which had already cleared by the authorities such as H600 OneStep[®] sterilization wrap.

2. Start the V-PRO Low Temperature Sterilization System using the Non Lumen Cycle according to the instructions provided by the manufacturer.
3. Keep the sterilization wrap together with other sterilized surgical instruments in a sterile item storage area.
4. Before next use, check whether the probe has defects such as peeling, rifts, bumps, cracks, or liquid spill. If such defects exist, the probe has reached the end of its service life. In this case, stop using it and contact the Mindray service department.


5.3.2 Sterilization using STERRAD Low Temperature Sterilization System

1. Place the probe into a APTIMAX® Instrument Tray and wrap the probe with sterilization wrap which had already cleared by the authorities such as Halyard Health Sterilization Wrap H400.
2. Start the STERRAD Low Temperature Sterilization System using the STANDARD Cycle according to the instructions provided by the manufacturer.
3. Keep the sterilization wrap together with other sterilized surgical instruments in a sterile item storage area.
4. Before next use, check whether the probe has defects such as peeling, rifts, bumps, cracks, or liquid spill. If such defects exist, the probe has reached the end of its service life. In this case, stop using it and contact the Mindray service department.

5.3.3 Sterilization using Solution

1. Select an appropriate sterilant to sterilize the probe. For detail, see "5.4.1 Validated Cleaner and Sterilant/Sterilization System".
2. Follow the manufacturer's instructions to prepare and use the sterilant. Prepare a sterilant by using sterile distilled or softened water when necessary.
3. Soak the probe insertion part in the sterilant solution and shake the probe properly to remove bubbles on the surface of the probe. For the probe soaking duration, see the sterilant manufacturer's instructions.

Only the section from distal end to the hard shaft can be immersed in the sterilant solution.

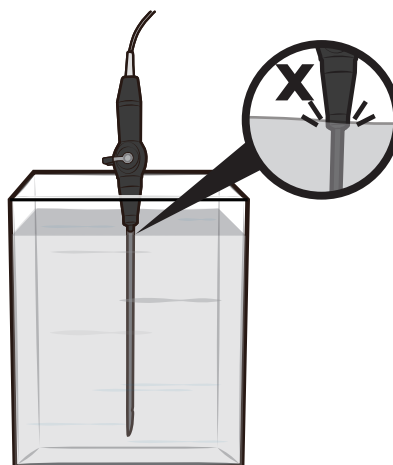

4. Rinse the probe insertion part thoroughly with plenty of sterile distilled water (about 7.5L) at room temperature for about 1 minute to remove the residual sterilant. Or follow the rinsing method specified by the manufacturer.
5. Dry the probe with a piece of disposable sterile lint-free soft cloth.
Do not dry the probe by heating.
6. Check whether the probe has defects such as peeling, rifts, bumps, cracks, or liquid spill. If such defects exist, the probe has reached the end of its service life. In this case, stop using it and contact the Mindray service department.
7. Store the probe in a cool and dry environment.

CAUTION

- **The lens may be discolored; the label on the transducer may fade. These are not abnormalities.**
- **Repeated sterilization will eventually damage the transducer, please check the transducer's performance periodically.**
- **Sterilizing incorrectly or with chemicals not recommended by Mindray will void the warranty.**

5.4 List of Cleaner, Disinfectant and Sterilant/Sterilization System

The manual release time is different from the version update time of the Mindray product, so the list in this manual may not be the latest version. If you cannot find the information in this list, contact Mindray Customer Service Department or sales representative.

5.4.1 Validated Cleaner and Sterilant/Sterilization System

Only the following cleaners, and sterilant/sterilization system are validated by Mindray to clean and sterilize the laparoscope transducers. For the biological effectiveness and the correct use of the sterilant, see the information of the manufacturer.

Type	Item
Cleaner	MetriZyme
	Liquinox
	Prolystica 2X Concentrate Enzymatic Cleaner
	DDN9
Sterilant	Cidex Activated Dialdehyde Solution
Sterilization System	V-PRO Low Temperature Sterilization System
	STERRAD® Low Temperature Sterilization System

5.4.2 Material Compatible Disinfectant

Type	Item
Disinfectant	Anioxyde 1000
	Anios Clean Excel D
	Bodedex Forte
	Cidex OPA TM
	Gigasept AF
	Korsolex Extra
	Perasafe
	Revital-Ox [®] Resert [®] High Level Disinfectant/UltrOx [™] High-Level disinfectant
	Sani Cloth HB
	Sekusept Aktiv
	Tristel Trio
	Virex II 256

NOTE

These disinfectants are only compatible with the probe materials, but the efficacy of realizing the appropriate level of disinfection has not been validated by Mindray.

6 Cleaning and Sterilizing the Needle-guided Bracket

NOTE:

Disposable components are packaged sterile and are single-use only. Do not use if integrity of packaging is violated or if expiration date has passed. Please use the disposable components compliant with the relevant local regulations.

CAUTION

- Needle-guided brackets whose name starts with NGB are reusable, and need thorough cleaning and sterilization before and after each biopsy.
- Follow local regulations when selecting and using the disinfectant.
- Repeated sterilization may degrade the safety and performance of the needle-guided bracket. Before use, please check whether the needle-guided bracket has defects such as deformation and rusting. If such defects exist, the bracket has reached the end of its service life. In this case, stop using it and contact the Mindray service department.
- It is recommended to use immersion sterilization for plastic needle-guided brackets and high-pressure steam sterilization for metal needle-guided brackets.
- For detailed operations about the cleaning solvent, sterilant and high-temperature steam sterilizer, see the respective operator's manuals provided by the manufacturer.

6.1 Cleaning

Perform the following procedure:

1. Wear a pair of gloves to prevent infection.
2. After use, immerse the needle-guided bracket in clean water immediately to prevent dirt from drying. Wipe the entire surface of the needle-guided bracket by using a piece of disposable lint-free soft cloth to remove coarse dirt.
3. Prepare a cleaning solvent (enzymatic or neutral pH detergent, e.g., liquinox, MetriZyme) by using clean water in accordance with the operator's manual provided by the manufacturer.
4. Detach all the detachable parts of the needle-guided bracket and immerse the needle-guided bracket and all its parts fully in the cleaning solvent for at least 1 minute or a period specified by the manufacturer.

5. Immerse the needle-guided bracket and all its parts fully in the cleaning solvent. Wipe and wash the surface and connecting parts of the needle-guided bracket gently by using a soft brush until no dirt is visible. Place the needle-guided bracket inside an ultrasonic cleaner and perform ultrasonic cleaning for 3–5 minutes.
6. Rinse the needle-guided bracket thoroughly by using a large amount of clean water (about 7.5 L/2 gallons) at room temperature for about 1 minute to remove the residual dirt and cleaning solvent.
7. Wipe away the water on the needle-guided bracket by using a piece of disposable lint-free soft cloth.
8. Inspect the needle-guided bracket. If visible dirt still exists, repeat the preceding steps to wash the bracket until it is all clean.

6.2 Sterilization

6.2.1 Sterilization with a Sterilant

Perform the following procedure:

1. Wear a pair of gloves to prevent infection.
2. Clean thoroughly in accordance with the cleaning procedure before sterilization.
3. Prepare a sterilant by using sterile distilled or softened water when necessary.

Table 6-1 Recommended sterilization solution

Chemical name	Trade name	Procedures
Glutaraldehyde (2.4%)	Cidex Activated Dialdehyde Solution (applicable for FDA region only)	Refer to the instructions provided by the solution manufacturer for details.
22% Hydrogen Peroxide 4.5% Peroxyacetic Acid	Minnicare liquid disinfectant (applicable for Canada only)	
Glutaraldehyde (2.6%)	Metricide	

4. Immerse the needle-guided bracket fully in the sterilant and shake the bracket appropriately to remove any bubbles on the surface. Use a syringe to draw an appropriate amount of sterilant and inject the sterilant into the hole to remove the bubbles inside the hole if necessary.

For details about the immersion duration, see the operator’s manual provided by the manufacturer.

5. After sterilization, wash the needle-guided bracket thoroughly by using a large amount of sterile distilled water (about 7.5 L/2 gallons) at room temperature for about 1 minute to remove the residual sterilant.
6. Dry the needle-guided bracket with a piece of sterile disposable lint-free soft cloth.
7. Store the needle-guided bracket in a cool, clean and dry environment.

6.2.2 High-Pressure Steam Sterilization

Perform the following procedure:

1. Wear a pair of gloves to prevent infection.
2. Clean thoroughly in accordance with the cleaning procedure before sterilization.

3. Package the needle-guided bracket in accordance with the sterilization requirements of surgical instruments.
4. Place the packaged needle-guided bracket inside a high-temperature steam sterilizer and perform sterilization. The sterilization parameters are 121 °C and 30 minutes for a gravity displacement steam sterilizer and are 132 °C and 4 minutes for a dynamic-air-removal steam sterilizer.
5. Take out the sterilization package after sterilization and dry it in an oven at 60 °C for 20 minutes to 30 minutes.

Keep the sterilization package together with other sterilized surgical instruments in a sterile item storage area.

This page intentionally left blank.

7 Cleaning and Disinfecting the Main Unit

7.1 Cleaning

WARNING

Before cleaning the main unit, be sure to turn off the power and disconnect the power cord from the outlet. If you clean the system while the power is “On”, it may result in electric shock.

Especially, ME series/TE series/Hepatus series ultrasound system have passed puncture test and leakage current test, and thus can be cleaned in power-on status.

CAUTION




Do not spill water or other liquid into the system while you perform the cleaning. Otherwise it may result in malfunction or electric shock.

NOTE:

- Do not use chemical solvents, or acid or alkaline solution to clean the main unit.
- Do not use hydrocarbon glass cleaner or cleaner for OA (Office Automation) equipment to clean the monitor. These substances may cause deterioration of the monitor.
- Use a soft brush to brush away dust attached to all visible sockets or interfaces (such as probe sockets, sockets or interfaces in the IO panel and power supply panel). Do not use a cloth and water.
- Keyboard on the control panel should be cleaned periodically, otherwise, keys may be blocked by dirt and buzzer dings, and thus keys don't work.

7.1.1 Cleaning the Display

Perform the following steps to clean the display of ultrasound system (including monitor and touch screen).

1. Power off and unplug the system.
 - For TE series/Hepatus series ultrasound system, you can clean the monitor in power-on status: tap  in the top-right corner of the screen and select  to lock the system for 10 seconds while the power is “On”.
 - For ME series ultrasound system, you can clean the touch screen in power-on status: tap > bar in the left side of the touch screen to enter exam main menu, and select  to lock the system for 10 seconds while the power is “On”.
2. Wear medical gloves.

3. Clean the display surface with a soft dry cloth. Remaining stains should be wiped away using a cloth with a little neutral detergent or clean water and then leave the display to air dry.


Do not dry the display by heating.

For disinfection information, see "7.2 Disinfection".

7.1.2 Cleaning the Control Panel

Perform the following steps to clean the control panel.

1. Power off and unplug the system.

For ME series ultrasound system, you can clean the control panel in power-on status: tap > bar in the left side of the touch screen to enter exam main menu, and select  to lock the system for 10 seconds while the power is "On".

2. Wear medical gloves.
3. Wipe the surface of control panel (including keys, buttons and sliders) with dry and soft cloth. Or wipe off difficult-to-remove soils by using soft cloth dampened with a bit of mild soapy water, and wipe the surface with dry and soft cloth or allow it to air dry. If it is difficult to clean the operation panel, remove the encoder caps, and then clean it with mild soapy water.
4. Allow the control panel to air dry.



Do not dry the control panel by heating.

For disinfection information, see "7.2 Disinfection".

7.1.3 Cleaning the Cover

Perform the following steps to clean the cover.

1. Power off and unplug the system.

TE series/Hepatus series ultrasound system has passed puncture test and leakage current test, and thus can be cleaned in power-on status. In power-on status, tap  in the top-right corner of the screen and select  to lock the system for 10 seconds.

2. Wear medical gloves.
3. Use a soft dry cloth to clean the system's cover. If the system is dirty, moisten the soft cloth with neutral soapy water, wipe away any stains.
4. Allow the cover to air dry.

Do not dry the cover by heating.

For disinfection information, see "7.2 Disinfection".

7.1.4 Cleaning Other Parts

For details on cleaning other parts of the main unit, such as trackball and dust-proof cover, refer to the basic volume of ultrasound system operator's manual.

7.2 Disinfection

WARNING

- Before disinfecting the main unit, be sure to turn off the power and disconnect the power cord from the outlet. Disinfecting the main unit while the power is “On” may result in electric shock.

Especially, ME series ultrasound system equipped with trackpad, TE series and Hepatus series ultrasound system have passed puncture test and leakage current test, and thus can be disinfected in power-on status.

- Use only Mindray approved disinfectants and methods listed in this section to disinfect the main unit. Warranty does not cover damage caused by unapproved substances or methods.
- Do not mix disinfectants, as hazardous gases may result.
- We make no claims regarding the efficacy of the listed chemicals or methods as a means for controlling infection. For the method to control infection, consult your hospital’s infection control officer or epidemiologist.

CAUTION

- Never immerse any part of the main unit in liquids or allow liquid to enter the interior.
- Any contact of disinfectants with connectors or metal parts may cause corrosion.
- Do not pour or spray any liquid directly on the main unit or permit fluid to seep into connections or openings. If you spill liquid on the main unit, disconnect the power supply, dry the main unit, and contact your service personnel.
- Never use abrasive materials (such as steel wool or silver polish), or erosive cleaners (such as acetone or acetone-based cleaners).
- Dilute and use the disinfectants according to the manufacturer’s instructions.
- Check the system after cleaning and disinfection. If there is any sign of damage, remove it from use.

NOTE:

- Disinfectants listed in this section are used for disinfecting the housing of the main unit and the monitor only, not for disinfecting the probes.
- Wear medical gloves to prevent transfer of soils and infection.
- Follow local regulations when selecting and using the disinfectant.

Different types of ultrasound systems (such as cart-based, hand-carried and touch screen) vary in materials of cover, display and control panel, and thus their resistance to disinfectants are different. Therefore, different ultrasound systems are compatible with different disinfectants.

7.2.1 Identifying the Parts to Disinfect

For TE series/Hepatus series ultrasound system

The ultrasound system supports disinfection of the main unit cover and monitor. The areas highlighted in gray color of the ultrasound system as shown in Figure 7-2 and Figure 7-3 can be disinfected.

Do not disinfect handle, or any visible sockets or interfaces (such as probe sockets, ventilation holes, dust-proof cover, loudspeaker, sockets or interfaces in the I/O panel and power supply panel). Take TE series (15-inch screen) as an example, see the Figure 7-2.

Figure 7-2 Parts cannot be disinfected for TE series (15-inch screen)



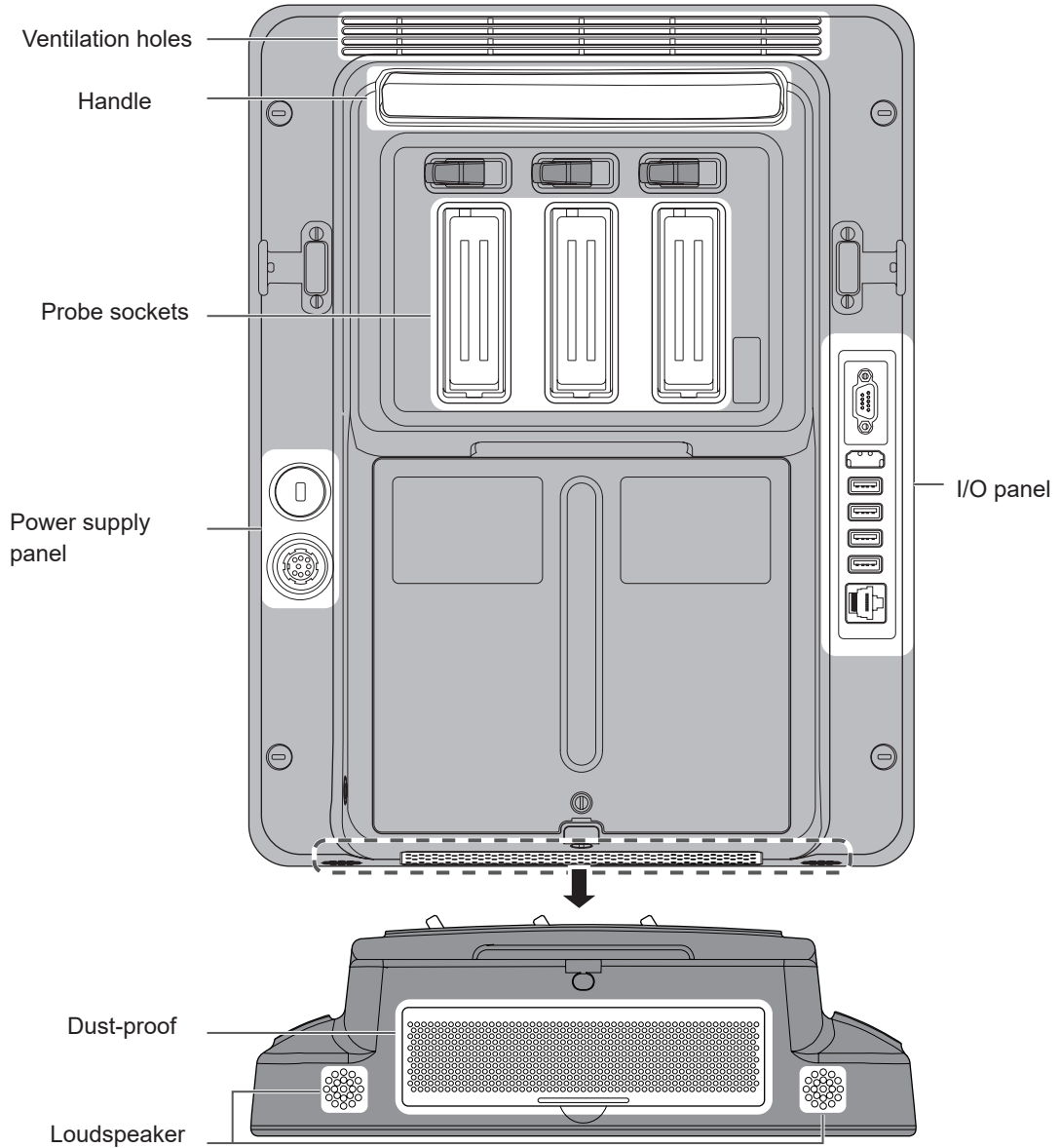


Figure 7-3 Parts cannot be disinfected for TE series (21.5-inch screen)

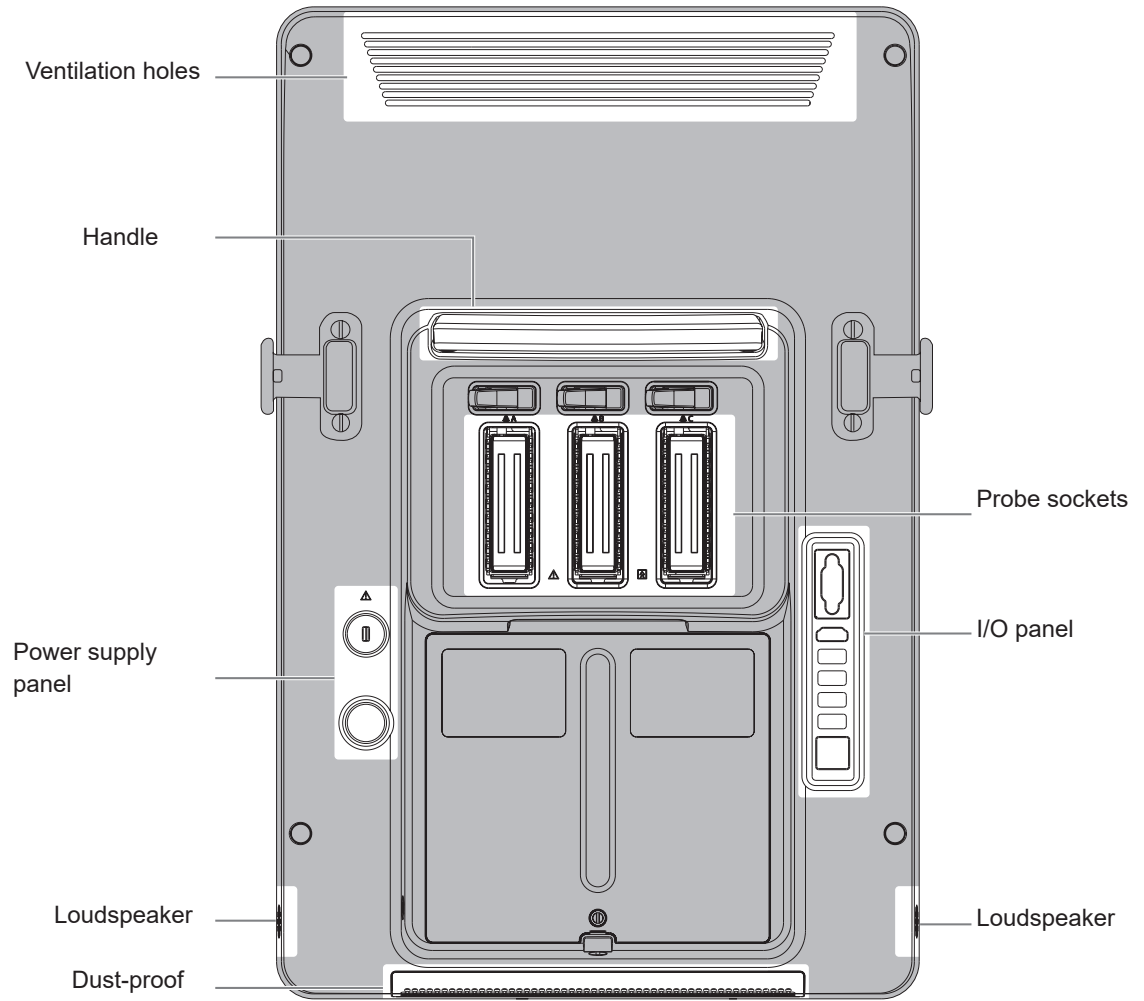

For UMT-400&UMT-400Plus Trolley

Only the trolley handle (UMT-400 and UMT-400Plus) can be disinfected. Other parts (including the height adjusting lever) cannot be disinfected (the diagram of the UMT-400 is taken as an example).

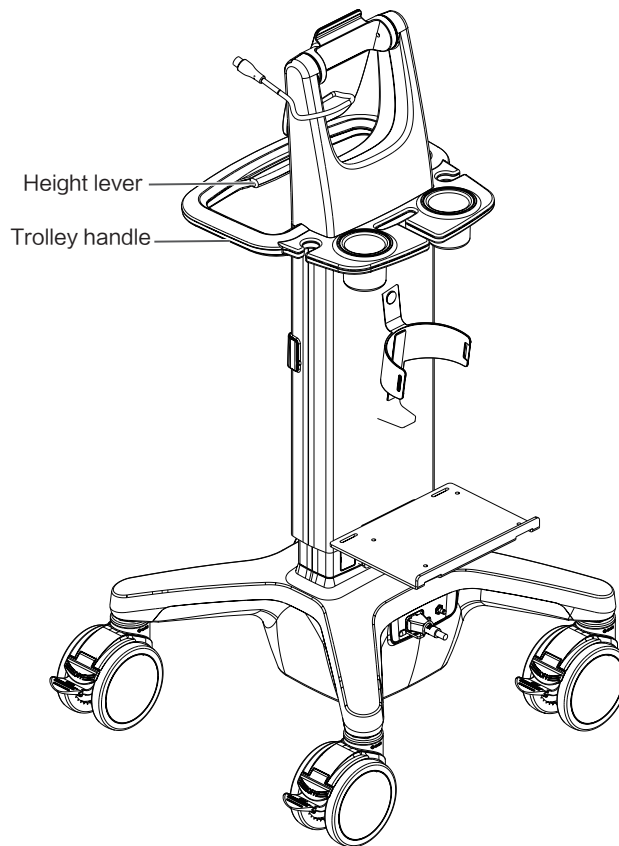


For TEX series ultrasound system

The ultrasound system supports disinfection of the main unit cover and monitor.

The areas highlighted in gray color of the ultrasound system as shown in Figure 7-4 can be disinfected.


Do not disinfect trolley back handle, manufacturer logo, any visible sockets or interfaces (such as probe sockets, ventilation holes, loudspeaker, I/O interface and power supply panel).

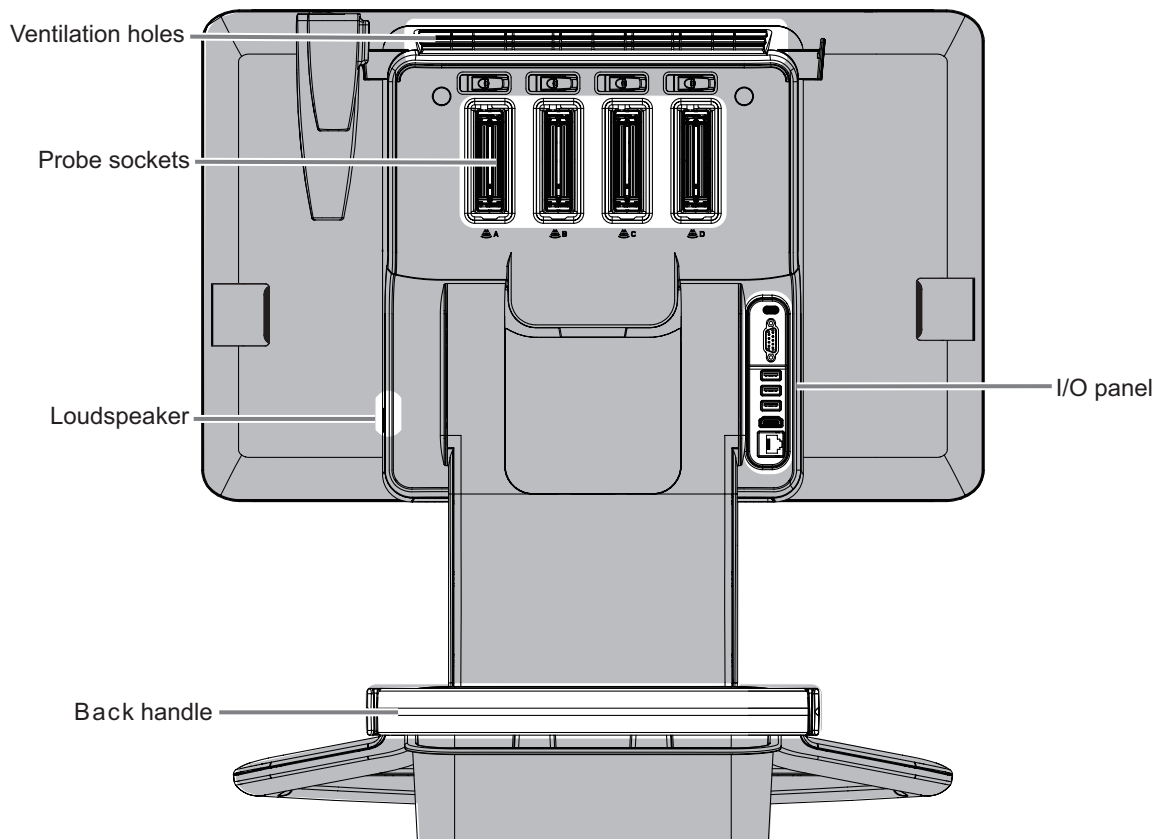Figure 7-4 Parts cannot be disinfected for TEX series**For ME series ultrasound system equipped with trackpad**

The areas highlighted in gray color of the ultrasound system as shown in Figure 6-2 can be disinfected.

Do not disinfect the top cover, bottom cover, side panels around the main unit, manufacturer logo, any visible sockets or interfaces (such as probe socket, ventilation holes, dust-proof cover, loudspeaker, sockets or interfaces in the I/O panel and power supply panel).

Figure 7-5 Parts to be disinfected for ME series equipped with trackpad

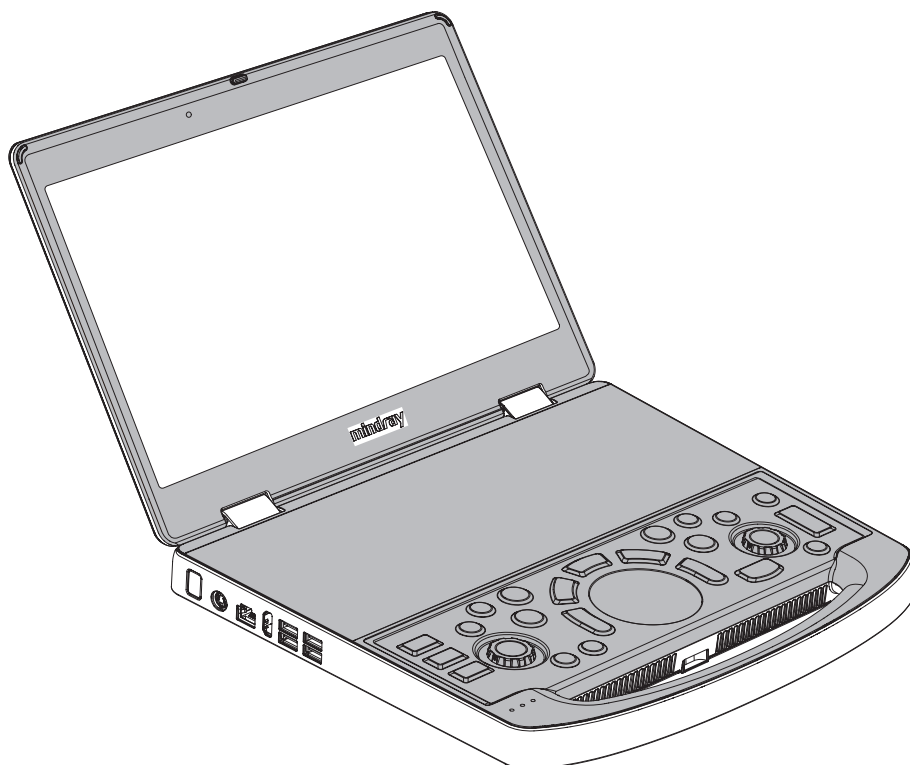

7.2.2 Compatible Disinfectants

Disinfectants compatible for TE series (15-inch screen) and Trolley (UMT-400)

Manufacturer	Trade name	Type
Advanced Ultrasound Solutions Inc.	SONO™ ULTRASOUND WIPES	Wipe
Parker laboratories Inc	Protex Disinfectant Spray	Spray
Professional Disposables International Inc	Sani-Cloth® Plus	Wipe
Bode Chemie Gmbh	Mikrobac Tissues	Wipe
GAMA Healthcare Ltd.	Clinell Universal wipes/Clinell® Surface Wipes	Wipe
Schulke	Schulke mikrozyd® Sensitive Wipes	Wipe
/	0.5% sodium hypochlorite	Solution
/	75% alcohol	Solution
Professional Disposables International Inc	Sani-Cloth® BLEACH	Wipe
Dr.Schumacher GmbH	Cleanisept Wipes	Wipe
Parker Laboratories Inc.	protex ultra disinfectant wipes	Wipe
Laboratoires ANIOS	WIP ANIOS CLEAN UP	Wipe
Laboratoires ANIOS	WIP ANIOS Premium	Wipe

Disinfectants compatible for TE series (21.5-inch screen) and Trolley (UMT-400Plus)

Manufacturer	Product Name	Type
Professional Disposables International Inc	Sani-Cloth® BLEACH	Wipe
	Super Sani-Cloth®	Wipe
	Sani-Cloth® Prime	Spray
	Sani-Cloth® Plus	
Laboratoires ANIOS	WIP ANIOS CLEAN UP	Wipe
/	75% alcohol	Solution
/	70% isopropyl alcohol	Solution
/	5% hydrogen peroxide	Solution
/	0.5% sodium hypochlorite	Solution
Laboratoires ANIOS	WIP ANIOS Premium	Wipe
Bode Chemie Gmbh	Mikrobac Tissues	Wipe
Parker Laboratoires Inc.	Protex Disfectant Spray	Wipe
Advanced Ultrasound Solutions Inc.	SONO™ ULTRASOUND WIPES	Wipe
Schulke	Schulke mikrozyd® Sensitive Wipes	Wipe
GAMA Healthcare Ltd.	Clinell Universal wipes/Clinell® Surface Wipes	Wipe
Parker Laboratories Inc.	protex ultra disinfectant wipes	Wipe
Metrex Research	CAVIWIPES	Wipe
Dr.Schumacher GmbH	Cleanisept Wipes	Wipe

Disinfectants compatible for Hepatus series

Manufacturer	Trade name	Type
Advanced Ultrasound Solutions Inc.	SONO™ ULTRASOUND WIPES	Wipe
Parker laboratories Inc	Protex Disinfectant Spray	Spray
Professional Disposables International Inc	Sani-Cloth® Plus	Wipe
Bode Chemie Gmbh	Mikrobac Tissues	Wipe
GAMA Healthcare Ltd.	Clinell Universal wipes/Clinell® Surface Wipes	Wipe
Professional Disposables International Inc	Sani-Cloth® AF3	Wipe
Diversey	VIREX II 256	Solution
Schulke	Schulke mikrozyd® Sensitive Wipes	Wipe
/	0.5% sodium hypochlorite	Solution
/	75% alcohol	Solution
/	5% hydrogen peroxide	Solution
Diversey	VIREX TB	Solution

Disinfectants compatible for ME series equipped with trackpad

Manufacturer	Trade name	Type
/	75% alcohol	Solution
/	99.5% isopropyl alcohol	Solution
Advanced Ultrasound Solutions Inc.	SONO™ ULTRASOUND WIPES	Wipe
Schulke	Schulke mikrozid® Sensitive Wipes	Wipe
Bode Chemie Gmbh	Mikrobac Tissues	Wipe
PDI	Sani-Cloth® Plus	Wipe
GAMA Healthcare Ltd.	Clinell Universal wipes/Clinell® Surface Wipes	Wipe
Parker Laboratories Inc.	protex ultra disinfectant wipes	Wipe
Metrex Research	CAVIWIPES	Wipe
Dr.Schumacher GmbH	Cleanisept Wipes	Wipe

Disinfectants compatible for TEX series

Manufacturer	Trade name	Type
Advanced Ultrasound Solutions Inc.	SONOTM ULTRASOUND WIPES	Wipe
PDI Inc.	Sani-Cloth® Plus	Wipe
PDI Inc.	Sani-Cloth® BLEACH	Wipe
PDI Inc.	Sani-Cloth® Prime	Wipe
PDI	Super Sani-Cloth	Wipe
Bode Chemie Gmbh	Mikrobac Tissues	Wipe
Diversey	VIREX TB	Solution
Schulke	Schulke mikrozid® Sensitive Wipes	Wipe
Dr.Schumacher GmbH	CLEANISEPT WIPES	Wipe
Parker Laboratories Inc.	PROTEX ULTRA	Wipe
Laboratoires ANIOS	wipe anios clean up	Wipe
/	99.5% isopropyl alcohol	Solution
/	75% alcohol	Solution
/	2.7%~3.3% hydrogen peroxide	Solution
/	0.5% sodium hypochlorite	Solution
Diversey Inc	OXIVIR TB WIPES	Wipe
Metrex Research	CAVIWIPES	Wipe
Dr.Schumacher GmbH	Cleanisept Wipes	Wipe
GAMA Healthcare Ltd	Clinell Universal wipes/Clinell® Surface Wipes	Wipe
Parker Laboratories Inc.	protex ultra disinfectant wipes	Wipe
Professional Disposables International Inc	Sani-Cloth® AF3	Wipe
Professional Disposables International Inc	Sani-Cloth® HB	Wipe
/	VIREX II 256	Solution

7.2.3 Disinfection Procedure

Perform the following steps to disinfect the ultrasound system: TEX series, TE series, Hepatus series and ME series equipped with trackpad.

After cleaning, disinfect the main unit as follows.

- 1.** Wear medical gloves to prevent infection.
- 2.** Disinfect the identified parts that can be disinfected with the compatible disinfectant.
Follow the disinfectant manufacturer's recommended contact time and method.
- 3.** Remove any residue with a water-moistened soft cloth on the main unit.
- 4.** Wipe off water on the main unit using clean cloth or gauze.

Do not dry the main unit by heating.

This page intentionally left blank.

8 Cleaning Other Accessories

8.1 Cleaning the Holders

Perform the following steps to clean probe holders and coupling gel holders.

1. Use dry soft cloth to wipe off the dust attached to inside, outside or gap of probe holder or gel holder. As to small intra-cavity probe holder or its gap, use the soft brush to brush the dust or stain.
2. Remained stain attached to inside, outside of holder should be washed out by cloth with a little soapy water and then allow it to air dry after take it out.
3. Gel warmer: Take the gel warmer out after pull out the power supply cable, use dry soft cloth to wipe off the dust attached to inside and outside, then brush the dust in the gel warmer or brush the stain with a little soap water and allow it to air dry at last.

8.2 Cleaning the ECG Cables

See the accompanying ECG manual for details.

8.3 Cleaning the Peripherals

Do the cleaning maintenance according to your actual peripheral configuration; items which are not configured can be skipped.

Color and B/W video printer

First wipe off dust or stain attached to the cover of printer with soft dry cloth, then clean the inside of printer. Be sure to do the cleaning maintenance according to the operation manual if necessary.

Graph/text printer

First wipe off dust or stain attached to the cover of printer with soft dry cloth, then clean the inside of printer. Be sure to do the cleaning maintenance according to the operation manual if necessary.

Footswitch

Use soft dry cloth with a little mild soap water to wipe off the dust or stain attached to the pedals or cable of foot switch.

Barcode reader

First use soft dry cloth to wipe off dust attached to glass panel of the reader, then the dust or strain attached to cable and bracket. Refer to barcode reader operator's manual for details.

This page intentionally left blank.

Appendix A Compatible Probe Cleaners and Disinfectants

NOTE:

The content in the list is written based on ultrasound systems manufactured by Mindray. The versions of the ultrasound systems will be updated, and the list may be not updated in time. Therefore, if the desired information cannot be found in this list, contact Mindray customer service department or sales representative.

A.1 Cleaners

- a1: MetriZyme
- a2: Tristel Pre-Clean Wipes
- a3: Liquinox
- a4: Revital-OX Enzymatic Detergent
- a5: MetriSponge
- a6: Prolystica 2X Concentrate Enzymatic Cleaner
- a7: Endozime and Endozime Sponge
- a8: klenzyme
- a9: ANIOSYME 5
- a10: DDN9

	Probe	Cleaners
Convex	C5-1s/C5-1E/C5-1U/C5-1/C5-2/C5-2s/C5-2E/C6-2E/C6-2/C6-2s/C6-2P/C11-3E/C11-3s/C11-3U/C11-3	a1, a2, a3, a4, a5, a6, a7, a9, a10
	C6-2Gs/C6-2GE/C6-2GU/SC7-1U/SC9-2U/SL10-3U/SCM7-1U/SLM10-3U/SC6-1GU/C6-1	a1, a2, a3, a4, a5, a6, a7, a9, a10
	C7-3E/3C1/3C1s/3C1P/6C2/6C2P/6C2s/35C20EA/35C50EA/35C50EB/35C50P/65C15EAV/65C15EA	a1, a2, a3, a4, a5, a6, a7, a9, a10
	3C5/3C5s/3C5A/3C5P	a1, a2, a3, a4, a5, a6, a7, a9, a10
	SC5-1U/SC5-1E	a1, a2, a3, a4, a5, a6, a7
	SC6-1U/SC6-1E/SC6-1s/SC5-1NE/SC5-1Ns/SC5-1N	a1, a2, a3, a4, a5, a6, a7, a9, a10
	SC8-2U/SC8-2E/SC8-2s	a1, a8
	65EC10EA/65EC10EB/65EC10EC	a1, a2, a3, a4, a5, a6, a7, a9
	65EC10ED	a1, a2, a3, a4, a5, a6, a7, a9, a10
	C4-1/C4-1U/C4-1s	a1, a3, a4, a5, a6, a7
	C9-3Ts	a1, a3, a4, a5, a6, a7

- a1: MetriZyme
- a2: Tristel Pre-Clean Wipes
- a3: Liquinox
- a4: Revital-OX Enzymatic Detergent
- a5: MetriSponge
- a6: Prolystica 2X Concentrate Enzymatic Cleaner
- a7: Endozime and Endozime Sponge
- a8: klenzyme
- a9: ANIOSYME 5
- a10: DDN9

Probe		Cleaners
Linear	L7-3/L7-3s/L7-3E/7L4/7L4s/7L4A/7L4P/7L4B/7L4Bs/ 7L4BP/7L5/7L5P/7L5s/7L6/7L6s/75L38EA/ 75L38EB/75L53EA/75L60EA/75L38P/10L4/10L4s	a1, a2, a3, a4, a5, a6, a7, a9, a10
	65EL60EA/6LE7/6LE7s/6LE7P/6LE5V/6LE5Vs/6LE5VP/ 7LT4/7LT4s/7LT4P/7LT4E/50L60EAV/65L50HAV/ 75L50EAV/75LT38EA	a1, a2, a3, a4, a5, a6, a7, a9
	L9-3/L9-3U/L9-3E/L9-3s/L12-3/L12- 3E/L13-3s/L13-3/L12-4/ L12-4s/L14-6N/L14-6Ns/L14-6NP/L14-6NE/L14-6WE/ L14-6Ws/L14-6WU	a1, a2, a3, a4, a5, a6, a7, a9, a10
	L10-3E/L10-3s	a1
	L11-3U/L11-3VNs/L13-3N/L13-3Ns/L14-3s/ L14-3W/L14-3WU/L14-3Ws/L14-3WE	a1, a2, a3, a4, a5, a6, a7, a9, a10
	L11-4/L11-4s/L14-6/L14-6s/L14-6P/10L24EA	a1, a2, a3, a4, a5, a6, a7, a9, a10
	L16-4HE/L16-4Hs/L16-4HU	a8
	LM14-6E/LM14-6s/LM16-4U	a1, a2, a3, a4, a5, a6, a7, a9, a10
	L20-5U/L20-5s/L20-5E	a1,a3, a4, a5, a6, a7, a9, a10
	L12-3RCs/L12-3VNs	a9
	L13-3WE/L13-3WU/L13-3Ws/L15-3WE/L15-3WU/ L15-3Ws/L18-5WU/LM18-5WU/LM24-6WU	a9, a10
	L14-5sp	a1, a3, a4, a5, a6, a7
	L14-5WU/L14-5WE/L14-5Ws	a1, a3, a4, a5, a6, a7
	L30-8U/L33-8U/L33-8s	a8
Phased	P4-2/P4-2s/P4-2E/P4-2NE/P4-2Ns/P10-4E/P10-4s/P10-4/ P10-4U/2P2/2P2s/2P2P	a1, a2, a3, a4, a5, a6, a7, a9, a10
	P7-3/P7-3s/P7-3E/P7-3U/P7-3P	a1, a8
	P12-4/P12-4s	a1
	SP5-1U/SP5-1s/SP5-1E/SP5-1	a1, a2, a3, a4, a5, a6, a7, a9, a10
	SP5-1Ns/SP5-1N	a1, a2, a3, a4, a5, a6, a7, a9, a10
	LFP5-1s/LFP5-1U/LFC5-1s/LFC5-1U	a10
	P8-2s/P8-2/P8-2U/P8-2P/SP9-2U/SPM6-1U/XP5-1U	a1, a2, a3, a4, a5, a6, a7, a9, a10
Endo- cavity	V11-3/V11-3BE/V11-3B/V11-3WE/V11-3Ws/V11-3E/ V11-3s	a1, a2, a3, a4, a5, a6, a7, a9, a10
	V11-3HU/V11-3HE/V11-3Hs/V11-3H/V11- 3HB/V11-3HBs/V11-3HBE/SV10-2U	a1, a2, a3, a4, a5, a6, a7, a9, a10
	V10-4/V10-4s/V10-4B/V10-4Bs/V10-4BP/ 6CV1/6CV1s/6CV1P	a1, a2, a3, a4, a5, a6, a7, a9, a10
Biplane	CB10-4/CB10-4P/CB10-4A/CB10-4E/CB10- 4s/6LB7/6LB7s/6LB7P/6LB7E/65EB10EA	a1, a2, a3, a4, a5, a6, a7, a9
	ELC13-4U/ELC13-4s/ELC10-4	a9

- a1: MetriZyme
- a2: Tristel Pre-Clean Wipes
- a3: Liquinox
- a4: Revital-OX Enzymatic Detergent
- a5: MetriSponge
- a6: Prolystica 2X Concentrate Enzymatic Cleaner
- a7: Endozime and Endozime Sponge
- a8: klenzyme
- a9: ANIOSYME 5
- a10: DDN9

Probe		Cleaners
4D	D7-2/D7-2E/D7-2s/SD8-1/SD8-1E/SD8-1s/SD8-1U/ D6-2/D6-2P/D6-2EA/D6-2B/D6-2A	a1, a2, a3, a4, a5, a6, a7, a9
	D6-2NE/DL14-3U	a1
	DE11-3U/DE11-3s/DE11-3E/DE11-3/DE11-3WE/ DE11-3Ws/DE11-3WU	a1, a2, a3, a4, a5, a6, a7
	DE10-3WU/DE10-3WE (only for the probe with gray strain relief)	a8

A.2 Disinfectants

A.2.1 Wipes

b1: CLEANISEPT® WIPES	b18: Accel TB Wipes
b2: mikrozyd® AF Wipes Jumbo	b19: CaviWipes 1
b3: PROTEX™ DISINFECTANT Wipes	b20: Tuffle 5
b4: Sani-Cloth® Plus	b21: Sani-Cloth Active
b5: SONO™ ULTRASOUND WIPES	b22: Septiwipes
b7: Tristel Sporicidal Wipes	b23: Mikorbac Tissues
b8: Tristel Rinse Wipes	b24: Sani-Cloth Germicidal Wipes
b9: Clinell Universal wipes/Clinell® Surface Wipes	b25: WIP'ANIOS CLEAN'UP
b10: mikrozyd® Sensitive Wipes	b26: CaviWipes XL
b11: Wip' Anios premium	b27: OXIVIR™ WIPES
b12: ultrasound probe cleaning wipes	b28: Oxivir 1 Wipes
b13: Sani-Cloth AF3 (gray)	b29: PDI SANI-CLOTH BLEACH WIPES
b14: Protex Ultra Wipes	b30: wip anios excel
b15: Sani-Cloth HB	b31: Virusolve® + Pronte all'uso Wipes
b16: CaviWipes	b32: Super Sani-Cloth®
b17: Dispatch Towels	b33: SANI-CLOTH® PRIME WIPES

	Probe	Wipes
Convex	C5-1s/C5-1E/C5-1U/C5-1/C5-2/C5-2s/C5-2E/C6-2E/ C6-2/C6-2s/C6-2P/C11-3E/C11-3s/C11-3U/C11-3	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b32, b33
	C6-2Gs/C6-2GE/C6-2GU/SC7-1U/SC9-2U/SL10-3U/SCM7-1U/SLM10-3U/SC6-1GU/C6-1	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	C7-3E/3C1/3C1s/3C1P/6C2/6C2P/6C2s/35C20EA/ 35C50EA/35C50EB/35C50P/65C15EAV/65C15EA	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	3C5/3C5s/3C5A/3C5P	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	SC5-1U/SC5-1E	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b16, b18, b19, b24, b25, b26, b27, b28, b29
	SC6-1U/SC6-1E/SC6-1s/SC5-1NE/SC5-1Ns/SC5-1N	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	SC8-2U/SC8-2E/SC8-2s	b15, b25
	65EC10EA/65EC10EB/65EC10EC	b1, b2, b3, b4, b5, b7, b8, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31
	65EC10ED	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	C4-1/C4-1U/C4-1s	b4, b5, b13, b14, b15, b16, b17, b18, b26, b32, b33

- | | |
|--|--|
| b1: CLEANISEPT® WIPES | b18: Accel TB Wipes |
| b2: mikroqid® AF Wipes Jumbo | b19: CaviWipes 1 |
| b3: PROTEX™ DISINFECTANT Wipes | b20: Tuffle 5 |
| b4: Sani-Cloth® Plus | b21: Sani-Cloth Active |
| b5: SONO™ ULTRASOUND WIPES | b22: Septiwipes |
| b7: Tristel Sporicidal Wipes | b23: Mikorbac Tissues |
| b8: Tristel Rinse Wipes | b24: Sani-Cloth Germicidal Wipes |
| b9: Clinell Universal wipes/Clinell® Surface Wipes | b25: WIP'ANIOS CLEAN'UP |
| b10: mikroqid® Sensitive Wipes | b26: CaviWipes XL |
| b11: Wip' Anios premium | b27: OXIVIR™ WIPES |
| b12: ultrasound probe cleaning wipes | b28: Oxivir 1 Wipes |
| b13: Sani-Cloth AF3 (gray) | b29: PDI SANI-CLOTH BLEACH WIPES |
| b14: Protex Ultra Wipes | b30: wip anios excel |
| b15: Sani-Cloth HB | b31: Virusolve® + Pronte all'uso Wipes |
| b16: CaviWipes | b32: Super Sani-Cloth® |
| b17: Dispatch Towels | b33: SANI-CLOTH® PRIME WIPES |

Probe		Wipes
Convex	C9-3Ts	b16, b26
Linear	L7-3/L7-3s/L7-3E/7L4/7L4s/7L4A/7L4P/7L4B/7L4Bs/ 7L4BP/7L5/7L5P/7L5s/7L6/7L6s/75L38EA/ 75L38EB/75L53EA/75L60EA/75L38P/10L4/10L4s	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	65EL60EA/6LE7/6LE7s/6LE7P/6LE5V/6LE5Vs/ 6LE5VP/7LT4/7LT4s/7LT4P/7LT4E/50L60EAV/ 65L50HAV/75L50EAV/75LT38EA	b1, b2, b3, b4, b5, b7, b8, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31
	L9-3/L9-3U/L9-3E/L9-3s/L12-3/L12- 3E/L13-3s/L13-3/L12-4/ L12-4s/L14-6N/L14-6Ns/L14-6NP/L14-6NE/L14-6WE/ L14-6Ws/L14-6WU	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b32, b33
	L10-3E/L10-3s	b25
	L11-3U/L11-3VNs/L13-3N/L13-3Ns/L14-3s/ L14-3W/L14-3WU/L14-3Ws/L14-3WE	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	L11-4/L11-4s/L14-6/L14-6s/L14-6P/10L24EA	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	LM14-6E/LM14-6s/LM16-4U	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	L20-5U/L20-5s/L20-5E	b4, b5, b13, b14, b15, b16, b17, b18, b25, b26, b31
	L12-3RCs/L12-3VNs	b1, b4, b5, b11, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	L13-3WE/L13-3WU/L13-3Ws/L15-3WE/L15-3WU/ L15-3Ws/L18-5WU/LM18-5WU/LM24-6WU	b13, b25, b29, b30, b31, b32, b33
	L14-5sp	b4, b5, b13, b14, b15, b16, b17, b18, b26, b32, b33
	L14-5WU/L14-5WE/L14-5Ws	b4, b5, b13, b14, b15, b16, b17, b18, b26, b32, b33

b1: CLEANISEPT® WIPES	b18: Accel TB Wipes
b2: mikroqid® AF Wipes Jumbo	b19: CaviWipes 1
b3: PROTEX™ DISINFECTANT Wipes	b20: Tuffle 5
b4: Sani-Cloth® Plus	b21: Sani-Cloth Active
b5: SONO™ ULTRASOUND WIPES	b22: Septiwipes
b7: Tristel Sporicidal Wipes	b23: Mikorbac Tissues
b8: Tristel Rinse Wipes	b24: Sani-Cloth Germicidal Wipes
b9: Clinell Universal wipes/Clinell® Surface Wipes	b25: WIP'ANIOS CLEAN'UP
b10: mikroqid® Sensitive Wipes	b26: CaviWipes XL
b11: Wip' Anios premium	b27: OXIVIR™ WIPES
b12: ultrasound probe cleaning wipes	b28: Oxivir 1 Wipes
b13: Sani-Cloth AF3 (gray)	b29: PDI SANI-CLOTH BLEACH WIPES
b14: Protex Ultra Wipes	b30: wip anios excel
b15: Sani-Cloth HB	b31: Virusolve® + Pronte all'uso Wipes
b16: CaviWipes	b32: Super Sani-Cloth®
b17: Dispatch Towels	b33: SANI-CLOTH® PRIME WIPES

	Probe	Wipes
Phased	P4-2/P4-2s/P4-2E/P4-2NE/P4-2Ns/P10-4E/ P10-4s/P10-4/P10-4U/2P2/2P2s/2P2P	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	P7-3/P7-3s/P7-3E/P7-3U/P7-3P	b3, b4, b15, b25, b29
	P12-4/P12-4s	b5, b25, b29
	SP5-1U/SP5-1s/SP5-1E/SP5-1	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	SP5-1Ns/SP5-1N	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b32, b33
	LFP5-1s/LFP5-1U/LFC5-1s/LFC5-1U	b1, b4, b5, b11, b13, b16, b18, b19, b24, b26, b27, b28, b32, b33
	P8-2s/P8-2/P8-2U/P8-2P/SP9-2U/SPM6-1U/XP5-1U	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	i3P/i3PA/a3P-w	b29
Endo-cavity	V11-3/V11-3BE/V11-3B/V11-3WE/ V11-3Ws/V11-3E/V11-3s	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b32, b33
	V11-3HU/V11-3HE/V11-3Hs/V11-3H/V11- 3HB/V11-3HBs/V11-3HBE/SV10-2U	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	V10-4/V10-4s/V10-4B/V10-4Bs/V10-4BP/ 6CV1/6CV1s/6CV1P	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33

- | | |
|--|--|
| b1: CLEANISEPT® WIPES | b18: Accel TB Wipes |
| b2: mikroqid® AF Wipes Jumbo | b19: CaviWipes 1 |
| b3: PROTEX™ DISINFECTANT Wipes | b20: Tuffle 5 |
| b4: Sani-Cloth® Plus | b21: Sani-Cloth Active |
| b5: SONO™ ULTRASOUND WIPES | b22: Septiwipes |
| b7: Tristel Sporicidal Wipes | b23: Mikorbac Tissues |
| b8: Tristel Rinse Wipes | b24: Sani-Cloth Germicidal Wipes |
| b9: Clinell Universal wipes/Clinell® Surface Wipes | b25: WIP'ANIOS CLEAN'UP |
| b10: mikroqid® Sensitive Wipes | b26: CaviWipes XL |
| b11: Wip' Anios premium | b27: OXIVIR™ WIPES |
| b12: ultrasound probe cleaning wipes | b28: Oxivir 1 Wipes |
| b13: Sani-Cloth AF3 (gray) | b29: PDI SANI-CLOTH BLEACH WIPES |
| b14: Protex Ultra Wipes | b30: wip anios excel |
| b15: Sani-Cloth HB | b31: Virusolve® + Pronte all'uso Wipes |
| b16: CaviWipes | b32: Super Sani-Cloth® |
| b17: Dispatch Towels | b33: SANI-CLOTH® PRIME WIPES |

Probe		Wipes
Biplane	CB10-4/CB10-4P/CB10-4A/CB10-4E/CB10-4s/6LB7/6LB7s/6LB7P/6LB7E/65EB10EA	b1, b2, b3, b4, b5, b7, b8, b10, b11, b12, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31
	ELC13-4U/ELC13-4s/ELC10-4	b1, b4, b5, b11, b16, b18, b19, b24, b26, b27, b28, b30, b31
4D	D7-2/D7-2E/D7-2s/SD8-1/SD8-1E/SD8-1s/SD8-1U/D6-2/D6-2P/D6-2EA/D6-2B/D6-2A	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b29, b30, b31, b32, b33
	D8-2E/D8-2U	b10, b16, b19, b20, b21
	D8-4U	b4, b10, b16, b19, b20
	DE10-3/DE10-3E/DE10-3U/DE10-3s	b10, b16, b19, b20, b21, b32, b33
	DE11-3U/DE11-3s/DE11-3E/DE11-3/DE11-3WE/DE11-3Ws/DE11-3WU	b1, b2, b3, b4, b5, b7, b8, b9, b10, b11, b12, b13, b16, b18, b19, b24, b25, b26, b27, b28, b32, b33
	DE10-3WU/DE10-3WE(only for the probe with gray strain relief)	b1, b12, b15, b16, b20, b21, b22, b23, b24

A.2.2 Sprays

c1: Oxivir ^{TM/MC} Tb	c9: Protex Spray
c2: PI-SPRAY II	c10: CaviCide
c3: Surfa'safe	c11: T-Spray
c4: TRANSEPTIC	c12: Indican Form
c5: PROTEX TM DISINFECTANT SPRAY	c13: Transeptic Spray
c7: Tristel Duo	c14: CAVICIDE 1
c8: IODOCLEAN	c15: OXIVIR 1
c9: Protex Spray	c16: WIP'ANIOS SPOR'ACTIV
	c17: Accel TB Liquid
	c18: Sani-HyPerCide GERMICIDAL SPRAY
	c19: Sani-24 GERMICIDAL SPRAY

Probe		Spray
Convex	C5-1s/C5-1E/C5-1U/C5-1/C5-2/C5-2s/C5-2E/C6-2E/ C6-2/C6-2s/C6-2P/C11-3E/C11-3s/C11-3U/C11-3	c1, c2, c3, c5, c7, c8, c14, c15, c16, c17
	C6-2Gs/C6-2GE/C6-2GU/SC7-1U/SC9-2U/SL10- 3U/SCM7-1U/SLM10-3U/SC6-1GU/C6-1	c1, c2, c3, c5, c7, c8, c14, c15, c16, c17
	C7-3E/3C1/3C1s/3C1P/6C2/6C2P/6C2s/35C20EA/ 35C50EA/35C50EB/35C50P/65C15EAV/65C15EA	c1, c2, c3, c4, c5, c7, c8, c14, c15, c16, c17
	3C5/3C5s/3C5A/3C5P	c1, c2, c3, c4, c5, c7, c8, c14, c15, c16, c17
	SC5-1U/SC5-1E	c1, c2, c3, c5, c7, c14, c15, c16, c17
	SC6-1U/SC6-1E/SC6-1s/SC5-1NE/SC5-1Ns/SC5-1N	c1, c2, c3, c5, c7, c8, c14, c15, c16, c17
	SC8-2U/SC8-2E/SC8-2s	c4, c11, c16
	65EC10EA/65EC10EB/65EC10EC	c1, c2, c3, c4, c5, c7, c14, c15, c16, c17
	65EC10ED	c1, c2, c3, c4, c5, c7, c8, c14, c15, c16, c17
	C4-1/C4-1U/C4-1s	c2, c9

- | | |
|-----------------------------------|--------------------------------------|
| c1: Oxivir™/MC Tb | c9: Protex Spray |
| c2: PI-SPRAY II | c10: CaviCide |
| c3: Surfa'safe | c11: T-Spray |
| c4: TRANSEPTIC | c12: Indican Form |
| c5: PROTEX™
DISINFECTANT SPRAY | c13: Transeptic Spray |
| c7: Tristel Duo | c14: CAVICIDE 1 |
| c8: IODOCLEAN | c15: OXIVIR 1 |
| c9: Protex Spray | c16: WIP'ANIOS SPOR'ACTIV |
| | c17: Accel TB Liquid |
| | c18: Sani-HyPerCide GERMICIDAL SPRAY |
| | c19: Sani-24 GERMICIDAL SPRAY |

Probe		Spray
Linear	L7-3/L7-3s/L7-3E/7L4/7L4s/7L4A/7L4P/7L4B/7L4Bs/ 7L4BP/7L5/7L5P/7L5s/7L6/7L6s/75L38EA/75L38EB/ 75L53EA/75L60EA/75L38P/10L4/10L4s	c1, c2, c3, c4, c5, c7, c8, c14, c15, c16, c17
	65EL60EA/6LE7/6LE7s/6LE7P/6LE5V/6LE5Vs/ 6LE5VP/7LT4/7LT4s/7LT4P/7LT4E/50L60EAV/ 65L50HAV/75L50EAV/75LT38EA	c1, c2, c3, c4, c5, c7, c14, c15, c16, c17
	L9-3/L9-3U/L9-3E/L9-3s/L12-3/L12-3E/L13-3s/ L13-3/L12-4/L12-4s/L14-6N/L14-6Ns/ L14-6NP/L14-6NE/L14-6WE/L14-6Ws/L14-6WU	c1, c2, c3, c5, c7, c8, c14, c15, c16, c17
	L10-3E/L10-3s	c5
	L11-3U/L11-3VNs/L13-3N/L13-3Ns/L14-3s/ L14-3W/L14-3WU/L14-3Ws/L14-3WE	c1, c2, c3, c5, c7, c8, c14, c15, c16, c17
	L11-4/L11-4s/L14-6/L14-6s/L14-6P/10L24EA	c1, c2, c3, c5, c7, c8, c14, c15, c16, c17
	LM14-6E/LM14-6s/LM16-4U	c1, c2, c3, c4, c5, c7, c8, c14, c15, c16, c17
	L20-5U/L20-5s/L20-5E	c2, c9
	L12-3RCs/L12-3VNs	c1, c3, c14, c15, c16, c17
	L13-3WE/L13-3WU/L13-3Ws/L15-3WE/L15-3WU/ L15-3Ws/L18-5WU/LM18-5WU/LM24-6WU	c7, c16
	L14-5sp	c2, c9
	L14-5WU/L14-5WE/L14-5Ws	c2, c9, c11
L9-3PAU	c18, c19	
Phased	P4-2/P4-2s/P4-2E/P4-2NE/P4-2Ns/P10-4E/ P10-4s/P10-4P/P10-4U/2P2/2P2s/2P2P	c1, c2, c3, c4, c5, c7, c8, c14, c15, c16, c17
	P7-3/P7-3s/P7-3E/P7-3U/P7-3P	c2, c4, c5, c16
	P12-4/P12-4s	c1, c2, c5, c16
	SP5-1U/SP5-1s/SP5-1E/SP5-1	c1, c2, c3, c5, c7, c8, c14, c15, c16, c17
	SP5-1Ns/SP5-1N	c1, c2, c3, c5, c7, c8, c14, c15, c16, c17
	LFP5-1s/LFP5-1U/LFC5-1s/LFC5-1U	c1, c3, c14, c15, c17
	P8-2s/P8-2/P8-2U/P8-2P/SP9-2U/SPM6-1U/XP5-1U	c1, c2, c3, c5, c7, c8, c14, c15, c16, c17
	i3P/i3PA/a3P-w	c19

c1: Oxivir™ ^{MC} Tb	c9: Protex Spray
c2: PI-SPRAY II	c10: CaviCide
c3: Surfa'safe	c11: T-Spray
c4: TRANSEPTIC	c12: Indican Form
c5: PROTEX™ DISINFECTANT SPRAY	c13: Transeptic Spray
c7: Tristel Duo	c14: CAVICIDE 1
c8: IODOCLEAN	c15: OXIVIR 1
c9: Protex Spray	c16: WIP'ANIOS SPOR'ACTIV
	c17: Accel TB Liquid
	c18: Sani-HyPerCide GERMICIDAL SPRAY
	c19: Sani-24 GERMICIDAL SPRAY

Probe		Spray
Endo-cavity	V11-3/V11-3BE/V11-3B/V11-3WE/V11-3Ws/V11-3E/V11-3s	c1, c2, c3, c4, c5, c7, c8, c14, c15, c16, c17
	V11-3HU/V11-3HE/V11-3Hs/V11-3H/V11-3HB/V11-3HBs/V11-3HBE/SV10-2U	c1, c2, c3, c5, c7, c8, c14, c15, c16, c17
	V10-4/V10-4s/V10-4B/V10-4Bs/V10-4BP/6CV1/6CV1s/6CV1P	c1, c2, c3, c4, c5, c7, c8, c14, c15, c16, c17
Biplane	CB10-4/CB10-4P/CB10-4A/CB10-4E/CB10-4s/6LB7/6LB7s/6LB7P/6LB7E/65EB10EA	c1, c2, c3, c4, c5, c7, c14, c15, c16, c17
	ELC13-4U/ELC13-4s/ELC10-4	c1, c3, c7, c14, c15, c17
4D	D7-2/D7-2E/D7-2s/SD8-1/SD8-1E/SD8-1s/SD8-1U/D6-2/D6-2P/D6-2EA/D6-2B/D6-2A	c1, c2, c3, c4, c5, c7, c8, c14, c15, c16, c17
	D8-2E/D8-2U	c2, c10
	D8-4U	c2, c10, c11
	DE10-3/DE10-3E/DE10-3U/DE10-3s	c2, c10, c11
	DE11-3U/DE11-3s/DE11-3E/DE11-3/DE11-3WE/DE11-3Ws/DE11-3WU	c1, c2, c3, c5, c7, c8, c14, c15, c17
	DE10-3WU/DE10-3WE (only for the probe with gray strain relief)	c2, c11, c12, c13
	DE10-3WU/DE10-3WE/DE10-3Ws (only for the probe with white strain relief)	c18

A.2.3 Solutions

d1: CIDEX OPA	d15: Metricide OPA Plus
d2: Cidex Activated Dialdehyde Solution	d16: Cidex Plus
d3: Minncare® Cold Sterilant	d17: Gigasept AF
d4: Ster-Bac	d18: Osvan
d5: Triacid-N	d19: Neojodin
d6: Revital-Ox® Resert® High Level Disinfectant/UltrOx™ High-Level disinfectant	d20: Milton
d7: Gigasept® PAA concentrate	d21: hibitane
d8: DESCOTON extra	d22: Sterihyde
d9: Gigasept® FF(neu)	d23: Metricide 14
d10: ANIOXYDE 1000	d24: Sekusept plus
d11: SALVANIOS pH10	d25: Wavicide-01
d12: Cavicide Liquid	d26: SALVANIOS pH7
d13: Metricide	d27: Minncare liquid disinfectant
d14: Metricide 28	d28: Virusolve® + Concentrate
	d29: Virex II 256

	Probe	Solutions
Convex	C5-1s/C5-1E/C5-1U/C5-1/C5-2/C5-2s/C5-2E/ C6-2E/C6-2/C6-2s/C6-2P/C11-3E/C11-3s/ C11-3U/C11-3	d1, d4, d5, d6, d7, d8, d9, d10, d11, d12, d26, d29
	C6-2Gs/C6-2GE/C6-2GU/SC7-1U/SC9-2U/ SL10-3U/SCM7-1U/SLM10-3U/SC6-1GU/C6-1	d1, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d15, d26, d27, d28, d29
	C7-3E/3C1/3C1s/3C1P/6C2/6C2P/6C2s/ 35C20EA/35C50EA/35C50EB/ 35C50P/65C15EAV/65C15EA	d1, d2, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d27, d28, d29
	3C5/3C5s/3C5A/3C5P	d1, d2, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d27, d28, d29
	SC5-1U/SC5-1E	d1, d4, d6, d12, d15
	SC6-1U/SC6-1E/SC6-1s/SC5-1NE/SC5-1Ns/ SC5-1N	d1, d4, d5, d6, d7, d10, d11, d12, d15, d26, d28, d29
	SC8-2U/SC8-2E/SC8-2s	d1, d9, d16, d20, d25
	65EC10EA/65EC10EB/65EC10EC	d1, d2, d3, d4, d5, d6, d12, d13, d14, d15, d27, d28
	65EC10ED	d1, d2, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d28, d29
	C4-1/C4-1U/C4-1s	d1, d2, d6, d12, d13, d14, d15
C9-3Ts	d1, d6, d12, d13, d14, d15	

d1: CIDEX OPA	d15: Metricide OPA Plus
d2: Cidex Activated Dialdehyde Solution	d16: Cidex Plus
d3: Minncare® Cold Sterilant	d17: Gigasept AF
d4: Ster-Bac	d18: Osvan
d5: Triacid-N	d19: Neojodin
d6: Revital-Ox® Resert® High Level Disinfectant/UltrOx™ High-Level disinfectant	d20: Milton
d7: Gigasept® PAA concentrate	d21: hibitane
d8: DESCOTON extra	d22: Sterihyde
d9: Gigasept® FF(neu)	d23: Metricide 14
d10: ANIOXYDE 1000	d24: Sekusept plus
d11: SALVANIOS pH10	d25: Wavicide-01
d12: Cavicide Liquid	d26: SALVANIOS pH7
d13: Metricide	d27: Minncare liquid disinfectant
d14: Metricide 28	d28: Virusolve® + Concentrate
	d29: Virex II 256

Probe		Solutions
Linear	L7-3/L7-3s/L7-3E/7L4/7L4s/7L4A/7L4P/7L4B/7L4Bs/7L4BP/7L5/7L5P/7L5s/7L6/7L6s/75L38EA/75L38EB/75L53EA/75L60EA/75L38P/10L4/10L4s	d1, d2, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d27, d28, d29
	65EL60EA/6LE7/6LE7s/6LE7P/6LE5V/6LE5Vs/6LE5VP/7LT4/7LT4s/7LT4P/7LT4E/50L60EAV/65L50HAV/75L50EAV/75LT38EA	d1, d2, d3, d4, d5, d6, d12, d13, d14, d15, d27, d28
	L9-3/L9-3U/L9-3E/L9-3s/L12-3/L12-3E/L13-3s/L13-3/L12-4/L12-4s/L14-6N/L14-6Ns/L14-6NP/L14-6NE/L14-6WE/L14-6Ws/L14-6WU	d1, d4, d5, d6, d7, d8, d9, d10, d11, d12, d26, d29
	L10-3E/L10-3s	d1, d25
	L11-3U/L11-3VNs/L13-3N/L13-3Ns/L14-3s/L14-3W/L14-3WU/L14-3Ws/L14-3WE	d1, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d15, d26, d27, d28, d29
	L11-4/L11-4s/L14-6/L14-6s/L14-6P/10L24EA	d1, d4, d5, d6, d7, d10, d11, d12, d15, d26, d28, d29
	L16-4HE/L16-4Hs/L16-4HU	d1, d2
	LM14-6E/LM14-6s/LM16-4U	d1, d2, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d27, d28, d29
	L20-5U/L20-5s/L20-5E	d1, d2, d6, d12, d13, d14, d15, d28, d29
	L12-3RCs/L12-3VNs	d12, d28
	L13-3WE/L13-3WU/L13-3Ws/L15-3WE/L15-3WU/L15-3Ws/L18-5WU/LM18-5WU/LM24-6WU	d28, d29
	L14-5sp	d1, d2, d6, d12, d13, d14, d15
	L14-5WU/L14-5WE/L14-5Ws	d1, d2, d6, d12, d13, d14, d15
L30-8U/L33-8U/L33-8s	d2	

d1: CIDEX OPA	d15: Metricide OPA Plus
d2: Cidex Activated Dialdehyde Solution	d16: Cidex Plus
d3: Minncare® Cold Sterilant	d17: Gigasept AF
d4: Ster-Bac	d18: Osvan
d5: Triacid-N	d19: Neojodin
d6: Revital-Ox® Resert® High Level Disinfectant/UltrOx™ High-Level disinfectant	d20: Milton
d7: Gigasept® PAA concentrate	d21: hibitane
d8: DESCOTON extra	d22: Sterihyde
d9: Gigasept® FF(neu)	d23: Metricide 14
d10: ANIOXYDE 1000	d24: Sekusept plus
d11: SALVANIOS pH10	d25: Wavicide-01
d12: Cavicide Liquid	d26: SALVANIOS pH7
d13: Metricide	d27: Minncare liquid disinfectant
d14: Metricide 28	d28: Virusolve® + Concentrate
	d29: Virex II 256

Probe		Solutions
Phased	P4-2/P4-2s/P4-2E/P4-2NE/P4-2Ns/ P10-4E/P10-4s/P10-4/P10-4U/2P2/2P2s/2P2P	d1, d2, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d27, d28, d29
	P7-3/P7-3s/P7-3E/P7-3U/P7-3P	d1, d2, d11, d13, d16, d20, d25
	P12-4/P12-4s	d1, d2, d3, d4, d5, d27
	SP5-1U/SP5-1s/SP5-1E/SP5-1	d1, d4, d5, d6, d7, d10, d11, d12, d15, d26, d28, d29
	SP5-1Ns/SP5-1N	d1, d4, d5, d6, d7, d8, d9, d10, d11, d12, d26, d29
	LFP5-1s/LFP5-1U/LFC5-1s/LFC5-1U	d12, d29
Endo-cavity	P8-2s/P8-2/P8-2U/P8-2P/SP9-2U/SPM6-1U/XP5-1U	d1, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d15, d26, d27, d28, d29
	V11-3/V11-3BE/V11-3B/V11-3WE/V11-3Ws/ V11-3E/V11-3s	d1, d2, d3, d4, d5, d6, d10, d11, d12, d13, d14, d15, d26, d27, d29
	V11-3HU/V11-3HE/V11-3Hs/V11-3H/V11-3HB/V11-3HBs/V11-3HBE/SV10-2U	d1, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d15, d26, d27, d28, d29
Pencil	V10-4/V10-4s/V10-4B/V10-4Bs/V10-4BP/ 6CV1/6CV1s/6CV1P	d1, d2, d4, d5, d6, d7, d8, d9, d10, d11, d12, d13, d14, d15, d26, d28, d29
Biplane	CW2s/CW5s/CW5	d1, d2, d4, d5
	CB10-4/CB10-4P/CB10-4A/CB10-4E/CB10-4s/6LB7/6LB7s/6LB7P/6LB7E/65EB10EA	d1, d2, d3, d4, d5, d6, d12, d13, d14, d15, d27, d28
4D	ELC13-4U/ELC13-4s/ELC10-4	d1, d12, d15, d28
	4CD4/4CD4s/D6-2E	d1, d2, d4
	D7-2/D7-2E/D7-2s/SD8-1/SD8-1E/SD8-1s/SD8-1U/ D6-2/D6-2P/D6-2EA/D6-2B/D6-2A	d1, d2, d3, d4, d5, d6, d10, d11, d12, d13, d14, d15, d26, d27, d28
	D6-2NE/DL14-3U	d1, d18, d19, d20, d21, d22
	D8-2E/D8-2U	d1, d2, d9
	D8-4U	d1, d2, d16, d17
	DE10-3/DE10-3E/DE10-3U/DE10-3s	d1, d2, d16, d17
	DE11-3U/DE11-3s/DE11-3E/DE11-3/ DE11-3WE/DE11-3Ws/DE11-3WU	d1, d3, d4, d5, d6, d7, d8, d9, d10, d11, d12, d15, d26, d27
	DE10-3WU/DE10-3WE (only for the probe with gray strain relief)	d1, d2, d15, d16, d17, d23, d24, d25
DE10-3WU/DE10-3WE/DE10-3Ws (only for the probe with white strain relief)	d1	

A.2.4 Devices

- e1: TrophonSonex-HL (Used with Trophon/Trophon2)
 e2: Germitec UV-C
 e3: Vaporized Hydrogen Peroxide (Used with V-PRO Low Temperature Sterilization System)
 e4: STERRAD® system

	Probe	Devices
Convex	C5-1s/C5-1E/C5-1U/C5-1/C5-2/C5-2s/C5-2E/C6-2E/C6-2/C6-2s/ C6-2P/C11-3E/C11-3s/C11-3U/C11-3	e1, e2
	C6-2Gs/C6-2GE/C6-2GU/SC7-1U/SC9-2U/SL10- 3U/SCM7-1U/SLM10-3U/SC6-1GU/C6-1	e1, e2
	C7-3E/3C1/3C1s/3C1P/6C2/6C2P/6C2s/35C20EA/35C50EA/ 35C50EB/35C50P/65C15EAV/65C15EA	e1, e2
	3C5/3C5s/3C5A/3C5P	e1, e2
	SC5-1U/SC5-1E	e1, e2
	SC6-1U/SC6-1E/SC6-1s/SC5-1NE/SC5-1Ns/SC5-1N	e1, e2
	SC8-2U/SC8-2E/SC8-2s	e1
	65EC10EA/65EC10EB/65EC10EC	e1, e2, e3
	65EC10ED	e1, e2, e3
	C4-1/C4-1U/C4-1s	e2
C9-3Ts	e1, e4	
Linear	L7-3/L7-3s/L7-3E/7L4/7L4s/7L4A/7L4P/7L4B/7L4Bs/7L4BP/7L5/7L5P/ 7L5s/7L6/7L6s/75L38EA/75L38EB/75L53EA/75L60EA/75L38P/10L4/ 10L4s	e1, e2
	65EL60EA/6LE7/6LE7s/6LE7P/6LE5V/6LE5Vs/6LE5VP/7LT4/7LT4s/ 7LT4P/7LT4E/50L60EAV/65L50HAV/75L50EAV/75LT38EA	e1, e2, e3
	7LT4s (only for the socket with black cover)	e4
	L9-3/L9-3U/L9-3E/L9-3s/L12-3/L12-3E/L13-3s/L13-3/L12-4/L12-4s/ L14-6N/L14-6Ns/L14-6NP/L14-6NE/L14-6WE/L14-6Ws/L14-6WU	e1, e2
	L10-3E/L10-3s	e1
	L11-3U/L11-3VNs/L13-3N/L13-3Ns/L14-3s/ L14-3W/L14-3WU/L14-3Ws/L14-3WE	e1, e2
	L11-4/L11-4s/L14-6/L14-6s/L14-6P/10L24EA	e1, e2
	L16-4Hs (only for the socket with black cover)	e4
	LM14-6E/LM14-6s/LM16-4U	e1, e2
	L12-3RCs/L12-3VNs	e1, e2
L14-5sp	e1, e2	
L14-5WU/L14-5WE/L14-5Ws	e2	
Phased	P4-2/P4-2s/P4-2E/P4-2NE/P4-2Ns/P10-4E/P10-4s/P10-4/P10-4U/ 2P2/2P2s/2P2P	e1, e2
	P7-3/P7-3s/P7-3E/P7-3U/P7-3P	e1
	P12-4/P12-4s	e1
	SP5-1U/SP5-1s/SP5-1E/SP5-1	e1, e2
	SP5-1Ns/SP5-1N	e1, e2
	P8-2s/P8-2/P8-2U/P8-2P/SP9-2U/SPM6-1U/XP5-1U	e1, e2

- e1: TrophonSonex-HL (Used with Trophon/Trophon2)
 e2: Germitec UV-C
 e3: Vaporized Hydrogen Peroxide (Used with V-PRO Low Temperature Sterilization System)
 e4: STERRAD® system

Probe		Devices
Endo-cavity	V11-3/V11-3BE/V11-3B/V11-3WE/V11-3Ws/V11-3E/V11-3s	e1, e2, e3
	V11-3HU/V11-3HE/V11-3Hs/V11-3H/V11-3HB/ V11-3HBs/V11-3HBE/SV10-2U	e1, e2
	V11-3HBs V11-3Hs (only for the socket with black cover)	e4
	V10-4/V10-4s/V10-4B/V10-4Bs/V10-4BP/6CV1/6CV1s/6CV1P	e1, e2, e3
Biplane	CB10-4/CB10-4P/CB10-4E/CB10-4s	e1, e2, e3
	6LB7/6LB7s/6LB7P/6LB7E/65EB10EA	e2, e3
4D	D7-2/D7-2E/D7-2s/SD8-1/SD8-1E/SD8-1s/SD8-1U/ D6-2/D6-2P/D6-2EA/D6-2B/D6-2A	e1
	D6-2NE/DL14-3U	e1
	D8-2E/D8-2U	e1
	DE10-3/DE10-3E/DE10-3U/DE10-3s	e1, e2
	DE11-3U/DE11-3s/DE11-3E/DE11-3/DE11-3WE/DE11-3Ws/ DE11-3WU	e1, e3
	DE10-3WU/DE10-3WE (only for the probe with gray strain relief)	e1, e2

NOTE:

For V-PRO Low Temperature Sterilization System:

- The probe should be placed into a STERIS Sterilization Tray and wrapped with sterilization wrap. In Canada/FDA region, the STERIS Sterilization Tray and sterilization wrap should be cleared by the FDA/HC, such as H600 OneStep® sterilization wrap.
- Start the sterilization system using the Non Lumen Cycle according to the instructions provided by the manufacturer.

A.2.5 Powders

f1: Rely+On PeraSafe

Probe		Powders
4D	D8-2E/D8-2U	f1
	D8-4U	f1
	DE10-3/DE10-3E/DE10-3U/DE10-3s	f1
	DE10-3WU/DE10-3WE (only for the probe with gray strain relief)	f1

This page intentionally left blank.

Appendix B Composition

B.1 Active Ingredients of the Cleaner

Cleaner	Active Ingredients
MetriZyme/MetriSponge:	Proteinase subtilisin
Revital-OX Enzymatic Detergent:	Citric acid, Triethanolamine, Ethanolamine, Ethoxylated coconut oil alkyl amine, Subtilisins (proteolytic enzymes), Glycerine
Endozime and Endozime Sponge:	Subtilisins (proteolytic enzymes)
Prolystica 2X Concentrate Enzymatic Cleaner/klenzyme/ Liquinox/Tristel Pre-Clean Wipes:	enzymatic detergent
ANIOSYME 5:	ionic surfactants, sequestering agent, stabilising agent, enzymatic complex, Excipients
DDN9	N PROPIONATE, N-DIDECYL—N-METHYL-POLY(OXYETHYL)AMMONIUM TETRAPOTASSIUM ETHYLENEDIAMINETETRAACETATE EDETIC ACID

B.2 Active Ingredients of the Disinfectant

Disinfectant	Active Ingredients
Tristel Trio Wipes System/Tristel Jet/Tristel Duo/Tristel Sporidical Wipes:	chlorine dioxide
IODOCLEAN	sodium thiosulfate and excipients
Ster-Bac/PI-SPRAY II/mikrozid® Sensitive Wipes/Clinell Universal Wipes/Sani-Cloth HB/Sani-Cloth Active/T-Spray/Mikorbac Tissues/ Sani-Cloth Germicidal Wipes/ SALVANIOS pH7/SALVANIOS pH10/Protex Ultra Wipes/ CaviWipes XL/CAVICIDE 1:	Quaternary Ammoniums
Tristel Rinse Wipes:	deionized water
Cidex OPA:	0.55% Ortho-phthlaldehyde
DESCOTON extra/Wavicide-01:	glutaraldehyde
Oxivir ^{TM/MC} Tb:	0.5% hydrogen peroxide

Disinfectant	Active Ingredients
Sani-Cloth® Plus:	n-Alkyl dimethyl benzyl ammonium chloride n-Alkyl ethylbenzyl ammonium chloride
CaviWipes:	isopropanol, ethylene glycol monobutyl ether (2-butoxyethanol), diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride, water
CaviWipes 1:	isopropanol, ethanol, ethylene glycol monobutyl ether (2-butoxyethanol), didecyldime-thylammonium chloride, water
Dispatch Towels:	Sodium hydroxide, Sodium metasilicate, Sodium hypochlorite
SONO™ ULTRASOUND WIPES:	octyl decyl dimethyl ammonium chloride; dioctyl dimethyl ammonium chloride; didecyl dimethyl ammonium chloride; dimethyl benzyl ammonium chloride
Minnicare® Cold Sterilant/Minnicare liquid disinfectant:	22% Hydrogen Peroxide, 4.5% Peroxyacetic Acid
mikrozid® AF Wipes Jumbo:	25% ethanol; 35% propan-1-ol
CLEANISEPT® WIPES:	0.25g didecyldimethylammoniumchloride 0.5g quaternary ammonium compounds, benzyl-C12-16-alkyldimethyl-, chlorides
Wip'Anios premium:	didecyldimethylammonium chloride 1.4mg/g, polyhexamethylene biguanide hydrochloride 0.96mg/g
Cidex Activated Dialdehyde Solution:	glutaraldehyde
TRANSEPTIC:	isopropyl alcohol, chlorhexidine gluconate
Protex Spray:	Quaternary ammonium compounds di-C8-10-alkyldimethyl, chlorides Water
gigasept® FF(neu):	0.11g succindialdehyde; 0.3g dimethoxytetrahydrofuran; < 5% anionic surfactant; non-ionic surfactants; anti-corrosion compounds; fragrance
gigasept® PAA concentrate:	peracetic acid (5%); hydrogen peroxide: acetic acid; potassium hydroxide: corrosion inhibitor
Protex™ Disinfectant Spray/ Protex™ Disinfectant Wipes:	octyl decyl dimethyl ammonium chloride; dioctyl dimethyl ammonium chloride; didecyl dimethyl ammonium chloride; dimethyl benzyl ammonium chloride
Triacid-N:	N-Dodecylpropan-1,3-diamin; propan-2-ol; isotridecanol, ethoxylated; non-ionic detergent
Surfa'safe:	didecyldimethylammonium chloride 1.4mg/g, polyhexamethylene biguanide hydrochloride 0.96mg/g
Revital-Ox® Resert® High Level Disinfectant/TrophonSonex-HL:	hydrogen peroxide

Disinfectant	Active Ingredients
ANIOXYDE 1000:	3% Hydrogen Peroxide
Sani-Cloth AF3:	Quaternary ammonium chlorides.
Metricide OPA Plus:	Ortho-phthlaldehyde
Metricide/Metricide 14/Metricide 28:	Glutaraldehyde
CaviCide:	isopropanol, ethanol, ethylene glycol monobutyl ether (2-butoxyethanol), diisobutylphe-noxyethoxyethyl dimethyl benzyl ammonium chloride, water
Gigasept AF:	didecyldimethylammonium chloride, glycine, aminoalkyl derivs tridecylpolyethylenglycoether.
Rely+On PeraSafe:	disodium carbonate, citric acid
Cidex Plus:	glutaraldehyde
Sonogel:	carbomer, polyacrylat
SteriHyde:	Glutaraldehyde
Osvan:	Ammonia benzalkonium
Neojodin:	povidone-iodine
Milton:	sodium hypochlorite
Hibitane:	Hlorhexidine
WIP'ANIOS CLEAN'UP:	Association of surfactants, excipients
OXIVIR 1/OXIVIR™ WIPES/Oxivir 1 Wipes/Accel TB Liquid/Accel TB Wipes:	Hydrogen peroxide
WIP'ANIOS SPOR'ACTIV:	Hydrogen peroxide and peroxyacetic acid mixture stabilized
PDI SANI-CLOTH BLEACH WIPES:	Sodium hypochlorite
wip anios excel:	Didecyldimethylammonium chloride, non-ionic surfactants, sequestering agent, excipients
Virusolve® + Pronte all'uso Wipes:	Polycarboxylate, non ionic emulsifier, alcohol, Didecyldimethyl ammonium Chloride, Bis(3-aminopropyl)dodecylamine
Virusolve® + Concentrate:	2-Aminoethanol, Didecyldimethyl ammonium Chloride, Propan-2-ol, Potassium Carbonate
Super Sani-Cloth®	Quaternary Ammoniums
SANI-CLOTH® PRIME WIPES	Quaternary Ammoniums
STERRAD® system	Hydrogen peroxide plasma
Sani-HyPerCide GERMICIDAL SPRAY	hydrogen
Sani-24 GERMICIDAL SPRAY	Quaternary Ammoniums, Ethanol
Virex II 256	Quaternary Ammoniums

This page intentionally left blank.

Appendix C Registered Disinfectants in Canada

Drug Identification Number /License Number	Name
74736	V-PRO Low Temperature Sterilization System
2981	STERRAD [®] system
02286467	Revital-Ox [®] Resert [®] High Level Disinfectant
100935	Trophon/Trophon2
01963996	Metricide
01964461	Metricide 28
02197170	Wavicide-01
02277484	Minnicare liquid disinfectant
02161656	Cavicide Liquid
02247354	Super Sani-Cloth [®]
02517000	SANI-CLOTH [®] PRIME WIPES

This page intentionally left blank.

Appendix D Registered Disinfectants in FDA Region

Registration Number	Name	Microbicidal Method	
FDA 510(k)	K190103	V-PRO Low Temperature Sterilization System	Sterilization
	K924434	Cidex Activated Dialdehyde Solution	High-level disinfectant and Sterilization
	K030004	CIDEX OPA	High-level disinfectant
	K923744	Cidex Plus	High-level disinfectant and Sterilization
	K080420	Revital-Ox [®] Resert [®] High Level Disinfectant	High-level disinfectant
	K091022	UltrOx [™] High-Level disinfectant	High-level disinfectant
	K103059	Trophon	High-level disinfectant
	K173865	Trophon2	High-level disinfectant
	K930284	Metricide	High-level disinfectant and Sterilization
	K931052	Metricide 28	High-level disinfectant and Sterilization
	K140703	Metricide OPA Plus	High-level disinfectant
	K914749	Wavicide-01	High-level disinfectant and Sterilization
	K954142	STERRAD [®] system	Sterilization

	Registration Number	Name	Microbicidal Method
EPA	9480-6	Sani-Cloth® Plus	Low-level disinfectant
	70627-56	Oxivir™/MC Tb	Low-level disinfectant
	9480-9	Sani-Cloth AF3 (gray)	Low-level disinfectant
	46781-8	CAVIWIPES	Low-level disinfectant
	46781-6	CAVICIDE	Low-level disinfectant
	70627-60	OXIVIR™ WIPES	Low-level disinfectant
	46781-12	CAVICIDE 1	Low-level disinfectant
	9480-4	Super Sani-Cloth®	Low-level disinfectant
	9480-12	SANI-CLOTH® PRIME WIPES	Low-level disinfectant
	9480-14	Sani-HyPerCide GERMICIDAL SPRAY	Low-level disinfectant
	42182-9-9480	Sani-24 GERMICIDAL SPRAY	Low-level disinfectant
	9480-8	PDI SANI-CLOTH BLEACH WIPES	Low-level disinfectant
	70627-24	Virex II 256	Low-level disinfectant

