# MR Urine Quality Control

## Summary

MR Urine Quality Control is an electrolyte reference urine control used to monitor the urine mode performance of ion-selective electrode on ISE Modules on automatic chemistry analyzers. Daily use of this urine control provides quality control data for confirming the precision and accuracy of urine sample tests.

## Specifications

High 1x100 mL, Low 1x100 mL

## Appearance and Components

MR Urine Quality Control should be clear and transparent liquid without precipitate, suspension and floccule.

Components: Sodium Chloride, Potassium Chloride, Deioned Water, Surfactant and Preservatives.

## Applications

MR Urine Quality Control applies to BS series chemistry analyzers manufactured by SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD.

## Storage and Stability

MR Urine Quality Control is stable when stored at 2°C-10°C at least until the expiration date printed on the label. The control vial should be kept away from light. After each use, promptly replace the cap and ensure the vial is capped tightly. DO NOT FREEZE.

Once opened, this product will be stable at least for 8 weeks when stored tightly capped at 2°C-10°C.

## PRECAUTION

MR Urine Quality Control is intended for *in vitro* diagnostic use only by trained personnel.

## How to use this product

Use this product on the chemistry analyzer as instructed in the instrument manual. The target values are as follows:

Lot.No.: 016025005

Unit: mmol/L	Na <sup>+</sup>	K <sup>+</sup>	Cl <sup>-</sup>
High	119.6-161.6	37.81-51.15	117.7-159.1
Low	63.8-86.2	16.85-22.79	63.0-85.2

## Limitations

The product only applies to the ISE module of BS series chemistry analyzers. In addition, the warranted results only come from processing tests strictly as instructed in the instrument manual and storing the product correctly.

## Warnings

- MR Urine Standard is for In Vitro Diagnostic use only.
- Consult instructions for using MR Urine Standard.
- This product should not be used past the expiration date.
- If any appearance change occurs, like a cloudy or turbid appearance shows up, do not use this vial of the product any more.
- Wear proper personal protective equipment (e.g. gloves, lab coat, etc.) and follow safe laboratory procedures when handling this product in the laboratory.
- This product must be handled by skilled/trained medical professionals.
- Preservative contained. Do not swallow. Avoid contact with skin and mucous membranes.
- Dispose of any discarded material in accordance with the requirements of your local government regulations.
- Material safety data sheet is available for professional user on request.

## Company Contact

Manufacturer: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Address: Mindray Building, Keji 12th Road South, Hi-tech Industrial Park, Nanshan, ShenZhen 518057, P.R.China.

Website: www.mindray.com.cn

Tel: +86 755 26582479 26582888

Fax: +86 755 26582934 26582500

## EC-Representative:

Shanghai International Holding Corp. GmbH (Europe)

Address: Eiffestraße 80 20537 Hamburg, Germany

Tel: 0049-40-2513175

Fax: 0049-40-255726

In vitro diagnostic medical device

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Üretici: Shenzhen Mindray Bio-Medical Electronics Co.,Ltd.
Shanghai International Holding Corp. GmbH (Avrupa)