
HCG Calibration falure

IVD International Clinical Application Department

Elena Yamshchikova CAS IVS SPb

2024/9/4

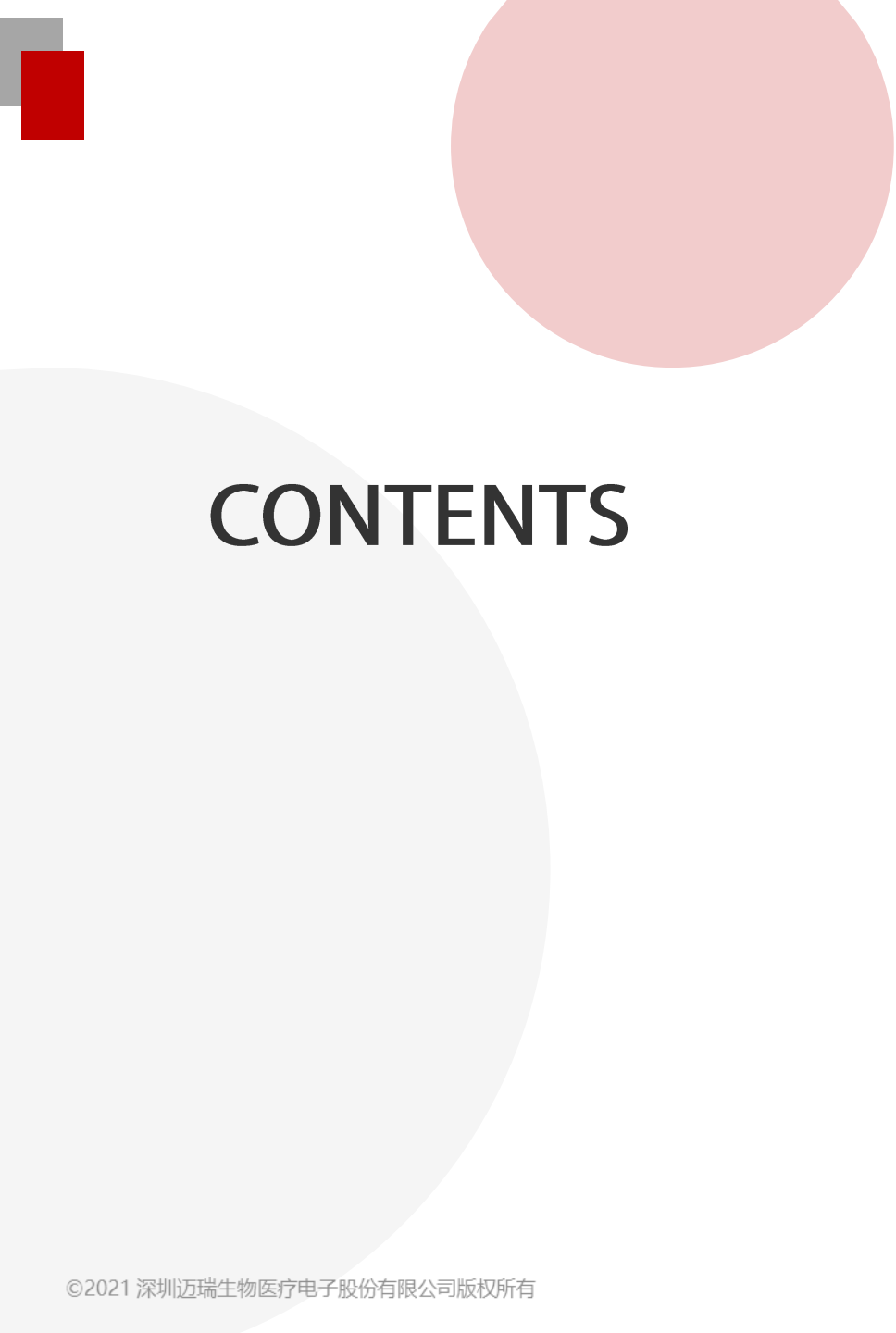CONTENTS

01. Case Background

02. Case Ideas

03. Case Solution

04. Case Summary

Case Background

Background

Ideas

Solution

Summary

Require:

- Customer use HCG reagent lot 230201 (exp. Date 27.03.2024) on CL-1200i;
- Calibrate current lot of reagent with new lot of calibrator (lot number 230901, expiration date 6.10.2024, open 1.02.2024);
- Calibration failure with flag SLO, DEV

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

Follow :

- Ask about any problem with other tests;
- Ask about system errors and performance system check;
- Ask about procedure of calibrator and reagent maintenance, like time of warming ets:
- Ask calibration results
- Ask QC results for this test;

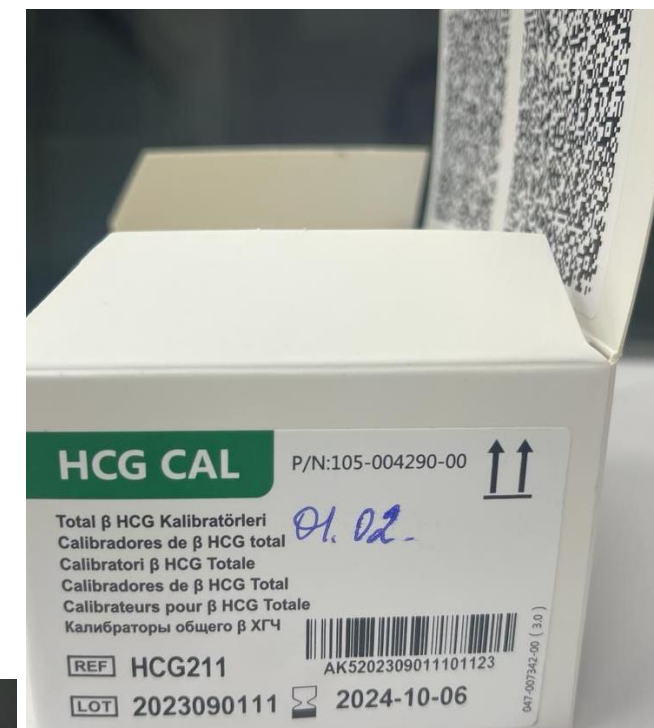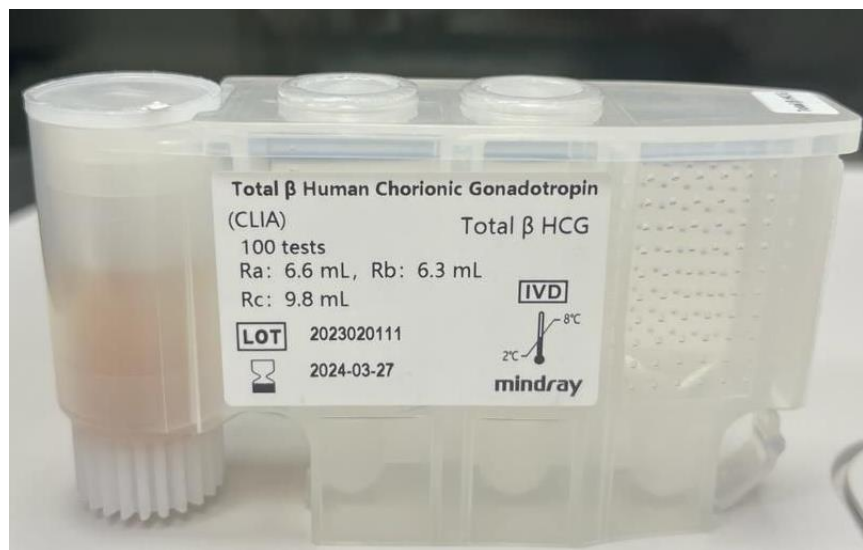Calibration results :

- RLU of calibrators C1 and C2 significantly decreased from target

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
QC results :

- QC results display trend to decrease results of rests;
- There is last reagent bottle if this lot at customer laboratory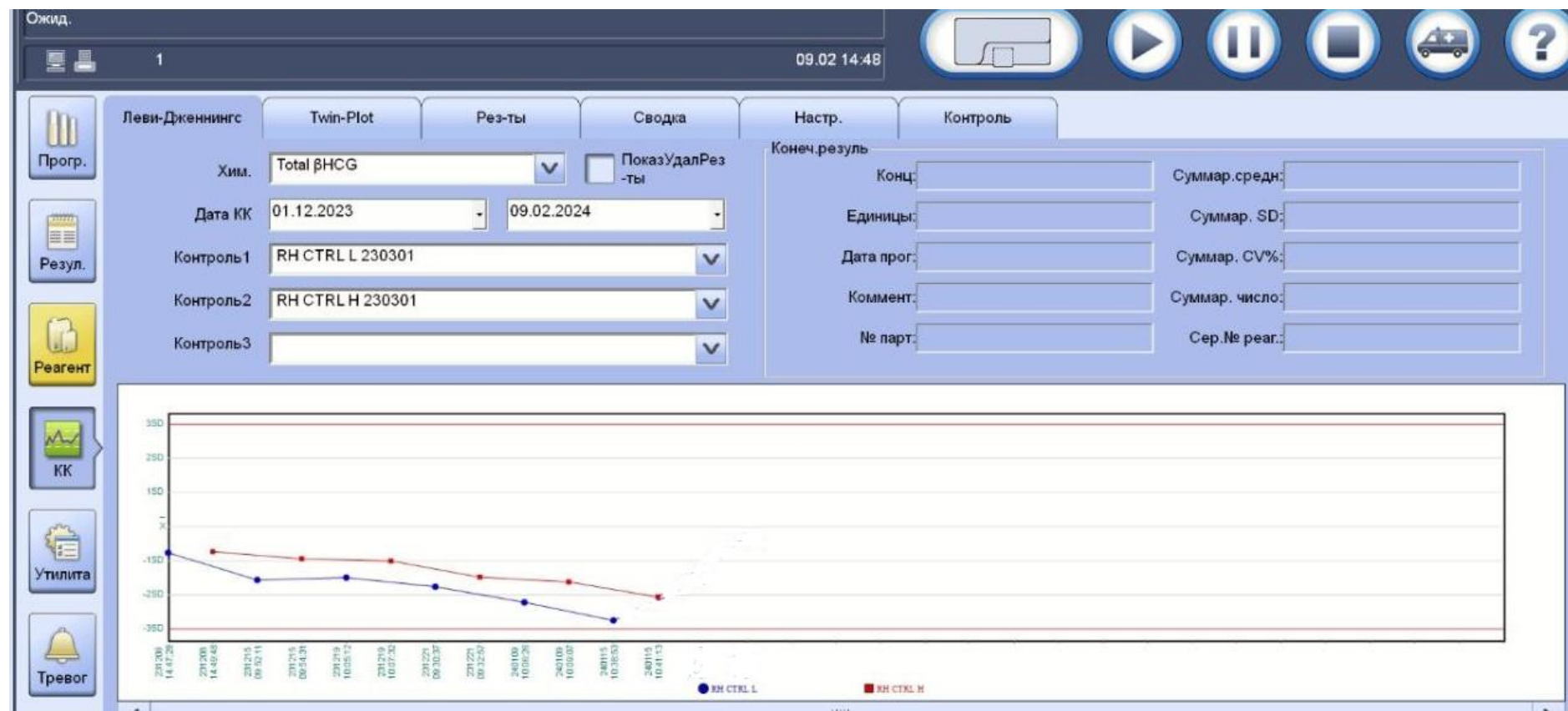


Case analysis:

- No problem with other tests;
- System status of analyzer is ok: no error message, no alarms;
- Effect check is ok;
- Expiration date of reagent is Ok;
- QC results display trend to decrease results of rests;
- Calibration RLU decrease;
- Consumables status is Ok;
- QC lot is the same, do not change;
- Last reagent bottle of this lot on customer side

Case ideas:

- CAL Failure with SLO and DEV flag, which means
DEV: "Calibrator RLU deviation out of range. Check failed"
- SLO: "Calibration slope check failed"
- First suspect the reagent problem

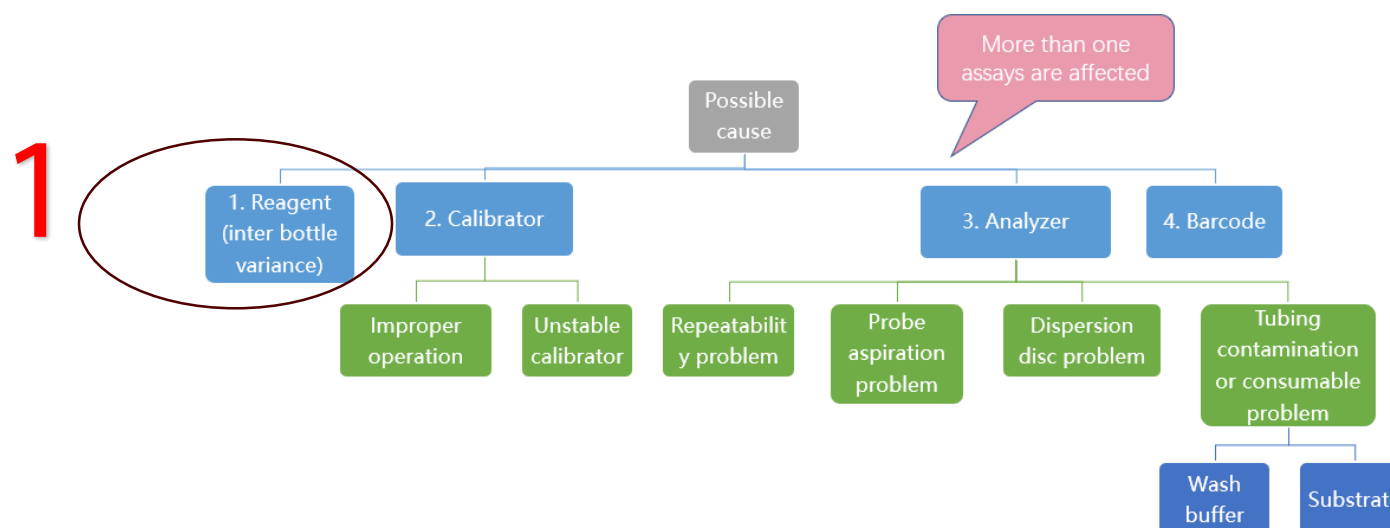Take new reagents :

- Customer use new fresh lot of reagents and re-calibrate with the same lot of calibrator;
- Calibration successfully complete;
-

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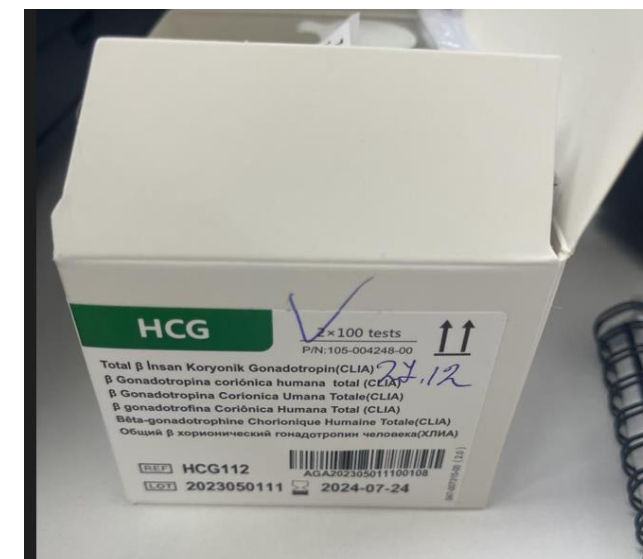
QC results display more convenient results:

Result:

- Customer successfully calibrate test;
- Pay attention to reagent maintenance. Reasons to decrease RLU could be:
 - Out of reagent expiration date
 - Out of on-board stability reagent
 - Repeatedly put on and off reagent of analyzer could lead to reagent deterioration
 - Reagent storage should be in appropriate temperature (+2+8)

Thanks !

mindray迈瑞