

Ultrasound Journal

The Levator Trauma in Childbirth


Case 2



Special thanks to: Ru Yanhui, Sheng Xuan, Yin Hong, Department of Ultrasound of Shandong Maternal and Child Health Hospital, Medical Imaging Center of Shandong Maternal and Child Health Hospital

The levator ani muscle is believed to be an important support for lower urinary tract function and pelvic organ. In recent studies, it is shown that major levator trauma happens in 15–35% of vaginally parous women, when using MRI and transperineal ultrasound for the diagnosis [1,2]. The main methods of clinical assessment of levator trauma include transvaginal palpation, Magnetic Resonance Imaging, and pelvic floor ultrasound. Generally speaking, MRI has limitations such as cost and accessibility, and the fact that the imaging is static, not dynamic [3]. Pelvic floor ultrasound, on the contrary, has the advantages of shorter examination time, more economic, real-time display, intuitive and multi-plane display of the levator trauma location, etc. It has become an increasingly important method to assess levator trauma.

The main scanning methods include transperineal/intraluminal 2D scanning in parasagittal view, 3D/4D volume imaging, and tomographic ultrasound called ipage*.

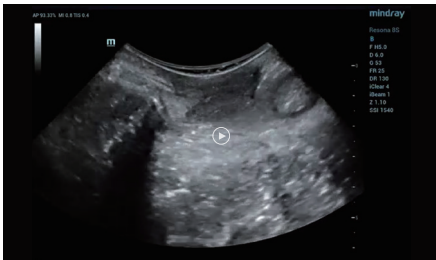

Video 1. Transperineal 2D ultrasound imaging of levator ani muscle with abdominal volume probe in parasagittal view
*Video is available through the web version

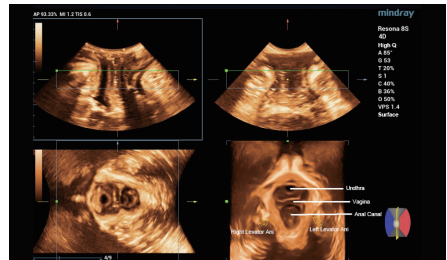


Figure 1. Pelvic floor ultrasound volume imaging of levator ani muscle


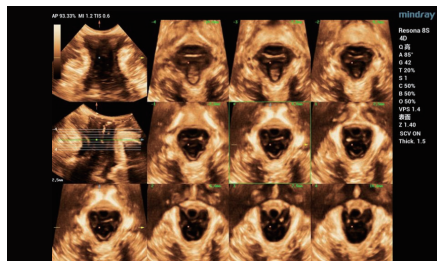


Figure 2. Ipage* of normal levator ani muscle, adopts a 3 × 3 image mode, shows that bilateral levator ani muscles are symmetrical with uniform echo. The middle three images represent the open, closing, and closed state of pubic symphysis respectively, with a layer spacing of 2.5 mm.

Case Introduction:

A 30-year-old female, G1P1, vaginal delivery at 39+ weeks. Negative-pressure vacuum extraction and left perineotomy were applied during delivery. The weight of the newborn was 3370 g. Pelvic floor ultrasound was carried out 42 days after delivery.

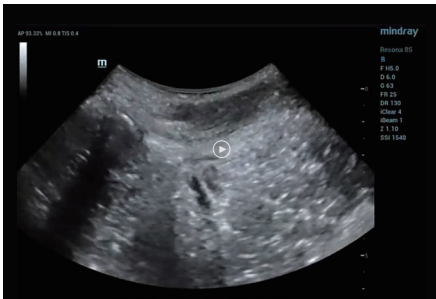

Video 2. Transperineal 2D scanning of left-sided avulsion in parasagittal view (right side first and then left side)

*Video is available through the web version

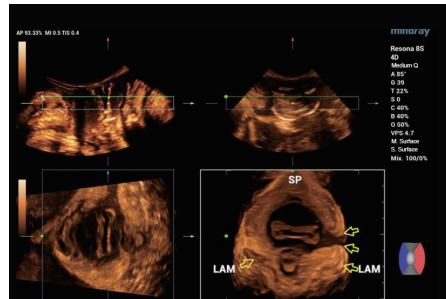


Figure 3. Volume imaging of left-sided avulsion



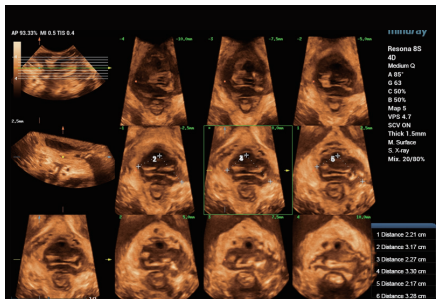


Figure 4. Ipage* of left-sided avulsion

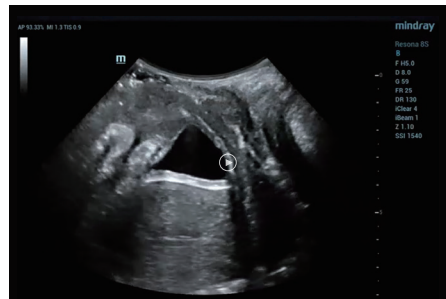

Video 3. Bladder prolapse on Valsalva

*Video is available through the web version

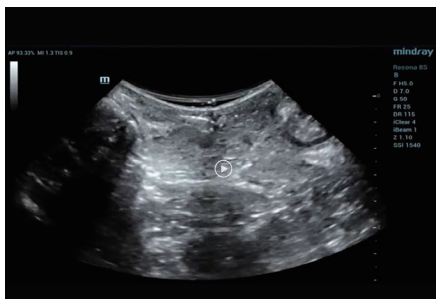

Video 4. Review at 9 months postpartum

*Video is available through the web version

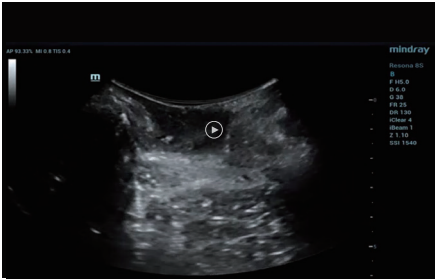


Figure 5. Ipage* at 9 months postpartum

Discussion:

Differential Diagnosis:

1. False positive caused by gas inside and outside the probe sleeve:




Video 5. False positive of levator avulsion caused by gas inside and outside the probe sleeve. Replacing the probe sleeve and evenly applying a large amount of gelinside and outside the sleeve can help with differential diagnosis.

*Video is available through the web version

2. Congenital agenesis of levator ani muscle: The development of levator ani muscle is abnormal, mostly bilateral and symmetrical, presented as thin levator ani. Volume imaging is helpful for differential diagnosis.

3. The levator ani muscle is pushed by the perineal subcutaneous hematoma.

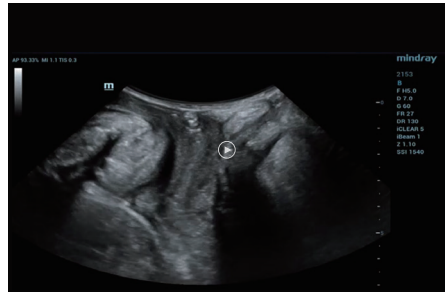

Video 6. Heterogeneous hypoechoogenicity between left levator ani and vagina

*Video is available through the web version

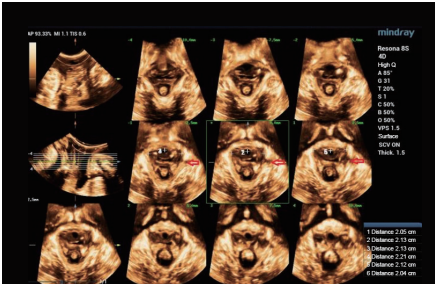


Figure 6. False positive of left-sided avulsion caused by the gas inside and outside the probe sleeve

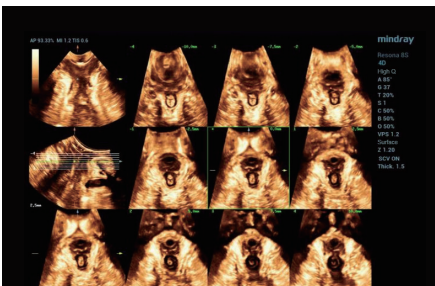


Figure 7. After eliminating gas interference in the same patient

Key Points of Ultrasound Diagnosis of Levator Trauma:

1. 2D scanning: The levator ani muscle fibers are completely or partially interrupted; the edge of the broken end is coarse. The levator ani muscles are asymmetrical on both sides.

2. Volume imaging: The levator hiatus is asymmetric and deviated to the injured side.

3. Ipage*: If more than three slices show levator avulsion on one or both sides, the diagnosis of levator trauma may be considered.

Precautions for Scanning:

1. When observing the integrity of the muscle, the scanning must start from the starting

point to the ending point of the muscle along the direction of the muscle fibers, so as to avoid omission.

2. Combine intracavitary probe and superficial probe that have high frequency to help diagnose when the diagnosis is not clear.
3. If levator trauma cannot be excluded, the pelvic floor ultrasound should be repeated after 1–2 months.

Treatment:

Patients are advised to avoid fatigue and start rehabilitation exercise 3 months postpartum. At present, the main treatment methods include:

1. Rehabilitation treatment, such as pelvic floor electrical stimulation and vaginal dumbbell exercise, can alleviate some clinical symptoms.

2. Pelvic reconstructive surgery

The long-term efficacy of these treatment methods still requires further evaluation.

References:

- [1] DeLancey J, Kearney R, Chou Q, Speights S, Binno S. The appearance of levator ani muscle abnormalities in magnetic resonance images after vaginal delivery. *Obstet Gynecol* 2003;101(1):46–53.
- [2] Dietz HP, Lanzarone V. Levator trauma after vaginal delivery. *Obstet Gynecol* 2005;106(4):707–12.
- [3] Hans Peter Dietz, Pelvic floor ultrasound: a review. *American Journal of Obstetrics & Gynecology* 2010(4); 321-334

healthcare within reach

MINDRAY SOCIAL MEDIA CHANNELS

LinkedIn



www.mindray.com

P/N: ENG-Ultrasound Journal-210285X6P-20220216

©2021 Shenzhen Mindray Bio-Medical Electronics Co.,Ltd. All rights reserved.

mindray
healthcare within reach