

## CONTENT

To Whom It May Concern,

Below content is only for your information.

### Introduction:

We have reviewed the applicability to Mindray products of the Microsoft Windows security patches released in Dec, 2025. The following CVEs have been evaluated:

### CVE Identifiers

- CVE-2025-64680 Windows DWM Core Library Elevation of Privilege Vulnerability
- CVE-2025-64679 Windows DWM Core Library Elevation of Privilege Vulnerability
- CVE-2025-64678 Windows Routing and Remote Access Service (RRAS) Remote Code Execution Vulnerability
- CVE-2025-64673 Windows Storage VSP Driver Elevation of Privilege Vulnerability
- CVE-2025-64670 Windows DirectX Information Disclosure Vulnerability
- CVE-2025-64661 Windows Shell Elevation of Privilege Vulnerability
- CVE-2025-64658 Windows File Explorer Elevation of Privilege Vulnerability
- CVE-2025-62573 DirectX Graphics Kernel Elevation of Privilege Vulnerability
- CVE-2025-62572 Application Information Service Elevation of Privilege Vulnerability
- CVE-2025-62571 Windows Installer Elevation of Privilege Vulnerability
- CVE-2025-62570 Windows Camera Frame Server Monitor Information Disclosure Vulnerability
- CVE-2025-62569 Microsoft Brokering File System Elevation of Privilege Vulnerability
- CVE-2025-62567 Windows Hyper-V Denial of Service Vulnerability
- CVE-2025-62565 Windows File Explorer Elevation of Privilege Vulnerability
- CVE-2025-62549 Windows Routing and Remote Access Service (RRAS) Remote Code Execution Vulnerability
- CVE-2025-62474 Windows Remote Access Connection Manager Elevation of Privilege Vulnerability
- CVE-2025-62473 Windows Routing and Remote Access Service (RRAS) Information Disclosure Vulnerability
- CVE-2025-62472 Windows Remote Access Connection Manager Elevation of Privilege Vulnerability
- CVE-2025-62470 Windows Common Log File System Driver Elevation of Privilege Vulnerability
- CVE-2025-62469 Microsoft Brokering File System Elevation of Privilege Vulnerability
- CVE-2025-62468 Windows Defender Firewall Service Information Disclosure Vulnerability
- CVE-2025-62467 Windows Projected File System Elevation of Privilege Vulnerability
- CVE-2025-62466 Windows Client-Side Caching Elevation of Privilege Vulnerability
- CVE-2025-62465 DirectX Graphics Kernel Denial of Service Vulnerability
- CVE-2025-62464 Windows Projected File System Elevation of Privilege Vulnerability
- CVE-2025-62463 DirectX Graphics Kernel Denial of Service Vulnerability

- CVE-2025-62462 Windows Projected File System Elevation of Privilege Vulnerability
- CVE-2025-62461 Windows Projected File System Elevation of Privilege Vulnerability
- CVE-2025-62458 Win32k Elevation of Privilege Vulnerability
- CVE-2025-62457 Windows Cloud Files Mini Filter Driver Elevation of Privilege Vulnerability
- CVE-2025-62456 Windows Resilient File System (ReFS) Remote Code Execution Vulnerability
- CVE-2025-62455 Microsoft Message Queuing (MSMQ) Elevation of Privilege Vulnerability
- CVE-2025-62454 Windows Cloud Files Mini Filter Driver Elevation of Privilege Vulnerability
- CVE-2025-62221 Windows Cloud Files Mini Filter Driver Elevation of Privilege Vulnerability
- CVE-2025-59517 Windows Storage VSP Driver Elevation of Privilege Vulnerability
- CVE-2025-59516 Windows Storage VSP Driver Elevation of Privilege Vulnerability
- CVE-2025-55233 Windows Projected File System Elevation of Privilege Vulnerability
- CVE-2025-54100 PowerShell Remote Code Execution Vulnerability

For more details, please refer to the Microsoft website:  
<https://portal.msrc.microsoft.com/en-us/security-guidance>.

### Impacted Mindray Products:

The following table lists the impacted device and those hotfixes determined to be applicable to each device:

Product	OS	Hotfix	Download website	Necessary Pre-installed patch
BeneVision CMS	Windows 10 Professional SP1 64bit 1809	KB5071544	<a href="#">windows10.0-kb5071544-x64_5fa88343a6db3dcae998e3197250446316995ee7.msu</a>	KB5005112 KB5003243 KB4587735
	Windows Server 2019	KB5071544	<a href="#">windows10.0-kb5071544-x64_5fa88343a6db3dcae998e3197250446316995ee7.msu</a>	KB5005112 KB5003243 KB4587735
	Windows 10 1607 for 32-bit	KB5071543	<a href="#">windows10.0-kb5071543-x86_cd23c0885d23abc88a3caab4a2352ea6e41a85ac.msu</a>	KB4498947 KB4132216
	Windows 10 1607 for x64-based	KB5071543	<a href="#">windows10.0-kb5071543-x64_025ec75233c046b89159ab58b47f216652240872.msu</a>	KB4498947 KB4132216
	Windows Server 2016	KB5071543	<a href="#">windows10.0-kb5071543-x64_025ec75233c046b89159ab58b47f216652240872.msu</a>	KB4498947 KB4132216
BeneVision CMS Viewer	Windows 10 Professional SP1 64bit 1809	KB5071544	<a href="#">windows10.0-kb5071544-x64_5fa88343a6db3dcae998e3197250446316995ee7.msu</a>	KB5005112 KB5003243 KB4587735
	Windows 10 1607 for 32-bit	KB5071543	<a href="#">windows10.0-kb5071543-x86_cd23c0885d23abc88a3caab4a2352ea6e41a85ac.msu</a>	KB4498947 KB4132216

	Windows 10 1607 for x64- based	KB5071543	<a href="#">windows10.0-kb5071543- x64_025ec75233c046b89159ab58b47f216652240872.msu</a>	KB4498947 KB4132216
Hypervisor X CMS	Windows 10 Professional SP1 64bit 1809	KB5071544	<a href="#">windows10.0-kb5071544- x64_5fa88343a6db3dcae998e3197250446316995ee7.msu</a>	KB5005112 KB5003243 KB4587735
	Windows Server 2019	KB5071544	<a href="#">windows10.0-kb5071544- x64_5fa88343a6db3dcae998e3197250446316995ee7.msu</a>	KB5005112 KB5003243 KB4587735
	Windows 10 1607 for 32-bit	KB5071543	<a href="#">windows10.0-kb5071543- x86_cd23c0885d23abc88a3caab4a2352ea6e41a85ac.msu</a>	KB4498947 KB4132216
	Windows 10 1607 for x64- based	KB5071543	<a href="#">windows10.0-kb5071543- x64_025ec75233c046b89159ab58b47f216652240872.msu</a>	KB4498947 KB4132216
eGateway	Windows 10 Professional SP1 64bit 1809	KB5071544	<a href="#">windows10.0-kb5071544- x64_5fa88343a6db3dcae998e3197250446316995ee7.msu</a>	KB5005112 KB5003243 KB4587735
	Windows Server 2019	KB5071544	<a href="#">windows10.0-kb5071544- x64_5fa88343a6db3dcae998e3197250446316995ee7.msu</a>	KB5005112 KB5003243 KB4587735
	Windows 10 1607 for x64- based	KB5071543	<a href="#">windows10.0-kb5071543- x64_025ec75233c046b89159ab58b47f216652240872.msu</a>	KB4498947 KB4132216
	Windows Server 2016	KB5071543	<a href="#">windows10.0-kb5071543- x64_025ec75233c046b89159ab58b47f216652240872.msu</a>	KB4498947 KB4132216
MLDAP Server	Windows 10 1607 for x64- based	KB5071543	<a href="#">windows10.0-kb5071543- x64_025ec75233c046b89159ab58b47f216652240872.msu</a>	KB4498947 KB4132216
	Windows Server 2016	KB5071543	<a href="#">windows10.0-kb5071543- x64_025ec75233c046b89159ab58b47f216652240872.msu</a>	KB4498947 KB4132216
	Windows Server 2019	KB5071544	<a href="#">windows10.0-kb5071544- x64_5fa88343a6db3dcae998e3197250446316995ee7.msu</a>	KB5005112 KB5003243 KB4587735
BeneVision Mobile Server	Windows Server 2016	KB5071543	<a href="#">windows10.0-kb5071543- x64_025ec75233c046b89159ab58b47f216652240872.msu</a>	KB4498947 KB4132216
	Windows Server 2019	KB5071544	<a href="#">windows10.0-kb5071544- x64_5fa88343a6db3dcae998e3197250446316995ee7.msu</a>	KB5005112 KB5003243 KB4587735
iView	Windows 10 1607 for x64- based	KB5071543	<a href="#">windows10.0-kb5071543- x64_025ec75233c046b89159ab58b47f216652240872.msu</a>	KB4498947 KB4132216

### Conclusion and Recommendation:

We have validated that the Mindray products of the latest version can perform to specification with the applicable patches applied to the OS. It is recommended that the applicable patches defined in the table above should be installed on the affected Mindray products.

If you need information on how to obtain the approved patches or have any question, please feel free to contact Mindray regional service engineer or Mindray Service Department (email: [service@mindray.com](mailto:service@mindray.com)).

Thank you for your kind attention and cooperation.

Sincerely yours,  
Mindray Service Department  
Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Release Time: 2026-01-04