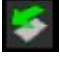




Importing Configurations Mindray iPM Patient Monitor

How to download iPM Standard configurations from the Mindray UK Website:

- Visit www.mindrayuk.com/education
- Find the iPM Standard Configuration zip file and download onto PC. Please note an encrypted USB stick cannot be used
- Unzip the file and copy the **MRPMCFG** folder and the **PAT** folder to a USB stick


To import the configuration from the USB drive to the monitor, follow this procedure:

- Connect the USB drive to the monitor's USB port
- Ensure the  icon appears on the bottom left hand-side of the touch screen to confirm USB detection
- Select the **Main Menu** quick key → select **Maintenance** → select **Manage Configuration**
- Input the password **315666** and select **OK**
- Ensure the **Department** is set to **General**
- If the **Department** is not set to **General**, select **Change Department** → select **General** → select **OK**
Please Note: Changing the Department will delete all current user configurations

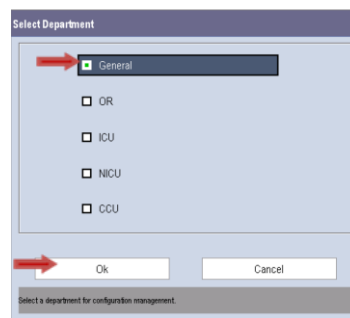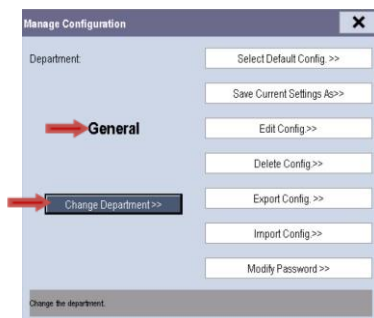


These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions. The illustrations may not necessarily reflect the setup or data displayed on your device.


- Select **Import Config** → select the **Configurations** and **User Maintenance Settings** to import iPM.STANDARD (Adu), iPM.STANDARD (Ped), iPM.STANDARD (Neo) → select **Import**

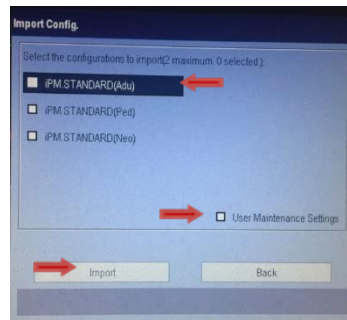

To set the default configuration, follow this procedure:

- In the **Manage Configuration** menu choose **Select Default Config** → select **Load Specified Config**
- Press the area labelled **Defaults** located next to **Default Adult Config / Default Ped Config / Default Neo Config** → select appropriate configuration → select **Back** → select **X** to exit the menus
-

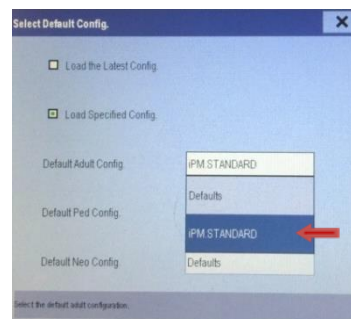


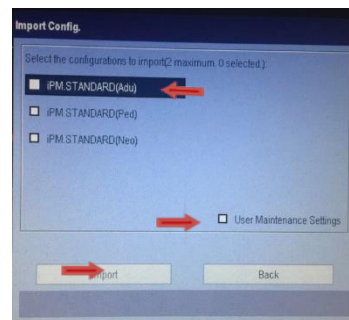
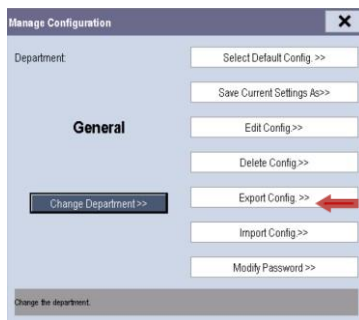
- Exit all menus by selecting **X** until you reach the **Main Menu**
- In the **Main Menu** select **Load Configuration** → select the config you wish to load → select **Load** → select **Yes**
- Select the  icon the bottom left hand-side of the touch screen → select **OK** when the warning menu appears stating 'Are you sure to unload the USB drive?'


Remove the USB drive from the monitor's USB port

These guidelines do **NOT** replace the instructions for use and all users should refer to the appropriate **Operator's Manual** for detailed instructions. The illustrations may not necessarily reflect the setup or data displayed on your device.

To export the configuration to the USB drive from the monitor, follow this procedure:

- Connect the USB drive to the monitor's USB port
- Ensure the  icon appears on the bottom left hand-side of the touch screen to confirm USB detection
- Select the **Main Menu** quick key → select **Maintenance** → select **Manage Configuration**
- Input the password **315666** and select **OK**
- Select **Export Configuration** → select the Configurations and **User Maintenance Settings** to Export → select **Export** → select **Back**

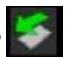

- When the configurations have been exported, the **Manage Configuration** Menu appears → Select **X** to exit all menus
- Select the  icon the bottom left hand-side of the touch screen → select **OK** when the warning menu appears stating 'Are you sure to unload the USB drive?'
-

Remove the USB drive from the monitor's USB port