




# Anaesthetic Gas Module CO<sub>2</sub> Measure & Standby modes Quick Guide

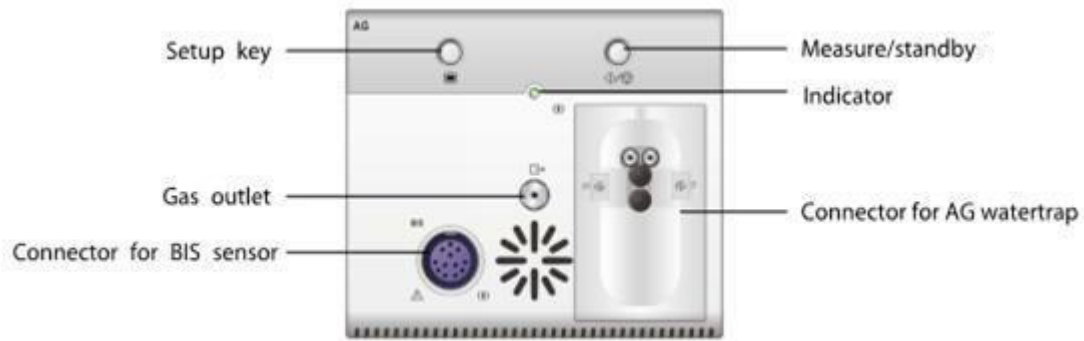

## To enter or exit the Standby mode manually

- Select the   hard key on the AG module (above the water trap) to enter or exit **Measure/Standby** mode
- Select **Operating Mode** in the **AG Setup** menu and toggle between **Measure** and **Standby**

## Information displayed when the Anaesthetic Gas Module is in Standby mode



When the Anaesthetic Gas Module is placed in standby mode, the sample intake pump sets to a sample flow rate of zero and the word **Standby** appears on the monitor display

When exiting **Standby** mode by pressing the   above the water trap, the CO<sub>2</sub> intake pump defaults to the preset flow rate

**Please Note:** The system configuration may be set to automatically place the Anaesthetic Gas Module into standby mode following a predetermined period of inactivity

These guidelines do NOT replace the instructions for use and all users should refer to the appropriate Operator's Manual for detailed instructions.  
The Illustrations may not necessarily reflect the setup or data displayed on your device.