



Anaesthetic Gas Module Measure & Standby modes Quick Guide

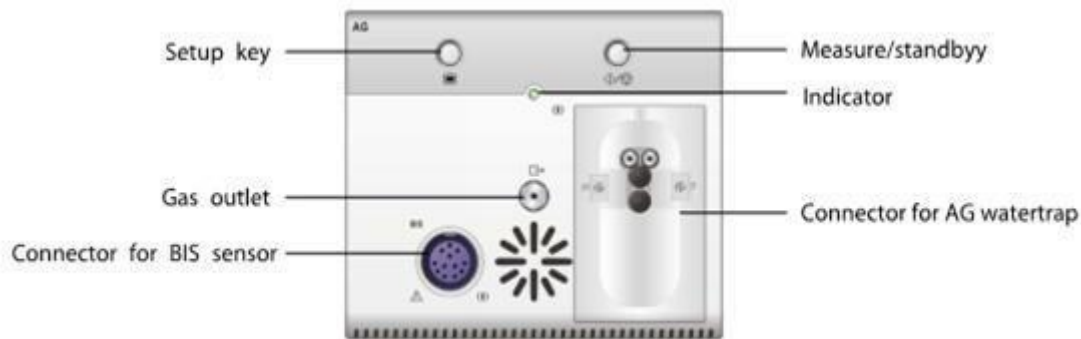

To enter or exit the Standby mode manually

- Select the  hard key on the AG module (above the water trap) to enter or exit **Measure / Standby** mode
- Select **Operating Mode** in the **CO₂ Setup** menu and toggle between **Measure** and **Standby**

Information displayed when the AG module is in Standby mode



When the CO₂ Module is placed into standby mode, the sample intake pump sets to a sample flow rate of zero and the word **Standby** appears on the monitor display

When exiting **Standby** mode by pressing  the sample intake pump defaults to the preset flow rate

Please Note: The system configuration may be set to automatically place the CO₂ Module into standby mode following a predetermined period of inactivity